



COMPRESSION THAT STANDS UP TO LYMPHEDEMA

DEFY GRAVITY



Help your patients stand up to lymphedema with
a **complete range of compression products** designed
with them in mind.

JOBST® Educational Resources

The JOBST® Compression Institute serves as a resource for healthcare providers and clinicians seeking further education about vein diseases and lymphedema. This website provides literature, videos, and links to our national affiliates. Additionally, the JOBST® Compression Institute offers a variety of nationwide workshops and continuing education classes to train you on JOBST® products

and proper measuring and fitting techniques.

Register at <http://www.jobstcompressioninstitute.com>.


To connect with a JOBST® Medical Advisor, please call our Customer Service team at **1-800-537-1063**.



PREFER TO SHOP ONLINE?

JOBST® eShop

- Place your JOBST® Custom Lymphology orders online
- Intelligent patient management
- Easy measurement documentation
- Time efficient ordering
- Voice enabled order form documentation with JOBST® Xpert Voice

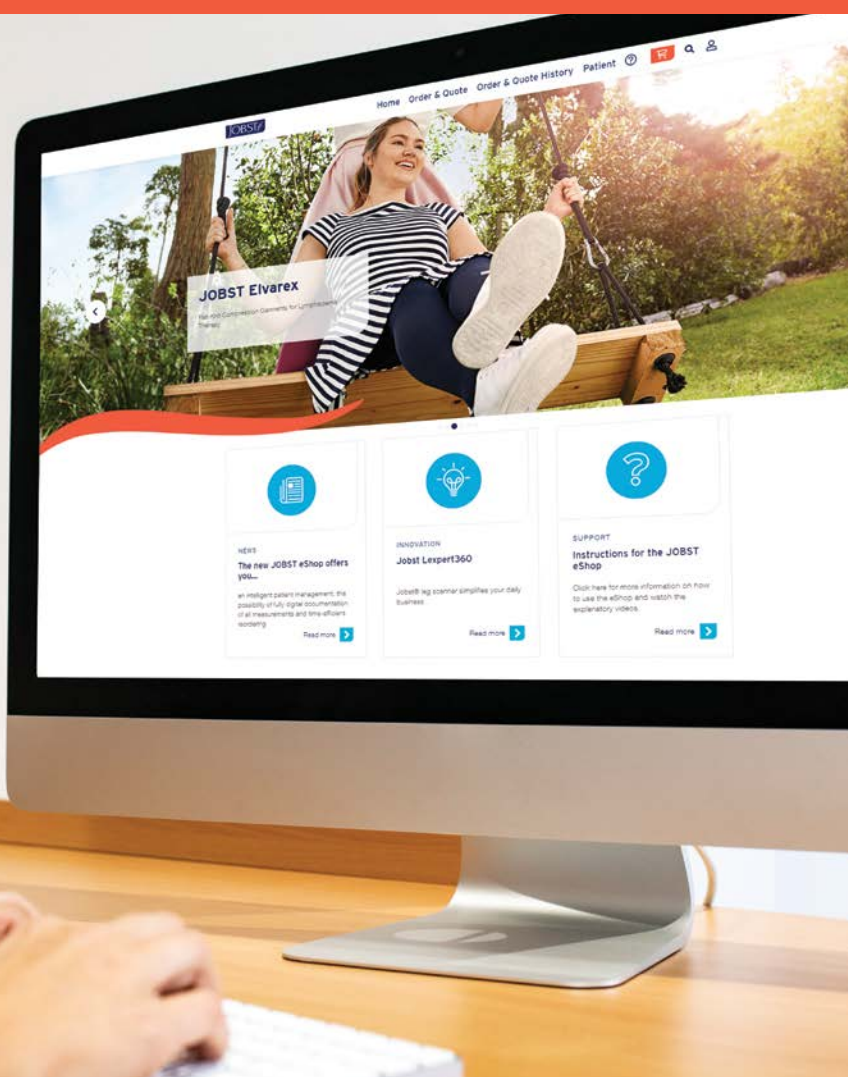

To get started, visit <https://eshop.jobst-usa.com>

Questions?

Email us at HMS-Elvarex-Orders@essity.com or call us at **800-537-1063**.

THE COMPLETE SOLUTION FOR LYMPHEDEMA

You are passionate about finding the right products for your patients, as you don't want them to be held down by their condition. JOBST® offers a complete portfolio of products to support you and your patients.

JOBST® Educational Resources / Custom eShop	2	JOBST® Custom Seamed	15
Why JOBST®?	4	JOBST® SoftFit™	
Compression Bandages		Band Variations.....	16
JOBST® Compri2.....	5	Functional Zones for JOBST® Elvarex®, JOBST® Elvarex® Soft and JOBST® Confidence.....	17
JOBST® FarrowWrap®	6-7	JOBST® Relax	22
JOBST® Daytime Garments Decision Tree.....	8	JOBST® JoViPak®	23-24
JOBST® Elvarex®	9	Bellisse®	
JOBST® Elvarex® Soft.....	10	Breast and Chest Wall Pads.....	25
JOBST® Confidence.....	11	JOBST® JoViPads®	
JOBST® Elvarex® Plus		JOBST® JoViLiner Arm and Leg	
JOBST® Elvarex® Soft Seamless	12	JOBST® PitPaks.....	26
JOBST® Custom Seamless Soft		JOBST® Donning Aids & Accessories	27
JOBST® Bellavar®	13		
JOBST® Bella™ Lite			
JOBST® Bella™ Strong	14		



WHY JOBST®?



TRUSTED BRAND

- JOBST® is the #1 physician-recommended brand of medical compression ready-to-wear legwear in the US* for the prevention, treatment and management of venous and lymphatic conditions.
- Conrad Jobst, mechanical engineer and industry pioneer, dedicated his life and expertise to inventing graduated compression garments in 1950.

INNOVATION

- JOBST® incorporates customer and healthcare professional involvement in the development of products. From focus groups to wash-and-wear tests, all garments go through a rigorous design process that ensures they provide the highest efficacy, safety, and satisfaction for our customers.
- Innovation is at the heart of our product excellence. Product development efforts are directed towards delivering the highest quality product with the highest assurance of medical efficacy.



PARTNERSHIPS

- JOBST® is an exclusive sponsor of the JOBST® Research Grant in Venous and Lymphatic Diseases in association with the American Venous Forum Foundation.
- JOBST® supports the National Lymphedema Network (NLN), Lymphatic Education and Research Network (LE&RN), and Brylan's Feat Foundation.
- JOBST® sponsors the JOBST® Lymphatic Research Grant through the Boston Lymphatic Symposium supporting research in lymphatics and lymphatic disease.
- Key JOBST® stakeholders are part of the Lymphedema Advocacy Group. Their mission is to advocate for improved insurance coverage for the diagnosis and treatment of the disease.



BOSTON LYMPHATIC
SYMPOSIUM

EDUCATION

- JOBST® Compression Institute serves as a valuable resource for fitters, clinicians, healthcare providers and caregivers interested in learning more about compression therapy and lymphatic and venous disorders.
- Our product experts, called Medical Advisors, and our devoted team of Sales Professionals, provide customized education and solutions for both fitters and therapists.



SOCIAL RESPONSIBILITY



- Every year, JOBST® donates hundreds of compression garments to healthcare organizations who share our mission to help improve patient quality of life. Whether it is funding education through organizations such as Camp Watch Me, the NLN Garment Fund, or the LE&RN, or helping those recover after a devastating natural disaster, JOBST® does not hesitate to lend a helping hand.

COMPRESSION BANDAGES

A complete range of bandage products for the management of lymphedema. Designed with the patient and therapist in mind for optimal Complete Decongestive Therapy (CDT) outcomes.

MULTI-LAYER BANDAGING



Comprilan®

- Short stretch compression bandage
- Provides high working pressure and low resting pressure
 - Easy-care and durable



Elastomull®

- Soft, comfortable bandage
- Precision fit in areas with constant movement
 - Air-permeable and comfortable to wear



Tricofix®

- Lightweight stockinette
- Protects the skin surface under compression, casts, or lymphedema wrapping
 - 100% cotton



CompriFoam®

- Open-cell foam bandage
- 100% polyurethane foam
 - Reusable



Isoband®

- Elastic multipurpose bandage
- Short stretch with longitudinal elasticity for high working pressure
 - Reusable
 - Porous weave provides good ventilation to the skin



Artiflex®

- Non-woven padding bandage
- Used in lymphology bandaging
 - Provides padding around bony or irregular shapes
 - Conformable and air-permeable
 - Supports even distribution of pressure

JOBST® COMPRI2 2-Layer Bandaging

The JOBST® Compri2 is a 2-layer bandage system* specifically designed for easy application. It provides graduated and reliable compression for up to seven days to help achieve effective edema reduction and promote wound healing.



1st Layer - Non Woven Padding

- **Stretchable** - simplifies application
- **Active Moisture Management** - helps keep skin dry and comfortable
- **Customized Padding** - allows for anatomical adjustments and additional protection to sensitive areas



2nd Layer - Short Stretch Bandage

- **Cohesive** - delivers reliable compression
- **Adjustable** - allows modification during application
- **Visual Indicator** - offers simple and effective application support

*As JOBST® Compri2 does not include a wound dressing, any wounds must be treated with a suitable wound dressing before the compression bandage is applied.

COMPRESSION WRAPS

JOBST® FARROWWRAP®

A guide to selecting the right JOBST® FarrowWrap®

APPLICATION TIME:

**43%
FASTER**

using JOBST® FarrowWrap®
compared to 4-layer
compression bandages.

COMPRESSION LEVEL

20-30 mmHg

LITE

Fits in RTW sizing
chart?

YES

LITE

YES

LITE TTF

NO

Fits in Trim-to-Fit
sizing chart?

NO

LITE CUSTOM

30-40 mmHg

Presence of lobules
or shape distortions?

YES

CLASSIC

Fits in RTW
sizing chart?

YES

CLASSIC

NO

NO

CLASSIC CUSTOM

Presence of
venous ulcer?

YES

4000

BASIC

STRONG

NO

STRONG

Fits in RTW
sizing chart?

YES

STRONG RTW

NO

Fits in Trim-To-Fit
sizing chart

YES

STRONG TTF

NO

STRONG CUSTOM

PATIENT SATISFACTION:

90%

of patients valued JOBST®
FarrowWrap® due to the ease
of attending self-hygiene.

COMPRESSION WRAPS

JOBST® FARROWWRAP®

A FarrowWrap® style to suit every need

JOBST® FarrowWrap® STRONG (30-40 mmHg*)


Premium fabric with strong compression and a soft, inner layer suitable for patients with good limb shape. Recommended for patients with moderate to severe edema. Available in Trim-to-Fit.

Colors


Tan




Footpiece^{1,2,3}

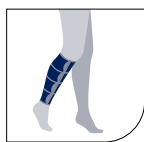

Legpiece^{1,2,3}

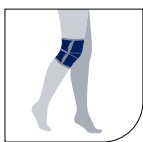

Kneepiece^{1,6}

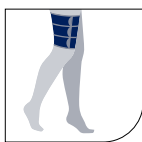

Thighpiece^{1,2,4}



Armpiece
(Custom)

FarrowWrap
4000



JOBST® FarrowWrap® CLASSIC (30-40 mmHg*)


Made from a more rigid material than JOBST® FarrowWrap® STRONG. Ideal for patients with large lobules, skin folds, or shape distortions.

Colors


Tan




Footpiece^{1,2}

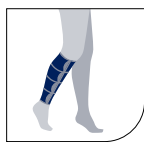

Legpiece^{1,2}

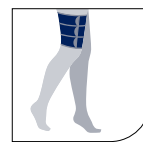

Thighpiece^{1,2,4}

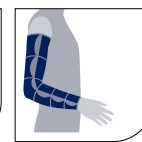

Armpiece²
(Custom)

Improved
VELCRO® with
rounded corners.
Landing strip at
each strap.

JOBST® FarrowWrap® BASIC (30-40 mmHg*)


A value fabric with quality compression. The double-sided VELCRO® allows the band length to be trimmed as the limb reduces.

Colors

Tan



Footpiece¹

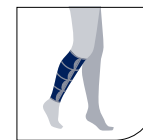

Legpiece¹
(Includes FarrowHybrid Sock)


JOBST® FarrowWrap® 4000 (30-40 mmHg*)

JOBST® FarrowWrap® 4000 is ideal for patients with moderate to severe venous and lymphatic conditions, with or without a venous leg ulcer. The inner sleeve and 4-band juxtaposing design allow for easy application. Anatomically contoured to help prevent gapping.

Colors

Tan

Black

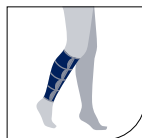

Legpiece¹
(Includes FarrowHybrid Sock)

JOBST® FarrowWrap® LITE (20-30 mmHg*)


A double-laminated, durable, and breathable fabric especially suitable for home-care use/self-management. Available in Trim-to-Fit.

Colors


Tan




Footpiece^{1,2,3}

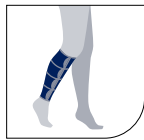


Legpiece^{1,2,3}

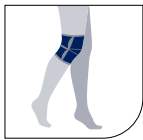

Kneepiece^{1,5,6}

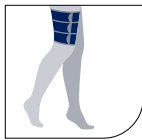

Thighpiece^{1,2,4}

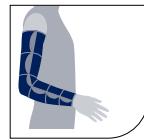

Armpiece^{1,2,3}



Hand Gauntlet**

The JOBST® FarrowWrap® Hand Gauntlet manages swelling in the hand and can be used on either the left or right hand. Includes removable 4 mm and 8 mm foam piece



Toe Cap**

(15-20 mmHg*, 20-30 mmHg*)

Trimable Ready-To-Wear toe cap for managing swelling in the forefoot and toes. Can be used on either foot, and is available in black and tan.



FarrowHybrid ADI Foot Compression Sock

Below knee high liner with 20-30 mmHg of compression in the foot and ankle area. Included with BASIC and 4000 Legpiece.



Easy Donning

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Short-Stretch Technology

Short-Stretch Technology provides low-resting and high-working pressures to enhance lymphatic and venous return.



Adjustable

VELCRO® brand hook & loop system allows patients to respond better to fluctuating edema by simply tightening or loosening the wrap.



50% Band Overlap

Optimal width of bands - combined with 50% overlap of bands from bottom to top - provide ideal compression.



Liner Options

- Standard Liner
- TG Soft Liner
- FarrowHybrid Liner

Additional Accessories

- Garmet Grip
- VELCRO® Pack
- Short-Strech Wrap Extra Band

*The average compression for a medium sized ankle.

**Hand Gauntlet and trimmable RTW Toe Cap for managing mild to moderate edema.

1. RTW (Ready-To-Wear) 2. CM (Custom Made) 3. TTF (Trim-To-Fit) 4. Thighpiece must be worn with Kneepiece 5. STRONG kneepiece can be combined with CLASSIC garment 6. Kneepieces included with Ready-to-Wear Thighpieces. Kneepieces can also be ordered separately

FLAT-KNIT DECISION TABLE:

JOBST® Confidence vs JOBST® Elvarex® Soft vs JOBST® Elvarex®

Body Shape			
Stage	Mild (I)	Moderate (II)	Severe (III)
Patient Description	First time user, newly diagnosed, preventative treatment, Lipedema/ Lymphedema	First time user, low tolerance to stiff fabric, desire to “feel” the compression, Lipedema/ Lymphedema	Persistent/aggressive edema, Lymphedema (not best for Lipedema)
Skin Folds	None	Minimal	Significant
Tissue	Proportionate, symmetrical, athletic, controlled/maintained edema	Adipose present, beginning development of fibrosis	Significant adipose and/or hard, fibrotic
User Need	Light & Soft material	Soft & Flexible material	Firm & Strong material
JOBST® Product Recommendation	JOBST® Confidence		JOBST® Elvarex® Soft
			JOBST® Elvarex®

DAYTIME GARMENTS


JOBST® ELVAREX®

JOBST's Gold Standard

JOBST® Elvarex® represents the brand's gold standard in custom daytime garments for the management of lymphedema. These flat-knit garments deliver exceptional comfort and containment, even for the most severe cases. Featuring a **natural rubber latex core***, Elvarex® offers outstanding structure and control for patients across a wide range of shapes and sizes.

KEY FEATURES

- Natural rubber latex core provides superior containment and allows garment to return to original shape for extended wear
- Flat-knit structure bridges skin folds instead of working into them
- Seven compression classes for optimal therapeutic outcomes
- Excellent breathability for comfort and patient compliance
- Vast product assortment covers a diverse range of specifications and requirements of patients
- SPF 80 UV protection

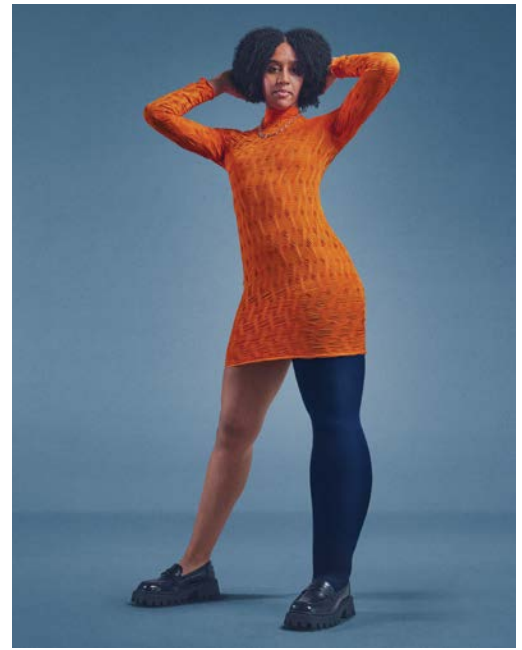

EXTREMITIES/COLORS


Upper Limb

Beige	Black	Navy	Caramel	Cranberry	Cherry	Hazelnut	Bronze
-------	-------	------	---------	-----------	--------	----------	--------

Lower Limb

Beige	Black	Navy	Cocoa	Cranberry	Gray	Henna**	Graphite**
Denim**	Stone	Aubergine	Caramel	Bronze			

STYLES

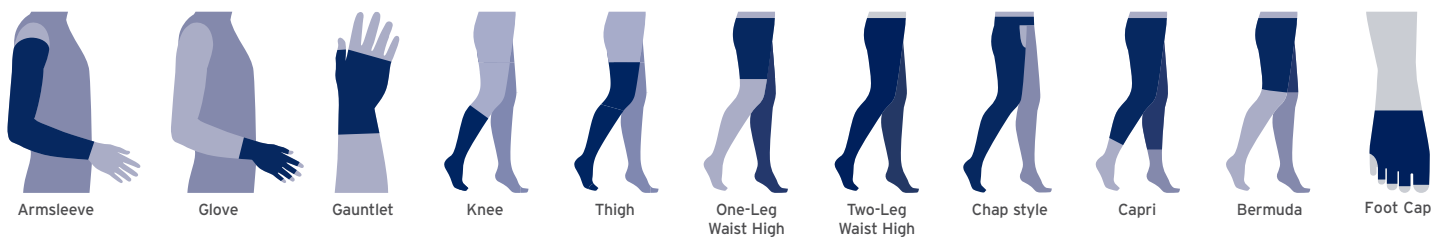

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 2 Forte ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg	CCL 3 Forte ▶ 34-46 mmHg
CCL 2 ▶ 23-32 mmHg	CCL 4 ▶ 49-70 mmHg
CCL 3 ▶ 34-46 mmHg	CCL 4 Super ▶ 60-90 mmHg

*CAUTION: These products contain natural rubber latex which may cause allergic reactions.

**Not available in Foot Cap

DAYTIME GARMENTS

JOBST® ELVAREX® SOFT

Gentle and Not Made with Natural Rubber Latex

JOBST® Elvarex® Soft is designed for patients seeking a softer, gentler compression garment. The flat-knit fabric supports easier donning and meets the specific needs of many elderly and pediatric patients, as well as patients with lipedema. Despite its soft and forgiving feel, Elvarex® Soft still provides the containment needed for patients with mild to moderate lymphedema.

KEY FEATURES

- Soft, skin-friendly flat-knit fabric with excellent breathability for comfort
- Latex-free construction and easy donning for improved daily wear
- Anatomically correct fit helps prevent rolling, digging, and discomfort
- Proven efficacy for mild to moderate lymphedema with appropriate fabric stiffness
- Available in a broad assortment of upper and lower extremity styles
- Optional silicone bands provide additional grip and secure hold



EXTREMITIES/COLORS


Upper Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry	Honey
-------	-------	------	-------	------	-----------	--------	-------

Lower Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry
-------	-------	------	-------	------	-----------	--------

STYLES

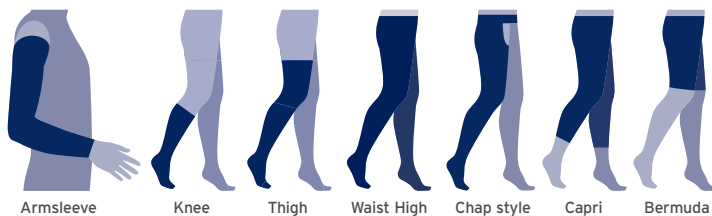

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 3 ▶ 34-46 mmHg



DAYTIME GARMENTS

JOBST® CONFIDENCE


Next Generation of Made-to-measure Flat-knit Compression Garments

JOBST® Confidence helps patients with mild to moderate lymphedema and lipedema feel unrestricted with a garment uniquely tailored to their individual shape. Using a new generation of flat-knit technology, JOBST® Confidence enables a precise anatomical fit that delivers both comfort and a greater degree of movement.


KEY FEATURES

- Innovative 3D Contour Fit technology adapts to individual body shapes for a highly personalized anatomical fit
- Supple, conforming flat-knit fabric provides firm support while allowing greater freedom of movement
- Advanced double-layer moisture management system helps wick moisture and maintain all-day comfort
- Functional zones* support natural mobility and reduce restriction
- New design options—including decorative seams, motivational prints, and expanded colors—allow personalization without compromising support
- Panty styles feature ergonomic seam placement for increased comfort; men's versions available with dedicated fit options
- Not made with natural rubber latex


*Available for knee, ankle, and elbow

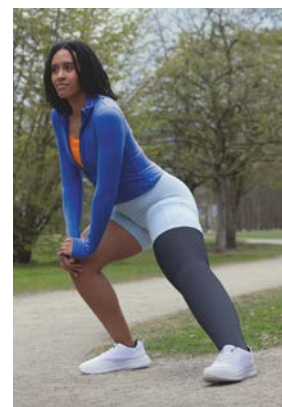

COLORS

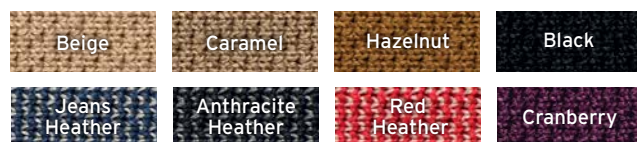

MOTIVATIONAL WAISTBAND

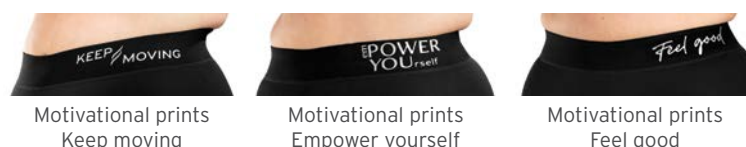

SEAMS




DESIGN OPTIONS



COMFORT OPTIONS



STYLES

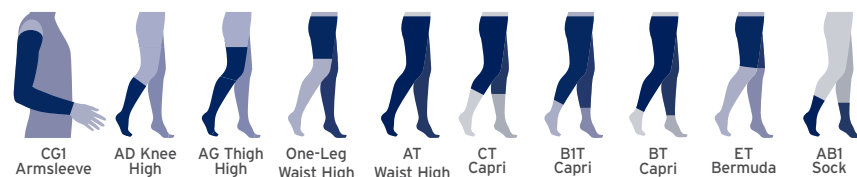

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg

CCL 2 ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg

CCL 2 ▶ 23-32 mmHg

CCL 3 ▶ 34-46 mmHg

PATIENTS PREFER THE COMFORT OF JOBST® CONFIDENCE

RANGE OF MOTION:

85% of patients said they can move freely with JOBST® Confidence.

MOISTURE MANAGEMENT:

92% of patients agreed JOBST® Confidence keeps the skin dry.

GENERAL WEARING COMFORT:

84% of patients evaluated the comfort of JOBST® Confidence as good/very good.

PATIENT SATISFACTION:

7/10 patients said they would recommend JOBST® Confidence.

DAYTIME GARMENTS

JOBST® ELVAREX® PLUS*

The Cutting Edge of Flat-Knit Technology

JOBST® Elvarex® Plus garments combine innovative 3D knitting with seamless flat-knit technology to deliver advanced comfort and performance. Constructed with a material that contains natural rubber latex, these seamless designs help reduce potential irritation between the fingers and toes and at the base of the thumb—providing patients with both the comfort they want and the containment they need.

Caution: This product contains natural rubber latex which may cause allergic reactions.




Patented 3D Technology

KEY FEATURES

- Unique 3D knitting technology provides a “second-skin” feel and reproduces natural contours for a better anatomical fit
- Seamless construction reduces the risk of pressure marks and supports greater mobility during daily activities
- Open-loop knitting structure promotes a high degree of air permeability for enhanced comfort

STYLES

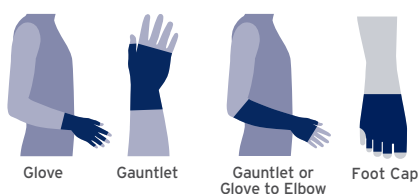

COMPRESSION CLASSES


CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)

CCL 1 ▶ 18-21 mmHg (Foot Cap)


CCL 2 ▶ 23-32 mmHg

EXTREMITIES/COLORS

Upper Limb

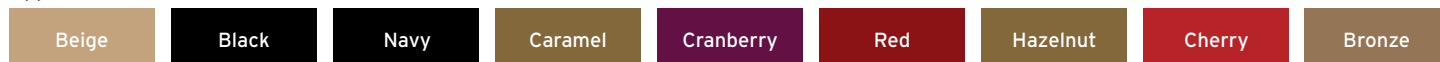

Lower Limb

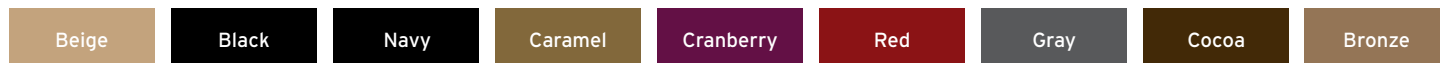

JOBST® ELVAREX® SOFT SEAMLESS

Soft, Seamless Comfort for Hands and Toes


JOBST® Elvarex® Soft Seamless combines all the benefits of Elvarex® Soft with a seamless construction designed specifically for gloves and toe caps. These soft, flat-knit garments are made without natural rubber latex and provide a smooth, irritation-reducing fit by eliminating seams between the fingers and toes. This design helps patients experience both the comfort they want and the performance they need.



KEY FEATURES

- Anatomically shaped 3D construction provides a precise fit and supports effective edema control in the web spaces
- Seamless, flat edge finish at fingers and toes helps reduce irritation, decrease bulk, and prevent curling
- Soft, breathable flat-knit fabric with dual-covered inlay yarn ensures controlled compression and comfortable all-day wear

STYLES

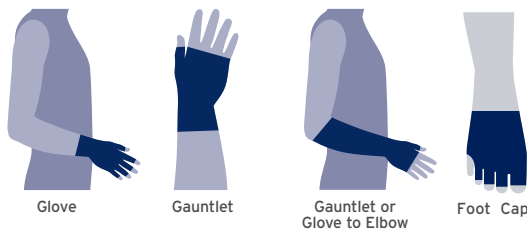

COMPRESSION CLASSES

CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)

CCL 1 ▶ 18-21 mmHg (Foot Cap)

CCL 2 ▶ 23-32 mmHg

EXTREMITIES/COLORS

Upper Limb



Lower Limb



DAYTIME GARMENTS

JOBST® CUSTOM SEAMLESS SOFT

Custom-Made Compression Hosiery

JOBST® Custom Seamless Soft is a custom-made circular knit garment that looks and feels like hosiery. Appropriate for mild to moderate edema and vascular disorders, these garments wick away moisture and help keep skin cool and dry. With Custom Seamless Soft, patients do not have to compromise on fit or style to wear medical grade compression.


KEY FEATURES

- Multi-fiber double-covered inlay yarns increase softness and improve donning
- Refined opacity helps cover blemishes and scars
- High-stretch core yarns increase recoil (stretch and recovery) to help prevent sliding
- Reinforced foot zones increase strength and durability with added comfort

COLORS



STYLES

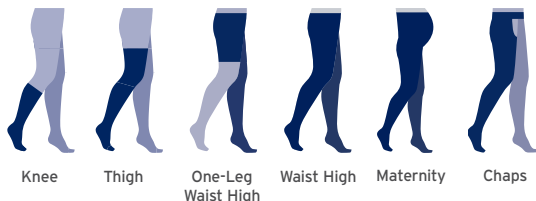

COMPRESSION CLASSES



JOBST® BELLAVAR®

Firm Circular Knit Containment

JOBST® Bellavar® is a seamless, custom-made circular-knit compression garment designed for effective management of moderate to severe leg edema. Constructed from fine-meshed, breathable material, Bellavar® delivers skin-friendly comfort while its firm fabric provides controlled stretch and resistance. The garments offer a higher level of containment than JOBST® Custom Seamless Soft and are available in a variety of made-to-measure styles to meet individual patient needs.

KEY FEATURES

- Firm, custom-made circular knit construction provides controlled stretch and resistance for managing moderate to severe edema
- Seamless design offers a higher level of containment than JOBST®
- Custom Seamless Soft for effective leg edema management
- Breathable, fine-meshed material improves comfort by allowing air passage and supporting skin-friendly wear
- Flexible, durable fabric enables easier donning and long-lasting daily use


COLORS



STYLES



COMPRESSION CLASSES

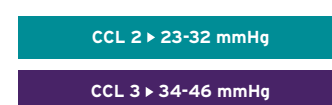

DAYTIME GARMENTS

JOBST® BELLA™ LITE

The Entry Point to JOBST® Quality

The JOBST® Bella™ Lite Ready-to-Wear Armsleeve, Gauntlet, and Glove help maintain limb size for lymphedema patients. Worn individually or together, these comfortable garments provide gradient compression to support patient management following trauma, surgery, or radiation therapy.



KEY FEATURES

- Soft, lightweight, and air-permeable knit construction for comfortable daily wear
- Fits a wide range of arm and hand shapes, with a silicone dot band option available
- Reduced compression at the wrist provides an effective and comfortable fit when the armsleeve and gauntlet are worn together
- Trimmable glove design allows customization to individual finger lengths


COLORS

Beige

Black**

Brown**

STYLES

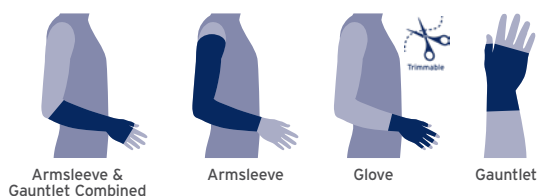

COMPRESSION CLASSES

15-20 mmHg*

20-30 mmHg*

*All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet). Not all colors and styles may be available in all compression ranges.

**Available in select styles.

JOBST® BELLA™ STRONG

Comfortable, Cost-Effective Compression

JOBST® Bella™ Strong upper-extremity garments offer a ready-to-wear solution for patients who need to manage their lymphedema but do not require the containment of a flat-knit custom garment. Bella™ Strong makes JOBST® quality more accessible through a broad range of sizes, lengths, compression classes, and value-driven pricing.

KEY FEATURES

- Wide range of sizes and lengths accommodates a broad spectrum of patient dimensions
- Easy to don, with armsleeve options available with an included donning device
- Comfort Flat wrist seam helps reduce rolling and supports a smooth, comfortable fit
- Soft, cotton-like feel enhances comfort for everyday wear

COLORS

Beige

Black

COMPRESSION CLASSES



15-20 mmHg*

20-30 mmHg*

30-40 mmHg**



STYLES

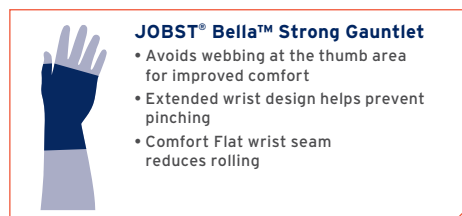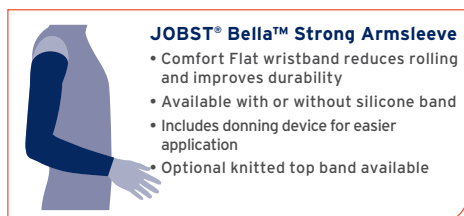

*All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet).

**Bella™ Strong Gauntlet and Glove styles not available in 30-40 mmHg.

DAYTIME GARMENTS

JOBST® CUSTOM SEAMED

Nearly Limitless Solutions for Unique Challenges

JOBST® Custom Seamed garments were the first of their kind to manage lymphedema, and our ability to solve unique and complex compression challenges continues today. These warp-knit, custom-made garments are available for virtually any body part and in countless configurations. Each garment is precisely sewn by hand in Hickory, North Carolina to ensure exceptional accuracy, durability, and individualized fit for a wide range of patient needs.

KEY FEATURES

- Durable warp-knit fabric provides long-lasting wear for both basic and complex compression needs
- Computer-designed pattern supports precise gradient compression
- Hand-sewn construction ensures a custom fit for any body part and any size
- Variety of customizable options, including zippers, linings, and silicone bands

COLORS

Beige

Black

COMPRESSION CLASS

15-65+ mmHg




STYLES



Our founder, Conrad Jobst, had venous insufficiency like many of our consumers. He refused to accept that there was nothing doctors could do for him. From illness came innovation.

He loved swimming, where he felt weightless in the water. Inspired by that feeling, he created a sock that squeezed his leg, just as the water did. No matter where he went, he no longer felt weighed down.

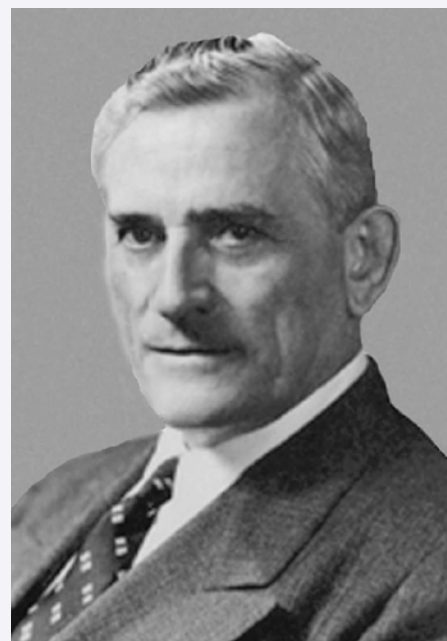

DAYTIME GARMENTS

JOBST® SOFTFIT™

Our innovative JOBST® SoftFit™ technology is a unique system of silicone yarns knitted into the welt of our compression garments. It provides gentle, seamless friction on the skin, decreasing the chance of constriction and slipping down.

LOOKS GOOD, FEELS GOOD

With the SoftFit™ band, garments are not only visually appealing and more comfortable to wear, but they also provide precisely the right amount of pressure where it is needed.

IMPROVED COMPLIANCE, IMPROVED QUALITY OF LIFE

Compression garments that feel comfortable lead to greater patient compliance.

MORE OPTIONS, HAPPIER PATIENTS

SoftFit™ Technology is offered in a wide variety of compression garments and styles.



BAND VARIATIONS

Product	Dotted Silicone	Lace Silicone	SoftFit	Sensitive	Ribbed Fleece Waistband, Adjustable Waistband	MicroDot	Floral Waistband
Elvarex® Upper Extremity	2.5 cm, on top, inside 5 cm, on top, inside		2.5 cm, on top (CCL 1 and 2 only)			5cm, on top	
Elvarex® Lower Extremity	2.5 cm, on top, inside 5 cm on top, inside 5 cm on top, waistband		5 cm, on top (Knee High only, CCL 1 thru 3)	5 cm, on top (Waist High only)	5cm, on top (Waist High only)	5cm, on top	
Elvarex® Soft Upper Extremity	2.5 cm, on top, inside 5cm, on top only		2.5 cm, on top			5cm, on top	
Elvarex® Soft Lower Extremity	2.5 cm, on top (Knee High only) 5 cm, on top		5 cm, on top (Knee High only)		(Waist High only)	5cm, on top	
Confidence	5 cm, on top (Thigh High only) 2.5 cm on top for Armsleeve		5 cm, on top (Knee High only) 2.5 cm on top for Armsleeve				(Waist High only)
Bella™ Lite Armsleeve	5 cm, on top						
Bella™ Strong Armsleeve	5 cm, on top						
Custom Bellavar	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)					
Custom Seamless Soft	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)	5 cm, on top, (Knee High only) (CCL 1 and 2 only)	5 cm, on top, (Thigh High only)			
Custom Seamed	2.5 cm, on top 5 cm, on top			5 cm, on top			

FUNCTIONAL ZONES FOR JOBST® ELVAREX®, ELVAREX® SOFT & CONFIDENCE*

Functional Zones are created using highly specialized yarns and advanced knitting techniques. These targeted zones increase comfort through improved flexibility and breathability while maintaining medical effectiveness. They are designed to enhance mobility around the elbow, knee, and ankle by adapting to movement and reducing wrinkling, all while retaining effective compression.

JOBST® FUNCTIONAL ZONES

- Created using highly specialized yarns and advanced knitting techniques
- Increased breathability and flexibility for more comfortable wear
- Designed to improve fit and enhance comfort around joints
- Reduce wrinkling when bending while maintaining effective compression
- Adjust flexibly to movement to support greater mobility

Functional Zone
"Bias top end"



Functional Zone
"Elbow Comfort Zone"



Functional Zone
"Pressure Panty
Elvarex®
(body-bandage)"



Functional Zone
"Top Comfort Zone"



Functional Zone
"Knee Comfort Zone"



Functional Zone
"T-Heel"



Functional Zones
"Slant Foot"
"Hallux Valgus"

*Caution: These product contain natural rubber latex which may cause allergic reactions.

DAYTIME GARMENTS

JOBST® ELVAREX®

Custom-Made Garments

AVAILABLE STYLE AND OPTIONS

I. UPPER EXTREMITY GARMENTS

A. Glove/Gauntlet



AC1 Glove or Gauntlet
• with or without fingers
• with or without thumb



AE Glove or Gauntlet
• with or without fingers
• with or without thumb
• normal elbow termination or outside higher



Zipper Options
• lining at back of hand
• lining inside of hand
• lining on top of arm sleeve
• measure point C-D
• measure point E-G



Pocket Options
• back of hand
• back of palm

Requires pocket width and length

B. Armsleeve



CG1 Armsleeve or CG Armsleeve with no slant top



AG1 Armsleeve with Glove or Gauntlet Attached



AH or CH Armsleeve w/ Shoulder Cap
• Normal Shoulder Cap
• Full Compression Shoulder Cap



Arm Sleeve Strap
• without bra/shoulder strap
• bra strap
• bra strap/velcro



Silicone Dotted Band (on top)
• 2.5cm
• 5cm



Silicone Dotted Band (inside)
• 2.5cm
• 5cm

DAYTIME GARMENTS

JOBST® ELVAREX®

Custom Made Garments

AVAILABLE STYLE AND OPTIONS

2. LOWER EXTREMITY GARMENTS

A. Knee High



AD Knee High

- open toes
- closed toes
- straight foot
- slant foot



Band Options

- without band
- SoftFit
- Dotted silicone band inside
- Dotted silicone band 2.5cm on top
- Micro-dotted silicone 5cm on top



Zipper Options

- Lining Inside
- Lining Outside
- Lining Front



Other Options

- Lateral Rise
- T-Heel
- Profile (standard, inside, outside)
- Lining
- Label

B. Thigh High



AG Thigh High

- open toes
- closed toes
- straight foot
- slant foot



Lateral Rise

- Outside Higher

Top Comfort Zone also available



No Lateral Rise

- Straight Top



Band Options

- Without Volant
- With Volant
- Dotted Silicone Band Inside
- Dotted Silicone Band 2.5cm on top
- Dotted Silicone Band 5cm on top
- Micro-dotted Silicone Band 5cm on top

Other Options

- Knee Comfort Zone
- T-Heel
- Profile (standard, inside, outside)
- Lining
- Label

DAYTIME GARMENTS

JOBST® ELVAREX®

Custom-Made Garments

AVAILABLE STYLE AND OPTIONS

2. LOWER EXTREMITY GARMENTS

C. Chaps/Panty



AG-T or CG-T Chap

- Left
- Right
- Pair



AT Waist High/Panty



One Leg Panty (with body bandage and leg extension)

- Styles in BG-HT, B1G-HT, CG-HT



Panty Part Option

- Elvarex Male Panty with fly
- Elvarex Male Panty without fly



Capri Panty

- Styles in BT, B1T, CT



Bermuda FT



Silicone Band - Inside

- 2.5cm or 5cm silicone band inside at 1/2 or 3/4 length



Silicone Band - Slip Form

- Multiple length options for silicone band pieces in slipform
- 10cm, 12cm, 15cm, 20cm, 25cm, 30cm



Silicone Band - Leg Extension

- Leg Extension with 2.5cm or 5cm silicone band



D. Foot/Toe Cap



Slant foot with open toe is standard for Elvarex®



Straight Foot



Closed Toe

- Made from soft material with no compression.



T-Heel

- For reduced sensation of pressure in the foot area. The special knit construction in the heel and instep spreads the pressure evenly and reduced the formation of creases.



Profile

- Pads are used to alleviate pressure on the ankle by distributing compression.



DAYTIME GARMENTS

JOBST® ELVAREX®

Custom Made Garments

AVAILABLE STYLE AND OPTIONS

2. LOWER EXTREMITY GARMENTS

D. Foot/Toe Cap Continued



AA Foot Cap
• Without toes
• 4 Toes
• 5 Toes

E. Possible Combinations



Knee High + Capri Style



Thigh High + Bermuda Shorts



Knee High + Thigh Sleeve + Bermuda Shorts

NIGHTTIME GARMENTS

JOBST® RELAX

Redefining Lymphedema Night Care

JOBST® Relax is a revolutionary nighttime compression garment designed for more than just sleeping. Its monofilament spacer fabric technology creates a lightweight, flexible, and slim-line design that helps maintain edema reduction and counteract fluid accumulation at night. Made with advanced flat-knit techniques and CoolMax® yarns, JOBST® Relax is highly breathable and comfortable, supporting greater patient compliance throughout the evening hours. The innovative Comfort Spacer Layer enhances heat and moisture management while promoting freedom of movement for a better night's rest.



COMFORT SPACER LAYER




KEY FEATURES

- Custom-made flat-knit construction provides gradient compression therapy and stays securely in place
- Comfort Spacer Layer enhances breathability, heat management, and nighttime comfort
- Textured surface with monofilament spacer supports stimulation of lymph flow while at rest
- Soft, flexible, and slim-line fabric reduces wrinkling into skin folds and promotes freedom of movement during sleep

COLORS




STYLES

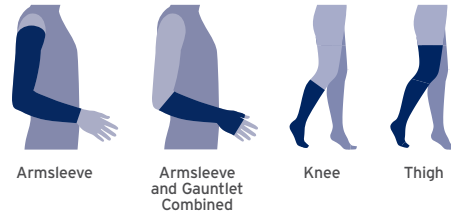

COMPRESSION CLASSES



* 20-30 mmHg compression is only available in lower extremity.

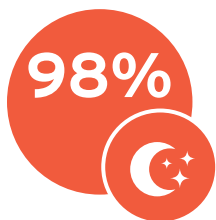


BETTER QUALITY OF SLEEP:

98% of patients rated quality of sleep **good or very good** compared to when wearing previous night-time products.



EDEMA CONTROL:

Many patients reported a major improvement in their lymphedema the next morning, with 90% of patients describing handling of day compression to be **good or very good** after using JOBST® Relax at night.

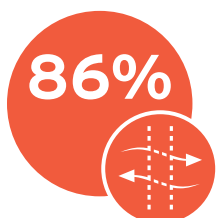

BREATHABILITY:

86.8% rated JOBST® Relax as **good or very good** in terms of avoiding heat generation and perspiration, respectively.



NIGHTTIME GARMENTS

JOBST® JOVIPAK®

Handmade Lymphedema Night Care

JOBST® JoViPak® garments offer an easy and comfortable alternative to nighttime bandaging for patients with moderate to severe lymphedema. Their unique stitched channeling and blended-density chip foam construction are designed to stimulate lymph flow, address fibrotic tissue, and manage even complex edema presentations. Available in both ready-to-wear and custom designs for virtually every part of the body, each garment is handmade with care, and can be paired with the JOBST® JoViJacket for enhanced nighttime compression therapy.

KEY FEATURES


- Chip-foam blend and stitched channeling create multiple pressure points designed to stimulate lymph flow and soften fibrotic tissue
- Available in both Ready-to-Wear and Custom configurations with styles for virtually every body area
- Comfortable, breathable construction offers an easier alternative to nighttime bandaging
- Durable materials and up to four types of foam provide long-lasting performance without unpleasant chemical odors

COLORS

Cotton




Polartec® Power Dry®

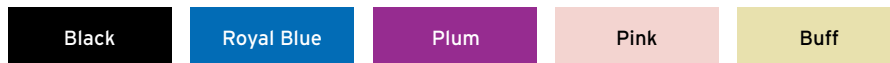

PowerNet



Super PowerNet

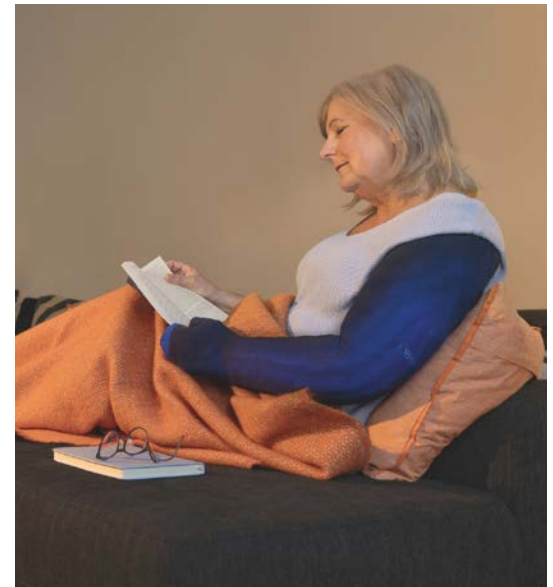

Continuous Flow Channeling Technology

- Aims to support lymph fluid drainage

Polartec® Power Dry® Fabric

Garments made with Polartec® Power Dry® fabric use special microfiber yarns and a unique weave to disperse moisture and heat across a wide surface area.

- Helps quick drying of fabric

Outer fabric surface

Inner next-to-skin surface

Blended Foam Construction

Blended-density foam creates multiple pressure points throughout the garment.

- The varying, prolonged pressure on fibrotic tissue may improve patient outcomes

Moisture-Managing Yarns

Smooth, highly durable yarns are optimized to help move moisture away from the skin.

- Designed for comfortable wearing

Garments made of **Organic Cotton/Lycra®*** may be a better choice for patients with skin sensitivity to polyester and for products with **Velcro®** closures, as Velcro® can snag **Polartec® Power Dry®** fabric.

*Lycra® is a registered trademark of Invista. Polartec® and Power Dry® are registered trademarks of Polartec®, LLC.

NIGHTTIME GARMENTS

Head and Neck

- Standard & Extended Chin Strap
- Half & Full Face Mask
- Neck Pads
- Eye Pad

Torso

- Shoulder-Torso Sleeve
- Vest
- Busti

Genital area

- Boxer
- Boxer Capri
- Female & Male Genital Pads
- Custom Leg (DG1)

Upper Extremities

- Classic Arm Sleeve
- Two-Piece Arm Sleeve
- Stitched Finger Glove/
Palm Pad/Dorsal Pad

Lower extremities

- Classic Full Leg
- Classic Lower Leg
- Innaboot Full Leg &
Lower Leg
- Hip-Hugger
- Custom Leg (AF1)
- ADVI

JOBST® JoViPads and JOBST® PitPaks can be used in treatment on any area of the body.



NIGHTTIME GARMENTS

BELLISSE®

Supporting Healing After Surgery

Bellisse® is uniquely designed to promote healing after breast surgery by providing uniform, gentle compression around the torso. When used in conjunction with JOBST® JoViPads®, Bellisse® encourages drainage of edema in the chest walls, which can aid in recovery.

KEY FEATURES

- Ready-to-wear and available in a wide range of sizes
- Exceptional adjustability with a front zipper for easy donning and a variable-position back closure
- Attractive shape that can be worn comfortably day and night
- Soft against the skin, smooth while sleeping, and unobtrusive under clothing
- Extender can be used for adjustment by adding three additional rows of eye loops
- Made of Polyester / Nylon / Spandex



COLORS

Buff

Black


SIZING RANGE

32 A/B - 50 DD/E*

*Please check order form with measuring instructions to determine the correct size.

BREAST AND CHEST WALL PADS


JOBST® JoViPads® provide targeted support for managing breast and chest wall edema after surgery. These soft, foam-filled pads are designed to fit under the Bellisse® or other compression garments to promote lymphatic flow and protect sensitive tissue. The assortment includes options that address postoperative needs such as unilateral or bilateral mastectomy, lumpectomy, lateral or inferior breast edema, axillary support, and chest wall fibrosis. All pads are made from Polartec® Power Dry® fabric with a polyurethane foam core to offer comfort, breathability, and gentle pressure where it is most needed.

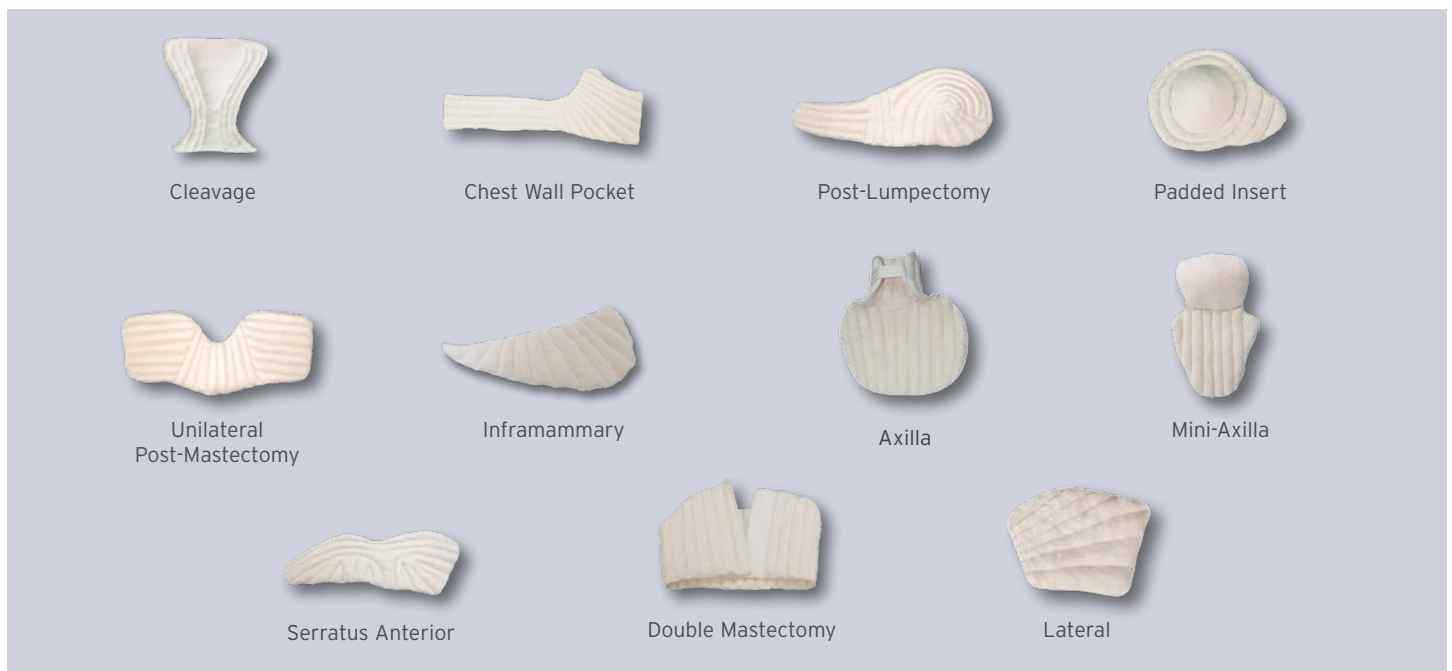

NIGHTTIME GARMENTS

JOBST® JOVIPADS®

JOBST® JoViPads® provide localized pressure and comfortable cushioning to help manage lymphedema, other edema, and fibrosis. Their blended foam and stitched channeling create multiple pressure points under compression to stimulate lymph flow and soften fibrotic tissue. These pads come in a variety of anatomical shapes to support areas such as joints, bony prominences, the chest wall, limbs, the foot dorsum, and other focused regions affected by swelling or fibrosis. Each pad is made with soft organic cotton and polyurethane foam for breathable, comfortable support.

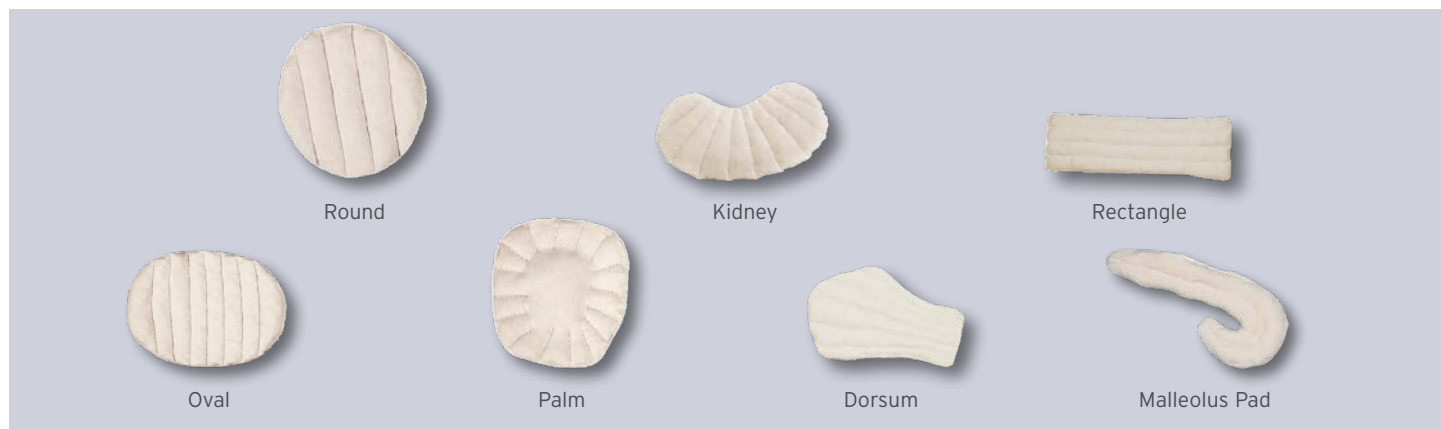

JOVILINER ARM



Extends from the distal interphalangeal joints (DIPs) to the axilla and includes stitching between each finger. Designed for use under short-stretch bandaging or pump appliances.


JOVILINER LEG



Full-leg design with Continuous Flow Channels extending from the tip of the toes to the groin. Intended for use under short-stretch bandaging or pump appliances.

JOBST® PITPAKS

PitPaks are designed to deliver focused, localized pressure for patients with dense or stubborn fibrotic tissue related to lymphedema. Filled with natural cherry pits, they create multiple pressure points under compression to help stimulate lymph flow and soften hardened tissue. PitPaks are available in a range of anatomically oriented shapes to address localized problem areas throughout the body. Each pad is constructed from soft organic cotton and cherry-pit filling and should be used only under the supervision of a qualified therapist or medical professional.

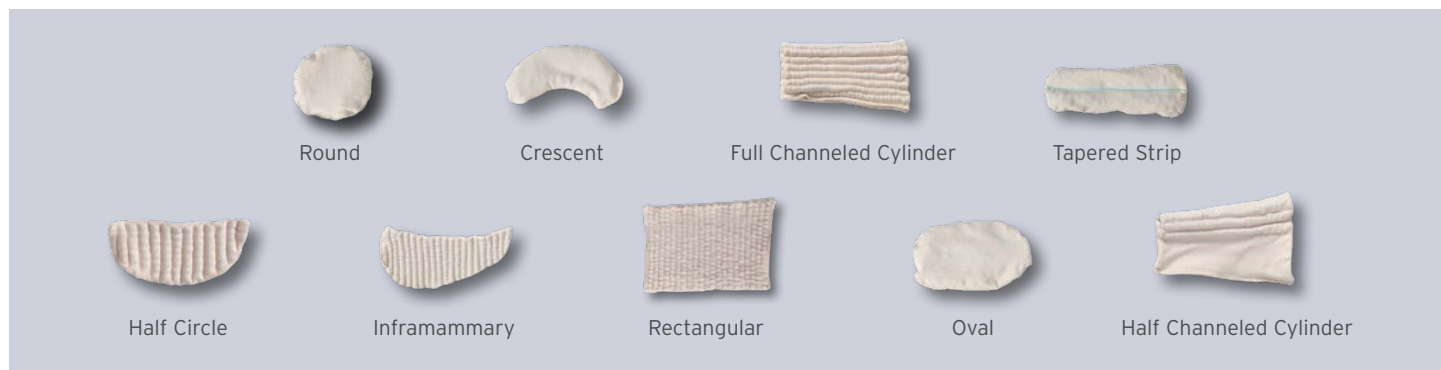

ACCESSORIES



JOBST® Donning Gloves*

Donning Gloves are designed to help create friction and provide grip during the application of compression garments. They support proper donning by allowing the clinician or patient to position and distribute the fabric evenly on the leg or arm. The gloves are easy to use and help reduce garment slippage or damage during application.



Cutimed® ACUTE

Cutimed® ACUTE 10% Urea Cream Mousse is an effective moisturizer for very dry skin and can be applied to targeted areas or the entire body. It helps reduce skin roughness, absorbs quickly, and is suitable for patients with very dry skin, including those with diabetes. The formulation contains natural moisturizers and leaves the skin feeling hydrated and comfortable.



Dycem®

Dycem® non-slip material provides an improved grip and increased friction to support easier donning or doffing of compression garments. When placed on the floor or a tabletop, it creates a stable, non-slip surface that patients can push against while applying an arm or leg garment. The material offers reliable hold and resistance during donning and can be easily cleaned with soap and water to renew its non-slip properties.

ARION APPLICATION AIDS

Arion Application Aids assist with the donning and doffing of upper-extremity and lower-extremity compression garments. They can be used with both open-ended and closed-ended garments and are especially helpful for patients who experience tenderness or weakness in the limb used for donning, or for those who are not strong enough to apply the garment independently.

PUTTING ON & TAKING OFF

Arion Magnide® 2in1

For putting on and taking off closed-toe compression stockings and tights.



Arion Sim-Slide®

For putting on and taking off open-toe compression stockings and tights.



PUTTING ON

Arion Easy-Slide

User-friendly donning aid for open-toe compression stockings and tights.



Arion Easy-Slide Arm

User-friendly application aid makes putting on compression arm sleeves quick and easy.



Arion Magnide®

Donning aid designed for closed-toe compression stockings and tights.



Customer Service Hours

Monday - Friday
8:00am - 6:00pm ET
(+1) 800 537 1063

Contact Information

Bellavar, Custom Seamless Soft, Elvarex®, Confidence & Relax Orders	HMS-Elvarex-Orders@essity.com
Bellavar, Custom Seamless Soft, Elvarex®, Confidence & Relax Returns	HMS-Elvarex>Returns@essity.com
FarrowWrap®	Farrow.Orders@essity.com
JoViPak® Custom	Info.JoViPak@essity.com*
JoViPak® Ready-to-Wear	Jovi.Orders@essity.com
Custom Seamed	SeamedOrders@essity.com*

*Hours may vary

Available On-Line 24 hours per day, 365 Days per Year

Standard Products: shop.hm.essity.us
Custom Products: eshop.jobst-usa.com

Sorbact® is a registered trademark of ABIGO Medical AB.
Bellisse® is a registered trademark of Lightning2, LLC.
VELCRO® is a registered trademark of Velcro BVBA.
CoolMax® is a registered trademark of INVISTA.
Dycem® is a registered trademark of Dycem Ltd.
Easy-Slide® is a registered trademark of Arion International.

The BSN Medical Inc., Reimbursement Disclaimer:

The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA LLC., as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at <https://www4.palmettogba.com/pdac.dmecls/>. The Product Classification List is a searchable database containing products that have received HCPCS coding verification from the PDAC and their corresponding HCPCS Codes. BSN Medical Inc., however, makes no representations as to the accuracy of the information contained within the PDAC Database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered. RV: 12/24

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,
an Essity brand



BSN Medical Inc., an Essity company
5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 980 498 8980
To order toll-free: Tel. (+1) 800 537 1063

52209 R20 ©2026 BSN Medical Inc. B26