

## JOBST® Compri2 Application Technique\*



JOBST® Compri2			
REF No.	Compression Level	Size	QTY
76271-00	Regular 30-40mmHg	18 cm - 25 cm	1 Kit
76271-01	Regular 30-40mmHg	25 cm - 32 cm	1 Kit
76271-02	Lite 20-30mmHg	18 cm - 25 cm	1 Kit
76271-03	Lite 20-30mmHg	25 cm - 32 cm	1 Kit
76271-04	Regular 30-40 mmHg	18 cm - 25 cm	6 Pack
76271-05	Regular 30-40 mmHg	25 cm - 32 cm	6 Pack
76271-06	Lite 20-30 mmHg	18 cm - 25 cm	6 Pack
76271-07	Lite 20-30 mmHg	25 cm - 32 cm	6 Pack



1 Apply the compression padding with the ankle at a 90-degree angle. Start by wrapping it around the foot, starting at the base of the toes. Continue wrapping proximally around the foot.



2 At the heel, wrap the padding in a figure of 8 around the malleolus, before wrapping below to cover the heel. If necessary, bony prominences can be padded with padding off-cuts.



3 Continue bandaging at 50% overlap with slight tension applied to the padding. Finish the padding at two finger-widths below the knee crease.




4 Remove excess material and press the end of the padding lightly.



5 An indicator on the compression bandage helps to ensure correct tension is applied by changing from an oval to a circle



6 Without adding tension, start by wrapping the compression bandage around the foot at the base of the toes, leaving some padding exposed. Pass the first wrap twice around before moving proximal.

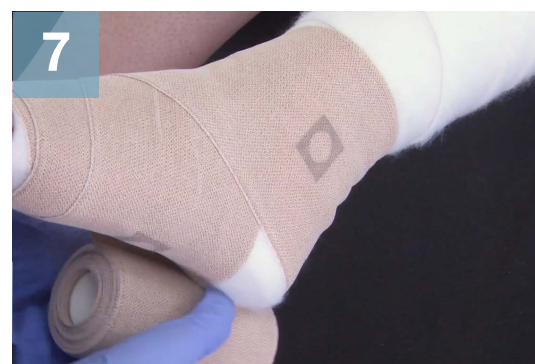

7 Maintain the ankle in a 90-degree angle while passing the bandage around the malleolus and back across the heel, creating a figure 8 at the heel. Ensure the heel is fully covered.



8 Applying tension to the bandage from the ankle, continue wrapping the compression bandage around the leg, overlapping by 50%. Keep the bandage under even tension whereby the indicators will appear as circles. 50% of the indicator will be visible with the correct overlap.



9 Finish the bandage slightly below the padding and cut off the excess bandage material. Press the compression bandage lightly to ensure it adheres well to itself. For removal, unravel the bandage and padding.



Application Video