



DEFY GRAVITY



JOBST® COMPRESSION

Compression Wrap Therapy

CONTENTS

Compression Wrap Therapy

SHORT-STRETCH COMPRESSION WRAP SOLUTIONS

JOBST® FarrowWrap® **140**

Product Range

Footpiece

Toe Cap

Legpiece

Kneepiece

Thighpiece

Liners & Accessories

Upper Limb

JOBST® FarrowWrap® **APPLICATION TECHNIQUES** **162**

JOBST® FarrowWrap® **CARE GUIDELINES** **164**

JOBST® FarrowWrap®

THE EASY CHOICE TO SUPPORT PATIENT SELF-CARE AND EMPOWERMENT

Give your patients the tools they need to better manage their swelling and respond confidently to fluctuating oedema levels, with a solution that takes the guesswork out of applying compression safely.

JOBST® FarrowWrap® garments are a range of adjustable compression devices that are available in a variety of styles, fabrics, sizes and compression classes. Made with durable materials, they are designed to help reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers. They are designed to provide the same strong support as multi-layer bandaging but in a simplified, intuitive design.

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Easy Donning

Short-stretch technology provides low resting and high working pressures to enhance lymphatic and venous return.



Short Stretch Technology

VELCRO® brand hook & loop system allows patients to respond better to fluctuating oedema by simply tightening or loosening the wrap.




Adjustable

Discreet design is less bulky than multi-layer bandaging and can be worn under normal clothing.



Low Profile Design

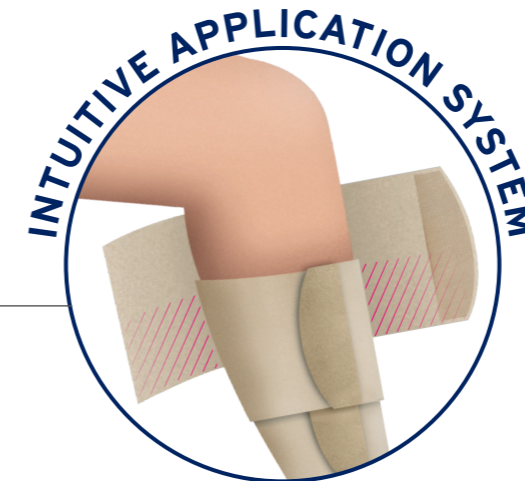

Overlapping bands replicate the principles of multi-layer bandaging.

Apply and adjust confidently

JOBST® FarrowWrap® was engineered for easy application and can offer all the benefits of multi-layer bandaging without the complexity of application. The overlapping material supports consistent self-application without the need for specialised intervention by health care providers. The overlap provides the support needed to efficiently control oedema and help reduce the risk of swelling forming between the bands.



THE JOBST® FarrowWrap®

Product Range

LITE



20-30 mmHg

Indicated for mild to moderate chronic oedema/lymphoedema.

A double laminated fabric suitable for mild to moderate compression needs, making it especially suitable for palliative and paediatric patients. The fabric is both durable and breathable.

- Conforms to limb shape
- Good choice for patients who have neuropathy or peripheral vascular disease (PVD)
- Recommended for paediatric patients
- Suitable for sensitive skin and as such palliative care patients
- Machine washable

STRONG



30-40 mmHg

Indicated for moderate to severe chronic oedema/lymphoedema with a good limb shape.

Premium fabric with strong compression and soft inner layer suitable for patients with a good limb shape and moderate to severe oedema.

- Less rigid than CLASSIC
- Contoured bands conform to the limb and resist sliding
- Highly breathable and durable
- Suitable for minor skin folds and distortion
- Material less likely to pill or fray
- Machine washable

CLASSIC



30-40 mmHg

Indicated for moderate to severe chronic oedema/lymphoedema.

The original material that has been clinically proven over many years of use on thousands of patients worldwide. The rigid nature of the material offers performance beyond the compression range making it excellent for patients with moderate to severe compression levels.

- Suitable for oedema with skin folds and irregular limb shapes
- Durable, rigid fabric with a comfortable feel
- Reinforced spine for easier donning
- Less likely to roll down and will not work into skin folds
- Hand wash only

Product Assortment

| | Footpiece | Legpiece | Thighpiece | Armpiece | Gauntlet |
|--|-----------|----------|------------|----------|----------|
| JOBST® FarrowWrap® LITE Mild to Moderate 20-30mmHg | | | | | |
| JOBST® FarrowWrap® STRONG JOBST® FarrowWrap® CLASSIC Moderate to Strong 30-40mmHg | | | | | |

| | Non-compressive Liners | FarrowHybrid® Compressive Liner | Toe Cap |
|--|------------------------|---------------------------------|---------|
| Accessories | | | |
| JOBST® FarrowWrap® GarmentGrip™ Roll JOBST® FarrowWrap® Short-Stretch | | | |



EASY-TO-USE COMPRESSION SOLUTIONS HELP TO PROMOTE INDEPENDENT SELF-CARE

Compression therapy is a vital component in the management of chronic swelling from venous and lymphatic conditions. Conventional treatment focuses on reducing symptoms with multi-layer bandaging.

For certain patients, this management system can be difficult to maintain independently. It is therefore important to offer these patients suitable compression solutions that are efficient and easy-to-use, to help improve therapy outcomes and empower patients to confidently care for their own needs.

Choosing the right compression¹:

- Easy to apply and remove**
- Is comfortable for the patient and stays in place**
- Offers variety in patient choice**
- Delivers therapeutic compression and has high stiffness**
- Is aesthetically acceptable**

¹Harding, K., Dowsett, C., Flax, L. et al. Simplifying venous leg ulcer management: consensus recommendations. Wounds International, 2015. <https://tinyurl.com/jrxrcrj> (accessed 26 February 2017). Properties of an ideal compression system.

INNOVATIVE SOLUTIONS DESIGNED FOR IMPROVED SELF-CARE: **ADJUSTABLE COMPRESSION DEVICES**

Give your patients the choice and variety needed to take control of their own care and support them to achieve better therapy outcomes with adjustable compression devices.

In oedema management, adjustable compression devices (or compression wraps) can be used as a first-choice treatment option and offer an alternative to multi-layer bandaging or hosiery.

JOBST® FarrowWrap has short-stretch properties similar to multi-layering bandaging, delivering low resting pressure and high working pressure. As a result, they may create peak pressures that produce a massaging effect and stimulate lymph flow. JOBST® FarrowWrap are designed for easy use, enabling patients to independently apply and adjust the compression according to their needs and tolerance.

Which of your patients would benefit from **JOBST® FarrowWrap**?



OEDEMA

Patients with swelling resulting from lymphoedema and venous oedema:

- With fluctuations in limb swelling
- In decongestive and/or maintenance phases



LARGER BODY SIZES

Patients unable to don or unsuitable for compression garments/bandages through:

- Severe obesity
- Overweight or larger body sizes
- Irregular shape of limbs
- Skin folds and lobules

VENOUS LEG ULCER

Patients with open lesions between the knee and the ankle:

- Most common type of leg ulcer, accounting for more than 90% of all cases
- Can develop after a minor injury, where persistently high pressure in the veins of the legs has damaged the skin

PHYSICAL LIMITATIONS

Patients unable to apply or remove compression hosiery/bandaging due to:

- Arthritis or weak hand strength
- Back problems
- Sensitive or fragile skin at risk of breakdown

JOBST® FarrowWrap® FOOTPIECE




LITE Footpiece



STRONG Footpiece



CLASSIC Footpiece

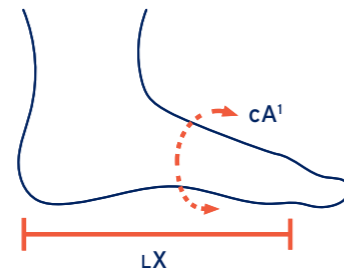

How to Measure Footpiece Size Chart

| | XSmall | Small | Medium | Large | XLarge |
|---------------------|------------|------------|------------|------------|------------|
| cA¹ Mid-foot | 22 - 24 cm | 25 - 27 cm | 28 - 30 cm | 31 - 34 cm | 35 - 40 cm |
| LX Regular | 16 - 17 cm | 17 - 18 cm | 19 - 20 cm | 20 - 21 cm | 22 - 23 cm |
| LX Long | 18 - 19 cm | 19 - 20 cm | 21 - 22 cm | 22 - 23 cm | 24 - 25 cm |

Circumference A¹ = Mid-foot circumference taken around the middle of the foot.

Length X = Foot length is measured from the most posterior aspect of the heel to the base of the 5th toe.

Circumference measurements must be within +/- 0.5cm.

Ordering Information

| | Product Code | Size | Length | Colour |
|----------|---|--------|---------|--------|
| | JOBST® FarrowWrap® Lite (20-30mmHg) Footpiece | | | |
| | 76652-05 | XSmall | Regular | Tan |
| | 76652-06 | Small | Regular | Tan |
| | 76652-07 | Medium | Regular | Tan |
| | 76652-08 | Large | Regular | Tan |
| | 76652-09 | XLarge | Regular | Tan |
| | 76652-00 | XSmall | Long | Tan |
| | 76652-01 | Small | Long | Tan |
| | 76652-02 | Medium | Long | Tan |
| | 76652-03 | Large | Long | Tan |
| 76652-04 | XLarge | Long | Tan | |
| | JOBST® FarrowWrap® Strong (30-40mmHg) Footpiece | | | |
| | 76646-04 | XSmall | Regular | Tan |
| | 76646-05 | Small | Regular | Tan |
| | 76646-06 | Medium | Regular | Tan |
| | 76646-07 | Large | Regular | Tan |
| | 76646-08 | XLarge | Regular | Tan |
| | 76646-09 | XSmall | Long | Tan |
| | 76646-00 | Small | Long | Tan |
| | 76646-01 | Medium | Long | Tan |
| | 76646-02 | Large | Long | Tan |
| 76646-03 | XLarge | Long | Tan | |
| | JOBST® FarrowWrap® Classic (30-40mmHg) Footpiece | | | |
| | 76656-05 | XSmall | Regular | Tan |
| | 76656-06 | Small | Regular | Tan |
| | 76656-07 | Medium | Regular | Tan |
| | 76656-08 | Large | Regular | Tan |
| | 76656-09 | XLarge | Regular | Tan |
| | 76656-00 | XSmall | Long | Tan |
| | 76656-01 | Small | Long | Tan |
| | 76656-02 | Medium | Long | Tan |
| | 76656-03 | Large | Long | Tan |
| 76656-04 | XLarge | Long | Tan | |

All JOBST FarrowWrap Footpieces are supplied as single units and without a liner.

JOBST® Farrow® TOE CAP


Ready-to-wear Toe Cap which can be worn on either right or left foot.

Fine seams, with ultra thin fabric, allows air to pass to enhance patient comfort.

Toe lengths can be easily trimmed with scissors to the required length.

Toe Cap Size Chart

| Size | cA |
|--------|------------|
| XSmall | 20 - 22 cm |
| Small | 22 - 24 cm |
| Medium | 24 - 26 cm |
| Large | 26 - 28 cm |

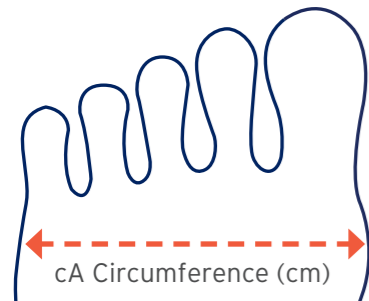

Ordering Information

| Size | Product Code | Compression | Colour |
|--------|--------------|-------------|--------|
| XSmall | 72947-00 | 20-30 mmHg | Tan |
| Small | 72947-01 | 20-30 mmHg | Tan |
| Medium | 72947-02 | 20-30 mmHg | Tan |
| Large | 72947-03 | 20-30 mmHg | Tan |

All JOBST Farrow Toe Caps are supplied as single units.



JOBST® FarrowWrap® LEGPIECE




LITE Legpiece



STRONG Legpiece



CLASSIC Legpiece

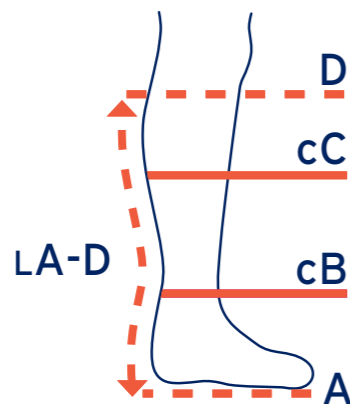

How to Measure Legpiece Size Chart

| | XSmall | Small | Medium | Large | XLarge |
|---------------------|------------|------------|------------|------------|------------|
| cC Calf | 36 - 43 cm | 42 - 50 cm | 48 - 58 cm | 53 - 63 cm | 58 - 68 cm |
| cB Ankle | 21 - 25 cm | 25 - 30 cm | 30 - 36 cm | 36 - 42 cm | 42 - 50 cm |
| LA-D Regular | 33 - 37 cm | 35 - 39 cm | 37 - 41 cm | 39 - 43 cm | 39 - 43 cm |
| LA-D Tall | 38 - 41 cm | 40 - 43 cm | 42 - 45 cm | 44 - 47 cm | 44 - 47 cm |

Circumference C = Calf circumference taken at the widest part of the calf.

Circumference B = Ankle circumference taken at the narrowest part of the ankle.

Length A-D = Floor to two finger widths below crease of knee. Measure posteriorly following the contour of the leg.

Ordering Information

| | Product Code | Size | Length | Colour |
|----------|--|--------|---------|--------|
| | JOBST® FarrowWrap® Lite (20-30mmHg) Legpiece | | | |
| | 76654-00 | XSmall | Regular | Tan |
| | 76654-01 | Small | Regular | Tan |
| | 76654-02 | Medium | Regular | Tan |
| | 76654-03 | Large | Regular | Tan |
| | 76654-04 | XLarge | Regular | Tan |
| | 76654-05 | XSmall | Tall | Tan |
| | 76654-06 | Small | Tall | Tan |
| | 76654-07 | Medium | Tall | Tan |
| | 76654-08 | Large | Tall | Tan |
| 76654-09 | XLarge | Tall | Tan | |
| | JOBST® FarrowWrap® Strong (30-40mmHg) Legpiece | | | |
| | 76648-00 | XSmall | Regular | Tan |
| | 76648-01 | Small | Regular | Tan |
| | 76648-02 | Medium | Regular | Tan |
| | 76648-03 | Large | Regular | Tan |
| | 76648-04 | XLarge | Regular | Tan |
| | 76648-05 | XSmall | Tall | Tan |
| | 76648-06 | Small | Tall | Tan |
| | 76648-07 | Medium | Tall | Tan |
| | 76648-08 | Large | Tall | Tan |
| 76648-09 | XLarge | Tall | Tan | |
| | JOBST® FarrowWrap® Classic (30-40mmHg) Legpiece | | | |
| | 76657-00 | XSmall | Regular | Tan |
| | 76657-01 | Small | Regular | Tan |
| | 76657-02 | Medium | Regular | Tan |
| | 76657-03 | Large | Regular | Tan |
| | 76657-04 | XLarge | Regular | Tan |
| | 76657-05 | XSmall | Tall | Tan |
| | 76657-06 | Small | Tall | Tan |
| | 76657-07 | Medium | Tall | Tan |
| | 76657-08 | Large | Tall | Tan |
| 76657-09 | XLarge | Tall | Tan | |

All JOBST FarrowWrap Legpieces are supplied as single units with a pair of closed toe, knee high liners.


JOBST® FarrowWrap® KNEEPIECE



Adjustable Kneepiece, easy to apply & comfortable to wear.

Can be worn in combination with JOBST® FarrowWrap® Legpieces and Thighpieces, or as a single unit (e.g. in combination with an Elvarex knee high stocking).

Available in two fabrics: LITE and STRONG.

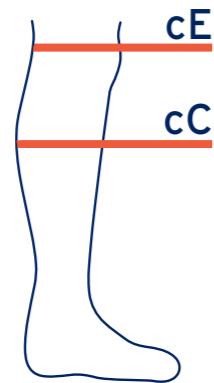

How to Measure

Kneepiece Size Chart

| | XSmall | Small | Medium | Large | XLarge |
|-----------------------------|------------|------------|------------|------------|------------|
| cE 5cm above Patella | 47 - 52 cm | 53 - 59 cm | 60 - 66 cm | 67 - 73 cm | 74 - 80 cm |
| cC Calf | 36 - 43 cm | 42 - 50 cm | 48 - 58 cm | 53 - 63 cm | 58 - 68 cm |

Circumference E = Knee circumference taken 5cm above the patella.

Circumference C = Calf circumference taken at the widest part of the calf.

JOBST FarrowWrap Kneepiece is automatically supplied with the Thighpiece. Kneepieces can be purchased as single units.

Ordering Information

| | Product Code | Size | Colour |
|--|--------------|--------|--------|
| JOBST® FarrowWrap® Lite (20-30mmHg) Kneepiece | | | |
| | 73367-00 | XSmall | Tan |
| | 73367-01 | Small | Tan |
| | 73367-02 | Medium | Tan |
| | 73367-03 | Large | Tan |
| | 73367-04 | XLarge | Tan |
| JOBST® FarrowWrap® Strong (30-40mmHg) Kneepiece | | | |
| | 73368-00 | XSmall | Tan |
| | 73368-01 | Small | Tan |
| | 73368-02 | Medium | Tan |
| | 73368-03 | Large | Tan |
| | 73368-04 | XLarge | Tan |



JOBST® FarrowWrap® THIGHPIECE




LITE Thighpiece



STRONG Thighpiece



CLASSIC Thighpiece

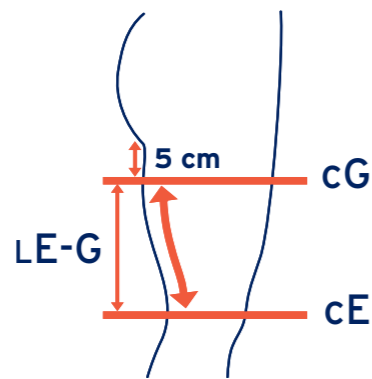

How to Measure Thighpiece Size Chart

| | XSmall | Small | Medium | Large | XLarge |
|----------------------|------------|------------|------------|------------|-------------|
| cG Groin | 60 - 70 cm | 65 - 75 cm | 70 - 80 cm | 75 - 90 cm | 85 - 100 cm |
| cE Above Knee | 47 - 52 cm | 53 - 59 cm | 60 - 66 cm | 67 - 73 cm | 74 - 80 cm |
| LE-G Short | 16 - 20 cm | 16 - 20 cm | 16 - 20 cm | 16 - 20 cm | 16 - 20 cm |
| LE-G Regular | 21 - 26 cm | 21 - 26 cm | 21 - 26 cm | 21 - 26 cm | 21 - 26 cm |
| LE-G Tall | 27 - 31 cm | 27 - 31 cm | 27 - 31 cm | 27 - 31 cm | 27 - 31 cm |

Circumference G = Thigh circumference 5cm below the gluteal fold (where the buttock meets the leg).

Circumference E = Knee circumference 5cm above the patella.

Length E-G = Measure the length of the thigh medially (inside leg) between E and G following the contour of the leg.

JOBST FarrowWrap Thighpiece is supplied as a single unit with the equivalent sized Kneepiece. All Classic Thighpieces are supplied with the equivalent Strong Kneepiece.

Ordering Information

| Product Code | Size | Length | Colour |
|--|--------|---------|--------|
| JOBST® FarrowWrap® Lite (20-30mmHg) Thighpiece | | | |
| 76655-05 | XSmall | Short | Tan |
| 76655-06 | Small | Short | Tan |
| 76655-07 | Medium | Short | Tan |
| 76655-08 | Large | Short | Tan |
| 76655-09 | XLarge | Short | Tan |
| 76655-00 | XSmall | Regular | Tan |
| 76655-01 | Small | Regular | Tan |
| 76655-02 | Medium | Regular | Tan |
| 76655-03 | Large | Regular | Tan |
| 76655-04 | XLarge | Regular | Tan |
| 76655-10 | XSmall | Tall | Tan |
| 76655-11 | Small | Tall | Tan |
| 76655-12 | Medium | Tall | Tan |
| 76655-13 | Large | Tall | Tan |
| 76655-14 | XLarge | Tall | Tan |
| JOBST® FarrowWrap® Strong (30-40mmHg) Thighpiece | | | |
| 76649-05 | XSmall | Short | Tan |
| 76649-06 | Small | Short | Tan |
| 76649-07 | Medium | Short | Tan |
| 76649-08 | Large | Short | Tan |
| 76649-09 | XLarge | Short | Tan |
| 76649-00 | XSmall | Regular | Tan |
| 76649-01 | Small | Regular | Tan |
| 76649-02 | Medium | Regular | Tan |
| 76649-03 | Large | Regular | Tan |
| 76649-04 | XLarge | Regular | Tan |
| 76649-10 | XSmall | Tall | Tan |
| 76649-11 | Small | Tall | Tan |
| 76649-12 | Medium | Tall | Tan |
| 76649-13 | Large | Tall | Tan |
| 76649-14 | XLarge | Tall | Tan |
| JOBST® FarrowWrap® Classic (30-40mmHg) Thighpiece | | | |
| 76658-06 | XSmall | Short | Tan |
| 76658-07 | Small | Short | Tan |
| 76658-08 | Medium | Short | Tan |
| 76658-09 | Large | Short | Tan |
| 76658-10 | XLarge | Short | Tan |
| 76658-00 | XSmall | Regular | Tan |
| 76658-01 | Small | Regular | Tan |
| 76658-02 | Medium | Regular | Tan |
| 76658-03 | Large | Regular | Tan |
| 76658-04 | XLarge | Regular | Tan |
| 76658-12 | XSmall | Tall | Tan |
| 76658-13 | Small | Tall | Tan |
| 76658-14 | Medium | Tall | Tan |
| 76658-15 | Large | Tall | Tan |
| 76658-16 | XLarge | Tall | Tan |

JOBST® FarrowWrap® LINERS & ACCESSORIES

JOBST® FarrowHybrid® Liner



FarrowHybrid® is a patented foot compression garment that provides compression in the foot up to the ankle.

- 20-30mmHg Compression
- From the ankle and above, it serves as a liner
- Available in standard and wide sizes
- Great alternative to a footpiece
- Low profile, easily fits into a shoe

How to Measure

Measure the mid-foot circumference A1.
Two mid-foot sizes available: 20-27cm (regular foot) and 28-35cm (wide foot).
Request the patient's shoe size (US Sizing).
The maximum calf circumference for FarrowHybrid® Liners is 60cm.

Accessories

| Product Code | Size | Style | Limb | Colour |
|---------------------------|----------------|----------------------|------------------|--------|
| JOBST® Accessories | | | | |
| 76672-00 | 14 cm x 80 cm | GarmentGrip | Thigh | Tan |
| 76672-01 | 14 cm x 120 cm | GarmentGrip | Thigh Long | Tan |
| 76672-03 | 20 cm x 120 cm | GarmentGrip | Thigh Long Wide | Tan |
| 76672-04 | 30 cm x 150 cm | GarmentGrip | Thigh Extra Wide | Tan |
| 76672-02 | 14 cm x 550 cm | GarmentGrip | Thigh Extra Long | Tan |
| 76673-00 | 10 cm x 150 cm | Farrow Short Stretch | | Tan |
| 76673-01 | 8 cm x 150 cm | Farrow Short Stretch | | Tan |

JOBST® Farrow GarmentGrip can be cut to size to fit the shape and contour of the body and VELCRO® brand hook-and-loop fasteners can be added to secure in place. For triangular shaped legs, it is recommended to be worn over the legpiece to secure the wrap system in place.

JOBST® Farrow GarmentGrip is washable and therefore reusable. It is available for the upper and lower limb in differing roll lengths and extra wide widths and can also be used to manage mid-line oedema.

FarrowHybrid Liners

| Product Code | Fabric | Size | Female Shoe Size | Male Shoe Size | Colour | Mid-foot Circumference A1 |
|---|------------|--------|------------------------|----------------------|--------|---------------------------|
| JOBST® Knee High Liner (20-30mmHg) | | | | | | Regular |
| 76664-00 | Hybrid ADI | Small | US Female Shoe 4 - 7 | US Male Shoe 3 - 6 | Beige | Mid-foot 20 - 27 cm |
| 76664-01 | Hybrid ADI | Medium | US Female Shoe 8 - 10 | US Male Shoe 7 - 9 | Beige | Mid-foot 20 - 27 cm |
| 76664-02 | Hybrid ADI | Large | US Female Shoe 11 - 13 | US Male Shoe 10 - 12 | Beige | Mid-foot 20 - 27 cm |
| 76664-03 | Hybrid ADI | XLarge | US Female Shoe 14 - 16 | US Male Shoe 13 - 15 | Beige | Mid-foot 20 - 27 cm |
| JOBST® Knee High Liner (20-30mmHg) | | | | | | Wide |
| 76664-12 | Hybrid ADI | Small | US Female Shoe 4 - 7 | US Male Shoe 3 - 6 | Beige | Mid-foot 28 - 35 cm |
| 76664-13 | Hybrid ADI | Medium | US Female Shoe 8 - 10 | US Male Shoe 7 - 9 | Beige | Mid-foot 28 - 35 cm |
| 76664-14 | Hybrid ADI | Large | US Female Shoe 11 - 13 | US Male Shoe 10 - 12 | Beige | Mid-foot 28 - 35 cm |
| 76664-15 | Hybrid ADI | XLarge | US Female Shoe 14 - 16 | US Male Shoe 13 - 15 | Beige | Mid-foot 28 - 35 cm |
| JOBST® Knee High Liner (20-30mmHg) | | | | | | Wide |
| 76664-16 | Hybrid ADI | Small | US Female Shoe 4 - 7 | US Male Shoe 3 - 6 | Black | Mid-foot 28 - 35 cm |
| 76664-17 | Hybrid ADI | Medium | US Female Shoe 8 - 10 | US Male Shoe 7 - 9 | Black | Mid-foot 28 - 35 cm |
| 76664-18 | Hybrid ADI | Large | US Female Shoe 11 - 13 | US Male Shoe 10 - 12 | Black | Mid-foot 28 - 35 cm |
| 76664-19 | Hybrid ADI | XLarge | US Female Shoe 14 - 16 | US Male Shoe 13 - 15 | Black | Mid-foot 28 - 35 cm |

Calf circumference up to 60cm.
JOBST FarrowHybrid Liners are supplied as a pair.

Non-compressive Lower Limb Liners

| Product Code | Type | Size | Calf Circumference | Colour |
|---------------------------------|----------------------|----------|--------------------|--------|
| JOBST® Knee High Liners | | | | |
| 76666-21 | Closed Toe | Standard | 19 - 48 cm | Beige |
| 76666-22 | Closed Toe | Wide | 48 - 70 cm | Beige |
| 76669-00 | Terry Cloth Open Toe | Small | < 40 cm | White |
| 76669-02 | Terry Cloth Open Toe | Medium | 41 - 70 cm | White |
| 76669-04 | Terry Cloth Open Toe | Large | 71 - 120 cm | White |
| JOBST® Thigh High Liners | | | | |
| 76670-00 | Terry Cloth Open Toe | Small | < 40 cm | White |
| 76670-02 | Terry Cloth Open Toe | Medium | 41 - 70 cm | White |
| 76670-04 | Terry Cloth Open Toe | Large | 71 - 120 cm | White |

JOBST non-compressive liners are supplied as a pair.
All JOBST FarrowWrap Legpieces are supplied with a pair of closed toe, knee high liners.
All JOBST FarrowWrap Thighpieces are supplied with a pair of open toe, thigh high liners.

JOBST® FarrowWrap® UPPER LIMB


JOBST® FarrowWrap® LITE Hand Gauntlet 20-30mmHg



JOBST® FarrowWrap LITE Hand Gauntlet is ambidextrous and comes with 4mm and 8mm foam squares for added compression to the dorsal hand. Suitable for mild to severe oedema.

Benefits:

1. Foam inserts help reduce oedema that can pool on the back of the hand
2. Ambidextrous, i.e. can be used on either hand
3. Durable and comfortable

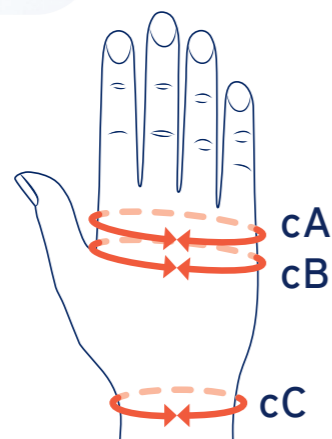

How to Measure Gauntlet Size Chart

| | XSmall | Small | Medium | Large |
|----|------------|------------|------------|------------|
| cA | 16 - 18 cm | 18 - 20 cm | 20 - 22 cm | 22 - 24 cm |
| cB | 17 - 19 cm | 19 - 21 cm | 21 - 23 cm | 23 - 26 cm |
| cC | 15 - 18 cm | 17 - 20 cm | 19 - 22 cm | 22 - 24 cm |

Ordering Information

| Product Code | Size |
|---|--------|
| JOBST® FarrowWrap® Lite (20-30mmHg) Gauntlet | |
| 76653-00 | XSmall |
| 76653-01 | Small |
| 76653-02 | Medium |
| 76653-03 | Large |



JOBST® FarrowWrap® UPPER LIMB

JOBST® FarrowWrap® LITE Armpiece 20-30mmHg




FarrowWrap LITE Armpiece is an anatomically contoured effective solution to control upper extremity oedema. It is easy to apply and remove with one hand and designed for mild to moderate oedema control. Light and comfortable with just one layer but with a robust short-stretch performance.

Liner is built into the armsleeve.
Product is anatomically designed for left and right arms.



How to Measure Armpiece Size Chart

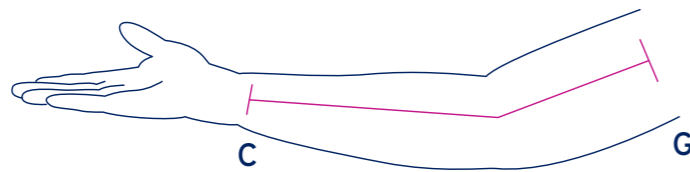

| Length C-G | |
|------------|---------------|
| Short | 40 cm - 43 cm |
| Regular | 44 cm - 47 cm |
| Long | 48 cm - 51 cm |

NOTE: Measure length along the lateral outer arm from the wrist to the axilla with a slight bend in the arm.

| | cC Wrist | cE Elbow | cG Axilla |
|--------|------------|------------|------------|
| | | | |
| Small | 14 - 18 cm | 20 - 27 cm | 22 - 31 cm |
| Medium | 16 - 21 cm | 25 - 34 cm | 29 - 39 cm |
| Large | 19 - 25 cm | 30 - 40 cm | 32 - 45 cm |

Ordering Information

| Product Code | Size | Option | Colour |
|---|--------|--------|----------------|
| JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece | | | Long |
| 76650-35 | Small | Left | Tan |
| 76650-00 | Small | Right | Tan |
| 76650-01 | Medium | Left | Tan |
| 76650-02 | Medium | Right | Tan |
| 76650-03 | Large | Left | Tan |
| 76650-04 | Large | Right | Tan |
| 76650-17 | Small | Left | Black |
| 76650-18 | Small | Right | Black |
| 76650-19 | Medium | Left | Black |
| 76650-20 | Medium | Right | Black |
| 76650-21 | Large | Left | Black |
| 76650-22 | Large | Right | Black |
| JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece | | | Regular |
| 76650-05 | Small | Left | Tan |
| 76650-06 | Small | Right | Tan |
| 76650-07 | Medium | Left | Tan |
| 76650-08 | Medium | Right | Tan |
| 76650-09 | Large | Left | Tan |
| 76650-10 | Large | Right | Tan |
| 76650-23 | Small | Left | Black |
| 76650-24 | Small | Right | Black |
| 76650-25 | Medium | Left | Black |
| 76650-26 | Medium | Right | Black |
| 76650-27 | Large | Left | Black |
| 76650-28 | Large | Right | Black |
| JOBST® FarrowWrap® Lite (20-30mmHg) Armpiece | | | Short |
| 76650-11 | Small | Left | Tan |
| 76650-12 | Small | Right | Tan |
| 76650-13 | Medium | Left | Tan |
| 76650-14 | Medium | Right | Tan |
| 76650-15 | Large | Left | Tan |
| 76650-16 | Large | Right | Tan |
| 76650-29 | Small | Left | Black |
| 76650-30 | Small | Right | Black |
| 76650-31 | Medium | Left | Black |
| 76650-32 | Medium | Right | Black |
| 76650-33 | Large | Left | Black |
| 76650-34 | Large | Right | Black |

FarrowWrap® FOOTPIECE APPLICATION TECHNIQUE

Applicable for the following fabrics:

- JOBST® FarrowWrap® Lite
- JOBST® FarrowWrap® Strong
- JOBST® FarrowWrap® Classic



1 Apply a liner. Either the open toe Soft Liner (white) or the closed toe Liner (beige).

With the Soft Liner, ensure you extend the fabric past the toes distally and past the knee proximally to allow sufficient fabric to fold back over the wraps once they have been applied. Unwrap the Footpiece and lay it on the floor with the hole for the heel closest to you.



2 Place the foot on top of the Footpiece, resting the heel over the hole in the Footpiece.



3 To secure the foot, hold the Footpiece flap closest to the toes on each side and pull both sides over the top of the foot to the opposite side. Apply at or near end stretch with the VELCRO®.

Now apply the remaining flap (the one closest to the ankle) in the same manner.



4 After securing the forefoot, hold one side of the ankle band and pull it over the top of the mid-foot to the opposite side and secure it in place at or near end stretch.



5 Repeat the same step with the other half of the ankle band on the opposite side.

When removing the Footpiece, fold the VELCRO® over on itself and back on the same band.



Care instructions - Strong and Lite fabric: Machine wash in cold water with a mild detergent, lay flat to dry, do not iron, do not dry clean.

Care instructions - Classic fabric: Hand wash in cold water with mild detergent, lay flat to dry, do not iron, do not dry clean.

FarrowWrap® LEGPIECE APPLICATION TECHNIQUE

Applicable for the following fabrics:

- JOBST® FarrowWrap® Lite
- JOBST® FarrowWrap® Strong
- JOBST® FarrowWrap® Classic



1 Apply a JOBST® Farrow liner and Footpiece if applicable.

Unwrap the Legpiece and lay the wrap on the floor behind your leg with the label side furthest back and facing up. The shortest bands will be closer to the foot.



2 Place the wrap with its spine positioned behind the leg starting from the ankle and finishing just below the knee.

Apply the top wrap as an anchor to hold the wrap in place. Then commence wrapping from the ankle up towards the knee.



3 Grab each side of the band closest to your ankle and pull forward wrapping the band around the lowest part of the leg at or near end stretch and affix with the VELCRO®. Undo the top band before applying the second last band to ensure the final band sits over the top. If wearing a Footpiece, the bottom band should overlap the Footpiece.



4 Repeat with each band moving up the leg, smoothing out any wrinkles as you go.

Try to ensure a 50% overlap between adjacent straps.

JOBST® FarrowWrap® CARE GUIDELINES



Wash your wrap/s weekly or more often if you experience wound leakage and wash your liner every day.

Prior to any washing fold the Velcro tabs back on themselves. This prevents the build-up of fluff, keeping the tabs in good condition and easy to apply.

Hand Washing

- After folding the tabs on themselves, wash with warm water (30°C) and a mild soap or detergent.
- Roll firmly in a towel to remove excess water.

Washing Machine

- After folding the tabs on themselves, place your wraps in a laundry bag
- Wash at 30°C using a mild soap or detergent.

Drying Your Wraps

- Dry flat and leave in a warm or shady room.



Do not tumble dry



Do not bleach



Do not iron



Do not use softeners



Do not dry clean

Fibre Content

| FABRIC/CCL | Nylon (Polyamide) | Spandex (Elastane) | Cotton | Polyester | Neoprene | Polyurethane |
|--|----------------------|-----------------------|--------|-----------|----------|--------------|
| JOBST® FarrowWrap® LITE 20-30mmHg | 82% | 18% | - | - | - | - |
| JOBST® FarrowWrap® STRONG 30-40mmHg | 82% | 18% | - | - | - | - |
| JOBST® FarrowWrap® CLASSIC 30-40mmHg | 32% | 5% | 47% | 16% | - | - |
| JOBST® Farrow Non-compressive Closed Toe Liner | 94% | 6% | - | - | - | - |
| JOBST® FarrowWrap® Non-compressive Terry Cloth Liner | - | 15% | 85% | - | - | - |
| JOBST® FarrowHybrid® ADI 20-30mmHg | 83% | 17% | - | - | - | - |
| JOBST® Farrow Toe Cap | 59% | 41% | - | - | - | - |
| JOBST® Farrow GarmentGrip Roll | 48% | 5% | - | - | 47% | - |
| JOBST® FarrowFoam Squares | - | - | - | - | - | 100% |