



DEFY GRAVITY



JOBST® COMPRESSION

Upper Limb Compression Garments

CONTENTS

Upper Limb Compression Garments

JOBST® BELLA LITE 4

Circular Knit Ready-To-Wear (RTW)

Armsleeve

Gauntlet

Combination Sleeve

Glove

JOBST® BELLA STRONG 6

Circular Knit Ready-To-Wear (RTW)

Armsleeve Regular

Armsleeve Long

Gauntlet

Flat Knit Glove

JOBST® ELVAREX 8

Flat Knit Ready-To-Wear (RTW)

Armsleeve with Bias Finish

Armsleeve with Bra Loop

Armsleeve with Body Strap

Gauntlet and Glove

MEASURING GUIDE 10

THINKING ABOUT EARLY INTERVENTION?


JOBST® BELLA™ LITE Circular Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Made from soft yarns providing high wearing comfort
- Breathable ultra-thin fabric
- Fine seams
- Fingers on glove can be cut to length without fraying
- All products are suitable for wearing on either hand/arm
- Latex free
- Available in beige and black



Ordering Guide


JOBST® Bella Lite Armsleeve

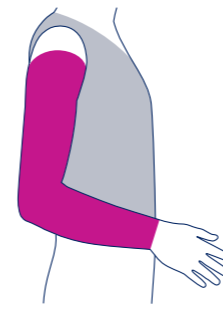

Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long	Colour
Product Code	75276-05	75276-00 75276-04	75277-05	75277-00 75277-04	75278-05	75278-00 75278-04	● Beige ● Black
cC	13.5 - 19.5 cm	13.5 - 19.5 cm	15.5 - 21.5 cm	15.5 - 21.5 cm	17.5 - 23.5 cm	17.5 - 23.5 cm	
cD	17.8 - 23.2 cm	17.8 - 23.2 cm	23.3 - 28.7 cm	23.3 - 28.7 cm	28.8 - 34.3 cm	28.8 - 34.3 cm	
cF	21.5 - 39.5 cm	21.5 - 39.5 cm	27 - 45 cm	27 - 45 cm	32.5 - 50.5 cm	32.5 - 50.5 cm	


Regular length C-G ≤ 43 cm. Long length C-G >43 cm.

JOBST® Bella Lite Gauntlet

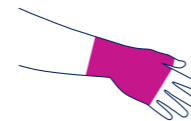

Size	Small	Medium	Large	Colour
Product Code	75282-00	75283-00	75284-00	● Beige ● Black
	75282-03	75283-03	75284-03	
cA	15 - 19 cm	19 - 22 cm	22 - 26 cm	
cC	14 - 20 cm	16 - 22 cm	18 - 24 cm	


JOBST® Bella Lite Combination Armsleeve

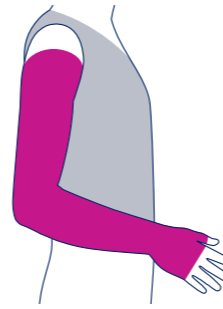

Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long
Product Code	77680-00	77680-03	77680-01	77680-04	77680-02	77680-05
cA	15 - 18.9 cm	15 - 18.9 cm	19 - 22 cm	19 - 22 cm	22.1 - 26 cm	22.1 - 26 cm
cD	17.8 - 23.2 cm	17.8 - 23.2 cm	23.3 - 28.7 cm	23.3 - 28.7 cm	28.8 - 34.3 cm	28.8 - 34.3 cm
cF	21.5 - 39.5 cm	21.5 - 39.5 cm	27 - 45 cm	27 - 45 cm	32.5 - 50.5 cm	32.5 - 50.5 cm

Regular length C-G ≤ 43 cm. Long length C-G >43 cm.

JOBST® Bella Lite Glove

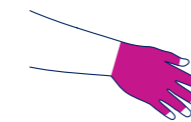

Size	1	2	3	4	5	6	7	8	Colour
Product Code	71026-00	71026-01	71026-02	71026-03	71026-04	71026-05	71026-06	71026-07	● Beige ● Black
	71026-10	71026-11	71026-12	71026-13	71026-14	71026-15	71026-16	71026-17	
cA	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm	
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm	

All JOBST Bella Lite products are supplied as single units and are finished without a silicone top band.



MILD TO MODERATE LYMPHOEDEMA

JOBST® BELLA™ STRONG


Circular Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Skin friendly multi fibre nylon material
- Utilises advanced comfort technology to regulate humidity, quickly wicks moisture, keeping you cool and comfortable
- Innovative sizing system - armsleeves can fit approx 80% of patients and gauntlets approx 90% of patients
- Available in beige
- The JOBST Bella Strong Glove is the same fabric as JOBST Elvarex Soft, offering a RTW flat knit glove.



Ordering Guide


JOBST® Bella Strong Armsleeve Regular Length

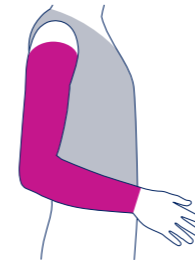

Size	1	2	3	4	5	6	7	8	9	10
Product Code	72736-01	72736-02	72736-03	72736-04	72736-05	72736-06	72736-07	72736-08	72736-09	72736-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

Finished with a 5cm silicone top band.


JOBST® Bella Strong Armsleeve Long Length

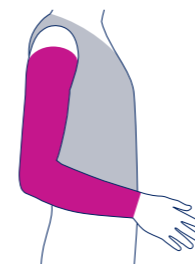


Size	1	2	3	4	5	6	7	8	9	10
Product Code	7237-01	7237-02	7237-03	7237-04	7237-05	7237-06	7237-07	7237-08	7237-09	7237-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

Finished with a 5cm silicone top band.

JOBST® Bella Strong Gauntlet

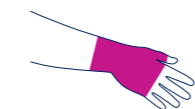

Size	1	2	3	4	5	6	7	8
Product Code	73694-40	73694-41	73694-42	73694-43	73694-44	73694-45	73694-46	73694-47
cA	15 - 17 cm	17 - 19 cm	17 - 19 cm	19 - 21 cm	19 - 21 cm	21 - 23 cm	21 - 23 cm	23 - 26 cm
cC	14 - 18 cm	14 - 18 cm	18 - 21 cm	14 - 18 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm	19 - 22 cm

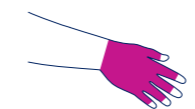

JOBST® Bella Strong Glove Flat Knit

Size	1	2	3	4	5	6	7	8
Product Code	75291-10	75291-11	75291-12	75291-13	75291-14	75291-15	75291-16	75291-17
cA	16.6 - 18.0 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm

All JOBST Bella Strong products are supplied as single units.



JOBST® ELVAREX - MODERATE LYMPHOEDEMA

JOBST® ELVAREX™ Flat Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for moderate oedema
- Available in beige

Two special features make JOBST® Elvarex™ the product of choice for lymphoedema treatment:

- “Double knit” exerts a light massaging effect on the initial lymphatics
- Low-stretch fabric


Other features include:

- Good skin tolerance due to air permeability (knitted construction), and cotton covering of all threads (cotton component against skin)
- Durable and comfortable to wear
- Graduated compression
- Flat knitting process with only one seam, provides good anatomical fit, especially for elbow region in sleeves
- Ease of donning due to short stretch thread and high cotton content




Ordering Guide

JOBST® Elvarex® Ready-To-Wear **Armsleeve Bias Finish** (no strap)

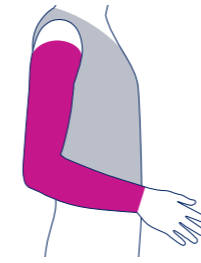

Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	76101-11	76101-10	76101-09	76101-08	76101-07	76101-06
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm

JOBST® Elvarex® Ready-To-Wear **Armsleeve with Bra Loop**

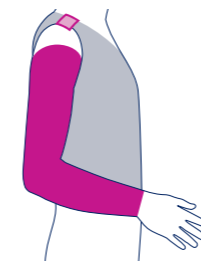


Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	79435-17	79435-16	79435-15	79435-14	79435-13	79435-12
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm

JOBST® Elvarex® Ready-To-Wear **Armsleeve with Body Strap**



Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	79435-23	79435-22	79435-21	79435-20	79435-19	79435-18
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm

JOBST® Elvarex® Ready-To-Wear **Gauntlet and Glove**

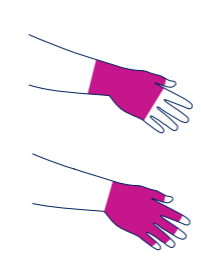


Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Glove Product Code	76091-17	76091-16	76091-15	76091-14	76091-13	76091-12
Gauntlet Product Code	76091-23	76091-22	76091-21	76091-20	76091-19	76091-18
cA (Metacarpal heads)	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm
cB (Widest Palm)	17 - 19 cm	19 - 21 cm	21 - 23 cm	17 - 19 cm	19 - 21 cm	21 - 23 cm
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 22 cm	16 - 18 cm	18 - 20 cm	20 - 22 cm
Thumb base	5 - 6 cm	6 - 7 cm	7 - 8 cm	5 - 6 cm	6 - 7 cm	7 - 8 cm
Finger 2 base	5 - 6 cm	6 - 7 cm	6 - 7 cm	5 - 6 cm	6 - 7 cm	6 - 7 cm
Finger 3 base	5 - 6 cm	6 - 7 cm	6.5 - 7 cm	5 - 6 cm	6 - 7 cm	6.5 - 7 cm
Finger 4 base	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm
Finger 5 base	4 - 5 cm	5 - 6 cm	6 - 7 cm	4 - 5 cm	5 - 6 cm	6 - 7 cm

All JOBST Elvarex RTW products are supplied as single units and are finished without a silicone top band.

Measuring Guide

JOBST® Bella Lite Armsleeves

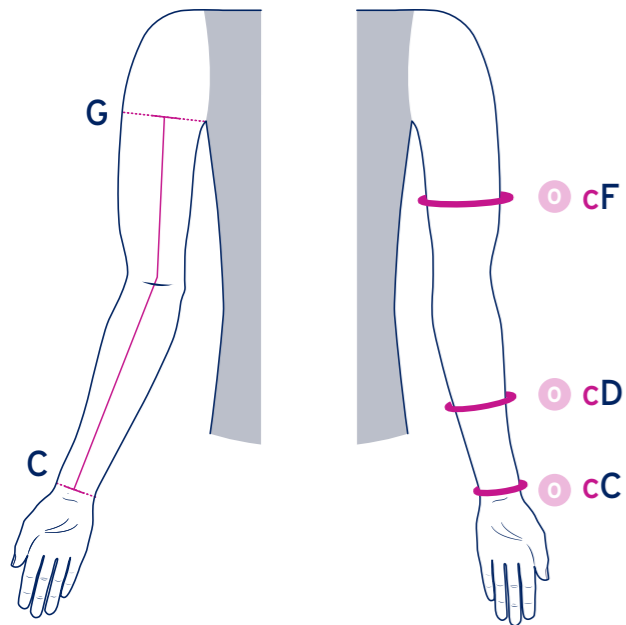


Length Measurements (C-G)

Regular up to 43 cm.
Long greater than 43 cm.

Circumference Measurements Bella Lite			
Size	Wrist cC	Mid Lower cD	Mid Upper cF
Small Reg	13.5 - 19.5 cm	17.8 - 23.2 cm	21.5 - 39.5 cm
Small Long	13.5 - 19.5 cm	17.8 - 23.2 cm	21.5 - 39.5 cm
Medium Reg	15.5 - 21.5 cm	23.3 - 28.7 cm	27 - 45 cm
Medium Long	15.5 - 21.5 cm	23.3 - 28.7 cm	27 - 45 cm
Large Reg	17.5 - 23.5 cm	28.8 - 34.3 cm	32.5 - 50.5 cm
Large Long	17.5 - 23.5 cm	28.8 - 34.3 cm	32.5 - 50.5 cm

JOBST® Bella Strong Armsleeves

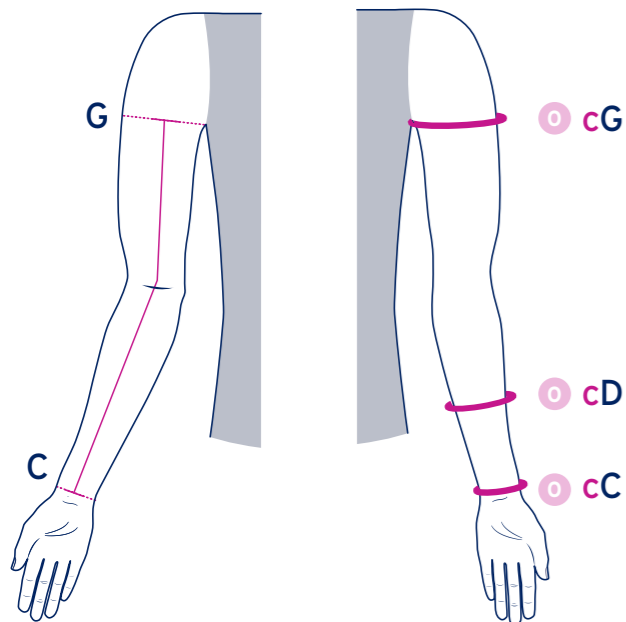

Measuring Circumference

- STEP 1** Measure axilla circumference **cG**
This leaves 1 or 2 choices.
- STEP 2** Measure mid lower circumference **cD**
This leaves 2 or 3 choices.
- STEP 3** Measure wrist circumference **cC**
This leaves 1 or 2 choices.


If there are still 2 options left, we recommend choosing the larger size for ease of donning, overall comfort and ultimate compliance.

Either size selection will deliver the desired therapy.

Circumference Measurements Bella Strong			
Size	Wrist cC	Mid Forearm cD	Axilla cG
1	14 - 18 cm	17 - 21 cm	22 - 31 cm
2	14 - 18 cm	17 - 21 cm	31 - 41 cm
3	14 - 18 cm	21 - 25 cm	25 - 34 cm
4	16 - 19 cm	21 - 25 cm	28 - 38 cm
5	17 - 20 cm	21 - 25 cm	34 - 43 cm
6	14 - 18 cm	25 - 29 cm	29 - 38 cm
7	17 - 20 cm	25 - 29 cm	33 - 43 cm
8	18 - 21 cm	25 - 29 cm	38 - 48 cm
9	16 - 19 cm	29 - 33 cm	32 - 42 cm
10	19 - 22 cm	29 - 33 cm	36 - 46 cm

Ready-To-Wear Circular Knit garment circumferences are measured with zero tension applied to the measuring tape.

JOBST® Bella Lite and Strong Gloves

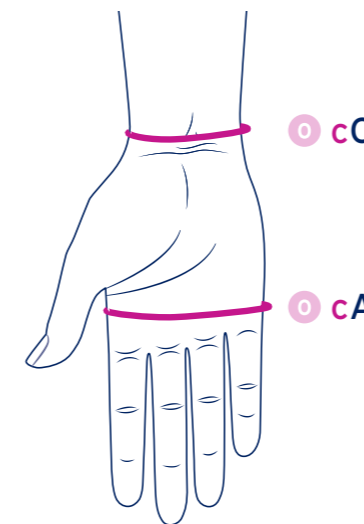


Measuring Circumference

- STEP 1** Measure palm circumference **cA**
This leaves 2 choices.
- STEP 2** Measure wrist circumference **cC**
Select Glove.

Circumference Measurements - Bella Lite and Bella Strong								
Size	1	2	3	4	5	6	7	8
cA	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm

JOBST® Bella Lite and Strong Gauntlets

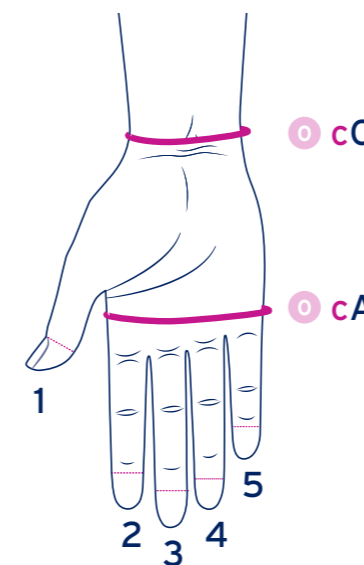

JOBST® Bella Strong Measuring Circumference

- STEP 1** Measure palm circumference
This leaves 2 choices.
- STEP 2** Measure wrist circumference
Select Glove.

Circumference Measurements Bella Lite		
Size	Palm cA	Wrist cC
Small	15 - 19 cm	14 - 20 cm
Medium	19 - 22 cm	16 - 22 cm
Large	22 - 26 cm	18 - 24 cm

Circumference Measurements Bella Strong		
Size	Palm cA	Wrist cC
1	15 - 17 cm	14 - 18 cm
2	17 - 19 cm	14 - 18 cm
3	17 - 19 cm	18 - 21 cm
4	19 - 21 cm	14 - 18 cm
5	19 - 21 cm	18 - 21 cm
6	21 - 23 cm	16 - 19 cm
7	21 - 23 cm	19 - 22 cm
8	23 - 26 cm	19 - 22 cm

TIPS


Mark the measurement points on the limb with a JOBST® Skin Marking Pencil



Take the corresponding circumference and length measurements using the JOBST® tape measure

Measuring Guide

Elvarex® Ready-To-Wear Gloves and Gauntlets

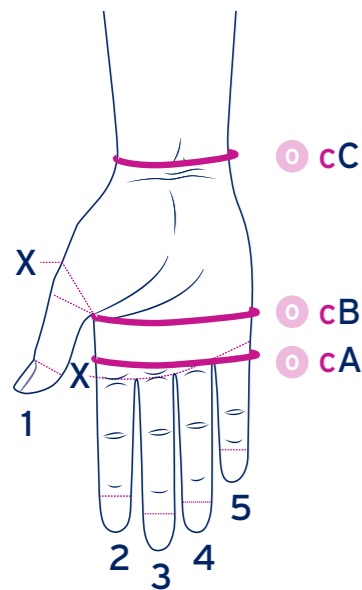


Measuring Circumferences

- STEP 1** The patient's hand should be placed palm down and the fingers should be slightly apart. As the hand is sensitive to pressure, the measuring tape should not be pulled tightly. Circumferential measures should follow the contour of the palm.
- STEP 2** Measure and record the circumference at A - over the MCP heads (as per diagram).
- STEP 3** Measure and record the circumference at B - distal to the thumb web (as per diagram).
- STEP 4** Measure and record the circumference at C, the wrist - distal to the styloid process (as per diagram).
- STEP 5** Measure and record the circumference at (X), the base of the thumb.
- STEP 6** If a glove with fingers is required, measure and record the circumference at (X) the base of each finger (as per diagram).
- Compare the measurements you have obtained with the following table to determine the correct size. Where measurements vary greatly, a custom-made glove is necessary.

Lengths A-B 3 cm, A-C 9 cm, A-C1 12 cm

Circumference Measurements			
Measurement Point	Small	Medium	Large
cA (Metacarpal heads)	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm
cB (Widest Palm)	17 - 19 cm	19 - 21 cm	21 - 23 cm
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 22 cm
Thumb base	5 - 6 cm	6 - 7 cm	7 - 8 cm
Finger 2 base	5 - 6 cm	6 - 7 cm	6 - 7 cm
Finger 3 base	5 - 6 cm	6 - 7 cm	6.5 - 7 cm
Finger 4 base	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm
Finger 5 base	4 - 5 cm	5 - 6 cm	6 - 7 cm

Elvarex® Ready-To-Wear Armsleeves

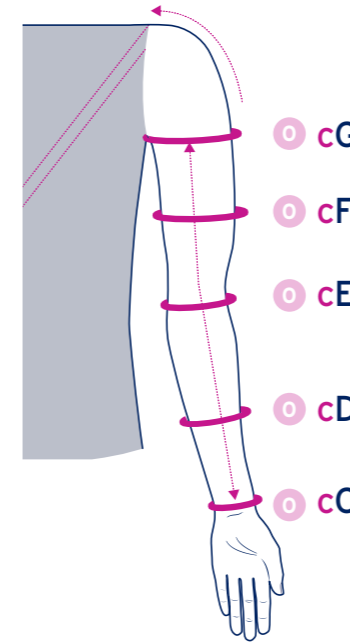

Length Measurements

Suitable for up to 42 cm for C-G, and 45 cm for C-G with attachment

Measuring Circumferences

- STEP 1** The patient should be seated, arm on table with palm facing upwards and elbow flexed to approximately 40°.
- STEP 2** Measure and record the circumference at C, the wrist. NOTE: At the wrist and elbow, the tape should never be pulled tightly, however, slightly firmer pressure may be used over fleshy areas, i.e. D, F and G points).
- STEP 3** Measure and record the circumference at D, the mid-point of the lower arm.
- STEP 4** Measure and record the circumference at E, the elbow. (Ensure the arm is slightly bent at [40°] before taking this measurement.)
- STEP 5** Measure and record the circumference F, the mid-point between the axilla and the elbow.
- STEP 6** Measure and record the circumference at G, (take the measurement approx 1 cm below axilla). Compare the measurements you have obtained with the following table to determine the correct size. Where measurements vary across all three sizes, or are smaller or larger than the tabled measurements, a custom-made sleeve will be required.

Circumference Measurements			
Measurement Point	Small	Medium	Large
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD (Mid Forearm)	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE (Elbow)	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF (Mid Upper arm)	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG (Axilla)	31 - 34 cm	34 - 38 cm	38 - 44 cm

Fibre Content

FABRIC/CCL	Spandex (Elastane)	Elastodiene (Latex)	Polyamide (Nylon)	Silicone
JOBST® Bella Lite Armsleeve 20-30mmHg	20%	-	80%	-
JOBST® Bella Lite Gauntlet 20-30mmHg	14%	-	86%	-
JOBST® Bella Lite Combined Armsleeve 20-30mmHg	21%	-	79%	-
JOBST® Bella Lite Glove 20-30mmHg	41%	-	59%	8%
JOBST® Bella Strong Armsleeve with Silicone Band 20-30mmHg	20%	-	72%	-
JOBST® Bella Strong Gauntlet 20-30mmHg	9%	-	91%	-
JOBST® Bella Strong Glove 20-30mmHg	35%	-	65%	-
JOBST® Elvarex 2 20-30mmHg	10%	20%	70%	-

Top bands contain silicone.

Caution: Products labelled on the packaging with  contain natural rubber (latex) which may cause allergic reactions.