



JOBST® COMPRESSION

Upper Limb Compression Garments

CONTENTS

Upper Limb Compression Garments

JOBST® **BELLA LITE** **4**

Circular Knit Ready-To-Wear (RTW)

Armsleeve

Gauntlet

Combination Sleeve

Glove

JOBST® **BELLA STRONG** **6**

Circular Knit Ready-To-Wear (RTW)

Armsleeve

Gauntlet

JOBST® **ELVAREX SOFT** **8**

Flat Knit Ready-To-Wear (RTW)

Armsleeve

Glove

Gauntlet

MEASURING GUIDE **10**


THINKING ABOUT EARLY INTERVENTION?

JOBST® BELLA™ LITE Circular Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Made from soft yarns providing high wearing comfort
- Breathable ultra-thin fabric
- Fine seams
- Fingers on glove can be cut to length without fraying
- All products are suitable for wearing on either hand/arm
- Latex free
- Available in beige and black




Ordering Guide

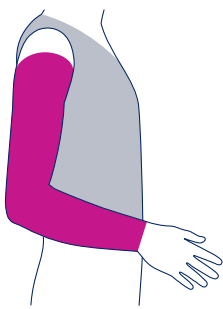

JOBST® Bella Lite Armsleeve


Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long	Colour
Product Code	75276-05	75276-00 75276-04	75277-05	75277-00 75277-04	75278-05	75278-00 75278-04	● Beige ● Black
cC	13.5 - 19.5 cm	13.5 - 19.5 cm	15.5 - 21.5 cm	15.5 - 21.5 cm	17.5 - 23.5 cm	17.5 - 23.5 cm	
cD	17.8 - 23.2 cm	17.8 - 23.2 cm	23.3 - 28.7 cm	23.3 - 28.7 cm	28.8 - 34.3 cm	28.8 - 34.3 cm	
cF	21.5 - 39.5 cm	21.5 - 39.5 cm	27 - 45 cm	27 - 45 cm	32.5 - 50.5 cm	32.5 - 50.5 cm	

Regular length C-G ≤ 43 cm. Long length C-G >43 cm.

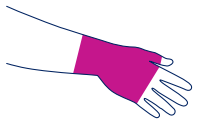

JOBST® Bella Lite Gauntlet


Size	Small	Medium	Large	Colour
Product Code	75282-00 75282-03	75283-00 75283-03	75284-00 75284-03	● Beige ● Black
cA	15 - 19 cm	19 - 22 cm	22 - 26 cm	
cC	14 - 20 cm	16 - 22 cm	18 - 24 cm	

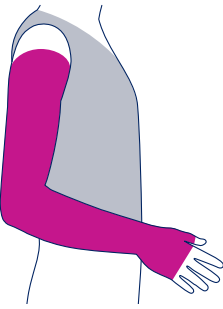

JOBST® Bella Lite Combination Armsleeve

Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long
Product Code	77680-00	77680-03	77680-01	77680-04	77680-02	77680-05
cA	15 - 18.9 cm	15 - 18.9 cm	19 - 22 cm	19 - 22 cm	22.1 - 26 cm	22.1 - 26 cm
cD	17.8 - 23.2 cm	17.8 - 23.2 cm	23.3 - 28.7 cm	23.3 - 28.7 cm	28.8 - 34.3 cm	28.8 - 34.3 cm
cF	21.5 - 39.5 cm	21.5 - 39.5 cm	27 - 45 cm	27 - 45 cm	32.5 - 50.5 cm	32.5 - 50.5 cm

Regular length C-G ≤ 43 cm. Long length C-G >43 cm.

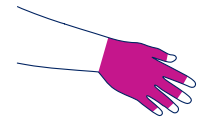

JOBST® Bella Lite Glove

Size	1	2	3	4	5	6	7	8	Colour
Product Code	71026-00 71026-10	71026-01 71026-11	71026-02 71026-12	71026-03 71026-13	71026-04 71026-14	71026-05 71026-15	71026-06 71026-16	71026-07 71026-17	● Beige ● Black
cA	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm	
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm	

All JOBST Bella Lite products are supplied as single units and are finished without a silicone top band.



MILD TO MODERATE LYMPHOEDEMA

JOBST® BELLA™ STRONG


Circular Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Skin friendly multi fibre nylon material
- Utilises advanced comfort technology to regulate humidity, quickly wicks moisture, keeping you cool and comfortable
- Innovative sizing system - armsleeves can fit approx 80% of patients and gauntlets approx 90% of patients
- Available in beige



Ordering Guide


JOBST® Bella Strong **Armsleeve Regular** Length

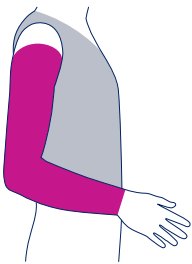

Size	1	2	3	4	5	6	7	8	9	10
Product Code	72736-01	72736-02	72736-03	72736-04	72736-05	72736-06	72736-07	72736-08	72736-09	72736-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

Finished with a 5cm silicone top band.


JOBST® Bella Strong **Armsleeve Long** Length

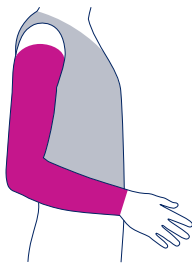

Size	1	2	3	4	5	6	7	8	9	10
Product Code	72737-01	72737-02	72737-03	72737-04	72737-05	72737-06	72737-07	72737-08	72737-09	72737-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

Finished with a 5cm silicone top band.

JOBST® Bella Strong **Gauntlet**

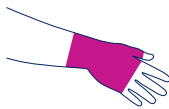

Size	1	2	3	4	5	6	7	8
Product Code	73694-40	73694-41	73694-42	73694-43	73694-44	73694-45	73694-46	73694-47
cA	15 - 17 cm	17 - 19 cm	17 - 19 cm	19 - 21 cm	19 - 21 cm	21 - 23 cm	21 - 23 cm	23 - 26 cm
cC	14 - 18 cm	14 - 18 cm	18 - 21 cm	14 - 18 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm	19 - 22 cm

All JOBST Bella Strong products are supplied as single units.



MILD TO
MODERATE
LYMPHOEDEMA

JOBST® ELVAREX™ SOFT
Flat Knit Ready-To-Wear (RTW)

- Indicated for mild to moderate oedema
- Available in CCL1 15-21mmHg and CCL2 23-32mmHg
- Flat knit fabric for greater containment and management of limb shape
- Soft, breathable fabric for exceptional comfort and adherence
- Moisture management technology to keep skin dry
- Graduated, short-stretch compression
- Ease of donning and doffing to support patient independence
- Available in beige and black




Ordering Guide

COMPRESSION LEVELS

15-21 mmHg / CCL1

23-32 mmHg / CCL2

JOBST® Elvarex® Soft Ready-To-Wear **Armsleeve (CG)**

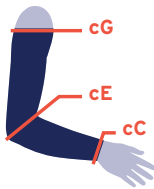


Size	1	2	3	4	5
cG (cm)	26 - 30	28 - 34	32 - 38	36 - 44	40 - 48
cE (cm)	22 - 26	24 - 28	26 - 32	29 - 35	32 - 40
cC (cm)	14 - 16	16 - 18	18 - 20	20 - 22	22 - 24

LCG Petite: <44 cm LCG Regular: >44 cm

Length	Top Band	Colour	Size 1	Size 2	Size 3	Size 4	Size 5
Regular	Standard	Beige	73619-00	73619-01	73619-02	73619-03	73619-04
Regular	Standard	Black	73619-05	73619-06	73619-07	73619-08	73619-09
Regular	SoftFit	Beige	73619-10	73619-11	73619-12	73619-13	73619-14
Regular	SoftFit	Black	73619-15	73619-16	73619-17	73619-18	73619-19
Petite	Standard	Beige	78101-02	78101-03	78101-04	78101-05	78101-06
Petite	Standard	Black	78101-07	78101-08	78101-09	78101-10	78101-11
Petite	SoftFit	Beige	78101-12	78101-13	78101-14	78101-15	78101-16
Petite	SoftFit	Black	78101-17	78101-18	78101-19	78101-20	78101-21
Regular	Standard	Beige	78100-02	78100-03	78100-04	78100-05	78100-06
Regular	Standard	Black	78100-07	78100-08	78100-09	78100-10	78100-11
Regular	SoftFit	Beige	78100-12	78100-13	78100-14	78100-15	78100-16
Regular	SoftFit	Black	78100-17	78100-18	78100-19	78100-20	78100-21
Petite	Standard	Beige	78102-03	78102-04	78102-05	78102-06	78102-07
Petite	Standard	Black	78102-08	78102-09	78102-10	78102-11	78102-12
Petite	SoftFit	Beige	78102-13	78102-14	78102-15	78102-16	78102-17
Petite	SoftFit	Black	78102-18	78102-19	78102-20	78102-21	78102-22

JOBST Elvarex Soft RTW armsleeve garments are supplied as a single unit.

JOBST® Elvarex® Soft Ready-To-Wear **Glove and Gauntlet (AC1)**

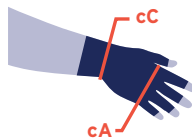


Size	Palm at base of fingers (cA)	Wrist (cC)
1	16.6 - 18 cm	15.6 - 16.7 cm
2	16.6 - 18 cm	16.7 - 18.1 cm
3	18 - 19.5 cm	15.9 - 17 cm
4	18 - 19.5 cm	17 - 18.5 cm
5	18 - 19.5 cm	18.5 - 20.4 cm
6	19.5 - 21.5 cm	17.2 - 18.7 cm
7	19.5 - 21.5 cm	18.7 - 20.6 cm
8	21.5 - 23.1 cm	18.3 - 20.1 cm

Style	Colour	Size 1	Size 2	Size 3	Size 4	Size 5	Size 6	Size 7	Size 8
Glove	Beige	73156-00	73156-01	73156-02	73156-03	73156-04	73156-05	73156-06	73156-07
Glove	Black	73156-08	73156-09	73156-10	73156-11	73156-12	73156-13	73156-14	73156-15
Gauntlet	Beige	73156-16	73156-17	73156-18	73156-19	73156-20	73156-21	73156-22	73156-23
Gauntlet	Black	73156-24	73156-25	73156-26	73156-27	73156-28	73156-29	73156-30	73156-31
Glove	Beige	73157-00	73157-01	73157-02	73157-03	73157-04	73157-05	73157-06	73157-07
Glove	Black	73157-08	73157-09	73157-10	73157-11	73157-12	73157-13	73157-14	73157-15
Gauntlet	Beige	73157-16	73157-17	73157-18	73157-19	73157-20	73157-21	73157-22	73157-23
Gauntlet	Black	73157-24	73157-25	73157-26	73157-27	73157-28	73157-29	73157-30	73157-31

JOBST Elvarex Soft RTW glove & gauntlet garments are supplied as a single unit.

Measuring Guide

JOBST® Bella Lite Armsleeves

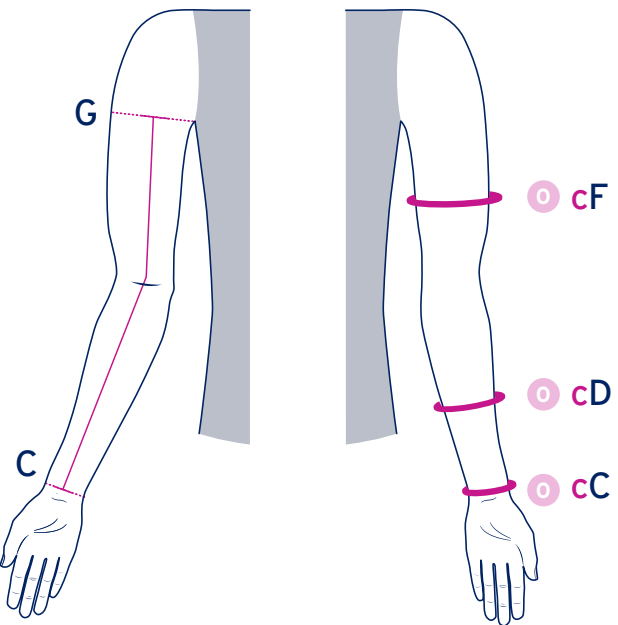


Length Measurements (C-G)

Regular up to 43 cm.
Long greater than 43 cm.

Circumference Measurements Bella Lite			
Size	Wrist cC	Mid Lower cD	Mid Upper cF
Small Reg	13.5 - 19.5 cm	17.8 - 23.2 cm	21.5 - 39.5 cm
Small Long	13.5 - 19.5 cm	17.8 - 23.2 cm	21.5 - 39.5 cm
Medium Reg	15.5 - 21.5 cm	23.3 - 28.7 cm	27 - 45 cm
Medium Long	15.5 - 21.5 cm	23.3 - 28.7 cm	27 - 45 cm
Large Reg	17.5 - 23.5 cm	28.8 - 34.3 cm	32.5 - 50.5 cm
Large Long	17.5 - 23.5 cm	28.8 - 34.3 cm	32.5 - 50.5 cm

JOBST® Bella Strong Armsleeves

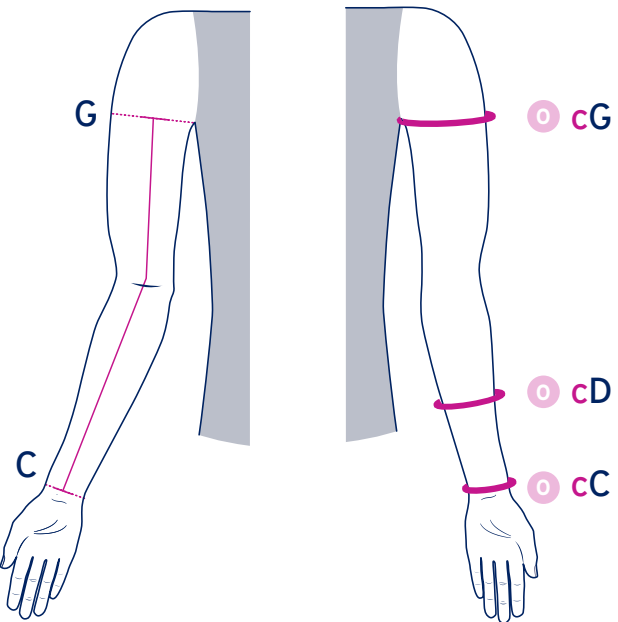


Measuring Circumference

- STEP 1** Measure axilla circumference **cG**
This leaves 1 or 2 choices.
- STEP 2** Measure mid lower circumference **cD**
This leaves 2 or 3 choices.
- STEP 3** Measure wrist circumference **cC**
This leaves 1 or 2 choices.
- If there are still 2 options left, we recommend choosing the larger size for ease of donning, overall comfort and ultimate compliance.
- Either size selection will deliver the desired therapy.**

Circumference Measurements Bella Strong			
Size	Wrist cC	Mid Forearm cD	Axilla cG
1	14 - 18 cm	17 - 21 cm	22 - 31 cm
2	14 - 18 cm	17 - 21 cm	31 - 41 cm
3	14 - 18 cm	21 - 25 cm	25 - 34 cm
4	16 - 19 cm	21 - 25 cm	28 - 38 cm
5	17 - 20 cm	21 - 25 cm	34 - 43 cm
6	14 - 18 cm	25 - 29 cm	29 - 38 cm
7	17 - 20 cm	25 - 29 cm	33 - 43 cm
8	18 - 21 cm	25 - 29 cm	38 - 48 cm
9	16 - 19 cm	29 - 33 cm	32 - 42 cm
10	19 - 22 cm	29 - 33 cm	36 - 46 cm

Ready-To-Wear Circular Knit garment circumferences are measured with zero tension applied to the measuring tape.

JOBST® Bella Lite Gloves

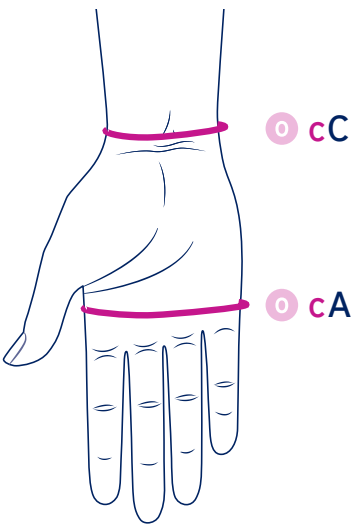


Measuring Circumference

- STEP 1** Measure palm circumference **cA**
This leaves 2 choices.
- STEP 2** Measure wrist circumference **cC**
Select Glove.

Circumference Measurements Bella Lite								
Size	1	2	3	4	5	6	7	8
cA	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm

JOBST® Bella Lite and Strong Gauntlets

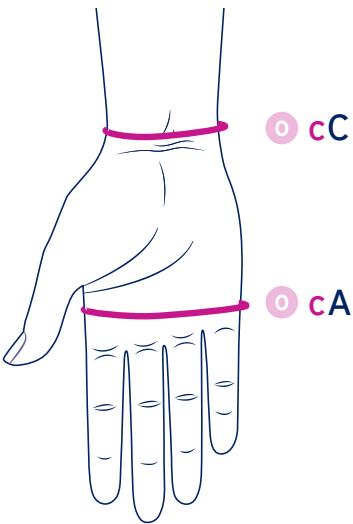

JOBST® Bella Strong Measuring Circumference

- STEP 1** Measure palm circumference
This leaves 2 choices.
- STEP 2** Measure wrist circumference
Select Glove.

Circumference Measurements Bella Lite		
Size	Palm cA	Wrist cC
Small	15 - 19 cm	14 - 20 cm
Medium	19 - 22 cm	16 - 22 cm
Large	22 - 26 cm	18 - 24 cm

Circumference Measurements Bella Strong		
Size	Palm cA	Wrist cC
1	15 - 17 cm	14 - 18 cm
2	17 - 19 cm	14 - 18 cm
3	17 - 19 cm	18 - 21 cm
4	19 - 21 cm	14 - 18 cm
5	19 - 21 cm	18 - 21 cm
6	21 - 23 cm	16 - 19 cm
7	21 - 23 cm	19 - 22 cm
8	23 - 26 cm	19 - 22 cm

TIPS


Mark the measurement points on the limb with a JOBST® Skin Marking Pencil



Take the corresponding circumference and length measurements using the JOBST® tape measure

Measuring Guide

Elvarex® Soft Ready-To-Wear Gloves and Gauntlets

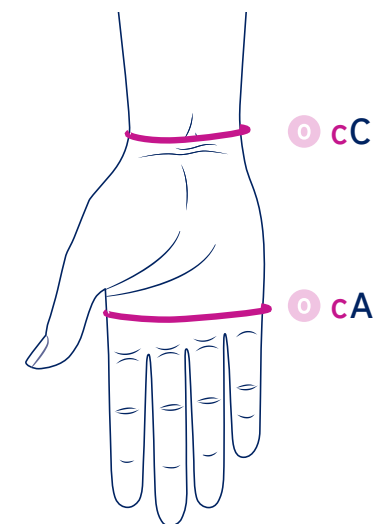


Measuring Circumference

- STEP 1** Measure palm circumference **cA**.
This leaves 2 choices.
- STEP 2** Measure wrist circumference **cC**.
Select Glove or Gauntlet.

Circumference Measurements Elvarex Soft RTW Gloves and Gauntlets

Size	Palm at base of fingers (cA)	Wrist (cC)
1	16.6 - 18 cm	15.6 - 16.7 cm
2	16.6 - 18 cm	16.7 - 18.1 cm
3	18 - 19.5 cm	15.9 - 17 cm
4	18 - 19.5 cm	17 - 18.5 cm
5	18 - 19.5 cm	18.5 - 20.4 cm
6	19.5 - 21.5 cm	17.2 - 18.7 cm
7	19.5 - 21.5 cm	18.7 - 20.6 cm
8	21.5 - 23.1 cm	18.3 - 20.1 cm

Elvarex® Soft Ready-To-Wear Armsleeves

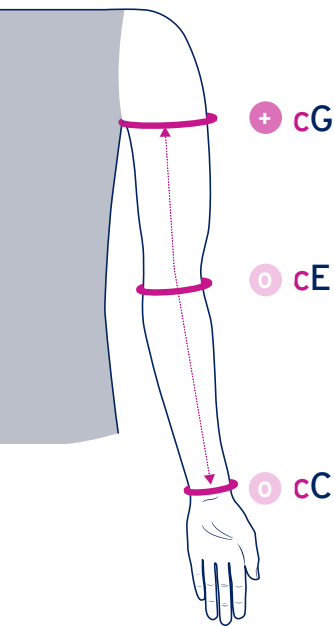

Length Measurements

Suitable for C-G lengths up to 44 cm for Petite, and C-G lengths greater than 44 cm for Regular.

Measuring Circumferences

- STEP 1** The patient should be seated, arm on table with palm facing upwards and elbow flexed to approximately 40°.
- STEP 2** Measure and record the circumference at C, the wrist.
NOTE: At the wrist and elbow, the tape should never be pulled tightly.
- STEP 3** Measure and record the circumference at E, the elbow. NOTE: Ensure the arm is slightly bent (approximately 40°) before taking this measurement.
- STEP 4** Measure and record the circumference at G. NOTE: Take the measurement approx 1 cm below axilla.

Compare the measurements you have obtained with the following table to determine the correct size.

Where measurements vary across all three sizes, or are smaller or larger than the tabled measurements, a custom-made sleeve will be required.

Circumference Measurements Elvarex Soft RTW Armsleeves

Size	1	2	3	4	5
cG (cm)	26 - 30	28 - 34	32 - 38	36 - 44	40 - 48
cE (cm)	22 - 26	24 - 28	26 - 32	29 - 35	32 - 40
cC (cm)	14 - 16	16 - 18	18 - 20	20 - 22	22 - 24

LCG Petite: <44 cm LCG Regular: >44 cm

Fibre Content

FABRIC/CCL	Spandex (Elastane)	Polyamide (Nylon)
JOBST® Bella Lite Armsleeve 20-30mmHg	20%	80%
JOBST® Bella Lite Gauntlet 20-30mmHg	14%	86%
JOBST® Bella Lite Combined Armsleeve 20-30mmHg	21%	79%
JOBST® Bella Lite Glove 20-30mmHg	41%	59%
JOBST® Bella Strong Armsleeve with Silicone Band 20-30mmHg	20%	72%
JOBST® Bella Strong Gauntlet 20-30mmHg	9%	91%
JOBST® Elvarex® Soft 15-21mmHg	16%	84%
JOBST® Elvarex® Soft 23-32mmHg	20%	80%

Selected products: Top bands contain silicone.

Caution: Products labelled on the packaging with  contain natural rubber (latex) which may cause allergic reactions.



JOBST®,
an Essity brand

Always read the
label and follow the
directions for use.

ESS0804
08/2026

Essity Health & Medical

PO Box 337, Mount Waverley,
Victoria, Australia 3149
medical.essity.com.au
jobst.com.au

JOBST® Customer Service AU
T 1300 998 810 F 1300 998 820
JOBST® Customer Service NZ
T 0508 998 810 F 0508 998 820