

FarrowWrap® Footpiece Application Techniques

Applicable for the following fabrics

JOBST® FarrowWrap® Lite

JOBST® FarrowWrap® Strong

JOBST® FarrowWrap® Classic



SCAN ME
for application video



1
Apply a liner. Either the open toe Soft Liner (white) or the closed toe Silver Liner (beige).

With the Soft Liner, ensure you extend the fabric past the toes distally and past the knee proximally to allow sufficient fabric to fold back over the wraps once they have been applied. Unwrap the footpiece and lay it on the floor with the hole for the heel closest to you.



2
Place the foot on top of the footpiece, resting the heel over the hole in the footpiece.



3
To secure the foot, hold the footpiece flap closest to the toes on each side and pull both sides over the top of the foot to the opposite side. Apply at or near end stretch with the VELCRO®.

Now apply the remaining flap (the one closest to the ankle) in the same manner.



4
After securing the forefoot, hold one side of the ankle band and pull it over the top of the mid-foot to the opposite side and secure it in place at or near end stretch.



5
Repeat the same step with the other half of the ankle band on the opposite side.

When removing the footpiece, fold the VELCRO® over on itself and back on the same band.



6
Care instructions (Strong and Lite fabric):
Machine wash in cold water with a mild detergent, tumble dry with no heat, do not iron, do not dry clean.

Care instructions (Classic fabric):
Hand wash in cold water with mild detergent, lay flat to dry, do not iron, do not dry clean.



JOBST®,
an Essity brand



Always follow the directions for use.

ESS0776 V2
05/2024

Essity Health & Medical

PO Box 337, Mount Waverley,
Victoria, Australia 3149
medical.essity.com.au
jobst.com.au

JOBST® Customer Service AU
T 1300 998 810 F 1300 998 820

JOBST® Customer Service NZ
T 0508 998 810 F 0508 998 820