

Selecting the right product Patients with Chronic Oedema/Lymphoedema

JOBST® FarrowWrap® is part of the JOBST® family of products, an extended variety of innovative treatment options available to meet the needs of patients in all therapy stages. JOBST® FarrowWrap® is versatile and easy to use and offers an alternative in the treatment of a lymphoedema patient.

Decongestion and Maintenance			
Therapy Stages	Mild Oedema	Moderate Oedema	Severe Oedema
Compression Wrap Systems Products	JOBST® FarrowWrap® Lite		
	JOBST® FarrowWrap® Strong		
	JOBST® FarrowWrap® Classic		

Application techniques videos

FarrowWrap®
Foot Piece



SCAN ME
for application video

FarrowWrap®
Leg Piece



SCAN ME
for application video

FarrowWrap®
Upper Limb Piece



SCAN ME
for application video

JOBST® FARROWWRAP®




Oedema management made simple through adjustable short-stretch wrap solutions

How to measure

Ready-to-wear pieces	XSmall	Small	Medium	Large	XLarge
C Calf	36-43 cm	42-50 cm	48-58 cm	53-63 cm	58-68 cm
B Ankle	21-25 cm	25-30 cm	30-36 cm	36-42 cm	42-50 cm
A-D Regular	35-39 cm	37-41 cm	39-43 cm	41-45 cm	42-46 cm
A-D Tall	40-44 cm	42-46 cm	44-48 cm	46-50 cm	47-51 cm


Length A-D = Floor to 2 finger widths below crease of knee. Measure posteriorly following contour of leg.
Calf circumference taken at the widest part of the calf - this will be measure B. Ankle circumference taken at the narrowest part of the ankle - this will be measure B.
Cuff circumference taken at the widest part of the cuff - this will be measure C. Ankle circumference taken at the heel to the base of the 5th toe.

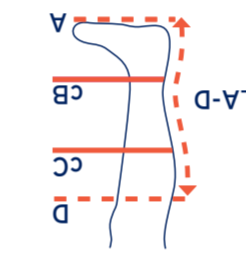

JOBST® FarrowWrap® RTW Foot Piece Size Chart

Ready-to-wear pieces	XSmall	Small	Medium	Large	XLarge
A1 Mid-foot	22-24 cm	25-27 cm	28-30 cm	31-34 cm	35-40 cm
X Regular	15-17 cm	16-19 cm	18-20 cm	19-22 cm	21-24 cm
X Tall	18-20 cm	20-22 cm	21-23 cm	23-25 cm	25-27 cm


The length is measured from the most posterior aspect of the heel to the base of the 5th toe. The midfoot measurement is a circumferential measurement taken at the midfoot point.

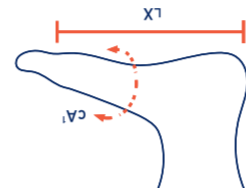

JOBST® FarrowWrap® RTW Thigh Piece Size Chart

Ready-to-wear pieces	XSmall	Small	Medium	Large	XLarge
G Groin	60-70 cm	65-75 cm	70-80 cm	75-90 cm	85-100 cm
E Above Knee	47-52 cm	53-59 cm	60-66 cm	67-73 cm	74-80 cm
E-G Short	16-20 cm	16-20 cm	16-20 cm	16-20 cm	16-20 cm
E-G Regular	21-26 cm	21-26 cm	21-26 cm	21-26 cm	21-26 cm
E-G Tall	27-31 cm	27-31 cm	27-31 cm	27-31 cm	27-31 cm


There are three measurements required for the Thigh piece. Above knee circumference - measure 5cm above the knee - this is your E measure. Groin circumference - measure at the girthal fold where the buttocks meets the leg, this will be your G measure. Thigh length - measure the length of the thigh medially (inside leg) between E and G following the contour of the leg, this will be your E-G measure.

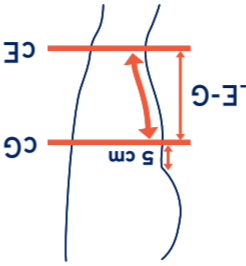

How to measure

Ready-to-wear pieces	XSmall	Small	Medium	Large
Wrist	14 - 18 cm	20 - 27 cm	22 - 31 cm	
Elbow	16 - 21 cm	25 - 34 cm	29 - 39 cm	
Axilla	19 - 25 cm	30 - 40 cm	32 - 45 cm	

NOTE: Measure length from the outside of the wrist to the axilla (G) with a slight bend in arm

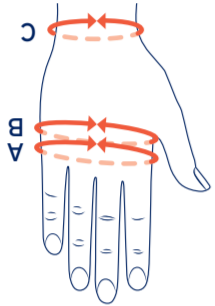

JOBST® FarrowWrap® RTW Gauntlet size chart

Ready-to-wear pieces	XSmall	Small	Medium	Large
A	16 - 18 cm	18 - 20 cm	20 - 22 cm	22 - 24 cm
B	17 - 19 cm	19 - 21 cm	21 - 23 cm	23 - 26 cm
C	15 - 18 cm	17 - 20 cm	19 - 22 cm	22 - 24 cm


Male Shoe Size	Female Shoe Size
3 - 6	4 - 7
7 - 9	8 - 10
10 - 12	11 - 13
13 - 15	14 - 16

Note the maximum calf circumference for Hybrid Liners is 60 cm

JOBST® FarrowWrap® Liners size chart

Open Toe Liners
Can accommodate the following calf circumferences:
Small (<40 cm), Medium (40-70 cm) and Large (70-120 cm).
Closed Toe Liners
Standard - Can accommodate calf circumferences to 60cm.
Wide - Can accommodate calf circumferences to 70cm.
Hybrid Liners
Measure the mid foot A circumference.
Two mid foot sizes available:
20-27cm (regular foot) and 28-35cm (wide foot).
Request the patients shoe size (US Sizing).

Size	CA
XSmall	20 - 22 cm
Small	22 - 24 cm
Medium	24 - 26 cm
Large	26 - 28 cm

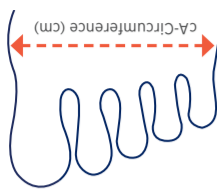

Toe Cap measurement chart

JOBST® FarrowWrap® Knee Piece Size Chart

Ready-to-wear pieces	XSmall	Small	Medium	Large	XLarge
5 cm above patella	47 - 52 cm	53 - 59 cm	60 - 66 cm	67 - 73 cm	74 - 80 cm
Calf	36 - 43 cm	42 - 50 cm	48 - 58 cm	53 - 63 cm	58 - 68 cm

The patient needs to be measured at two points, C (widest calf) and E.

