


PATIENTS WITH VENOUS LEG ULCERS WITH RECOMMENDED COMPRESSION THERAPY

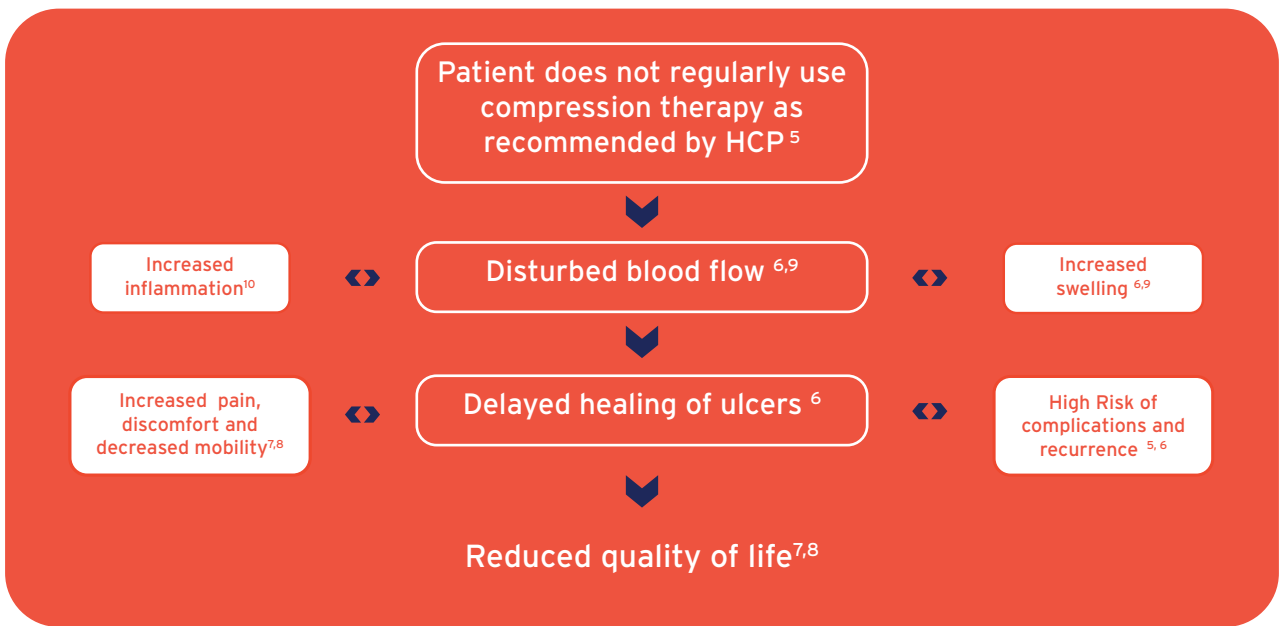

Trusted JOBST® Partners

You can purchase JOBST® Ready-To-Wear Compression garments from a range of online stockists and local pharmacies.

Scan the QR code for a list of all the online stockists



JOBST® How to Measure Videos

Please scan the QR code to view videos explaining how to measure for the following garments- Knee High, Thigh High and Waist High.



References:

1. O'Meara S, Cullum N, Nelson EA, Dumville JC. Compression for venous leg ulcers. Cochrane Database of Systematic Reviews 2012, Issue 11. Art. No.: CD000265. DOI: 10.1002/14651858.CD000265.pub3 <https://doi.org/10.1002/14651858.CD000265.pub3>
2. Nair B. Compression therapy for venous leg ulcers. Indian Dermatol Online J. 2014 Jul;5(3):378-82. doi: 10.4103/2229-5178.137822. PMID: 25165679; PMCID: PMC4144247. Indian Dermatology Online Journal
3. Marston WA, Carlin RE, Passman MA, Farber MA, Keagy BA. Healing rates and cost efficacy of outpatient compression treatment for leg ulcers associated with venous insufficiency. J Vasc Surg. 1999 Sep;30(3):491-8. doi: 10.1016/s0741-5214(99)70076-5. PMID: 10477642. <https://linkinghub.elsevier.com/retrieve/pii/S0741521499002748>
4. Finlayson K, Edwards H, Courtney M. The impact of psychosocial factors on adherence to compression therapy to prevent recurrence of venous leg ulcers. J Clin Nurs. 2010 May;19(9-10):1289-97. doi: 10.1111/j.1365-2702.2009.03151.x. PMID: 20500338. <https://doi.org/10.1111/j.1365-2702.2009.03151.x>
5. Raju S, Hollis K, Neglen P. Use of compression stockings in chronic venous disease: patient compliance and efficacy. Ann Vasc Surg. 2007 Nov;21(6):790-5. doi: 10.1016/j.avsg.2007.07.014. PMID: 17980798. <https://doi.org/10.1016/j.avsg.2007.07.014>
6. Christine Moffatt, Dheerendra Kommala, Nathalie Dourdin, Yoonhee Choe. Venous leg ulcers: patient concordance with compression therapy and its impact on healing and prevention of recurrence. International Wound Journal, 01 Oct 2009, 6(5):386-393 Venous leg ulcers: patient concordance with compression therapy and its impact on healing and prevention of recurrence - PubMed
7. Declan Patton, Pinar Avsar, Aicha Sayeh, Aglecia Budri, Tom O'Connor, Simone Walsh, Linda Nugent, Denis Harkin, Niall O'Brien, Jonathan Cayce, Michael Corcoran, Mario Gaztambide A meta-review of the impact of compression therapy on venous leg ulcer healing. Link: A meta-review of the impact of compression therapy on venous leg ulcer healing - PMC
8. Herber OR, Schnepf W, Rieger MA. A systematic review on the impact of leg ulceration on patients' quality of life. Health Qual Life Outcomes. 2007 Jul 25;5:44. doi: 10.1186/1477-7525-5-44. PMID: 17651490; PMCID: PMC1947954. A systematic review on the impact of leg ulceration on patients' quality of life - PMC
9. Shi C, Dumville JC, Cullum N, Connaughton E, Norman G. Compression bandages or stockings versus no compression for treating venous leg ulcers. Cochrane Database Syst Rev. 2021 Jul 26;7(7):CD013397. doi: 10.1002/14651858.CD013397.pub2. PMID: 34308565; PMCID: PMC8407020. Link: <https://pubmed.ncbi.nlm.nih.gov/34308565/>
10. Mosti, G. (2023). Compression Therapy in Ulcer Care. In: Maruccia, M., Papa, G., Ricci, E., Giudice, G. (eds) Pearls and Pitfalls in Skin Ulcer Management. Springer, Cham. https://doi.org/10.1007/978-3-031-45453-0_12 link: <https://rdcu.be/dZPMs>

