

A man with a grey beard and a light blue shirt is smiling while playing an acoustic guitar. He is wearing a white cast on his right leg, which is resting on a silver metal cooler. He is also wearing dark blue shorts and dark green sneakers. To his left, a woman with long brown hair, wearing a pink and white striped shirt and white pants, is sitting on a wooden picnic table and looking at him. In the background, another woman in a blue dress is clapping. The scene is outdoors near a lake with mountains in the distance under a blue sky with light clouds.

JOBST®

Feel confident and in
control of your
Self-Care

JOBST® FarrowWrap®

Your guide to independently managing oedema
with an Adjustable Compression Device.

Take control of your compression therapy



Living with a chronic condition means having to adapt your daily routine to suit new and sometimes complex needs, which can be challenging at times. In order to reach your health and wellbeing goals, it's important to play an active role in your care and find tools that help you effectively manage your condition, while enabling you to continue living life at your own pace.

When it comes to managing swelling from lymphatic or venous diseases, bandaging and hosiery are considered the most effective treatment options. However these can be difficult to independently manage for some people. Therefore, innovative solutions such as adjustable compression devices offer an alternative.

DID YOU KNOW ?

Wraps can be used in all stages of oedema management such as **decongestion therapy** to reduce limb volume and size, to **stabilize fluctuations** before moving onto compression garments and also in **maintaining** therapy results long-term, for those who have issues to apply and remove hosiery.

THE BENEFITS ARE

ALL YOURS

COMPRESSION WRAPS LET YOU TAKE CONTROL OF YOUR OWN CARE AND OFFER A RANGE OF BENEFITS:

- ✓ Designed for easy application and removal
- ✓ Suitable for all oedema management stages
- ✓ Deliver therapeutic compression levels
- ✓ Can be worn discreetly under clothing



Unwrapping

Adjustable Compression Devices

Taking control of your health can be made easier with innovative solutions like adjustable compression devices. They may offer a great alternative to multi-layer bandaging or hosiery for those who find it challenging to apply or adjust on their own, or for those who simply prefer

to have variety in their treatment options.

They are designed to be easy to use and discrete to wear under your clothing - which is perfect for those who want to continue living as normally as possible despite a change to their daily routines.

EASY TO ADAPT TO YOUR INDIVIDUAL NEEDS

Adjustable compression devices are designed to provide support similar to that of bandaging - but offer greater flexibility, choice and the ability to independently self-apply and adjust according to your needs and tolerance.



WHO BENEFITS FROM

COMPRESSION WRAPS?

WHEN COMPRESSION BANDAGING OR HOSIERY IS NOT SUITABLE, WRAPS CAN PROVIDE A FLEXIBLE TREATMENT OPTION FOR PATIENTS WITH:

- ✓ Swelling resulting from lymphoedema and venous oedema
- ✓ Venous leg ulcers
- ✓ Larger body sizes
- ✓ Physical limitations (eg: weak hand strength, back issues)

THE TECHNOLOGY THAT

MAKES WRAPS WORK

Short-Stretch technology refers to the force in which a compression wrap gives on the limb. When you are active and moving, the fabric provides **high working pressure** and can help improve venous flow and encourage better lymphatic fluid drainage. When you're less active and relaxing, the **low resting pressure** helps ensure that your wrap remains comfortable and safe to wear.

**ASK YOUR
HEALTHCARE
PROVIDER IF A
WRAP IS RIGHT
FOR YOU.**





JOBST® FarrowWrap - Compression that suits your lifestyle.

The JOBST FarrowWrap range of adjustable compression devices enables you to confidently care for your own needs and manage oedema with ease.

Made with durable fabrics, they help to reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers.

APPLY AND ADJUST

CONFIDENTLY

JOBST FarrowWrap was engineered for easy application and can offer all the benefits of multi-layer bandaging, without the tricky application. The adjustable VELCRO® brand bands allow you to quickly, safely and confidently respond to fluctuating oedema levels throughout the day. Simply tighten or loosen the wrap as needed.

The bands have an overlapping design, which not only provides the support needed to control oedema but also helps reduce the risk of swelling forming between the bands.



BENEFITS

Easy Donning

Easy-to-attach VELCRO® brand tabs support **easy donning and doffing.**

Short-Stretch Technology

Short-Stretch technology provides low resting and high working pressures to **enhance lymphatic and venous return.**

Adjustable

VELCRO® brand hook & loop system lets you **respond better to fluctuating oedema.** Simply tighten or loosen the wrap.

Low Profile Design

Discreet design is less bulky than multi-layer bandaging and **can be worn under normal clothing.**



JOBST® FarrowWrap

For confidence
with Compression

The JOBST FarrowWrap range of easy-to-apply compression wraps enables you to self-manage your oedema and empowers you with everyday confidence and comfort. Made from durable material, JOBST

FarrowWrap is designed to provide the same strong support as multi-layer bandaging but in a simplified, intuitive design. It helps to reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers.



JOBST FARROWWRAP

ARM PIECES

For oedema of the arms, the JOBST FarrowWrap Arm Piece is available in LITE fabric. The built in inner sleeve makes putting it on easy.



JOBST FARROWWRAP

LEG PIECES

For oedema effecting the lower limbs, the JOBST FarrowWrap leg pieces are available in a range of length types depending on your level of oedema, including: Classic, Lite, and Strong.

85 %
OF PATIENTS




A recent clinical study on the performance and safety of JOBST FarrowWrap found that 85% of patients were able to easily adapt the wrap to suit their individual needs.¹

73% of patients gave a positive assessment of JOBST FarrowWrap in terms of manageability, making it a suitable choice for managing the challenging issues associated with venous and lymphatic conditions.¹

¹ Observational Study JOBST® FarrowWrap® 2019, data on file

A style to suit
every need:

JOBST FarrowWrap is available in a range of styles, fabrics, sizes and compression classes. Contact your fitter to discuss your individual needs and preferences.

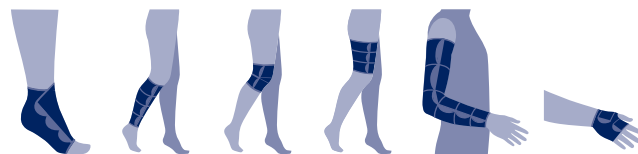



Top Tips for compression self-care



CHOOSE THE RIGHT SHOES

Safe footwear is important to reduce the risk of falling and to help protect your garment from any unnecessary wear and tear. These days, it's easy to find footwear options that are made with flexible material that can comfortably and fashionably accommodate your foot garment.



WATCH WHAT YOU EAT

Although no specific diet has been shown to be effective in chronic oedema management, following a healthy and balanced diet is generally recommended. Furthermore, obesity may worsen pre-existing lymphoedema.



HOW TO WASH YOUR WRAP

Before throwing your wrap into your washing machine*, fold the VELCRO® brand hook and loop fastener back on itself to avoid any sticky situations while washing. Following the provided care instructions will help ensure a longer life span for your wrap.



LOVE YOUR SKIN

Keep your skin properly hydrated with emollient creams, as dry or cracked skin can increase the risk of developing infections such as cellulitis. It's also important to ensure any wounds are thoroughly cleaned and covered with a suitable dressing before applying compression.

* JOBST FarrowWrap Classic must be handwashed



We're here
for you...

... AT EVERY

STAGE

If you have any questions or want to know more about how JOBST products can support you, visit:

www.jobst.com.au

WE CARE FOR YOU

**BSN medical Australia & New Zealand
an Essity company**

L6, 211 Wellington Road,
Mulgrave, Victoria Australia 3170
PO Box 337, Mount Waverley, Victoria 3149

JOBST® Customer Service Australia
T 1300 998 810 F 1300 998 820
www.bsnmedical.com.au

JOBST® Customer Service New Zealand
T 0508 998 810 F 0508 998 820
www.bsnmedical.co.nz

