

Fracture Management

Product Brochure



Delta-Cast®



About Essity

Essity is a leading global hygiene and health company. We are dedicated to improving well-being through our products and services. Sales are conducted in approximately 150 countries under the leading global brands TENA and Tork, and other strong brands, such as Actimove, Delta-Cast, JOBST, Knix, Leukoplast, Libero, Libresse, Lotus, Modibodi, Nosotras, Saba, Tempo, TOM Organic, Vinda and Zewa. Essity breaks barriers to well-being and contributes to a healthy, sustainable and circular society. More information at www.essity.com.

For more product information and advice please contact our Customer Service team who are always ready to answer your queries or take order details by telephone **0845 1223600**, or by email orders.uk@essity.com

Support

When faced with orthopaedic trauma, effective care for optimal healing is required. Delta-Cast is your trusted partner for broken bones and injured ligaments.

As the global market leader in casting with a strong brand heritage in fracture management, Delta-Cast has a comprehensive and innovative portfolio that you can rely on.

Delta-Cast takes pride in going the extra mile to support you through service and training so that you have the knowledge and tools to overcome any casting challenge that comes your way.

Delta-Cast – We know casting.



Education

Our commitment to professional education is illustrated with our Essity PATH educational platform. It underpins our belief that progress in patient care depends on continuing professional development and our educational initiatives are an integral part of our relationship with clinicians.

PATH is a gold standard education platform with a diverse selection of therapy areas and learning styles, to help you provide the best possible patient care.



**Professional Accredited Training
for Healthcare Professionals**

You choose the educational journey that suits you

Therapy Areas

Select from the following:

- Continence Care
- Lower Limb Management
- Lymphoedema Care
- Wound Management
- Orthopaedics
- Animal Healthcare


Topics

Then choose...

...from the available topics within each therapy area and the different learning styles

Learning Styles

- e-Learning
- Webcast
- Bitesize Learning
- Masterclasses and Events

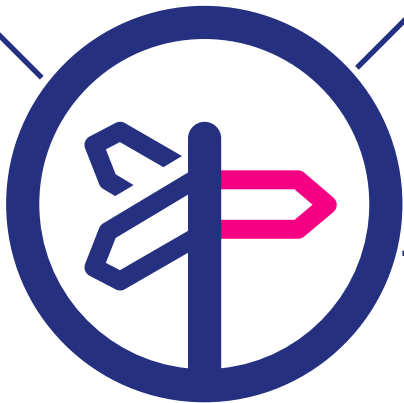

To find out more please contact your local Acute Specialist or contact us on:
Tel: **01482 670177** Email: customerconnect@essity.com

Digital Support

Essity's Orthopaedic product portfolio

Essity provide Orthopaedic solutions across the continuum of care from immobilisation with our Fracture Management range to bracing and supports with our Actimove brand to rehabilitations with our Physiotherapy range of tapes and strapping. A selection of these products have been shown in this catalogue, the full product portfolio can be found by scanning the QR code below. Please contact your Essity Acute Specialist or customerconnect@essity.com for further information or hard copies of the catalogues.



Practical Guide to Casting

To access the digital version of the Practical Guide to Casting, scan the QR code below. Please contact your Essity Acute Specialist or customerconnect@essity.com for further information on how to purchase hard copies.



Notes

Contents

Cast Removal		
Scissors	6	
Zip Stick	6	
Delta-Cast® Serene™	7	
Casting Tapes and Splinting		
Casting Tapes and Splinting	8	
Quick Guide - Application Techniques	9	
Plaster of Paris		
Gypsona®	10	
Specialist® E	11	
Biplatrix®	11	
Non-Fibreglass Casting Tape		
Delta-Cast® Conformable	13	
Delta-Cast® Elite	14	
Delta-Cast® Prints	15	
Fibreglass Casting Tape		
Delta-Lite® Plus	16	
Flexible Casting Tape		
Delta-Cast® Soft	17	
Synthetic Splinting		
Dynacast® Prelude	18	
Dynacast® Prelude Solo	18	
Dynacast® Prelude Strap Kits	19	
Dynacast® AS Ankle Splint	19	
Alternative Splinting		
Delta-Xpress®	20	
Undercast Padding		
Padding		
Soffban® Natural	22	
Velband®	22	
Soffban® Synthetic	23	
Delta-Rol® Synthetic	23	
Delta-Rol® 'S' Synthetic	24	
Water Resistant Cast Padding		
Delta-Dry®	24	
Delta-Dry® Pantaloons	25	
Stockinette		
Natural		
Jersey Standard	27	
Delta Terry-Net® C	27	
Synthetic		
Delta-Net® Stockinette	28	
Delta Terry-Net® S	28	
Delta Terry-Net® Thumb Spica	28	
Water Resistant Stockinette		
Delta-Dry® Stockinette	29	
Delta-Dry® Softliner	29	
Thermoplastics		
Thermoplastics		
Delta Thermo®	30	
Casting Accessories		
Accessories		
Tensoban® Pre-Tape Underwrap	32	
Tensoplast® Sport Cast Edge Tape	32	
Delta Terry-Net® Adhesive Felt Liner	33	
Delta Terry-Net® Cloth and Foam Liner	33	
Delta Terry-Net® Adhesive Fleece Edger	34	
Delta Terry-Net® Adhesive Hook and Stretch Loop Fabric	34	
Skin Traction Kits		
Tensoplast® Skin Traction Kits	35	
Casting Equipment		
Knee Rest	35	
Soffcrepe®	36	
Tensoplast® EAP	36	
Co-plus®	37	
Selection of Actimove®		
Artistep® S by Actimove®	38	
Actimove® Offloading Shoe	38	
Actimove® Cast Shoe Rocker Sole	39	
Actimove® Post-Op Shoe Open Toe	39	
Actimove® Cast Shoe Removable Toe Cover	40	
Actimove® Sling Comfort	41	
Actimove® Umerus Comfort	41	
Actimove® Mitella Comfort	42	



Delta-Cast® Serene™

Cast Remover

NEW

Meet the new cast removing device

- The static blade does not generate airborne particles, keeping the cast room air cleaner without the need for a vacuum system
- The weight of the balanced device rests on the cast – enabling almost effortless one-handed operation
- Due to its quiet operation, it allows for easier communication and reduced anxiety for patients
- Effortlessly cuts through up to 10 layers of synthetic cast material
- The blade lasts for up to 200 cuts, is quick to change and easy to clean
- Easy to move from room to room with no cumbersome vacuum system
- The concealed blade that does not produce vibrations or heat – making it less intimidating for patients, especially children
- The hidden static blade and protective foot and hidden blade, helping to mitigate the risk of cuts and burns



Cast Removal

From protection during removal through to full cast removal. This new range is available to you during 2025!

Zip Stick

NEW

Cast Removal Aid

Provides additional patient protection during cast removal whilst using scissors or an oscillating cast saw.



Code	Size	Pack Qty
72046-29	48.5cm x 2.5cm	1 x 1

Esmarch Scissors

NEW

Cast Removal Scissors

Made of high-quality stainless steel to provide precision performance. The blunt tip on the bottom blade provides increased patient safety. Excellent to cut bandages.



Code	Size	Pack Qty
72044-07	8 in	1 x 1

Code	Product	Pack Qty
76289-02	Delta-Cast® Serene™ (inc. all components)	1 x 1
76289-07	Delta-Cast® Serene™ Blades	1 x 10
76289-06	Delta-Cast® Serene™ Replacement footer plate*	1 x 1
76289-05	Delta-Cast® Serene™ Replacement Pin*	1 x 1
76289-12	Delta-Cast® Serene™ Replacement Power cable*	1 x 1

*Please note, replacement pin, footer plate and power supply are available if required, these are not a mandatory purchase

For **further information** or to book a **demonstration**, contact **customer services** or your **Essity Acute Specialist**



Casting Tapes and Splinting

Setting the industry benchmark in synthetic casting materials, the Delta range rises to the daily challenges of fracture management.

Delta-Lite Plus is a smooth, durable fibreglass casting tape that combines a fibreglass substrate and a polyurethane resin system. **Delta-Cast Conformable, Delta Cast Elite, Delta Cast Prints** are our non-fibreglass casting tapes, combining strength and rigidity with soft edges for enhance patient comfort. Flexible cast tape **Delta-Cast Soft** provides functional support to fractures and soft tissue injuries.

The **Dynacast Prelude** range and **Delta-Xpress** are ground-breaking splinting systems that allow you to immobilise limbs in a fraction of the time, compared to plaster of Paris slabs.

	Plaster of Paris	Delta-Cast® Conformable	Delta-Cast® Elite	Delta-Cast® Prints	Delta-Lite® Plus	Delta-Cast® Soft	Dynacast® Prelude	Dynacast® Prelude Solo
Substrate	POP	Poly	Poly	Poly	FG	Poly	FG	FG
Flexibility	R	SR	R	R	R	F	R	R

Quick Guide - Application Techniques

		Plaster of Paris Applications	Synthetic Casting	Soft Combination Casting	Focused Rigidity Casting (FRC)	Total Contact Casting (Below Knee)	DFU Slipper Casts	DFU Heel Cup Casts	Synthetic Splinting
Plaster of Paris Synthetic casting tapes	Plaster Of Paris (Gypsona®, Specialist® E, Biplatrix®)	✓				✓			
	Delta-Cast® Conformable		✓		✓	✓			
	Delta-Cast® Elite		✓			✓			
	Delta-Cast® Prints		✓			✓			
	Delta-Lite® Plus		✓	✓		✓	✓		
	Delta-Cast® Soft			✓		✓	✓	✓	
Synthetic splinting	Dynacast® Prelude Splint								✓
	Dynacast® Prelude Solo Splint			✓		✓	✓		✓
Strap kits	Dynacast® Prelude Strap Kits			✓	✓	✓	✓		✓
Padding	Soffban® Natural	✓	✓			✓			
	Velband®	✓							
	Soffban® Synthetic	✓	✓			✓			
	Delta-Rol®	✓	✓			✓			
	Delta-Rol® S		✓						
	Delta-Dry® Padding		✓						
Stockinette	Jersey Standard	✓	✓	✓	✓	✓	✓	✓	
	Delta Terry-Net® C		✓	✓	✓	✓	✓	✓	
	Delta-Net®	✓	✓	✓	✓	✓	✓	✓	
	Delta Terry-Net® S		✓	✓	✓	✓	✓	✓	
	Delta Terry-Net® Thumb Spica		✓	✓	✓				
	Delta-Dry®		✓						
	Delta-Dry® Softliner		✓	✓	✓				
Accessories	Tensoban® Pre-Tape		✓			✓			
	Tensoplast® Sport Cast Edge Tape		✓	✓	✓	✓	✓		✓
	Delta Terry-Net® Adhesive Felt Liner	✓	✓	✓	✓	✓	✓	✓	✓
	Delta Terry-Net® Adhesive Fleece Edger		✓	✓	✓	✓	✓		✓
	Delta-Net® Adhesive Hook and Stretch Loop			✓	✓	✓	✓		

Plaster of Paris



Gypsona, **Specialist E** and **Biplatrix** are our plaster of Paris products that suit all immobilisation needs. All formulations are available in rolls and slabs.

Gypsona®

Plaster of Paris

Gypsona® premium plaster of Paris bandage is made from a specially woven leno weave cloth, uniformly impregnated with high quality, creamy, fast setting plaster of Paris that produces a strong cast with an excellent finish. Available in roll and 5-ply slab format.

Interlocked leno weave gauze

- Provides excellent stability and conformability to the wet bandage
- Minimises plaster loss
- Provides strong finished cast

Smooth, creamy plaster formula

- Allows for a smooth aesthetically pleasing cast
- Improved moulding properties and excellent finish

Nicked bandage edge design

- Ensures minimal fraying and loose threads during application
- Supports a smoother cast finish

Round core

- Improved handling stability



Code	Size	Pack Qty	NHS SC Code
71985-04	5cm x 2.7m roll	1 x 48	EAA051
71985-05	7.5cm x 2.7m roll	1 x 48	EAA052
71985-00	10cm x 2.7m roll	1 x 48	EAA053
71985-02	15cm x 2.7m roll	1 x 24	EAA054
71985-03	20cm x 2.7m roll	1 x 24	EAA055
71988-14	10cm x 20m 5-ply slab dispenser	1 x 1	EAA023
71988-15	15cm x 20m 5-ply slab dispenser	1 x 1	EAA025
71988-16	20cm x 20m 5-ply slab dispenser	1 x 1	EAA027

LATEX
FREE
FORMULA

Specialist® E

Plaster of Paris

Specialist® E plaster of Paris has a creamy texture resulting in a smooth feel, and making it easy to apply. It forms a well fitting cast that is reassuringly tough and strong. It is the ideal choice for casting applications where a quick setting time is required. Available in roll and 5-ply slab format.

Smooth, creamy texture

- Results in a smooth, evenly finished cast
- Minimal plaster loss

Highly mouldable and strong

- Produces an excellent fit around difficult body contours and delivers a well laminated cast

Easy to apply and remove

- Able to 'window' the cast after only 10 minutes

Moisture proof packaging

- Protects and prolongs the product shelf life



Code	Size	Pack Qty	NHS SC Code
72260-00	5cm x 2.7m roll	1 x 12	EAA056
72260-01	7.5cm x 2.7m roll	1 x 12	EAA057
72260-02	10cm x 2.7m roll	1 x 12	EAA058
72260-04	15cm x 2.7m roll	1 x 12	EAA092
72260-05	20cm x 2.7m roll	1 x 12	EAA088
72259-00	10cm x 20m 5-ply slab dispenser	1 x 1	EAA007
72259-02	15cm x 20m 5-ply slab dispenser	1 x 1	EAA008
72259-03	20cm x 20m 5-ply slab dispenser	1 x 1	EAA009

Biplatrix®

Plaster of Paris

Biplatrix® is a very fast setting plaster of Paris bandage that consists of a cotton gauze substrate which is spread with gypsum and spooled on a round core. The bandages wet rapidly and evenly with no dry spots. The initial weight of plaster spread and it's low plaster loss combine to offer a smooth plaster and a durable cast.

Ultra fast setting time

- Working time of two minutes when activated in water (at maximum 25°C)
- Ideal for orthotic applications

High resistance and durability

- The high mass weight of Biplatrix® combined with low plaster loss provides a durable cast

Strong plaster formula

- Less creamy than other plaster of Paris, Biplatrix® provides a strong rigid plaster

Moisture resistant packaging

- Biplatrix® rolls are dual packed in a clear polythene film



Code	Size	Pack Qty	NHS SC Code
71995-24	5cm x 3m roll	1 x 40	EAA015
71995-05	8cm x 3m roll	1 x 40	EAA018
71995-06	10cm x 3m roll	1 x 40	EAA029
71995-09	15cm x 3m roll	1 x 40	EAA030
71995-11	20cm x 3m roll	1 x 20	EAA031
71995-15	10cm x 25m 4-ply slab dispenser	1 x 1	EAA032
71995-16	15cm x 25m 4-ply slab dispenser	1 x 1	EAA033
71995-17	20cm x 25m 4-ply slab dispenser	1 x 1	EAA034

LATEX
FREE
FORMULA

Casting Tapes and Splinting

Synthetic Casting Tapes

Delta-Cast® Conformable

Page 13

- Semi-Rigid – Polyester
- Used for removable casts which can be made and re-applied for great control
- 3D stretch helps to cover even difficult body contours

Delta-Cast® Elite

Page 14

- Rigid – Polyester
- Used for primary and secondary casting applications
- Provides a smooth and soft cast edges for comfort

Delta-Cast® Prints

Page 15

- Rigid – Polyester
- Used for primary and secondary casting applications.
- Offering a range of colourful alternative patterns for your patients enjoyment

Delta-Lite® Plus

Page 16

- Rigid – Fibreglass
- Used for primary casting applications and as reinforcing strips
- Good conformability making casting applications easier around all body contours

Delta-Cast® Soft

Page 17

- Flexible – Polyester
- Designed to provide functional support to fractures and soft tissue injuries
- Improved flexibility and mobility can help promote a faster healing process

Splints and Kits

Dynacast® Prelude

Page 18

- Fiberglass - Padded synthetic splinting system
- Ideal for trauma and post-op applications
- Initial set time of 3-5 minutes and weight bearing in 15 minutes
- No mess or waste

Dynacast® Prelude Solo

Page 18

- Fiberglass - Unpadded precut synthetic splinting system
- Ideal for trauma and post-op applications
- Ideal for theatre and A&E environments

Dynacast® Prelude Strap Kits

Page 19

- Latex-free hook and loop straps designed for use with Dynacast Prelude splints
- Allows the splint to remain in place without the need for a full bandage
- Each pack contains three straps

Dynacast® AS Ankle Splints

Page 19

- A custom mouldable solution for external ankle support
- Can be used throughout treatment from acute phase to rehabilitation
- Allows the use of normal footwear

Delta-Cast® Conformable

LATEX
FREE
FORMULA

Non-Fibreglass Cast Tape

Delta-Cast® Conformable has set a benchmark in standard and removable, non-fibreglass cast tapes. Delta-Cast® Conformable responds to the daily challenges of fracture management - it's distinctive construction provides rigid and semi-rigid casting options for primary and secondary casting applications where patient fit, comfort and compliance are critical. Delta-Cast® Conformable is the only UK cast tape with clinical evidence¹ to support its use for focused rigidity casting.



High performance 3-D stretch polyester bandage

- Outstanding conformability allows uninterrupted application without the need to tuck and fold, and without creasing
- 3-D stretch helps to cover even difficult body contours in one go to help produce wrinkle-free casts, without pressure points
- Long-lasting durability and resiliency for minimal cast breakdown

Patented resin formula

- Silicone and latex-free resin is designed to support ease of application and to ensure strong, long lasting lamination, offering:
 - Consistent and reliable set times
 - High lamination of layers and end lay down for optimum stability, strength and reliability
 - A smooth sensation during moulding which results in a smooth finish to the cast
 - Application with any standard examination gloves

Countless options

- Versatile rigidity enables the practitioner to generate either traditional total contact casts (TCC) or focused rigidity casts (FRC) with just one product
- Removable casts can be made and re-applied for greater control of the healing process

Code	Size	Colour	Pack Qty	NHS SC Code
72280-00	2.5cm x 1.80m	○ White	1 x 10	EAF2659
72280-01	5cm x 3.6m	○ White	1 x 10	EAF308
72280-02	7.5cm x 3.6m	○ White	1 x 10	EAF401
72280-03	10cm x 3.6m	○ White	1 x 10	EAF402
72280-04	12.5cm x 3.6m	○ White	1 x 10	EAF403
72281-00	5cm x 3.6m	● Pink	1 x 10	EAF495
72281-01	7.5cm x 3.6m	● Pink	1 x 10	EAF510
72281-02	10cm x 3.6m	● Pink	1 x 10	EAF521
72282-00	5cm x 3.6m	● Dark blue	1 x 10	EAF492
72282-01	7.5cm x 3.6m	● Dark blue	1 x 10	EAF504
72282-02	10cm x 3.6m	● Dark blue	1 x 10	EAF516
72283-00	5cm x 3.6m	● Yellow	1 x 10	EAF498
72283-01	7.5cm x 3.6m	● Yellow	1 x 10	EAF513
72283-02	10cm x 3.6m	● Yellow	1 x 10	EAF525
72284-00	5cm x 3.6m	● Orange	1 x 10	EAF494
72284-01	7.5cm x 3.6m	● Orange	1 x 10	EAF508
72284-02	10cm x 3.6m	● Orange	1 x 10	EAF518
72285-00	5cm x 3.6m	● Red	1 x 10	EAF497
72285-01	7.5cm x 3.6m	● Red	1 x 10	EAF512
72285-02	10cm x 3.6m	● Red	1 x 10	EAF524
72286-00	5cm x 3.6m	● Purple	1 x 10	EAF496
72286-01	7.5cm x 3.6m	● Purple	1 x 10	EAF511
72286-02	10cm x 3.6m	● Purple	1 x 10	EAF523
72287-00	5cm x 3.6m	● Green	1 x 10	EAF493
72287-01	7.5cm x 3.6m	● Green	1 x 10	EAF506
72287-02	10cm x 3.6m	● Green	1 x 10	EAF517
72288-00	5cm x 3.6m	● Black	1 x 10	EAF491
72288-01	7.5cm x 3.6m	● Black	1 x 10	EAF502
72288-02	10cm x 3.6m	● Black	1 x 10	EAF515
72289-10	5cm x 3.6m	Mixed†	1 x 10	EAF85185
72289-11	7.5cm x 3.6m	Mixed†	1 x 10	EAF665
72289-12	10cm x 3.6m	Mixed†	1 x 10	EAF85186

†Mixed packs contain two rolls each of dark blue, black, red, purple and pink

1. Petty, A. C., Wardman, C., (1998). A randomized, controlled comparison of adjustable focused rigidity primary casting technique with standard plaster of Paris/ synthetic casting technique in the management of fractures and other injuries. Journal of Orthopaedic Nursing. 2. 95 - 102

REACH UK Guidance

To comply to REACH UK guidance, healthcare professionals using synthetic splints and casts are to undertake safety education training.



Delta-cast provides free training to meet these requirements, and more. Access the training by registering to PATH [here](https://path.essity.uk/) or by typing <https://academieshm.essity.uk/> into your browser.



Delta-Cast® Elite

LATEX
FREE
FORMULA

Non-Fibreglass Cast Tape

Used for primary and secondary casting applications, **Delta-Cast® Elite** has been designed with ease of application and patient comfort in mind. Our patented non-fibreglass casting material combines strength and rigidity with soft edges for patient comfort.



Code	Size	Colour	Pack Qty	NHS SC Code
72279-00	5cm x 3.6m roll	○ White	1 x 10	EAF385
72279-01	7.5cm x 3.6m roll	○ White	1 x 10	EAF386
72279-02	10cm x 3.6m roll	○ White	1 x 10	EAF387
72279-03	12.5cm x 3.6m roll	○ White	1 x 10	EAF388
72278-00	5cm x 3.6m roll	● Black	1 x 10	EAF398
72278-01	7.5cm x 3.6m roll	● Black	1 x 10	EAF399
72278-02	10cm x 3.6m roll	● Black	1 x 10	EAF400
72276-00	5cm x 3.6m roll	● Dark blue	1 x 10	EAF389
72276-01	7.5cm x 3.6m roll	● Dark blue	1 x 10	EAF390
72276-02	10cm x 3.6m roll	● Dark blue	1 x 10	EAF391
72277-00	5cm x 3.6m roll	● Purple	1 x 10	EAF395
72277-01	7.5cm x 3.6m roll	● Purple	1 x 10	EAF396
72277-02	10cm x 3.6m roll	● Purple	1 x 10	EAF397
72275-00	5cm x 3.6m roll	● Red	1 x 10	EAF392
72275-01	7.5cm x 3.6m roll	● Red	1 x 10	EAF393
72275-02	10cm x 3.6m roll	● Red	1 x 10	EAF394
73455-00	5cm x 3.6m roll	● Pink	1 x 10	EAF534
73455-01	7.5cm x 3.6m roll	● Pink	1 x 10	EAF535
73455-02	10cm x 3.6m roll	● Pink	1 x 10	EAF536
73453-00	5cm x 3.6m roll	● Light blue	1 x 10	EAF584
73453-01	7.5cm x 3.6m roll	● Light blue	1 x 10	EAF585
73453-02	10cm x 3.6m roll	● Light blue	1 x 10	EAF586
73454-00	5cm x 3.6m roll	● Green	1 x 10	EAF587
73454-01	7.5cm x 3.6m roll	● Green	1 x 10	EAF588
73454-02	10cm x 3.6m roll	● Green	1 x 10	EAF589
73456-00	5cm x 3.6m roll	● Orange	1 x 10	EAF590
73456-01	7.5cm x 3.6m roll	● Orange	1 x 10	EAF591
73456-02	10cm x 3.6m roll	● Orange	1 x 10	EAF592

Non-fibreglass

- Provides smooth and soft cast edges for enhanced patient comfort

Highly conformable polyester casting material

- High strength, rigidity and durability
- Easily conforms to body contours without folding or tucking for increased patient comfort
- Smooth finish of the cast reduces potential snag on clothing and irritation of the skin

Consistent set-time

- Sets in approximately 3-5 minutes and is fully weight bearing after 20 minutes

Delta-Cast® Prints

LATEX
FREE
FORMULA

Non-Fibreglass Cast Tape

Delta-Cast® Prints is a range of colourful alternative patterned non-fibreglass cast tapes which give patients a variety of design options for their casts. The colourful designs can also help to overcome the anxiety of young and old patients in cast rooms.



Code	Size	Colour	Pack Qty	NHS SC Code
72273-22	5cm x 3.6m	● Army camouflage	1 x 10	EAF85360
72273-06	7.5cm x 3.6m	● Army camouflage	1 x 10	EAF85357
72273-02	5cm x 3.6m	● Dinosaur print	1 x 10	EAF85364
72273-07	7.5cm x 3.6m	● Pastel print	1 x 10	EAF85358
72273-08	5cm x 3.6m	● Football print	1 x 10	EAF85363
72273-09	7.5cm x 3.6m	● Football print	1 x 10	EAF85359
72273-05	5cm x 3.6m	● Teddy bear print	1 x 10	-
72273-20	5cm x 3.6m	● Paw print	1 x 10	EAF85349
72273-21	7.5cm x 3.6m	● Paw print	1 x 10	EAF85361
72273-03	5cm x 3.6m	● Winter print	1 x 10	-
72273-04	7.5cm x 3.6m	● Winter print	1 x 10	-
72273-40	5cm x 3.6m	● Pink camouflage	1 x 10	EAF85355
72273-41	7.5cm x 3.6m	● Pink camouflage	1 x 10	EAF85362

Colourful printed pattern

- The colourfast playful patterns make wearing a cast more fun and can help increase patient compliance

Highly conformable polyester casting material

- The high conformability supports confident, wrinkle-free, precise application, with even unrolling tension and good end lay down with secure lamination for long-term durability

Reliable and consistent working time

- The reliable and consistent working time window delivers a fully weight bearing cast after approximately 20 minutes

Robust, lightweight, smooth cast

- Rigid yet flexible material helps prevent constriction of muscle and tissue
- The smooth finish of the cast reduces snag on clothing and irritation of the skin

Available in eight patterns

- Providing the patient with a wide variety of choice



Delta-Lite® Plus

Fibreglass Cast Tape

Delta-Lite® Plus is a smooth, durable, fibreglass cast tape that combines a fibreglass substrate and a polyurethane resin system with tack free properties. The tack-free properties allow ease of application and moulding using standard examination gloves.



Code	Size	Colour	Pack Qty	NHS SC Code
73458-00	2.5cm x 1.8m roll	○ White	1 x 10	EAF435
73458-01	5cm x 3.6m roll	○ White	1 x 10	EAF443
73458-02	7.5cm x 3.6m roll	○ White	1 x 10	EAF450
73458-03	10cm x 3.6m roll	○ White	1 x 10	EAF463
73458-04	12.5cm x 3.6m roll	○ White	1 x 10	EAF466
73458-55	5cm x 3.6m roll	● Pink	1 x 10	EAF440
73458-56	7.5cm x 3.6m roll	● Pink	1 x 10	EAF447
73458-57	10cm x 3.6m roll	● Pink	1 x 10	EAF457
73458-20	5cm x 3.6m roll	● Dark blue	1 x 10	EAF437
73458-21	7.5cm x 3.6m roll	● Dark blue	1 x 10	EAF451
73458-22	10cm x 3.6m roll	● Dark blue	1 x 10	EAF453
73458-30	5cm x 3.6m roll	● Red	1 x 10	EAF442
73458-31	7.5cm x 3.6m roll	● Red	1 x 10	EAF449
73458-32	10cm x 3.6m roll	● Red	1 x 10	EAF461
73458-60	5cm x 3.6m roll	● Purple	1 x 10	EAF441
73458-61	7.5cm x 3.6m roll	● Purple	1 x 10	EAF448
73458-62	10cm x 3.6m roll	● Purple	1 x 10	EAF459
73458-35	5cm x 3.6m roll	● Light blue	1 x 10	EAF438
73458-36	7.5cm x 3.6m roll	● Light blue	1 x 10	EAF445
73458-37	10cm x 3.6m roll	● Light blue	1 x 10	EAF454
73458-45	5cm x 3.6m roll	● Black	1 x 10	EAF436
73458-46	7.5cm x 3.6m roll	● Black	1 x 10	EAF444
73458-47	10cm x 3.6m roll	● Black	1 x 10	EAF452
73458-25	5cm x 3.6m roll	● Green	1 x 10	EAF439
73458-26	7.5cm x 3.6m roll	● Green	1 x 10	EAF446
73458-27	10cm x 3.6m roll	● Green	1 x 10	EAF455
73458-65	5cm x 3.6m roll	● Yellow	1 x 10	EAF531
73458-66	7.5cm x 3.6m roll	● Yellow	1 x 10	EAF532
73458-67	10cm x 3.6m roll	● Yellow	1 x 10	EAF533
73458-75	5cm x 3.6m roll	● Orange	1 x 10	EAF2602
73458-76	7.5cm x 3.6m roll	● Orange	1 x 10	EAF2603
73458-77	10cm x 3.6m roll	● Orange	1 x 10	EAF2604



Smooth finish surface

- Non-abrasive finished cast enhances patient comfort

Excellent rigid support

- Creates a long-lasting, lightweight cast

High lamination

- Gives strength between cast layers
- Improved cast strength

Good conformability

- Makes casting application easier around all body contours
- No special application techniques required

Outstanding moulding

- Provides excellent anatomical support

Delta-Cast® Soft

Flexible Casting Tape

Delta-Cast® Soft is designed to provide functional support to fractures and soft tissue injuries. It's semi-rigidity allows for flexibility and mobility, helping to reduce immobilisation related problems such as muscle atrophy and joint stiffness. Improved flexibility and mobility can help promote a faster healing process.

Delta-Cast® Soft is constructed using a combination of high-tech polyester, polypropylene, and elastane yarns, together with a specially developed resin. This technology builds safety, functionality and comfort into a product offering all the advantages only a non-fibreglass casting tape can offer.



Three-way stretch / excellent conformability

- Ease of handling on application
- Allows for a consistent fit around difficult body contours with a wrinkle-free application, offering enhanced fit and support for the patient

Non-fibreglass

- Highly durable to maintain cast functionality, helping reduce time and costs for the cast room that can be associated with cast repairs

Range of colours and unique designs

- Available in patented paw print, heart or colour designs, offering patients of all ages a more playful, pleasant cast
- Helps support patient concordance and function

Versatile application

- Designed for both customised removable casts and circular applications, Delta-Cast® Soft can easily be trimmed to shape, producing an optimal fit for the patient
- Can be combined with hook and loop straps for removable applications, or with reinforcement materials for stronger support

100% Radiolucent

- Allows clear X-ray images for diagnosis, offering cost and time savings

Flexibility

- Can help to minimise some negative effects of total rigid immobilisation; muscle atrophy, joint stiffness and other immobilisation related issues
- Improved circulation can help towards a faster healing process and a shorter rehabilitation period

Ease of removal

- Can be removed by lifting and unpeeling
- No cast saw required. Can be removed using regular cast scissors

Excellent lamination

- Excellent lamination of layers and good end lay down produces a strong, well bonded cast



Code	Size	Colour	Pack Qty	NHS SC Code
72708-02	2.5cm x 1.8m roll	○ White	1 x 10	EAF659
72708-12	5cm x 3.6m roll	○ White	1 x 10	EAF660
72708-22	7.5cm x 3.6m roll	○ White	1 x 10	EAF661
72708-32	10cm x 3.6m roll	○ White	1 x 10	EAF657
72708-42	12.5cm x 3.6m roll	○ White	1 x 10	EAF658
72708-05	5cm x 3.6m roll	● Blue	1 x 10	EAF650
72708-15	7.5cm x 3.6m roll	● Blue	1 x 10	EAF651
72708-25	10cm x 3.6m roll	● Blue	1 x 10	EAF649
72708-08	5cm x 3.6m roll	● Red	1 x 10	EAF655
72708-18	7.5cm x 3.6m roll	● Red	1 x 10	EAF656
72708-28	10cm x 3.6m roll	● Red	1 x 10	EAF654
72708-50	5cm x 3.6m roll	● Pink	1 x 10	EAF663
72708-51	7.5cm x 3.6m roll	● Pink	1 x 10	EAF664
72708-52	10cm x 3.6m roll	● Pink	1 x 10	EAF662
72708-01	5cm x 3.6m roll	● Black	1 x 10	EAF647
72708-11	7.5cm x 3.6m roll	● Black	1 x 10	EAF648
72708-21	10cm x 3.6m roll	● Black	1 x 10	EAF646
72708-09	5cm x 3.6m roll	● Paw print	1 x 10	EAF652
72708-19	7.5cm x 3.6m roll	● Paw print	1 x 10	EAF653
72708-60	5cm x 3.6m roll	● Heart print	1 x 10	EAF668
72708-61	7.5cm x 3.6m roll	● Heart print	1 x 10	EAF669

Dynacast® Prelude

Synthetic Splint System

Dynacast® Prelude is a fibreglass splinting system, covered by a polypropylene padding. Stronger and more versatile than traditional plaster of Paris, it is ideal for trauma and post-op applications from acute through to rehabilitation phase.

Strong, light and durable

- Resin coated fibreglass provides a high strength splint which is resistant to breakdown
- Will not breakdown when wet

Quick setting and early weight bearing

- Dynacast® Prelude has an initial set of 3 - 5 minutes and reaches weight bearing strength in approximately 15 minutes

Code	Size	Colour	Pack Qty	NHS SC Code
71418-00	2.5cm x 4.6m roll	White	1 x 1	EAF85353
71420-00	5cm x 4.6m roll	White	1 x 1	EAF083
71421-00	7.5cm x 4.6m roll	White	1 x 1	EAF084
71415-00	10cm x 4.6m roll	White	1 x 1	EAF085
71416-00	12.5cm x 4.6m roll	White	1 x 1	EAF086
71417-00	15cm x 4.6m roll	White	1 x 1	EAF087
71419-00	20cm x 2.3m roll	White	1 x 1	EAM1844



LATEX
FREE
FORMULA

No mess or waste

- With no plaster mess, Dynacast® Prelude is ideally suited to the theatre and ward environment
- The continuous roll format ensures clinicians will only use what is required and provides versatility to choose the length of splint to suit the indication and patient

Patient comfort

- Soft polypropylene padding covers the fibreglass splint minimising the risk of pressure points
- Naturally aids moisture transmission away from the skin to enhance patient comfort

ILS (interlocking strength) technology

- Provides strength and durability
- Minimises wrinkles and creases in the finished splint

Dynacast® Prelude Solo

Unpadded Precut Splinting System

The Dynacast® Prelude Solo unpadded precut is a fibreglass splinting system that comes in a variety of widths and lengths, and is stronger and more versatile than traditional plaster of Paris slabs. Dynacast® Prelude Solo splints are easy to apply, clean, and are ideal for treating trauma and post-operative applications.



Code	Size	Colour	Pack Qty	NHS SC Code
72963-00	5cm x 25cm	White	1 x 10	EAF85356
72963-01	7.5cm x 30cm	White	1 x 10	EAZ687
72963-02	7.5cm 89cm	White	1 x 10	EAF85354
72963-03	10cm x 38cm	White	1 x 10	GRS000
72963-04	10cm x 76cm	White	1 x 10	EAF85348
72963-05	12.5cm x 76cm	White	1 x 10	EAF487
72963-06	15cm x 30cm	White	1 x 10	EAF488

Fibreglass splint

- Stronger than traditional plaster of Paris slabs
- Reduces weight and bulk to patient

Simple activation with no plaster dust

- Liner enables gloveless application
- No dust and mess associated with traditional plaster of Paris slabs
- Ideal for use in theatre and A&E environments

Slim profile

- Excellent mouldability

ILS (interlocking strength) technology


- Provides strength and durability
- Minimises wrinkles and creases in the finished splint

LATEX
FREE
FORMULA

Dynacast® Prelude Strap Kits

Acute Synthetic Splint System

Designed for use with **Dynacast® Prelude** splints in the rehabilitation phase, this patient-friendly product enables long-term use without the need for a full bandage, enabling easy splint removal. Each kit contains three latex-free hook and loop straps.

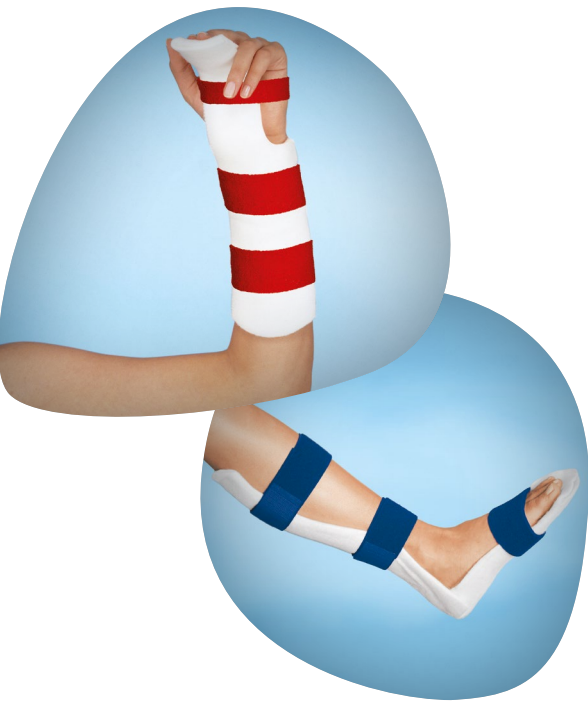

Code	Size	Colour	Pack Qty	NHS SC Code
71415-18	Upper extremity kit	Red	1 x 10	EAZ85078
71415-19	Lower extremity kit	Blue	1 x 10	EAZ85079

Dynacast® AS Ankle Splint

Acute Synthetic Splint System

Dynacast® AS Ankle Splint is a custom mouldable solution for external ankle support following acute injury or for chronic joint instability. This is a therapy solution that can be used throughout treatment from acute phase to rehabilitation.

Low profile

- Allows the use of normal footwear
- Fits easily into training shoes

Pre-made fibreglass stirrup splint with micro-perforated foam covering

- Ideal for A&E
- Strong, lightweight, breathable yet durable therapy solution

All in one treatment kit from acute to rehabilitation

- Oedema control
- Circumferential support
- Cold therapy in the acute stage
- U-shaped foam inserts



Code	Size	Colour	Pack Qty
71984-00	Paediatric kit	Blue	1 x 1
71361-00	Small / medium kit	Blue	1 x 1
71360-00	Medium / large kit	Blue	1 x 1
72834-00	Small / medium dispenser pack	Blue	1 x 10
72835-00	Medium / large dispenser pack	Blue	1 x 10

Delta-Xpress®

Alternative Splint System

Delta-Xpress® is a splinting system that allows limb immobilisation in a fraction of the time, compared to conventional splints or casts.

Delta-Xpress® offers four different designs to immobilise lower leg, arm, wrist and finger fractures. The prefabricated design is intuitive to use, while providing secure and precise immobilisation for short-term use.

Precise clinical positioning

- Pre-defined angles hold limb in optimal healing position
- Available in three sizes

Ready to use from the shelf

- Prefabricated design allows instant application
- No activation, no curing, no clean-up
- No need for water activation or even working space
- Instructions and icons support quick and correct application

Secure – safe and strong

- Rigid compound material
- Good conformability for individual fit
- Fixation with bandage or hook and loop straps

High performance design features

- Cushioned - padding inside for comfort and absorption
- Breathability - to help avoid skin maceration
- Size and shape - adjustable with standard scissors
- Convenient - splints come in three sizes with measuring scale and colour code on product
- Easy access to wounds

Fixation

- Can be secured by using a cohesive bandage such as Co-Plus – Page 37 or straps such as Dynacast® Prelude strap kits – Page 19

Wrist and Finger Splint

- Rigid stay for optimal wrist, hand and finger position
- The stay has an angle of 100° to support the flexion of the metacarpophalangeal joint (MCP) and stabilises the wrist in 30° dorsal extension

Code	Size	Pack Qty	NHS SC Code
72713-00	Small	1 x 10	EAZ153
72713-01	Medium	1 x 10	EAZ154
72713-02	Large	1 x 10	EAZ155

Lower Leg Splint

- Pre-adjusted 90° angle for ideal healing position of lower leg
- Ankle and foot rest inlay for optimal foot position
- Malleoli paddings for increased comfort

Code	Size	Pack Qty	NHS SC Code
72711-00	Small	1 x 10	EAZ85101
72711-01	Medium	1 x 10	EAZ148
72711-02	Large	1 x 10	EAZ149

Wrist Splint

- Rigid stay for optimal wrist position
- Wrist angle in 30° dorsal extension guarantees a functional therapy position

Code	Size	Pack Qty	NHS SC Code
72712-00	Small	1 x 10	EAZ150
72712-01	Medium	1 x 10	EAZ151
72712-02	Large	1 x 10	EAZ152

Elbow Splint

- Pre-adjusted 90° angle assures optimal healing position of upper arm, elbow and lower arm

Code	Size	Pack Qty	NHS SC Code
72710-00	Small	1 x 10	EAZ85103
72710-01	Medium	1 x 10	EAZ145
72710-02	Large	1 x 10	EAZ146



Undercast Padding

Essity offers a wide range of padding options for use in conjunction with both plaster of Paris and synthetic casting tapes.

Soffban® Natural and **Velband®** offer the benefits of a natural padding with excellent absorption, making them both an ideal product for theatre use.

Soffban® Synthetic, **Delta-Rol®** and **Delta-Rol® S** are synthetic materials that are non-absorbent and therefore retain excellent cushioning when wet to give increased patient comfort. **Delta-Rol® S** also incorporates a super fine foam backing, which removes the need to use separate pre-tape for a smooth finish. **Delta-Dry®** water resistant cast padding has a unique patented technology, which allows patients to shower and bathe as normal when used with synthetic cast tapes.

	Soffban® Synthetic	Soffban® Natural	Delta-Rol®	Delta-Rol® S	Velband®	Delta-Dry®
Natural (N) / Synthetic (S) / Water Resistant (WR)	S	N	S	S	N	WR

Soffban® Natural

Natural Viscose Padding

Soffban® Natural padding is suitable for use with both plaster of Paris and synthetic casts to aid comfort.

Highly absorbent natural viscose

- Suitable for use where bleeding may occur
- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity

Versatile

- Soffban® Natural is also suitable and widely used for venous leg ulcer multi-layer bandaging therapy as the first layer*



Code	Size	Pack Qty	NHS SC Code
71472-00	5cm x 2.7m roll	1 x 12	EPA071
71473-00	7.5cm x 2.7m roll	1 x 12	EPA073
71469-00	10cm x 2.7m roll	1 x 12	EPA075
71470-00	15cm x 2.7m roll	1 x 12	EPA077
71471-00	20cm x 2.7m roll	1 x 6	EPA079

Easy to apply

- Excellent conformability around body contours
- Easy to tear
- Feathers easily and clings to itself
- Blends to give a smooth even finish

Autoclavable

- Can be sterilised
- No need to remove the paper wrapping
- 100% viscose rayon

Soft

- Comfortable for the patient, soft and gentle against the skin

Excellent cushioning

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects from rigid casting material
- Also provides protection from the cast saw on cast removal

LATEX
FREE
FORMULA

Soffban® Synthetic

Synthetic Padding

Soffban® Synthetic padding is suitable for use with both plaster of Paris and synthetic casts.

Synthetic non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' is retained even if padding becomes wet

Easy to apply

- Excellent conformability around body contours
- Easy to tear
- Feathers easily and clings to itself
- Blends to give a smooth even finish

Excellent cushioning and soft feel

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal
- Comfortable for the patient, soft and gentle against the skin

Autoclavable

- Can be sterilised
- No need to remove the paper wrapping



Code	Size	Pack Qty	NHS SC Code
71464-01	5cm x 2.7m roll	1 x 12	EPA020
71467-01	7.5cm x 2.7m roll	1 x 12	EPA021
71486-05	10cm x 2.7m roll	1 x 12	EPA022
71465-01	15cm x 2.7m roll	1 x 12	EPA023
71466-01	20cm x 2.7m roll	1 x 6	EPA024

Delta-Rol®

Synthetic Padding

Delta-Rol® Synthetic padding is a non-absorbent synthetic material, it wicks moisture away from the skin and allows it to evaporate, resulting in a reduced risk of skin maceration. Suitable for use with plaster of Paris and synthetic casts.

Synthetic non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' of the padding is retained even when material becomes wet

Easy to apply

- Good conformability around body contours
- Gives a smooth even finish under the cast

Excellent cushioning and soft feel

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal
- Comfortable for the patient, soft and gentle against the skin

Autoclavable

- Can be sterilised
- No need to remove the paper wrapping



Code	Size	Pack Qty	NHS SC Code
72310-00	5cm x 2.75m roll	10 x 12	EGJ85034
72310-01	7.5cm x 2.75m roll	24 x 12	EGJ85025
72310-02	10cm x 2.75m roll	18 x 12	EGJ85039
72310-03	15cm x 2.75m roll	12 x 12	EGJ85026
72310-04	20cm x 2.75m roll	18 x 6	EGJ85027

LATEX
FREE
FORMULA

Velband®

Natural Rayon Padding

Velband® padding is a natural rayon fibre padding, highly absorbent, drawing moisture away from the skin. It has a unique three-layer structure and is ideal for use in theatre where absorbent cushioning is required.



Code	Size	Pack Qty	NHS SC Code
72315-00	5cm x 2.75m roll	12 x 12	EPA011
72315-01	7.5cm x 2.75m roll	20 x 12	EPA013
72315-02	10cm x 2.75m roll	16 x 12	EPA015
72315-04	15cm x 2.75m roll	10 x 12	EPA017
72315-06	20cm x 2.75m roll	15 x 6	EPA019
72315-30	10cm x 4.5m roll	7 x 12	EPA014

Unique construction

- Lightweight, breathable non-woven fabric
- The middle natural viscose layer gives excellent absorbent properties and cushioning

Highly absorbent

- Suitable for use where bleeding may occur*
- Absorbs exudate and moisture away from the skin
- Reduces the risk of skin maceration and sensitivity

Excellent cushioning

- Unique three-layer construction provides excellent cushioning for improved patient comfort
- Reduces risk of pressure sores

Autoclavable

- Can be sterilised for theatre use

LATEX
FREE
FORMULA

*A primary dressing should always be worn over open wounds

Delta-Rol® S

Synthetic Padding

Delta-Rol® S Synthetic is a unique combination of a synthetic undercast padding needled onto a thin polyurethane foam bandage. This exceptional combination delivers excellent conformability and cushioning together with the smooth, even finish associated with the use of a pre-tape over cast padding.



Code	Size	Pack Qty	NHS SC Code
72322-00	5cm x 2.75m roll	10 x 12	EGJ85040
72322-01	7.5cm x 2.75m roll	12 x 12	EGJ85033
72322-02	10cm x 2.75m roll	9 x 12	EGJ85044
72322-03	15cm x 2.75m roll	6 x 12	EGJ85031
72322-04	20cm x 2.75m roll	9 x 6	-

Non-absorbent material

- Wicks moisture away from the skin
- Reduces the risk of skin maceration and sensitivity
- Cushioning 'loft' of the padding is retained even if material becomes wet

Polyurethane foam lining

- Provides a smooth even finish on which to apply the cast

Easy to apply

- Good conformability around body contours
- Gives a smooth even finish under the cast
- Separate pre-tape not required

Excellent cushioning

- Padding structure creates a material which is light, deep and stable, with good airflow
- Protects the patient with a reduced risk of skin abrasions
- Also provides protection from the cast saw on cast removal

Delta-Dry®

Water Resistant Cast Padding

Delta-Dry® has a patented technology allowing patients to clean and rinse their cast and also bathe and shower as normal. Delta-Dry® is conformable and easy to apply, resulting in a comfortable cast.

Ideal for use with incontinent paediatric or elderly and less mobile patients when used with a synthetic cast tape.



Code	Size	Pack Qty	NHS SC Code
73443-00	5cm x 2.4m roll	1 x 12	EGJ85043
73443-01	7.5cm x 2.4m roll	1 x 12	EPA026
73443-02	10cm x 2.4m roll	1 x 12	EPA027

Water-resistant cast padding

- Allows patients to shower and bathe as normal
- Increases hygiene
- Reduces odour
- Improves patient compliance when wearing a cast

Easy to apply

- Good conformability around body contours
- No special tools are needed for application or removal

How does it work?

- The open knit structure, along with the water-resistant qualities of the fibres, allows the water to drain from the cast
- The majority of the water simply runs out of the cast. The remaining moisture evaporates through the cast, heated by normal body temperature
- Position the cast to allow water to drain from it
- Long arm and leg casts must be positioned as to not allow water to pool in the vicinity of the heel or elbow
- Casts will take approximately 30 minutes to 1 hour to completely dry out

Delta-Dry® Pantaloon

The Hygienic Hip Spica Cast Treatment Solution

This pre-shaped, soft and water-resistant liner has been developed using the patented Delta-Dry® technology to simplify the treatment of developmental dysplasia of the hip or femur fractures. The aim of the Delta-Dry® Pantaloon is to help support improved patient hygiene and ease of application for both patient and clinician.

Delta-Dry® Pantaloon maintains patient hygiene, as flushing the hip spica cast and even bathing is possible. For best results, only use Delta-Dry® Pantaloon in combination with water-resistant, synthetic cast material.

Delta-Dry technology

Water-resistant liner material that can help to increase patient and parent acceptance and potentially supports time and cost-savings

- Enables bathing and flushing
- Helps to avoid odour, itching and skin maceration
- Can help to allow expanded duration between cast changes (up to six weeks)

Pre-cut shape in three sizes

- Permits an easy application, a suitable fit with little creasing offering high wearing comfort and allowing safe and easy application

Quick dry system

High water vapour permeability for optimised hygienic care

- Dries within 90 minutes
- Helps patient's skin to breathe

Care and protection

Soft, flexible and cushioned for infant skin to offer high wearing comfort

- Protects bony prominences
- Provides safety during cast removal
- Comes with foldable flap and cast ends

X-ray translucency



Code	Size	Measurements	Pack Qty	NHS SC Code
73365-00	1	≤ 9.8" / ≤ 25 cm	1	EAJ930
73365-01	2	9.8 - 11.6" / 25 - 29.5 cm	1	EAJ929
73365-02	3	≥ 11.6" / ≥ 29.5 cm	1	EAJ928



Stockinette - Natural and Synthetic

Providing both natural and synthetic stockinette, Essity offers customers their product of choice. Stockinette can be used with both synthetic and plaster of Paris materials.

Delta-Net® and Jersey Standard are traditional stockinette offering patients protection and comfort.

Delta-Dry® Stockinette and Delta-Dry® Softliner used with synthetic casting material allows the patients to wash their hands, shower or bathe whilst in a cast. Delta Dry can be applied just like standard practice - no new techniques needed

	Jersey Standard	Delta-Net	Delta Terry-Net® C	Delta Terry-Net® S	Delta Terry-Net® Thumb Spica	Delta-Dry® Stockinette	Delta-Dry® Softliner
Cotton (C) / Synthetic (S) / Water Resistant (WR)	C	S	C	S	S	WR	WR

Jersey Standard

Natural Ribbed Stockinette


Jersey standard is a natural cotton stockinette. It is suitable for use with synthetic or plaster of Paris casting materials.

Natural cotton

- Soft and gentle to the patients skin with reduced risk of skin irritation

Autoclavable

- Compatible with steam and ethylene oxide sterilisation

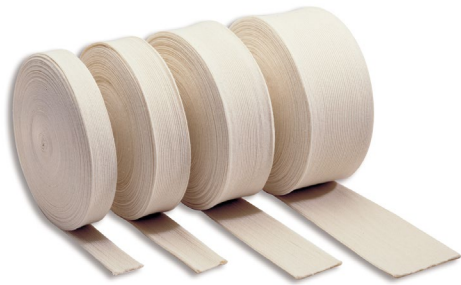

Code	Size	Colour	Pack Qty	NHS SC Code
72021-02	3cm x 25m roll	Natural	1 x 1	EGJ192
72021-03	5cm x 25m roll	Natural	1 x 1	EGJ138
72021-04	7cm x 25m roll	Natural	1 x 1	EGJ193
72021-05	10cm x 25m roll	Natural	1 x 1	EGJ140
72021-07	15cm x 25m roll	Natural	1 x 1	EGJ194
72021-08	20cm x 25m roll	Natural	1 x 1	EGJ195
72021-10	25cm x 25m roll	Natural	1 x 1	EGJ196

Delta Terry-Net® C

Natural Terry Cloth Stockinette

Delta Terry-Net® C is a cotton towelling stockinette that can be used underneath either synthetic or plaster of Paris casting materials where extra cushioning and protection required for the patient.

Natural cotton

- Soft and gentle to the patient's skin with reduced risk of skin irritation

Versatile

- The cotton towelling cast liner is ideal when used in conjunction with Focused Rigidity Casting (FRC) techniques

Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling



Code	Size	Colour	Pack Qty	NHS SC Code
72305-00	5cm x 13.7m	Natural	1 x 1	EGJ85050
72305-01	7.5cm x 13.7m	Natural	1 x 1	EGJ85049
72305-02	10cm x 13.7m	Natural	1 x 1	EGJ85052
72305-03	12.5cm x 13.7m	Natural	1 x 1	EGJ85051
72305-04	15cm x 13.7m	Natural	1 x 1	EGJ85053

Delta-Net® Stockinette

LATEX
FREE
FORMULA

Synthetic Stockinette

Delta-Net® is a soft, comfortable, knitted, synthetic stockinette suitable for use with synthetic or plaster of Paris materials.

Code	Size	Colour	Pack Qty	NHS SC Code
72307-05	2.5cm x 22.8m	White	1 x 1	EGJ207
72307-00	5cm x 22.8m	White	1 x 1	EGJ204
72307-01	7.5cm x 22.8m	White	1 x 1	EGJ205
72307-02	10cm x 22.8m	White	1 x 1	EGJ206
72723-00	2.5cm x 22.8m	Black	1 x 1	EGJ85021
72723-01	5cm x 22.8m	Black	1 x 1	EGJ85022
72723-02	7.5cm x 22.8m	Black	1 x 1	EGJ85023
72723-03	10cm x 22.8m	Black	1 x 1	EGJ85024

Synthetic material

- Stretches readily and easily conforms to body contours

Soft, comfortable knit

- To protect the patient's skin under casting materials

Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling



Delta Terry-Net® S

LATEX
FREE
FORMULA

Synthetic Terry Cloth Stockinette

Delta Terry-Net® S is a synthetic towelling stockinette that can be used underneath either synthetic or plaster of Paris casting materials where extra patient comfort is required.

Code	Size	Colour	Pack Qty	NHS SC Code
72306-00	5cm x 13.7m	White	1 x 1	EGJ85046
72306-01	7.5cm x 13.7m	White	1 x 1	EGJ85045
72306-02	10cm x 13.7m	White	1 x 1	EGJ85048
72306-03	12.5cm x 13.7m	White	1 x 1	EGJ85047

Terry knit design

- Provides increased cushioning and protection for the patient

Versatile

- Can be used in conjunction with Focused Rigidity Casting (FRC) techniques

Easy to use dispenser

- Easily cut to length as required
- Easy storage and handling



Delta Terry-Net® Thumb Spica

LATEX
FREE
FORMULA

Thumb Spica Cast Synthetic Liner

Delta Terry-Net® cast liner is designed for use underneath cast applications which include the thumb and other hand related injuries where extra patient comfort is required.

Code	Size	Colour	Pack Qty	NHS SC Code
72308-02	Short arm - 5cm	White	1 x 1	ESM85040
72308-03	Long arm - 5cm	White	1 x 1	ESM85041
72308-00	Short arm - 7cm	White	1 x 1	ESM85042
72308-01	Long arm - 7cm	White	1 x 1	ESM85043

Terry knit design

- Provides increased cushioning and protection for the patient

Integrated thumb piece

- Allows for quick and simple application

Available in two arm lengths

- Fits a variety of patient sizes



Delta-Dry®

Water-Resistant Stockinette and Softliner

Innovative and simple product solution to support your patients' increased hygiene needs.

Synthetic casts made using **Delta-Dry®** Stockinette and Softliner allow patients to wash their hands, shower or bathe whilst wearing their cast. Our synthetic water-resistant casts are also safe to use with both alcohol and water-based hand sanitisers to importantly maintain hand hygiene for those with upper limb immobilisation.

- Standard casting application: Delta-Dry® stockinette in conjunction with Delta-Dry® water resistant undercast padding can be applied just like standard stockinette and padding - no new techniques needed

Water-resistant with quick-dry system

- The patented Delta-Dry® material offers water-resistance
- The open knit structure of the fibres accelerate drainage after submerging in water
- The patient's body temperature evaporates residual moisture quickly through the airy mesh. This helps the cast and skin dry quickly, skin feels dry in about 90 minutes after daily hygiene

Boost hygiene and patient comfort

- The polypropylene-nylon-polyester mesh of Delta-Dry® products provides a soft, comfortable cushion on the patient's skin
- High breathability keeps skin dry and well aerated helping to reduce skin problems such as maceration, itching and odour
- Patients can also use with both alcohol and water-based hand sanitisers as needed, without compromising the integrity of the cast
- Helping to improve patient compliance and satisfaction

Apply Delta-Dry® using standard casting techniques

- Delta-Dry® Stockinette is applied the same way as conventional stockinette
- Delta-Dry® Softliner is applied using 2 layers with no padding required
- The highly conformable and stretchable material makes wrinkle-free application easily and quickly achievable
- Delta-Dry® can be removed using standard removal equipment
- Zip sticks are available for removal when using a traditional cast saw, refer to the IFU for further information



Code	Size	Pack Qty	NHS SC Code
Delta-Dry® Stockinette - Water Resistant Stockinette			
74564-00	2.5cm x 10m	1 x 1	EGJ85037
74564-01	5cm x 10m	1 x 1	EGJ85029
74564-02	7.5cm x 10m	1 x 1	EGJ85042
74564-03	10cm x 10m	1 x 1	EGJ85032
74564-04	12.5cm x 10m	1 x 1	EGJ85036
Delta-Dry® Softliner - Water Resistant Softliner			
72779-01	2.5cm x 10m	1 x 1	EGJ85030
72779-02	5cm x 10m	1 x 1	EGJ85028
72779-03	7.5cm x 10m	1 x 1	EGJ85041
72779-04	10cm x 10m	1 x 1	EGJ85035
72779-05	12.5cm x 10m	1 x 1	EGJ85038

Thermoplastics

Delta Thermo® with its elastic memory, ensures the ability to return to its original sheet form and is ideally suited for serial splinting.




Delta Thermo®

Elastic Memory Thermoplastic

Delta Thermo® is ideally suited for serial splinting and splints for both the upper and lower limbs.

LATEX
FREE
FORMULA

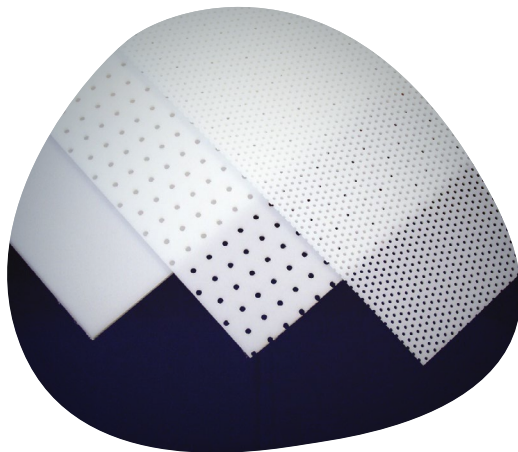

Elastic memory

- Ensures ability to return to its original sheet form
- Easily remouldable
- Suitable for serial splinting

Translucent appearance on reaching activation temperature

- Allows for precise positioning of the material on the area to be covered
- Prevents accidental self-bonding after heating

Self-finishing edges

Added patient comfort

Activation

- Heating time 1 - 2 minutes at a temperature of 60 - 70°C / 140 - 158°F
- Working time 3 - 4 minutes

Code	Size	Pack Qty
72312-06	1.6mm thick non-perf, 60cm x 45cm	1 x 2 sheets
72312-08	2.4mm thick non-perf, 60cm x 45cm	1 x 2 sheets
72312-10	3.2mm thick non-perf, 60cm x 45cm	1 x 2 sheets
72312-07	1.6mm thick perf, 60cm x 45cm	1 x 2 sheets
72312-09	2.4mm thick perf, 60cm x 45cm	1 x 2 sheets
72312-11	3.2mm thick perf, 60cm x 45cm	1 x 2 sheets



Stockinette - Natural and Synthetic

Casting Accessories

Essity offers a comprehensive range of accessories to complete various casting applications.

For more detailed product information including various fixation tapes, bandages and Orthopaedic braces and supports, please contact your Essity Acute Specialist or customer services Tel: **0345 122 3600**

Tensoban®

Pre-Tape Underwrap

Tensoban® is a very thin, polyurethane wrap, which can be used under adhesive bandages and tapes to protect the skin in taping and strapping techniques.

It can also be used as a pre-tape, applied on top of the padding layer in orthopaedic casting, to provide a smooth, compressed surface before the synthetic casting material is applied.



Thin polyurethane pre-tape

- Results in a smooth, compressed surface on which to apply synthetic casting material

Porous and water resistant

- Allows the skin to breathe
- Reduces the risk of skin maceration

Versatile

- Particularly useful when creating 'windows' for inspection and for easy conversion of casts into resting splints

Code	Size	Colour	Pack Qty	NHS SC Code
71500-08	7cm x 20m roll	White	1 x 12	EAZ026
71500-01	10cm x 20m roll	White	1 x 10	-

LATEX
FREE
FORMULA

Tensoplast® Sport

Cast Edge Tape

Tensoplast® Sport elastic edge tape with tramline adhesive provides soft edging to casts and splints, and is particularly useful for edging removable casts. The central, high-stretch section makes it easy to fold over cast edges.



Tramline adhesive

- Cleanly and easily adheres to casting materials and thermoplastics

Soft finish

- For increased patient comfort

Central high stretch section

- Easy to manoeuvre over the edge of the cast for a smooth, neat finish

Code	Size	Colour	Pack Qty	NHS SC Code
71548-00	3cm x 2.5m roll	White	1 x 1	EAZ027

LATEX
FREE
FORMULA

Delta Terry-Net® Adhesive Felt Liner

Undercast Cushioning

Delta Terry-Net® Adhesive Felt Liner is a sheet of 2mm semi-compressed felt, used when additional patient comfort is required. The felt is ideal for providing protection to any bony prominences under a cast.

High quality adhesive backing


- Secures felt where you require it

Soft finish

- For improved patient comfort

Sheet format

- Can be cut to the desired size and shape as required

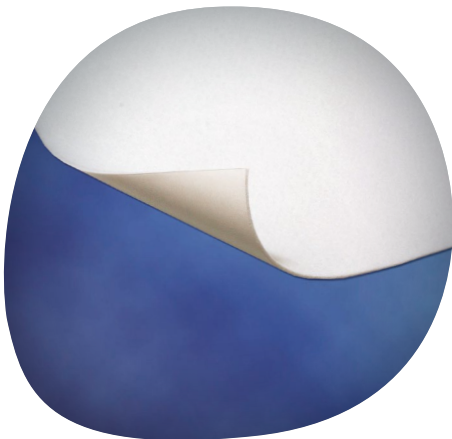

Code	Size	Colour	Pack Qty	NHS SC Code
72295-05	58cm x 100cm	White	1 x 1	GQB414

LATEX
FREE
FORMULA

Delta Terry-Net® Terry Cloth and Foam Liner

Undercast Cushioning


Delta Terry-Net® Terry Cloth and Foam Liner is a felt layer with foam sheet, which provides additional protection and cushioning for casting and bandaging techniques.

Soft finish

- For improved patient comfort

Sheet format

- Can be cut to the desired size and shape as required

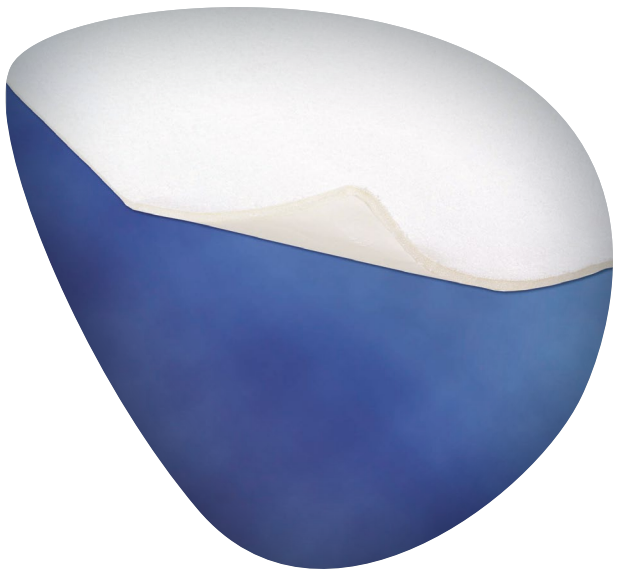

Code	Size	Colour	Pack Qty
72295-06	58cm x 100cm	White	1 x 1

LATEX
FREE
FORMULA

Delta Terry-Net® Adhesive Fleece Edger

Cast Edging Fleece

Delta Terry-Net® Adhesive Fleece Edger provides a soft, padded edging to casts and splints and is particularly useful for edging removable casts.



High stretch

- Makes it easy to pad around corners and contours resulting in a neater, more comfortable cast for the patient

High quality adhesive backing

- Secures fleece edger where you require it

Soft finish

- For improved patient comfort

Code	Size	Pack Qty	NHS SC Code
72295-13	3.2cm x13.5m black	1 x 1	-
72295-14	8cm x13.5m black	1 x 1	-

Delta-Net® Adhesive Hook and Stretch Loop

Closure System

Delta-Net® Adhesive Hook and Stretch Loop is an easy to use product, particularly suited for removable casting techniques.



High quality adhesive

- Readily adheres to casting materials and thermoplastics

Soft edge stretch loop fabric

- Provides increased patient comfort

Available in two widths

- Accommodates various casting requirements

Easy to use dispenser

- Easily cut to length
- Easy storage and handling

Code	Size	Pack Qty	NHS SC Code
72295-00	Hook fabric 2.5cm x 9.1m	1 x 1	EAZ100
72295-02	Hook fabric 5cm x 9.1m	1 x 1	EAZ85102
72295-01	Stretch loop 2.5cm x 13.7m	1 x 1	EAZ101
72295-03	Stretch loop 5cm x 13.7m	1 x 1	EAZ85100

Tensoplast® STK

Skin Traction Kits

Tensoplast® STK is a ready to use kit, including all items needed for the secure application of sustained traction weight support, and suitable for use with a variety of traction methods.

Tensoplast® STK is available in an adhesive and non-adhesive version.

Adhesive version

- Clear, low-allergy, pressure-sensitive adhesive for secure, reliable adhesion to the limb

Non-adhesive version

- Foam skin contact layer, breathable and suitable for tender skin

Padded foot plates

- Comfortable support of the foot

Conformable elastic bandage

- Ease of application to give secure positioning



Code	Size	Pack Qty	NHS SC Code
71544-00	Adhesive: adult	1 x 12	ESB151
71554-00	Adhesive: child	1 x 12	ESB153
71555-00	Non-adhesive: adult	1 x 12	ESC051
71556-00	Non-adhesive: child	1 x 12	ESC805

Knee Rest

Used for Lower Limb Casting Applications

Padded support to hold the knee at an angle of 10-15°, assisting in maintaining the foot in the correct position

Padded surface

- Gives comfort to the patient during casting application

Wipe Clean

- Help maintain infection control between patients

Order through customer services using
Orders.uk@essity.com



Code	Size	Pack Qty
73093-00	One size	1 x 1

Soffcrepe®

Type 2 Crepe Bandage

Soffcrepe® is a woven, non-sterile, non-adhesive elastic crepe bandage. It is designed to provide light compression in oedema management and light support in the treatment of minor soft tissue and joint injuries such as sprains and strains. Soffcrepe can also be used to secure dressings as well as fixation of casts or splints.

- Air-permeable
- Finished woven edges, non-fraying
- Anti-slip behaviour, inter-layer adhesion helps to keep the bandage in place
- Latex-free formula



Code	Size	Pack Qty	NHS SC Code
72090-15	5cm x 4.5m	1 x 12	ECA711
72090-16	7.5cm x 4.5m	1 x 12	ECA713
72090-17	10cm x 4.5m	1 x 12	ECA265
72090-18	15cm x 4.5m	1 x 6	ECA717

Tensoplast®

Elastic Adhesive Plaster

Tensoplast® EAP is a beige-coloured plain woven fabric designed for compressive strapping and taping of soft tissue injuries such as sprains and strains, joint stabilisation, posture-improvement and also elasto-compressive treatment in phlebology. Not suitable for use on patients with a known intolerance to latex or colophony.

- Lengthwise stretch allows the joints to move
- Easy to unwind
- Remains securely in place with continuous reliable adhesion
- Can be cut to length
- Contains dry and natural rubber latex



Code	Size	Pack Qty	NHS SC Code
72050-58	2.5cm x 4.5m	1 x 12	EHQ003

Co-plus®

Flexible Cohesive Bandage

Co-Plus® is a flexible cohesive bandage designed to provide short term compression for soft tissue injuries. Co-Plus maintains its original size and does not shrink over time, providing consistent support and compression. It's air-permeable materials allow the skin to breathe and reduce the potential for skin maceration. Not suitable for use on patients with a known intolerance to dry natural rubber latex.

- Adheres only to itself, not to dressing or clothes
- Secure, non- slip, retains cohesiveness in use
- Contains dry and natural rubber latex



Code	Size	Colour	Pack Qty	NHS SC Code
72100-08	5cm x 6.3m	Beige	1 x 36	ECD037
72100-09	7.5cm x 6.3m	Beige	1 x 24	ECD038
72100-04	10cm x 6.3m	Beige	1 x 18	ECD035
72100-05	15cm x 6.3m	Beige	1 x 12	ECD036
72101-09	5cm x 6.3m	White	1 x 36	ECD044
72101-10	7.5cm x 6.3m	White	1 x 24	ECD045
72101-05	10cm x 6.3m	White	1 x 18	ECD043
72102-04	5cm x 6.3m	Mixed*	1 x 36	ECD048
72102-03	7.5cm x 6.3m	Mixed*	1 x 24	ECD047
72102-02	10cm x 6.3m	Mixed*	1 x 18	ECD046
72102-23	10cm x 6.3m	Dark blue	1 x 18	ECD050
72102-05	10cm x 6.3m	Green	1 x 18	ECD049

*Mixed colour pack contains: blue, light blue, red, purple, neon pink and neon green.

Artistep® S by Actimove®

Actimove®

Cast Shoe

Non-slip walking sole that contributes to increasing the sense of security for patients with weight-bearing casts.

- Easy to apply and remove thanks to the adjustable straps
- Water resistant
- Lightweight
- Suitable for either the left or right foot
- Latex-free formula

Indications:

- For use with casts of the foot and ankle
- Surgery of the midfoot/forefoot
- Suitable to be used with a bandaged limb

LATEX
FREE
FORMULA

Code	Size	Shoe Size	NHS SC Code
65004-00	Paediatric Small	Childs 5	CVF85015
65004-01	Paediatric Medium	Childs 5.5 – 7.5	CVF85014
65004-02	Paediatric Large	Childs 8 - 11.5	CVF85000
65004-03	X Small	12 - 2	CVF85017
65004-04	Small	3 - 5.5	CVF85002
65004-05	Medium	6.5 - 7.5	CVF85001
65004-06	Large	8 - 9.5	CVF85016
65004-07	X Large	10 - 12	CVF85003



Actimove® Offloading Shoe

Offloading Footwear

Wide base for extra stability and the extended sole in the front area provides impact protection.

- 15° dorsi-angle shifts weight back to the rear foot area
- Non-slip sole for added walking security
- Water resistant
- Lightweight
- Suitable for either the left or right foot
- Latex-free formula

Indications:

- Post-surgical healing
- Wounds or ulcerations
- Pressure off-load of the forefoot
- Forefoot trauma

LATEX
FREE
FORMULA

Code	Size	Shoe Size	NHS SC Code
65004-20	X Small	12 - 2	CVF85004
65004-21	Small	3 - 5.5	CVF85018
65004-22	Medium	6.5 - 7.5	CVF85008
65004-23	Large	8 - 9.5	CVF85030
65004-24	X Large	10 - 12	CVF85007



Actimove® Cast Shoe Rocker Sole

Cast Shoe

The rocker sole helps ease pressure while walking and allows an easy roll through the gait pattern.

- Non-slip sole for added walking security
- Easy to apply and remove
- Water resistant
- Lightweight
- Suitable for either the left or right foot
- Latex-free formula

Indications:

- For use with casts of the foot and ankle
- Postoperative use
- Suitable to be used with a bandaged limb

LATEX
FREE
FORMULA

Code	Size	Shoe Size	NHS SC Code
65004-80	Paediatric	Childs 8 - 11.5	CVF85020
65004-81	X Small	12 - 2	CVF85031
65004-82	Small	3 - 5.5	CVF85013
65004-83	Medium	6.5 - 7.5	CVF85012
65004-84	Large	8 - 9.5	CVF85011
65004-85	X Large	10 - 12	CVF85010



Actimove® Post-Op Shoe Open Toe

Open Toe Shoe

Provides the foot with solid protection and easily accommodates bulky bandages.

- Non-slip sole for added walking security
- Water resistant
- Lightweight
- Suitable for either the left or right foot
- Latex-free formula

Indications:

- Provides support and protection for metatarsal stress fractures, broken toes or after foot surgery
- Post-operative stabilisation
- Post-operative healing
- Trauma care

LATEX
FREE
FORMULA

Code	Size	Shoe Size	NHS SC Code
65004-40	Paediatric	Childs 8 - 11.5	CVF85029
65004-44	X Small	12 - 2	CVF85027
65004-45	Small	3 - 5.5	CVF85028
65004-46	Medium	6.5 - 7.5	CVF85025
65004-47	Large	8 - 9.5	CVF85026
65004-48	X Large	10 - 12	CVF85009



Actimove® Cast Shoe Removable Toe Cover

Removable Toe Cover Shoe

Versatile shoe as it can be worn as closed toe or open toe.

- Protects the foot, casts and dressings
- Non-slip sole for added walking security
- Can be individually adjusted using the hook-and-loop straps
- Easy to apply and remove
- Water resistant
- Lightweight
- Suitable for either the left or right foot
- Latex-free formula

Indications:

- For use with casts of the foot and ankle
- Surgery of the midfoot/forefoot
- Lymphoedema, chronic oedema or venous leg ulcers

LATEX
FREE
FORMULA

Code	Size	Shoe Size	NHS SC Code
65004-60	Paediatric	Childs 8 - 11.5	CVF85023
65004-61	X Small	12 - 2	CVF85024
65004-62	Small	3 - 5.5	CVF85005
65004-63	Medium	6.5 - 7.5	CVF85006
65004-64	Large	8 - 9.5	CVF85021
65004-65	X Large	10 - 12	CVF85022
65004-66	XX Large	13.5 - 16	CVF85019



Actimove®

Actimove® Sling Comfort

Stable Arm Sling

- Designed to support the arm, elbow and shoulder by limiting the range of motion in the shoulder and elbow joints
- The wider 5.5cm sling provides comfort by reducing pressure on the back of the neck
- Dispenser box featuring a useful measuring guide aids easy application
- 28x T-tab hook and loop fastening included in dispenser box
- Radiolucent
- Metal-free
- Latex-free
- Machine washable at 40°C

LATEX
FREE
FORMULA

Code	Size	Measurements	Pack Qty
72859-18	Adjustable	5.5cm x 12m	1 x 2 rolls
73466-00			1 x 14 Y-tabs

Also available in different widths and lengths



Actimove®

Actimove® Umerus Comfort

Shoulder and Upper Arm Immobiliser

- For limited abduction, extension, internal and external rotation of the humerus
- Secure shoulder stabilisation through the waist strap that can be adjusted to suit most patients
- The wider 5.5cm sling provides comfort by reducing pressure on the back of the neck
- Soft thumb loop provides a stable and supported forearm position whilst still allowing full finger movement
- One-handed, self-application is achievable, aided by the hook and loop fastenings and the open and close pocket design
- Radiolucent
- Metal-free
- Machine washable at 40°C

Indications:

- Post-operative support
- Fracture management such as clavicle, scapular, humeral, and greater tuberosity fractures
- Joint dislocations such as acromioclavicular and sternoclavicular dislocations and shoulder luxation
- Inflammatory diseases such as calcific tendinitis of the rotator cuff
- Support while wearing a cast or splint
- Support for hemiplegic patients
- Minor instability of the shoulder



LATEX
FREE
FORMULA

Code	Size	Measurements	NHS SC Code
72819-94	X Small	< 33cm	GRW101
72819-39	Small	33cm – 37cm	GRW1121
72819-40	Medium	37cm – 40cm	GRW1122
72819-41	Large	40cm – 43cm	GRW1123
72819-42	X Large	> 43cm	GRW1129



Cast and Post-op Shoes

Actimove® Mitella Comfort



Stable Arm Support

- The 5.5cm sling width provides comfort by reducing pressure on the back of the neck
- Soft thumb loop provides a stable and supported forearm position whilst still allowing full finger movement
- Hook and loop fastenings enable one-handed self-application
- Suitable for both the left and right arm application
- Radiolucent
- Metal-free
- Machine washable at 40°C



**LATEX
FREE
FORMULA**

Code	Size	Measurements	NHS SC Code
72819-61	Small	< 37cm	GMB106
72819-62	Medium	37cm – 40cm	GMB107
72819-63	Large	40cm – 43cm	GMB108
72819-64	X Large	> 43cm	GMB109



Notes

[illegible]



Delta-Cast[®],
an Essity brand



Essity UK Ltd
Southfields Road, Dunstable,
Bedfordshire, LU6 3EJ

www.medical.essity.co.uk
Tel: 01482 670100 • Fax: 01482 670111
E-mail: orders.uk@essity.com