



JOBST® General Care Instructions

Washing Instructions

- Hand or gentle machine warm wash at 40 degrees celsius (Laundry bag recommended)
- Use mild liquid detergent
- Do not use powder detergent as it clogs elastic fibres
- Rinse well to remove all traces of detergent
- Do not bleach

For garments with silicone band:

The use of skin care products (such as skin creams, body lotion or powder) as well as various skin types (such as dry or peeling skin) and hair growth, may prevent the silicone band from keeping the thigh high in place. Therefore, moisturisers should be well absorbed before applying stockings.

We recommend the following steps to maximise the grip performance:

1. Turn the garment inside out before washing
2. Before the main wash, thoroughly hand wash the silicone band using a delicate fabric detergent. Concentrate on removing any lotions, hair or foreign material from the silicone.

Drying Instructions

- Dry JOBST® UltraSheer, JOBST® Travel Socks and JOBST® UlcerCARE™ liner either indoors or in the shade, laid flat
- JOBST® forMen Casual, ActiveWear, Sport, soSoft, forMen, Relief and UlcerCARE™ outer stocking can all be tumble dried at low temperature
- Do not wring



Do not bleach



Do not iron



Do not dry clean

Please refer to the product label and / or package insert for full instructions on the safe use of these products.

Always read the label and use only as directed. If symptoms persist, see your doctor / healthcare professional.