



DEFY GRAVITY

JOBST® COMPRESSION

Lower Limb Compression Garments

CONTENTS

Lower Limb Compression Garments

	EVERYDAY Ready-To-Wear	18
	JOBST® UltraSheer	
	JOBST® forMen	
	JOBST® Relief	
	SITUATIONAL Ready-To-Wear	24
	JOBST® ActiveWear	
	JOBST® Sport	
	JOBST® Maternity Opaque	
	JOBST® Travel	
	JOBST® Activa soSoft	
	MEDICAL Ready-To-Wear	32
	JOBST® Elvarex Soft	
	JOBST® UlcerCARE	
	JOBST® Comprinet pro	
	Accessories & Care Guidelines	40

WHAT TYPE OF COMPRESSION DOES MY PATIENT NEED?

1 Select the required compression


CCL 1: 15-20 mmHg	CCL 2: 20-30 mmHg	CCL 3: 30-40 mmHg
- Tired aching legs - Minor ankle, leg and foot swelling - Minor varicose during pregnancy - Long distance travel	- Tired aching legs - Moderate to severe varicose during pregnancy - Post vein surgery/post sclerotherapy - Moderate tendency of oedema - Superficial thrombophlebitis - Helps prevent recurrence of ulcerations	- Severe varicose veins - Lymphatic oedema - Post vein surgery/post sclerotherapy - Venous ulcers - Manages manifestations of Post Thrombotic Syndrome (PTS), Chronic Venous Insufficiency CVI/CVD - Aids in prevention of thrombotic syndrome

JOBST® Medical Legwear should not be worn and is contraindicated for the following conditions:

- Ischemia (eg. Advanced arterial disease)*
- Uncontrolled congestive heart failure
- Untreated septic phlebitis
- Phlegmasia cerulea dolens

*As part of a comprehensive assessment, a Doppler Ultrasound is used in the vascular screening process to clarify the vascular aetiology. A hand-held Doppler will assist in determining the degree, if any, of arterial disease. Using the Doppler the Ankle/Brachial Index (ABPI) can be calculated.

2 Select garment style

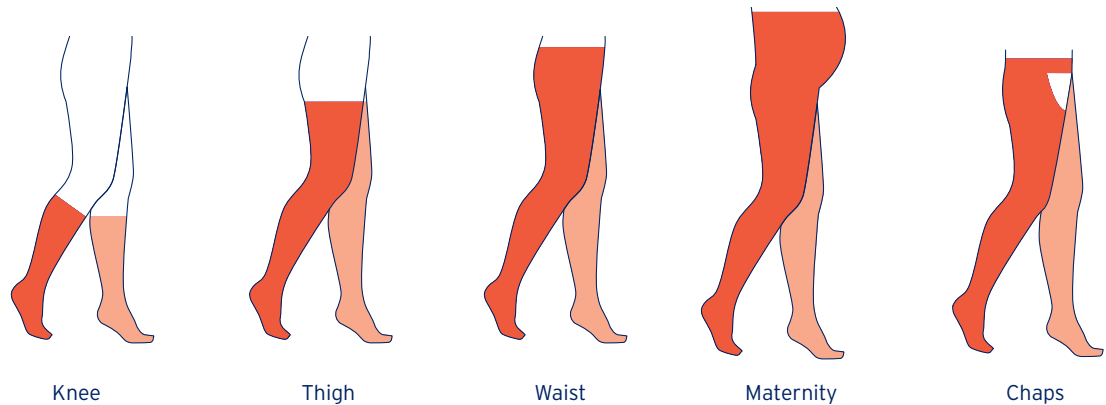


Some considerations when selecting the most appropriate style for your patient:

- Indication being treated/managed
- Lifestyle, mobility and age
- Ability to don and tolerate level of compression
- Skin Integrity
- Open toe or closed toe requirements
- Maternity Style only available in Maternity Opaque

3 Measure for JOBST® Medical Legwear

Measuring positions

cH - Hip Circumference at largest dimension around buttocks	cH
cG - Thigh Circumference at largest dimension	cG
cC - Calf Circumference at largest dimension	cC
cB - Ankle Circumference at smallest dimension	cB

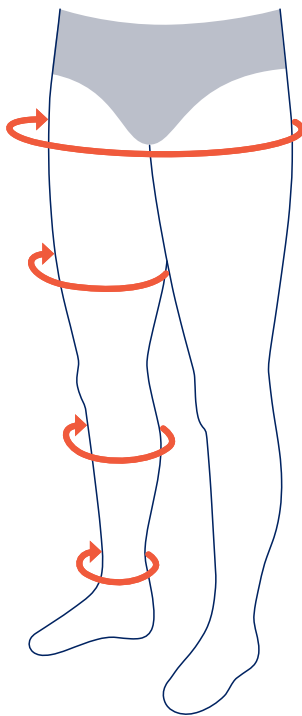

Provide the following measurements

Knee High - cB & cC
Thigh High - cB, cC & cG
Waist high - cB, cG & cH
Chap - cB, cC, cG, & cH

NOTE Each leg is measured independently. Measure early in the morning when possible as swelling often increases throughout in the day.

Sizing Chart for JOBST® UltraSheer, forMen, Relief, ActiveWear, Sport and Maternity Opaque


Style	Measurement	S	M	L	L Full Calf	XL	XL Full Calf
UNISEX	Ankle	18 - 21 cm	21 - 25 cm	25 - 29 cm	25 - 30 cm	29 - 33 cm	30 - 36 cm
	Calf	28 - 38 cm	30 - 42 cm	32 - 46 cm	46 - 61 cm	34 - 50 cm	46 - 61 cm
	Thigh	40 - 62 cm	46 - 70 cm	54 - 78 cm		60 - 81 cm	
	Hip	71 - 117 cm	76 - 127 cm	81 - 137 cm		102 - 166 cm	
MATERNITY	Ankle	18 - 21 cm	21 - 25 cm	25 - 29 cm			
	Thigh	40 - 62 cm	46 - 70 cm	54 - 78 cm			
	Hip	81 - 132 cm	86 - 142 cm	91 - 152 cm			

Knee High Length: Petite ≤ 38cm, Regular > 38cm. Thigh high length: ≥ 68.5cm.

JOBST® ULTRASHEER


Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Medical compression stockings help manage specific venous and lymphatic conditions
- Fine high quality yarns with an open mesh-like knitting structure allows air to pass for greater breathability
- Excellent moisture wicking for better patient compliance
- Shimmering appearance for a glossy, stylish look
- Thigh high silicone top bands that grip in style

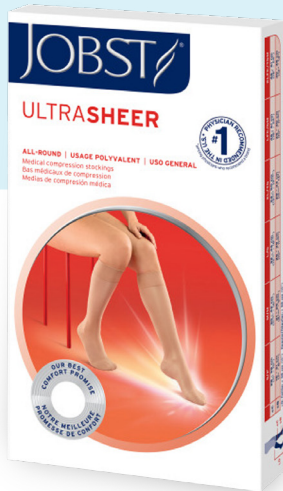

Sizing & Compression Classes

- CCL 1: 15-20 mmHg
- CCL 2: 20-30 mmHg

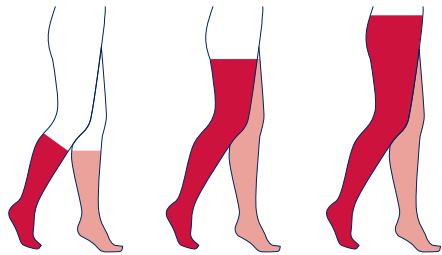

- Transparent: Sheerness for beautiful legs
- Breathable: Allows air to pass
- Shimmering: For a glossy look


Length	S		M		L		XL		Colour
	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	
Knee High	75165-30	75165-25	75166-30	75166-25	75167-30	75167-25	75168-30	75168-25	● Natural Regular
		75408-02		75409-02		75410-02		75415-02	● Natural Petite
		75165-04		75166-04		75167-04		75168-04	● Black
		75201-19		75202-19		75203-19		75204-19	● Natural Regular
		75435-02		75436-02		75437-02		75438-02	● Natural Petite
		75201-04		75202-04		75203-04		75204-04	● Black
Thigh High		75169-22		75170-22		75171-22		75172-22	● Natural
		75169-04		75170-04		75171-04		75172-04	● Black
		75205-23		75206-23		75207-23		75208-23	● Natural
		75205-04		75206-04		75207-04		75208-04	● Black
Sensitive Thigh High		77686-12		77686-13		77686-14		77686-15	● Natural
		77686-04		77686-05		77686-06		77686-07	● Black
		77687-12		77687-13		77687-14		77687-15	● Natural
		77687-04		77687-05		77687-06		77687-07	● Black
Waist High		75173-12		75174-12		75175-12		75176-12	● Natural
		75173-04		75174-04		75175-04		75176-04	● Black
		75209-13		75210-13		75211-13		75212-13	● Natural
		75209-04		75210-04		75211-04		75212-04	● Black

JOBST UltraSheer knee high and thigh high garments are supplied as a pair.
Only JOBST UltraSheer thigh high variants are finished with a silicone top band.

JOBST® FORMEN

Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- A unique knit structure
- Ribbed design offers a fashionable men's look dress sock
- Knee high garments include a SoftFit top band - a fine knitted-in silicone yarn for gentle, comfortable grip.
- Durable fabric designed for long-lasting compression
- Effective odour control
- Cushioned sole with anatomical left and right feet for improved fit
- JOBST® forMen offers therapy and style with unparalleled comfort and durability

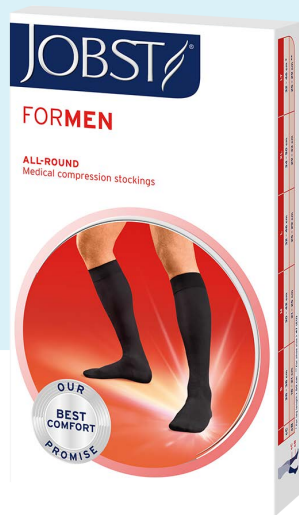

Sizing & Compression Classes

CCL 1: 15-20 mmHg

CCL 2: 20-30 mmHg

CCL 3: 30-40 mmHg

Durable

For long-term use

Odour Control

Helps to keep feet fresh

Cushioned Sole

Specially constructed cushioned sole

Length	S	M	L	LFC	XL	XLFC	Colour
	Closed Toe	Closed Toe	Closed Toe	Closed Toe Full Calf	Closed Toe	Closed Toe Full Calf	
Knee High	60523-40	60523-41	60523-42	60523-44	60523-43	60523-45	● Black
	60524-40	60524-41	60524-42	60524-44	60524-43	60524-45	● Khaki
	60529-40	60529-41	60529-42	60529-44	60529-43	60529-45	● Black
	60530-40	60530-41	60530-42	60530-44	60530-43	60530-45	● Khaki
	60535-40	60535-41	60535-42	60535-44	60535-43	60535-45	● Black
	60536-40	60536-41	60536-42	60536-44	60536-43	60536-45	● Khaki
Thigh High	60423-20	60423-21	60423-22		60423-23		● Black
	60423-60	60423-61	60423-62		60423-63		● Khaki
	60425-20	60425-21	60425-22		60425-23		● Black
	60425-60	60425-61	60425-62		60425-63		● Khaki
	60427-20	60427-21	60427-22		60427-23		● Black
	60427-60	60427-61	60427-62		60427-63		● Khaki

All JOBST forMen garments are supplied as a pair.
Knee high garments are finished with SoftFit silicone top band.
Thigh high garments are finished with a micro-dotted silicone top band.

JOBST® RELIEF

Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Universal graduated compression in a classical stocking appearance
- Reinforced areas in the foot make Relief stockings especially durable
- Circular knit and seamless providing comfort and support
- Unisex with open and closed toe options
- A wide top band promotes all day comfort



Sizing & Compression Classes

CCL 1: 15-20 mmHg

CCL 2: 20-30 mmHg

CCL 3: 30-40 mmHg

Durable
Long lasting fabric

Breathable
Allows air to pass

Helps minimise swelling
Improves blood circulation

S	S	M	M	L	L	LFC	LFC	XL	XL	XLFC	XLFC	Colour
Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe	Closed Toe	Open Toe Full Calf	Closed Toe Full Calf	Open Toe	Closed Toe	Open Toe Full Calf	Closed Toe Full Calf	
Relief Garments Knee High												
75339-20	75339-12	75340-20	75340-12	75341-20	75341-12	78649-04	75343-12	78649-03	75342-12	78649-05	75344-12	● Black
75339-10	75339-11	75340-10	75340-11	75341-10	75341-11	75343-10	75343-11	75342-10	75342-11	75344-10	75344-11	● Beige
78052-20	78052-32	78052-21	78052-33	78052-22	78052-34	78052-24	78052-36	78052-23	78052-35	78052-25	78052-37	● Black
78052-26	78052-38	78052-27	78052-39	78052-28	78052-40	78052-30	78052-42	78052-29	78052-41	78052-31	78052-43	● Black Petite
78050-04	78050-15	78051-04	78051-15	78052-04	78052-15	77975-15	77975-16	78053-04	78053-15	77976-15	77976-16	● Beige
78054-04	78054-15	78055-04	78055-15	78056-04	78056-15	77977-15	77977-16	78057-04	78057-15	77978-15	77978-16	● Beige
Relief Garments Thigh High												
78649-20	78649-34	78649-21	78649-35	78649-22	78649-36			78649-23	78649-37			● Black
78042-30	78042-07	78043-30	78043-31	78044-30	78044-07			78650-23	78045-07			● Black
78042-08	78042-09	78043-08	78043-09	78044-08	78044-09			78045-08	78045-09			● Beige
78059-08	78059-09	78047-08	78047-09	78048-08	78048-09			78049-08	78049-09			● Beige
Relief Garments Waist High												
78476-10	78476-11	78477-10	78477-11	78478-10	78478-11			78479-10	78479-11			● Beige
78480-10		78481-10		78482-10				78483-10				● Beige
Relief Garments Chaps												
78031-05		78032-05		78033-05				78034-05				● Beige L
78031-06		78032-06		78033-06				78034-06				● Beige R
78035-05		78036-05		78037-05				78041-05				● Beige L
78035-06		78036-06		78037-06				78041-06				● Beige R

JOBST Relief knee high and thigh high garments are supplied as a pair.
Only thigh high variants are finished with a dotted silicone top band.

JOBST® ACTIVEWEAR


Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Unisex, soft and easy to apply
- Cushioned foot and seamless toe help protect feet from shoe seams
- Dri-release technology - providing a dry and cooling effect



Sizing & Compression Classes

- CCL 1: 15-20 mmHg
- CCL 2: 20-30 mmHg
- CCL 3: 30-40 mmHg

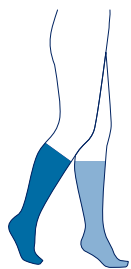

- Cushioning: Cushioned foot provides comfort
- Moisture Management: Helps to keep feet dry
- Soft Touch: Blend of soft yarns

Length	S	M	L	XL	Colour
	Closed Toe	Closed Toe	Closed Toe	Closed Toe	
Knee High	75147-03	75148-03	75149-03	75150-03	● Black
	75147-02	75148-02	75149-02	75150-02	○ White
	75155-03	75156-03	75157-03	75158-03	● Black
	75155-02	75156-02	75157-02	75158-02	○ White
	75411-03	75412-03	75413-03	75414-03	● Black
	75411-02	75412-02	75413-02	75414-02	○ White

JOBST ActiveWear garments are supplied as a pair and are finished without a silicone top band.

JOBST® SPORT


Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Soft fabric with a breathable knit provides a high level of comfort for sporting activities
- JOBST® moisture management system for moisture control of the feet
- Activated carbon fibres for effective odour control
- Extra flat toe seam for extended wearing comfort



Sizing & Compression Classes

- CCL 1: 15-20 mmHg

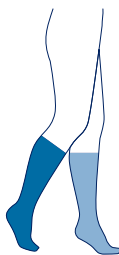

- Support Zone: Support zone for tendons
- Odour Control: Helps to keep feet fresh
- Moisture Management: Helps keep feet dry


Length	S	M	L	XL	Colour
	Closed Toe	Closed Toe	Closed Toe	Closed Toe	
Knee High	75289-70	75289-71	75289-72	75289-73	● Black
	75289-80	75289-81	75289-82	75289-83	■ Royal Blue
	75289-90	75289-91	75289-92	75289-93	■ Pink

JOBST Sport garments are supplied as a pair and are finished without a silicone top band.

JOBST® MATERNITY OPAQUE

Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Elastic SoftFlex Zone adapts to a woman's growing belly
- Moisture management system to provide air permeability through the foot
- Made from a soft blend of yarns with a particular focus on the wellbeing of pregnant women

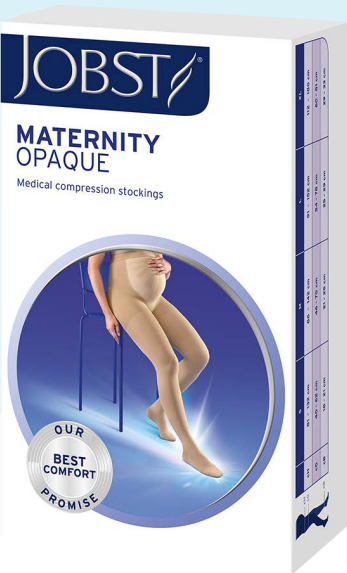

Sizing & Compression Classes

CCL 1: 15-20 mmHg

CCL 2: 20-30 mmHg

- Adapts to the growing belly
- Helps to keep feet dry
- Blend of soft yarns

Length	S	M	L	Colour	
	Closed Toe	Closed Toe	Closed Toe		
Maternity Waist High	60412-20	60412-21	60412-22		Caramel
	60412-70	60412-71	60412-72		Caramel
	60412-50	60412-51	60412-52		Black


JOBST® TRAVEL

Circular Knit
Ready-To-Wear (RTW)

- Graduated compression helps improve blood flow and reduce swelling
- Promotes leg health during long hours of inactivity
- Helps reduce swelling and aching legs, ankles and feet due to long periods of sitting or standing
- 15-20mmHg compression helps reduce the risk of deep vein thrombosis (Blood Clot)
- Improves circulation and energises legs



Sizing & Compression Classes

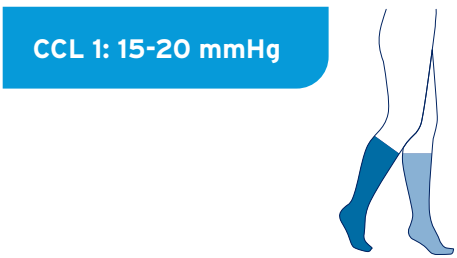

Improves blood circulation

Promotes leg health

Allows air to pass

Length	Size 1	Size 2	Size 3	Size 4	Size 5	Colour
	Closed Toe	Closed Toe	Closed Toe	Closed Toe	Closed Toe	
Knee High	78844-07	78845-07	78846-07	78847-07	79272-07	● Black
	78844-12	78845-12	78846-12	78847-12	79272-12	● Beige

JOBST Travel garments are supplied as a pair and are finished without a silicone top band.

JOBST® Travel Socks Measurement Table

Size	1	2	3	4	5
Ankle Circumference	18 - 21 cm	20 - 23 cm	22 - 25cm	24 - 27 cm	26 - 29 cm
Calf Circumference	27 - 37 cm	30 - 40 cm	33 - 43 cm	36 - 46 cm	39 - 49 cm
AUS Women's Shoe	4.5 - 6.5	7 - 9	9.5 - 11	11.5 - 13.5	-
AUS Men's Shoe	3.5 - 5.5	6 - 8	8.5 - 10	10.5 - 12	12.5 - 14
EU Shoe	37 - 38	39 - 40	41 - 42	43 - 44	45 - 46
UK Shoe	3 - 5	5 - 7	7 - 9	9 - 11	11 - 13




JOBST® ACTIVA SOSOFT

Circular Knit
Ready-To-Wear (RTW)

- JOBST® Activa soSoft compression socks provide graduated compression to help support circulation
- Before applying compression, a full leg assessment should be carried out
- With gentle compression of 8-15mmHg, JOBST® Activa soSoft compression socks can help tired and achy legs
- Suitable for longer periods of sitting and standing
- Provide prevention of and relief from minor oedema



Sizing & Compression Classes

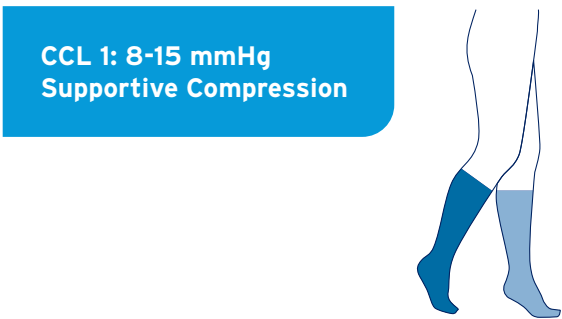

Length	S	M	L	XL	Colour
	Closed Toe	Closed Toe	Closed Toe	Closed Toe	
Knee High	60562-00	60562-01	60562-02	60562-03	● Black
	73680-12	60562-21	60562-22	60562-23	● Sand

JOBST Activa soSoft garments are supplied as a pair and are finished without a silicone top band.

Size	S	M	L	XL
Ankle Circumference	18 - 21 cm	21 - 25 cm	25 - 29 cm	29 - 33 cm
Calf Circumference	28 - 38 cm	30 - 42 cm	32 - 46 cm	34 - 50 cm
AUS Women's Shoe	4.5 - 6.5	7 - 9	9.5 - 11	11.5 - 13.5
AUS Men's Shoe	3.5 - 5.5	6 - 8	8.5 - 10	10.5 - 12

Shoe size alone is not an indicator for correct sizing. Measurements of calf and ankle circumference are mandatory for the proper performance of this product



JOBST® ELVAREX® SOFT

Flat Knit

Ready-To-Wear (RTW)

- Indicated for mild to moderate lymphoedema, lipoedema and vascular insufficiency
- Available in CCL1 15-21mmHg, CCL2 23-32mmHg and CCL3 34-46mmHg
- Flat knit fabric for greater containment and management of limb shape
- Soft, breathable fabric for exceptional comfort and adherence
- Moisture management technology to keep skin dry
- Graduated, short-stretch compression
- Ease of donning and doffing to support patient independence
- Available in beige and black



Sizing & Compression Classes


COMPRESSION LEVELS

15-21 mmHg / CCL1

23-32 mmHg / CCL2

34-46 mmHg / CCL3*

*CCL3 knee high & thigh high only

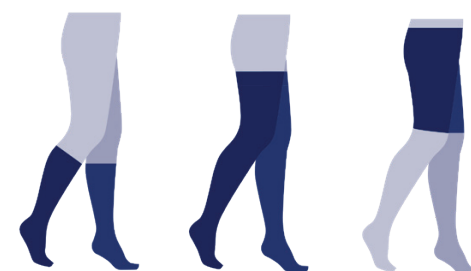


Soft Touch



Easy to Put On




SoftFit Technology

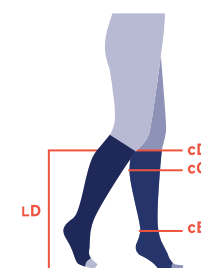

KNEE HIGH (AD)

Size	1	2	2 plus	3	3 plus	4	4 plus	5
cD (cm)	27 - 34	29 - 37	32 - 40	34 - 43	36 - 46	38 - 49	40 - 51	43 - 56
cC (cm)	28 - 35	30 - 38	33 - 41	35 - 44	37 - 47	39 - 50	41 - 54	44 - 58
cB (cm)	18 - 20	20 - 23	20 - 23	23 - 26	23 - 26	26 - 30	26 - 30	30 - 34


LD Petite: 35 - 40 cm LD Regular: 41 - 46 cm

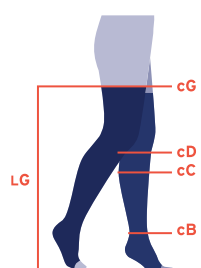

THIGH HIGH (AG)

Size	1	2	2 plus	3	3 plus	4	4 plus	5
cG (cm)	44 - 57	46 - 60	49 - 63	50 - 66	55 - 71	52 - 70	57 - 75	65 - 82
cD (cm)	27 - 34	29 - 37	32 - 40	34 - 43	36 - 46	38 - 49	40 - 51	43 - 56
cC (cm)	28 - 35	30 - 38	33 - 41	35 - 44	37 - 47	39 - 50	41 - 54	44 - 58
cB (cm)	18 - 20	20 - 23	20 - 23	23 - 26	23 - 26	26 - 30	26 - 30	30 - 34

LG Petite: 68 - 76 cm LG Regular: 77 - 85 cm

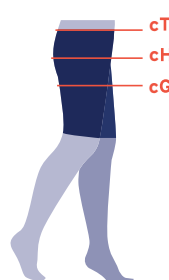

BERMUDA (ET)

Size	1	2	3	4	5
cT (cm)	70 - 85	80 - 95	85 - 105	90 - 110	100 - 120
cH (cm)	85 - 100	90 - 105	95 - 115	105 - 125	115 - 135
cG (cm)	44 - 57	46 - 60	50 - 66	52 - 70	65 - 82



JOBST® ELVAREX® SOFT Flat Knit

EXPLORE THE **READY-TO-WEAR** ASSORTMENT

COMPRESSION LEVELS

15-21 mmHg / CCL1

23-32 mmHg / CCL2

34-46 mmHg / CCL3*
*CCL3 knee high & thigh high only

KNEE HIGH (AD)

Length	Top Band	Toe Style	Colour	Size 1	Size 2	Size 2+	Size 3	Size 3+	Size 4	Size 4+	Size 5
Regular	Standard	Closed Toe	Beige	79546-00	79546-01	79546-05	79546-02	79546-06	79546-03	79546-07	79546-04
Regular	Standard	Closed Toe	Black	79546-08	79546-09	79546-13	79546-10	79546-14	79546-11	79546-15	79546-12
Regular	Standard	Open Toe	Beige	79546-32	79546-33	79546-37	79546-34	79546-38	79546-35	79546-39	79546-36
Regular	Standard	Open Toe	Black	79546-40	79546-41	79546-45	79546-42	79546-46	79546-43	79546-47	79546-44
Regular	SoftFit	Closed Toe	Beige	79546-16	79546-17	79546-21	79546-18	79546-22	79546-19	79546-23	79546-20
Regular	SoftFit	Closed Toe	Black	79546-24	79546-25	79546-29	79546-26	79546-30	79546-27	79546-31	79546-28
Regular	SoftFit	Open Toe	Beige	79546-48	79546-49	79546-53	79546-50	79546-54	79546-51	79546-55	79546-52
Regular	SoftFit	Open Toe	Black	79546-56	79546-57	79546-61	79546-58	79546-62	79546-59	79546-63	79546-60
Petite	Standard	Closed Toe	Beige	73605-00	73605-01	73605-05	73605-02	73605-06	73605-03	73605-07	73605-04
Petite	Standard	Closed Toe	Black	73605-08	73605-09	73605-13	73605-10	73605-14	73605-11	73605-15	73605-12
Petite	Standard	Open Toe	Beige	73605-32	73605-33	73605-37	73605-34	73605-38	73605-35	73605-39	73605-36
Petite	Standard	Open Toe	Black	73605-40	73605-41	73605-45	73605-42	73605-46	73605-43	73605-47	73605-44
Petite	SoftFit	Closed Toe	Beige	73605-16	73605-17	73605-21	73605-18	73605-22	73605-19	73605-23	73605-20
Petite	SoftFit	Closed Toe	Black	73605-24	73605-25	73605-29	73605-26	73605-30	73605-27	73605-31	73605-28
Petite	SoftFit	Open Toe	Beige	73605-48	73605-49	73605-53	73605-50	73605-54	73605-51	73605-55	73605-52
Petite	SoftFit	Open Toe	Black	73605-56	73605-57	73605-61	73605-58	73605-62	73605-59	73605-63	73605-60
Regular	Standard	Closed Toe	Beige	79559-24	79559-25	79559-29	79559-26	79559-30	79559-27	79559-31	79559-28
Regular	Standard	Closed Toe	Black	79559-32	79559-33	79559-37	79559-34	79559-38	79559-35	79559-39	79559-36
Regular	Standard	Open Toe	Beige	79559-56	79559-57	79559-61	79559-58	79559-62	79559-59	79559-63	79559-60
Regular	Standard	Open Toe	Black	79559-64	79559-65	79559-69	79559-66	79559-70	79559-67	79559-71	79559-68
Regular	SoftFit	Closed Toe	Beige	79559-40	79559-41	79559-45	79559-42	79559-46	79559-43	79559-47	79559-44
Regular	SoftFit	Closed Toe	Black	79559-48	79559-49	79559-53	79559-50	79559-54	79559-51	79559-55	79559-52
Regular	SoftFit	Open Toe	Beige	79559-72	79559-73	79559-77	79559-74	79559-78	79559-75	79559-79	79559-76
Regular	SoftFit	Open Toe	Black	79559-80	79559-81	79559-85	79559-82	79559-86	79559-83	79559-87	79559-84
Petite	Standard	Closed Toe	Beige	79558-24	79558-25	79558-29	79558-26	79558-30	79558-27	79558-31	79558-28
Petite	Standard	Closed Toe	Black	79558-32	79558-33	79558-37	79558-34	79558-38	79558-35	79558-39	79558-36
Petite	Standard	Open Toe	Beige	79558-56	79558-57	79558-61	79558-58	79558-62	79558-59	79558-63	79558-60
Petite	Standard	Open Toe	Black	79558-64	79558-65	79558-69	79558-66	79558-70	79558-67	79558-71	79558-68
Petite	SoftFit	Closed Toe	Beige	79558-40	79558-41	79558-45	79558-42	79558-46	79558-43	79558-47	79558-44
Petite	SoftFit	Closed Toe	Black	79558-48	79558-49	79558-53	79558-50	79558-54	79558-51	79558-55	79558-52
Petite	SoftFit	Open Toe	Beige	79558-72	79558-73	79558-77	79558-74	79558-78	79558-75	79558-79	79558-76
Petite	SoftFit	Open Toe	Black	79558-80	79558-81	79558-85	79558-82	79558-86	79558-83	79558-87	79558-84
Regular	Standard	Closed Toe	Beige	73552-24	73552-25	73552-29	73552-26	73552-30	73552-27	73552-31	73552-28
Regular	Standard	Closed Toe	Black	73552-32	73552-33	73552-37	73552-34	73552-38	73552-35	73552-39	73552-36
Regular	Standard	Open Toe	Beige	73552-56	73552-57	73552-61	73552-58	73552-62	73552-59	73552-63	73552-60
Regular	Standard	Open Toe	Black	73552-64	73552-65	73552-69	73552-66	73552-70	73552-67	73552-71	73552-68
Regular	SoftFit	Closed Toe	Beige	73552-40	73552-41	73552-45	73552-42	73552-46	73552-43	73552-47	73552-44
Regular	SoftFit	Closed Toe	Black	73552-48	73552-49	73552-53	73552-50	73552-54	73552-51	73552-55	73552-52
Regular	SoftFit	Open Toe	Beige	73552-72	73552-73	73552-77	73552-74	73552-78	73552-75	73552-79	73552-76
Regular	SoftFit	Open Toe	Black	73552-80	73552-81	73552-85	73552-82	73552-86	73552-83	73552-87	73552-84
Petite	Standard	Closed Toe	Beige	73579-24	73579-25	73579-29	73579-26	73579-30	73579-27	73579-31	73579-28
Petite	Standard	Closed Toe	Black	73579-32	73579-33	73579-37	73579-34	73579-38	73579-35	73579-39	73579-36
Petite	Standard	Open Toe	Beige	73579-56	73579-57	73579-61	73579-58	73579-62	73579-59	73579-63	73579-60
Petite	Standard	Open Toe	Black	73579-64	73579-65	73579-69	73579-66	73579-70	73579-67	73579-71	73579-68
Petite	SoftFit	Closed Toe	Beige	73579-40	73579-41	73579-45	73579-42	73579-46	73579-43	73579-47	73579-44
Petite	SoftFit	Closed Toe	Black	73579-48	73579-49	73579-53	73579-50	73579-54	73579-51	73579-55	73579-52
Petite	SoftFit	Open Toe	Beige	73579-72	73579-73	73579-77	73579-74	73579-78	73579-75	73579-79	73579-76
Petite	SoftFit	Open Toe	Black	73579-80	73579-81	73579-85	73579-82	73579-86	73579-83	73579-87	73579-84

JOBST Elvarex Soft RTW knee high garments are supplied as a pair.

THIGH HIGH (AG)

Length	Toe Style	Colour	Size 1	Size 2	Size 2+	Size 3	Size 3+	Size 4	Size 4+	Size 5
Regular	Closed Toe	Beige	73160-00	73160-01	73160-05	73160-02	73160-06	73160-03	73160-07	73160-04
Regular	Closed Toe	Black	73160-08	73160-09	73160-13	73160-10	73160-14	73160-11	73160-15	73160-12
Regular	Open Toe	Beige	73160-16	73160-17	73160-21	73160-18	73160-22	73160-19	73160-23	73160-20
Regular	Open Toe	Black	73160-24	73160-25	73160-29	73160-26	73160-30	73160-27	73160-31	73160-28
Petite	Closed Toe	Beige	73164-00	73164-01	73164-05	73164-02	73164-06	73164-03	73164-07	73164-04
Petite	Closed Toe	Black	73164-08	73164-09	73164-13	73164-10	73164-14	73164-11	73164-15	73164-12
Petite	Open Toe	Beige	73164-16	73164-17	73164-21	73164-18	73164-22	73164-19	73164-23	73164-20
Petite	Open Toe	Black	73164-24	73164-25	73164-29	73164-26	73164-30	73164-27	73164-31	73164-28
Regular	Closed Toe	Beige	73161-00	73161-01	73161-05	73161-02	73161-06	73161-03	73161-07	73161-04
Regular	Closed Toe	Black	73161-08	73161-09	73161-13	73161-10	73161-14	73161-11	73161-15	73161-12
Regular	Open Toe	Beige	73161-16	73161-17	73161-21	73161-18	73161-22	73161-19	73161-23	73161-20
Regular	Open Toe	Black	73161-24	73161-25	73161-29	73161-26	73161-30	73161-27	73161-31	73161-28
Petite	Closed Toe	Beige	73165-00	73165-01	73165-05	73165-02	73165-06	73165-03	73165-07	73165-04
Petite	Closed Toe	Black	73165-08	73165-09	73165-13	73165-10	73165-14	73165-11	73165-15	73165-12
Petite	Open Toe	Beige	73165-16	73165-17	73165-21	73165-18	73165-22	73165-19	73165-23	73165-20
Petite	Open Toe	Black	73165-24	73165-25	73165-29	73165-26	73165-30	73165-27	73165-31	73165-28
Regular	Closed Toe	Beige	73162-00	73162-01	73162-05	73162-02	73162-06	73162-03	73162-07	73162-04
Regular	Closed Toe	Black	73162-08	73162-09	73162-13	73162-10	73162-14	73162-11	73162-15	73162-12
Regular	Open Toe	Beige	73162-16	73162-17	73162-21	73162-18	73162-22	73162-19	73162-23	73162-20
Regular	Open Toe	Black	73162-24	73162-25	73162-29	73162-26	73162-30	73162-27	73162-31	73162-28
Petite	Closed Toe	Beige	73298-00	73298-01	73298-05	73298-02	73298-06	73298-03	73298-07	73298-04
Petite	Closed Toe	Black	73298-08	73298-09	73298-13	73298-10	73298-14	73298-11	73298-15	73298-12
Petite	Open Toe	Beige	73298-16	73298-17	73298-21	73298-18	73298-22	73298-19	73298-23	73298-20
Petite	Open Toe	Black	73298-24	73298-25	73298-29	73298-26	73298-30	73298-27	73298-31	73298-28

JOBST Elvarex Soft RTW thigh high garments are supplied as a pair.

BERMUDA (ET)

Gender	Colour	Size 1	Size 2	Size 3	Size 4	Size 5
Female	Beige	73318-00	73318-01	73318-02	73318-03	73318-04
Female	Black	73318-05	73318-06	73318-07	73318-08	73318-09
Male	Beige	73318-10	73318-11	73318-12	73318-13	73318-14
Male	Black	73318-15	73318-16	73318-17	73318-18	73318-19
Female	Beige	73616-00	73616-01	73616-02	73616-03	73616-04
Female	Black	73616-05	73616-06	73616-07	73616-08	73616-09
Male	Beige	73616-10	73616-11	73616-12	73616-13	73616-14
Male	Black	73616-15	73616-16	73616-17	73616-18	73616-19

JOBST Elvarex Soft RTW bermuda garments are supplied as a single unit.



JOBST® ULCERCARE




JOBST® UlcerCARE™ enables effective management of hard to heal wounds on the lower leg while offering ease of use for patients and care givers - the perfect combination for providing patients with an improved quality of life.

JOBST® UlcerCARE™ Compression System:

- Provides graduated compression of 40mmHg, the recommended pressure for healing these types of wounds
- This 2-in-1 stocking system is designed for patient self-management to facilitate easy changing of the wound dressing and to allow for improved personal hygiene (e.g. bathing and skin care)
- JOBST® UlcerCARE Liner provides mild compression (15-20mmHg) for the non-ambulatory patient to help manage lower leg oedema during bed rest
- Outer stocking comes with a zipper on the left or right side for comfortable closure on the opposite side of the wound. It is removed at night and applied first thing in the morning

Sizing & Compression Classes

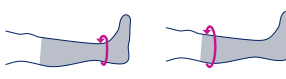Size			JOBST® UlcerCARE Zipper Stocking and Liners (1 stocking/2 liners)		JOBST® UlcerCARE Replacement Liners (3 liners)
	Ankle	Calf	Zipper on Left Side	Zipper on Right Side	
Small	18 - 21 cm	29 - 35 cm	60428-00	60428-20	60428-60
Medium	21 - 24 cm	34 - 41 cm	60428-01	60428-21	60428-61
Large	24 - 27 cm	40 - 47 cm	60428-02	60428-22	60428-62
X-Large	27 - 30 cm	46 - 54 cm	60428-03	60428-23	60428-63
XX-Large	30 - 33 cm	48 - 56 cm	60428-04	60428-24	60428-64
XXX-Large	33 - 36 cm	50 - 58 cm	60428-05	60428-25	60428-65
XXXX-Large	36 - 39 cm	52 - 60 cm	60428-06	60428-26	60428-66

JOBST UlcerCARE is supplied with one outer stocking and two liners without a silicone top band. Replacement liners are supplied in packs of three.



JOBST® COMPRINET® PRO

Graduated compression anti-embolism stockings

- Indicated for thrombosis prophylaxis in **surgical patients** and patients requiring prolonged periods of immobilisation
- Comprinet® pro anti-embolism stockings can help reduce the risk of thrombosis in immobile and surgical patients
- Comprinet® pro exerts pressure on the vessels from outside. This narrows the veins and, although the patient is lying supine, the blood flows at a faster rate back to the heart




Sizing & Compression Classes

The colour-coded sizing system allows rapid allocation of the right size.

The colours displayed at the inspection window correspond to the width (inner colour) and the length (outer colour) of the stocking.

Select the correct size garment based upon the measurement guide.

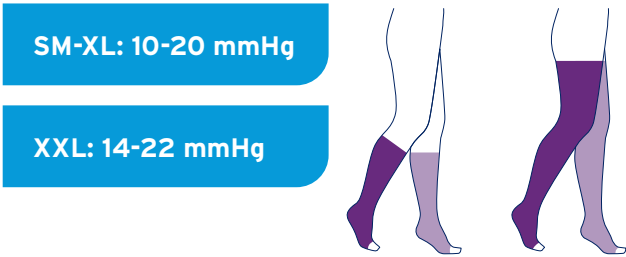

Comprinet pro Knee High

Size	Ankle Circumference	Colour Code	Product Code
Small	23 - 30 cm		77026-02
Medium	30 - 38 cm		77027-02
Large	38 - 46 cm		77028-02
X-Large	46 - 53 cm		77124-05
XX-Large	53 - 66 cm		77133-03

Comprinet pro Thigh High

Size	Ankle Circumference	Length	Colour Code	Product Code
Small Regular	23 - 30 cm	64 - 80 cm		46335-12
Small Long	23 - 30 cm	80 - 97 cm		46336-02
Medium Regular	30 - 38 cm	64 - 80 cm		46337-02
Medium Long	30 - 38 cm	80 - 97 cm		46338-02
Large Regular	38 - 46 cm	64 - 80 cm		46388-02
Large Long	38 - 46 cm	80 - 97 cm		46389-02
X-Large Regular	46 - 53 cm	64 - 80 cm		77125-02
X-Large Long	46 - 53 cm	80 - 97 cm		77128-02
XX-Large Regular	53 - 66 cm	64 - 80 cm		77130-01
XX-Large Long	53 - 66 cm	80 - 97 cm		77131-01

All JOBST Comprinet pro garments are supplied as a pair in boxes of 10 pairs and are finished without a silicone top band.

JOBST® ACCESSORIES AND CARE GUIDELINES

JOBST® DONNING AND DOFFING SOLUTIONS



Application of knee high stockings



Application of thigh high stockings



Doffing of thigh high stockings instructions

JOBST® ACCESSORIES




JOBST® Stocking Donner

Product Code: 79668-00

Due to the nature of compression stockings, people may find putting them on to be initially a challenging task. The **JOBST® Stocking Donner**, which features a **lightweight frame** and foam grip handles, is designed to aid in the application of all JOBST® Medical legwear.

JOBST® Application

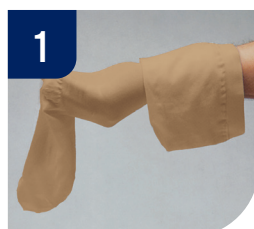

Put the hand inside the stocking and grip the heel.



Turn the stocking inside-out, whilst still holding onto the heel.



Pull the inside-out of the stocking over the front of the foot as far as the heel.



Gradually pull the stocking onto the calf, like a normal stocking.



Push the overlapping part of the stocking over the heel.



Fold over the stocking and pull without using force.



Make sure that the heel of the stocking is free of creases.



Pull the stocking up to the knee, thigh or waist; let it slide over the thumbs. Knee high socks or stockings should finish 2 finger width's below the knee crease at the back of the leg.



Place your JOBST® compression stocking inside the semi-circle frame and pull the top of the stocking down over the semi-circle. The heel of the stocking must face the back of your JOBST® Stocking Donner.

Continue to slide the stocking down over the semi-circle until the heel protrudes from the frame, and is centred and even.



Preferably whilst seated, grasp the padded handles and gently pull the stocking donner upwards whilst moving your leg slightly outwards, until the stocking is above the calf.



Whilst using a support for balance, insert your foot into the stocking until your foot is completely on the floor.

Helpful Tip: Sit down and hold the heel pocket against the frame as you slide your toes into the stocking.



Push the JOBST® Stocking Donner backward and down to free the stocking.

Be sure the heel of the stocking is properly placed on the foot. Then adjust the length and smooth out any wrinkles by stroking with the palm of your hands.

The knee band in knee high socks or stockings should stop two finger widths below the crease at the back of the knee.



ARION DONNING AND DOFFING AIDS

Slide solutions for medical compression garments

Product benefits

Easy to Use

- ✓ **Minimise friction** between compression garment and skin
- ✓ **Reduce physical strain** for both compression wearers and caregivers
- ✓ Promote **independence** of compression wearers



Safe to Apply

- ✓ **Reduce risk** of skin damage; can be especially beneficial in case of high skin sensitivity or leg ulcers
- ✓ Contribute to a **longer garment lifetime** by preventing damage from fingers and nails
- ✓ Help **maintain the original compression longer** and avoiding garments from overstretching and distortion



Simple Solutions

- ✓ Offer application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- ✓ Designed for different shapes and sizes of limbs
- ✓ Support the concordance to compression therapy



Arion Application Aids for your medical compression garments																																													
Donning and Doffing Aids for medical compression stockings and tights					Donning aid for armsleeves																																								
	 MD CE		 MD CE		 MD CE																																								
Open toe					—																																								
Closed toe					—																																								
Donning (On)/ Doffing (Off)	 ON																																												
Sizes	<table><thead><tr><th>Code</th><th>Size</th><th>Shoe size EU</th></tr></thead><tbody><tr><td>79658-03</td><td>Size XS</td><td>≤ 33</td></tr><tr><td>79658-04</td><td>Size S</td><td>34 - 36</td></tr><tr><td>79658-05</td><td>Size M</td><td>37 - 38.5</td></tr><tr><td>79659-03</td><td>Size L</td><td>39 - 41</td></tr><tr><td>79660-02</td><td>Size XL</td><td>≥ 42</td></tr></tbody></table>		Code	Size	Shoe size EU	79658-03	Size XS	≤ 33	79658-04	Size S	34 - 36	79658-05	Size M	37 - 38.5	79659-03	Size L	39 - 41	79660-02	Size XL	≥ 42	<table><thead><tr><th>Code</th><th>Size</th><th>Shoe size EU</th></tr></thead><tbody><tr><td>79571-00</td><td>Size M</td><td>≤ 38</td></tr><tr><td>79571-20</td><td>Size L</td><td>39 - 43</td></tr><tr><td>79571-02</td><td>Size XL</td><td>≥ 44</td></tr></tbody></table> XL also suitable for extra wide feet (circumference up to 28 cm) 		Code	Size	Shoe size EU	79571-00	Size M	≤ 38	79571-20	Size L	39 - 43	79571-02	Size XL	≥ 44	<table><thead><tr><th>Code</th><th>Size</th><th>Arm Length*</th></tr></thead><tbody><tr><td>79661-04</td><td>Size M</td><td>59 - 67 cm</td></tr><tr><td>75100-01</td><td>Size L</td><td>> 67 cm</td></tr></tbody></table> *Arm length measured from the tip of the middle finger to the armpit.		Code	Size	Arm Length*	79661-04	Size M	59 - 67 cm	75100-01	Size L	> 67 cm
Code	Size	Shoe size EU																																											
79658-03	Size XS	≤ 33																																											
79658-04	Size S	34 - 36																																											
79658-05	Size M	37 - 38.5																																											
79659-03	Size L	39 - 41																																											
79660-02	Size XL	≥ 42																																											
Code	Size	Shoe size EU																																											
79571-00	Size M	≤ 38																																											
79571-20	Size L	39 - 43																																											
79571-02	Size XL	≥ 44																																											
Code	Size	Arm Length*																																											
79661-04	Size M	59 - 67 cm																																											
75100-01	Size L	> 67 cm																																											
Video instructions: scan the QR code to watch the video	 		 		 																																								
Care instructions	 Wash by hand (30°C)	 Do not Bleach	 Tumble dry	 Dry with a towel	 Do not Iron	 Do not dry clean																																							



JOBST® DONNING GLOVES



JOBST® Donning Gloves help create friction and provide a better grip when you put your garments on. They support an even distribution of the fabric of your compression garments on the leg. Wearing donning gloves also reduces the risk of damage to the fabric.

Extra Tip

Besides donning aids and accessories, there are helpful tips and tricks for easier application. Make sure that your skin is dry before putting on your garments. Avoid using any greasy ointments or body lotions prior to applying compression stockings.

Supplied as a pair.

XS	S	M	L	XL	Colour
77400-27	77400-28	77400-29	77400-30	77400-31	● Blue

DO NOT:



Tumble dry



Bleach



Iron



Use softeners



Dry clean

JOBST® ROLL-ON ADHESIVE

An adhesive lotion to help hold garments in place

Suitable for daily use.

A single application with the roll-on-ball on the exposed skin around the arm or leg is sufficient. Then the edge of the garment is folded up and pressed against the skin.

For garments with a silicone edge, the adhesive lotion should not be applied directly under it.

Cleaning instructions: Remnants of the adhesive lotion can be easily washed off the skin or garment with water after removing the garment.

Supplied as a box of 12.

Product code: 79274-11



JOBST® GENERAL CARE INSTRUCTIONS

Washing Instructions

- It is recommended to wash JOBST® compression garments on a daily basis, or at least every other day.
- Use mild liquid detergent and a low spin cycle setting.
- Machine washable in warm water.
- Do not use laundry additives such as fabric softener, optical brightener or stain remover.
- Rinse compression garments well.
- Roll up the wet compression garments in a towel after washing and press out excess moisture. Do not wring.

For garments with a silicone band:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign materials.

Drying Instructions

- Dry JOBST® UltraSheer, JOBST® Travel and JOBST® UlcerCARE™ liner either indoors or in the shade, laid flat
- JOBST® forMen, ActiveWear, Sport, Activa soSoft, Relief, Maternity Opaque and UlcerCARE™ outer stocking can be tumble dried at low temperature
- Do not wring



Do not bleach



Do not iron



Do not dry clean

Please refer to the product label and / or package insert for full instructions on the safe use of these products.

Always read the label and follow the directions for use.

Fibre Content

FABRIC/CCL	Nylon (Polyamide)	Spandex (Elastane)	Silicone	Polyester	Other
JOBST® UltraSheer 15-20mmHg KH	75%	25%	-	-	-
JOBST® UltraSheer 15-20mmHg TH (Dotted Band)	66%	19%	15%	-	-
JOBST® UltraSheer 15-20mmHg THW (Sensitive)	75%	21%	4%	-	-
JOBST® UltraSheer 15-20mmHg WH	72%	28%	-	-	-
JOBST® UltraSheer 20-30mmHg KH	74%	26%	-	-	-
JOBST® UltraSheer 20-30mmHg TH (Dotted Band)	64%	24%	12%	-	-
JOBST® UltraSheer 20-30mmHg THW (Sensitive)	71%	26%	3%	-	-
JOBST® UltraSheer 20-30mmHg WH	73%	27%	-	-	-
JOBST® forMen 15-20mmHg KH (SoftFit)	72%	26%	2%	-	-
JOBST® forMen 15-20mmHg TH (Micro Dots)	75%	20%	5%	-	-
JOBST® forMen 20-30mmHg KH (SoftFit)	69%	29%	2%	-	-
JOBST® forMen 20-30mmHg TH (Micro Dots)	70%	26%	4%	-	-
JOBST® forMen 30-40mmHg KH (SoftFit)	67%	31%	2%	-	-
JOBST® forMen 30-40mmHg TH (Micro Dots)	66%	30%	4%	-	-
JOBST® Relief 15-20mmHg KH	80%	20%	-	-	-
JOBST® Relief 15-20mmHg TH (Dotted Band)	74%	18%	8%	-	-
JOBST® Relief 20-30mmHg KH CT	77%	23%	-	-	-
JOBST® Relief 20-30mmHg KH OT	75%	25%	-	-	-
JOBST® Relief 20-30mmHg TH (Dotted Band)	70%	23%	7%	-	-
JOBST® Relief 20-30mmHg / 30-40mmHg WH / CHAP OT	78%	22%	-	-	-
JOBST® Relief 20-30mmHg WH CT	75%	25%	-	-	-
JOBST® Relief 30-40mmHg KH CT	72%	28%	-	-	-
JOBST® Relief 30-40mmHg KH OT	70%	30%	-	-	-
JOBST® Relief 30-40mmHg TH (Dotted Band)	66%	28%	6%	-	-
JOBST® ActiveWear 15-20mmHg / 20-30mmHg	20%	9%	-	60%	11% Cotton
JOBST® ActiveWear 30-40mmHg	20%	12%	-	58%	10% Cotton
JOBST® Sport 15-20mmHg	60%	10%	-	30%	-
JOBST® Maternity Opaque 15-20mmHg	80%	20%	-	-	-
JOBST® Maternity Opaque 20-30mmHg	78%	22%	-	-	-
JOBST® Travel 15-20mmHg	55%	-	-	-	12% Elastan + 33% Cotton
JOBST® ACTIVA soSoft 8-15mmHg	92%	8%	-	-	-
JOBST® Elvarex® Soft 15-21mmHg	84%	16%	-	-	-
JOBST® Elvarex® Soft 23-32mmHg	80%	20%	-	-	-
JOBST® Elvarex® Soft 34-46mmHg	76%	24%	-	-	-
JOBST® UlcerCARE Liner	79%	16%	-	-	5% Silk
JOBST® UlcerCARE Outer Stocking (Zipper)	76%	24%	-	-	-
JOBST® Comprinet pro KH	80%	20%	-	-	-
JOBST® Comprinet pro TH	77%	19%	-	-	4% Neoprene

KH Knee High | **TH** Thigh High | **WH** Waist High | **CT** Closed Toe | **OT** Open Toe

Top bands contain silicone where indicated.

Caution: Products labelled on the packaging with  contain natural rubber (latex) which may cause allergic reactions.



JOBST®,
an Essity brand

Always read the
label and follow the
directions for use.

ESS0804
08/2026

Essity Health & Medical

PO Box 337, Mount Waverley,
Victoria, Australia 3149
medical.essity.com.au
jobst.com.au

JOBST® Customer Service AU
T 1300 998 810 F 1300 998 820
JOBST® Customer Service NZ
T 0508 998 810 F 0508 998 820