

Wound Care Product Catalogue

Solutions for
Health Care Providers



WOUND CARE

FIXATION - FILMS, TAPES AND BANDAGES

Hypafix® / Cover-Roll® Stretch – Non-Woven Adhesive Fixation Sheet	5
Hypafix® transparent – Waterproof Adhesive Transparent Film in Roll	6
Hypafix® skin sensitive – Soft and Secure Fixation for Fragile or Sensitive Skin	7
Leukomed® T – Waterproof Adhesive Transparent Sterile Film	7
Leukomed® IV – Waterproof Adhesive Transparent Sterile Film for IV	8
Leukoplast® Slek / Slek LF – Zinc Oxide Plastic Waterproof Adhesive Tape	9
Elastomull® – Non-Adhesive Fixation Bandage	9
Easifix® – Superior Non-Adhesive Fixation Bandage	10
Easifix® Cohesive – Self-Adhesive Fixation Bandage	10

POST-OP DRESSINGS

Leukomed® – Non-Woven Adhesive Sterile Dressing with Absorbent Pad	11
Leukomed® skin sensitive – For Wound Protection on Fragile or Sensitive Skin	12
Leukomed® T plus – Waterproof Adhesive Transparent Sterile Dressing with Absorbent Pad	13
Leukomed® Control – Transparent Post-Op Dressing with Absorbent Hydropolymer Gel Pad	13
Leukomed® Sorbact® – Antimicrobial Waterproof Transparent Sterile Dressing with Absorbent Pad	14

FIRST AID DRESSINGS

Leukoplast® elastic – Stretchable Lightweight Fabric Dressing	14
Leukoplast® soft white – Non-Woven Adhesive Dressing in Roll Format	15
Leukoplast® strong – Heavyweight Fabric Dressing	15
Leukoplast® barrier – Waterproof & Bacteria Proof Plastic Dressing	16
Leukoplast® universal – Water-Resistant Plastic Dressing	16

IMPREGNATED WOUND CONTACT LAYERS

Cuticell® Classic – Non-Adherent Dressing Impregnated with Paraffin	17
Cuticell® – Non-Adherent Dressing Impregnated with Non-Medicated Ointment	17
Cuticell® Contact – Silicone Wound Contact Layer	18

DEBRIDEMENT

Cutimed® DebriClean – Debridement Pad	18
---------------------------------------	----

ANTIMICROBIAL DRESSINGS

Cutimed® Sorbact® – DACC Antimicrobial Dressing	19
Hydrofera Blue® – Non-Cytotoxic, Antibacterial Wound Dressing	21

ABSORBENT DRESSINGS

Cutimed® Siltec – Foam Dressing with Silicone Wound Contact Layer	23
Cutimed® Sorbion® – Super-Absorbent Wound Dressing	24
Cutimed® Cavity – Absorbent Foam Dressing for Deep Wounds	25

PRESSURE OFFLOADING

Cutimed® Total Contact Cast Kit – Pressure Offloading Cast	25
--	----

WOUND CLOSURE

Leukoplast® Wound Closure Strip – Adhesive Closure Strips	26
---	----

HYDROGEL

Cutimed® Gel – Transparent Amorphous Hydrogel	26
---	----

HYPAFIX® / COVER-ROLL® STRETCH

NON-WOVEN ADHESIVE FIXATION SHEET



Hypafix® / Cover-Roll® stretch are indicated for wide-area fixation and widely used in place of surgical tapes to secure and retain primary dressings on surgical or wound sites. They provide light compression without causing constriction for joints and highly contoured parts of the body.

INDICATIONS

- Fixation of primary dressing fabric on small to large surgical or wound sites
- Ideal use for areas of body requiring flexibility such as elbows and knees
- Ideal base coverage for areas applying the patello-femoral taping technique

FEATURES AND BENEFITS

- Soft, stretchable non-woven polyester adapts well to the body
- Breathable
- Provides light compression without causing constriction
- Skin friendly adhesive holds dressing safely, even in the presence of moisture
- Easily cut to eliminate waste
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Case Contents	Selling Unit
Hypafix®				
2.5 cm x 10 m	7144300	1 roll	-	Box
5 cm x 10 m	7144301	1 roll	-	Box
10 cm x 10 m	7144302	1 roll	-	Box
15 cm x 10 m	7144303	1 roll	-	Box
20 cm x 10 m	7144304	1 roll	-	Box
Cover-Roll® Stretch				
5 cm x 1.8 m	4554700	1 roll	36 boxes	Case
10 cm x 1.8 m	4554800	1 roll	12 boxes	Case
15 cm x 1.8 m	4554900	1 roll	12 boxes	Case
20 cm x 1.8 m	4555000	1 roll	12 boxes	Case
30 cm x 1.8 m	4555100	1 roll	12 boxes	Case
5 cm x 9.2 m	4555200	1 roll	12 boxes	Case
10 cm x 9.2 m	4555300	1 roll	12 boxes	Case
15 cm x 9.2 m	4555400	1 roll	12 boxes	Case
20 cm x 9.2 m	4555500	1 roll	12 boxes	Case
30 cm x 9.2 m	4555600	1 roll	12 boxes	Case

HYPAFIX® TRANSPARENT

WATERPROOF ADHESIVE TRANSPARENT FILM IN ROLL



The innovative waterproof film of Hypafix® transparent is very thin and stretchable while maintaining superior strength and durability. It also feels soft and adapts well to the body contours like a second skin.

INDICATIONS

- Skin protection against friction and shearing
- Fixation of dressings, catheters, cannulae, patches and ointment

FEATURES AND BENEFITS

- Easy to apply with the unique red stripe system
- Waterproof
- Permeable to water vapour and oxygen
- Completely transparent for easy inspection without removing dressing
- Convenient roll format
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm x 10 m	7237800	1 roll	Box
10 cm x 10 m	7237801	1 roll	Box
15 cm x 10 m	7237802	1 roll	Box



HYPAFIX® SKIN SENSITIVE

SOFT & SECURE FIXATION FOR
FRAGILE OR SENSITIVE SKIN



Hypafix® skin sensitive is designed especially for use on patients with fragile or compromised skin. The soft silicone adhesive helps to protect against adhesive-related injuries.

INDICATIONS

- Ideal for aging, cortisone, and parchment skin and for neonatal care
- Skin protection against friction and shearing
- Fixation of dressings, catheters, cannulae, patches and ointment

FEATURES AND BENEFITS

- Soft, silicone, skin-friendly adhesive
- Atraumatic removal
- Easy repositioning
- Not made with natural rubber latex
- Convenient roll format

Dimensions	Code	Box Contents	Selling Unit
5 cm x 5 m	7996601	1 roll	Box
10 cm x 5 m	7996602	1 roll	Box
10 cm x 2 m	7996604	1 roll	Box

LEUKOMED® T

WATERPROOF ADHESIVE
TRANSPARENT STERILE FILM



Leukomed® T is a sterile, breathable, waterproof, transparent film dressing designed to cover catheter or wound sites while providing a barrier to external contaminants such as bacteria, blood and body fluids.

INDICATIONS

- Superficial wounds
- Post-surgical incisions
- Sterile fixation of wound dressings

FEATURES AND BENEFITS

- Easy to apply with the unique red stripe system
- Rounded edges prevent dressing from curling up
- Waterproof and barrier to bacteria
- Permeable to water vapour and oxygen
- Completely transparent for easy inspection without removing dressing
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
Hospital Pack			
7.2 cm x 5 cm	7238100	50 dressings	Box
8 cm x 10 cm	7238101	50 dressings	Box
10 cm x 12.5 cm	7238108	50 dressings	Box
11 cm x 14 cm	7238102	50 dressings	Box
10 cm x 25 cm	7238114	50 dressings	Box
15 cm x 20 cm	7238109	50 dressings	Box
15 cm x 25 cm	7238110	50 dressings	Box
Retail Pack			
7.2 cm x 5 cm	7238103	5 dressings	Box
8 cm x 10 cm	7238104	5 dressings	Box
10 cm x 12.5 cm	7238111	5 dressings	Box
11 cm x 14 cm	7238105	5 dressings	Box
15 cm x 20 cm	7238112	5 dressings	Box
15 cm x 25 cm	7238113	5 dressings	Box

LEUKOMED® IV

WATERPROOF ADHESIVE TRANSPARENT STERILE FILM FOR I.V.



When it comes to safety and reliability, choose Leukomed® IV film for securing I.V. catheters and cannulae. This I.V. dressing is completely transparent, bacteria proof and offers excellent adhesive properties.

INDICATIONS

- Fixation of cannulae and peripheral venous catheters
- Puncture sites

FEATURES AND BENEFITS

- Same advantages as Leukomed® T
- V-shaped slit makes fixation easier (except child size)
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
6 cm x 8 cm	7239000	50 dressings	Box
7 cm x 9 cm	7239005	50 dressings	Box
8.5 cm x 11.5 cm (oval)	7239003	50 dressings	Box
4.5 cm x 4.5 cm (child)	7239004	50 dressings	Box



LEUKOPLAST® SLEEK / SLEEK LF

ZINC OXIDE PLASTIC WATERPROOF ADHESIVE TAPE



Often referred to as pink tape, Leukoplast® SLEEK is indicated for situations where a reliable and conformable waterproof tape is required. Leukoplast® SLEEK is easy to unroll, easy to apply and can stay on throughout length of treatment.

INDICATIONS

- Fixation of dressings

FEATURES AND BENEFITS

- Strong & reliable
- Slight stretch allows the tape to mold to body contours
- Antimicrobial spools
- Not made with natural rubber latex (Leukoplast® SLEEK LF only)

Dimensions	Code	Box Contents	Selling Unit
Leukoplast® SLEEK			
2.5 cm x 3 m	7237907	12 rolls	Box
5 cm x 3 m	7237906	6 rolls	Box
Leukoplast® SLEEK LF			
2.5 cm x 3 m	7235906	12 rolls	Box
5 cm x 3 m	7235907	6 rolls	Box
2.5 cm x 3 m	7235908	10 spools	Pack
5 cm x 3 m	7235909	5 spools	Pack

ELASTOMULL®

NON-ADHESIVE FIXATION BANDAGE



Elastomull® offers a precision fit, even in areas where there is constant movement such as the elbow or knee.

INDICATIONS

- Fixation of dressings

FEATURES AND BENEFITS

- Elastic
- Air-permeable
- Comfortable to wear
- Not made with natural rubber latex

Dimensions*	Code	Box Contents	Selling Unit
4 cm x 4 m	2099	20 rolls	Box
6 cm x 4 m	2100	20 rolls	Box
8 cm x 4 m	2101	20 rolls	Box
10 cm x 4 m	2102	20 rolls	Box
12 cm x 4 m	2103	20 rolls	Box

*Stretched

EASIFIX®

SUPERIOR NON-ADHESIVE FIXATION BANDAGE



Easifix® is a durable fixation bandage, highly elastic and very comfortable. Easifix® helps to secure fixation even for awkward physical contours; it causes no restriction to functional movement or blood circulation.

INDICATIONS

- Fixation of dressings

FEATURES AND BENEFITS

- Light, soft and permits air circulation
- Skin friendly
- Autoclavable
- Durable elasticity, high recovery strength
- Not made with natural rubber latex

Dimensions*	Code	Box Contents	Selling Unit
2.5 cm x 4 m	7196101	20 rolls	Bag
5 cm x 4 m	7142910	20 rolls	Bag
7.5 cm x 4 m	7143003	20 rolls	Bag
10 cm x 4 m	7142804	20 rolls	Bag
15 cm x 4 m	7142911	20 rolls	Bag

*Stretched

EASIFIX® COHESIVE

SELF-ADHESIVE FIXATION BANDAGE



Easifix® Cohesive has remarkable body moulding properties and conforms to awkward bony prominences without impairing movement. The cohesive coating ensures that the bandage sticks only to itself and not the patient's skin or clothing. The bandage can be reapplied to the same patient, thus permitting routine wound inspection.

INDICATIONS

- Fixation of dressings
- Retention of splinting system
- Retention of compression bandages

FEATURES AND BENEFITS

- Air permeable
- Economical 20 m roll format
- Lightweight construction
- Conforms well to different body areas

Dimensions*	Code	Box Contents	Selling Unit
4 cm x 4 m	7143609	10 rolls	Box
6 cm x 4 m	7143610	10 rolls	Box
8 cm x 4 m	7143611	10 rolls	Box
10 cm x 4 m	7143612	10 rolls	Box
12 cm x 4 m	7143613	10 rolls	Box
4 cm x 20 m	7143614	1 roll	Box
6 cm x 20 m	7143615	1 roll	Box
8 cm x 20 m	7143616	1 roll	Box
10 cm x 20 m	7143617	1 roll	Box
12 cm x 20 m	7143618	1 roll	Box

*Stretched

LEUKOMED®

NON-WOVEN ADHESIVE STERILE DRESSING WITH ABSORBENT PAD



Leukomed® adhesive dressing is composed of an acrylic adhesive and non-woven substrate with a highly absorbent wound pad for secreting wounds. The adhesive which is not made with natural rubber latex, enables easy removal, and the non-adherent layer prevents adhesion to the wound.

INDICATIONS

- Post-surgical incisions
- Moderate to heavy drainage wounds
- Acute wounds like minor cuts, minor burns and abrasions

FEATURES AND BENEFITS

- Rounded edges prevent dressing from curling up
- Breathable
- Good absorption capacity
- Non-adherent wound pad doesn't stick to wound
- Skin friendly acrylic adhesive
- Highly conformable with two-way stretch
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
Hospital Pack			
7.2 cm x 5 cm	7238000	50 dressings	Box
8 cm x 10 cm	7238001	50 dressings	Box
8 cm x 15 cm	7238002	50 dressings	Box
10 cm x 20 cm	7238003	50 dressings	Box
10 cm x 25 cm	7238004	50 dressings	Box
10 cm x 30 cm	7238005	50 dressings	Box
10 cm x 35 cm	7238006	50 dressings	Box
Retail Pack			
7.2 cm x 5 cm	7238007	5 dressings	Box
8 cm x 10 cm	7238008	5 dressings	Box
8 cm x 15 cm	7238009	5 dressings	Box
10 cm x 20 cm	7238010	5 dressings	Box
10 cm x 25 cm	7238011	5 dressings	Box
10 cm x 30 cm	7238012	5 dressings	Box
10 cm x 35 cm	7238013	5 dressings	Box



LEUKOMED® SKIN SENSITIVE

ABSORBENT DRESSING FOR
WOUND PROTECTION ON
FRAGILE OR SENSITIVE SKIN



Leukomed® skin sensitive solutions combine atraumatic, almost pain-free removal with reliable adhesion. The reason is the use of silicone instead of acrylic or even rubber. Silicone offers great adhesion, but doesn't "glue" to the skin the way traditional adhesives do.

INDICATIONS

- Post-surgical incisions
- Ideal for aging, cortisone, fragile or parchment skin and for neonatal care
- Acute wounds like minor cuts, minor burns and abrasions

FEATURES AND BENEFITS

- Rounded edges prevent dressing from curling up
- Skin friendly with atraumatic removal
- Covers and protects wounds, such as closed surgical wounds, cuts and abrasions
- Absorbs blood and wound fluid
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm x 7.2 cm	7123904	5 dressings	Box
8 cm x 10 cm	7123905	5 dressings	Box
8 cm x 15 cm	7123906	5 dressings	Box
10 cm x 25 cm	7123907	5 dressings	Box



LEUKOMED® T PLUS

**WATERPROOF ADHESIVE
TRANSPARENT STERILE
DRESSING WITH ABSORBENT PAD**



Leukomed® T plus is a sterile, breathable, waterproof, transparent film dressing with a highly absorbent wound pad for secreting wounds.

INDICATIONS

- Post-surgical incisions
- Moderate to heavy drainage wounds
- Acute wounds like minor cuts, minor burns and abrasions

FEATURES AND BENEFITS

- Easy to apply with the unique red stripe system
- Rounded edges prevent dressing from curling up
- Waterproof and barrier to bacteria
- Permeable to water vapour and oxygen
- Convenient, all-in-one sterile dressing
- Good absorption capacity
- Non-adherent wound pad doesn't stick to wound
- Sterile and not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
Hospital Pack			
7.2 cm x 5 cm	7238200	50 dressings	Box
8 cm x 10 cm	7238201	50 dressings	Box
8 cm x 15 cm	7238202	50 dressings	Box
10 cm x 20 cm	7238214	50 dressings	Box
10 cm x 25 cm	7238203	50 dressings	Box
10 cm x 30 cm	7238204	50 dressings	Box
10 cm x 35 cm	7238205	50 dressings	Box
Retail Pack			
7.2 cm x 5 cm	7238206	5 dressings	Box
8 cm x 10 cm	7238207	5 dressings	Box
8 cm x 15 cm	7238208	5 dressings	Box
10 cm x 20 cm	7238215	5 dressings	Box
10 cm x 25 cm	7238209	5 dressings	Box
10 cm x 30 cm	7238210	5 dressings	Box
10 cm x 35 cm	7238211	5 dressings	Box

LEUKOMED® CONTROL

**FULLY TRANSPARENT
POST-OP DRESSING**



Leukomed® Control is a sterile, breathable, waterproof, transparent, conformable film that enables assessment of visible signs of infection while reducing the need for unnecessary dressing changes.

INDICATIONS

- Post-surgical incisions
- Post-operative dehisced wounds and lacerations
- Acute wounds like minor cuts, minor burns and abrasions

FEATURES AND BENEFITS

- Creates optimal moisture level and adapts to skin's elasticity
- Innovative hydropolymer technology that manages exudate
- Fully transparent to help monitor for early detection of wound infection
- Convenient, all-in-one sterile dressing
- Good absorption capacity
- Skin friendly acrylic adhesive, secure yet easy to remove
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm x 5 cm	7323000	10 dressings	Box
7 cm x 10 cm	7323001	10 dressings	Box
8 cm x 15 cm	7323002	10 dressings	Box
10 cm x 24 cm	7323003	5 dressings	Box
10 cm x 35 cm	7323005	5 dressings	Box

LEUKOMED® SORBACT®

ANTIMICROBIAL WATERPROOF TRANSPARENT STERILE DRESSING WITH ABSORBENT PAD



If surgical circumstances or patient risk factors increase the likelihood of an infection or if the wound is already contaminated, Leukomed® Sorbact® is the right choice. Purely by its physical mode of action, it binds microbes to the pad and helps to prevent critical colonization.

INDICATIONS

- Post-surgical incisions
- Post-operative dehisced wounds and lacerations
- Acute wounds like minor cuts, minor burns and abrasions

FEATURES AND BENEFITS

- Same advantages as Leukomed® T Plus
- Helps to minimize microbial colonization
- Effectively removes bacteria from the wound
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm x 7.2 cm	7619900	20 dressings	Box
8 cm x 10 cm	7619901	20 dressings	Box
8 cm x 15 cm	7619902	20 dressings	Box
10 cm x 20 cm	7619903	20 dressings	Box
10 cm x 25 cm	7619904	20 dressings	Box
10 cm x 30 cm	7619905	20 dressings	Box
10 cm x 35 cm	7619906	20 dressings	Box

LEUKOPLAST® ELASTIC

STRETCHABLE LIGHTWEIGHT FABRIC DRESSING



Leukoplast® elastic fabric adhesive bandages are made to absorb liquid fast. Featuring an extra-large wound pad capable of absorbing 10 times its own weight in water, the 360° adhesive surrounding the island pad helps seal the afflicted area.

INDICATIONS

- Minor acute wounds like cuts, lacerations, abrasions

FEATURES AND BENEFITS

- Available in 11 different shapes and sizes
- Perfect for use on knees, elbows, chins, noses and knuckles
- Elastic fabric moves with the curves and contours of body
- Sterile (except for toe shield and eye occluder)
- Not made with natural rubber latex (except for eye occluder)

	Dimensions	Code	Box Contents (dressings)	Selling Unit
	1.9 cm x 7.6 cm	7645617	100	Box
	2.5 cm x 7.6 cm	7645606	100	Box
	2.2 cm	7645616	100	Box
	2.5 cm x 3.1 cm	7645613	100	Box
	3.8 cm x 5 cm	7645612	100	Box
	5 cm x 7.6 cm	7645610	50	Box
	3.8 cm x 7.6 cm Knuckle	76456070	100	Box
	6.3 cm x 3.5 cm Toe	7645618	100	Box
	3.8 cm x 5.4 cm Fingertip	7645608	100	Box
	5 cm x 6.3 cm Fingertip	7645609	50	Box
	7.6 cm x 7.6 cm	7645611	50	Box
	4.8 cm x 6.3 cm Child eye	7646700	20	6 boxes
	5 cm x 7.5 cm Adult eye	7646600	20	6 boxes

LEUKOPLAST® SOFT WHITE

NON-WOVEN ADHESIVE DRESSING IN ROLL FORMAT



Leukoplast® soft white is made of highly flexible polyester to allow full freedom of movement, let the skin breathe and dry quickly if wet. Comfortable for the patient, it can stay in place for the duration of treatment.

INDICATIONS

- Minor acute wounds like cuts, lacerations, abrasions
- After blood drawing or vaccination

FEATURES AND BENEFITS

- Breathable
- Good absorption capacity
- Non-adherent wound pad doesn't stick to wound
- Skin-friendly acrylic adhesive
- Highly conformable with two-way stretch
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
4 cm x 5 m	7645000	1 roll	Box
6 cm x 5 m	7645001	1 roll	Box
8 cm x 5 m	7645002	1 roll	Box

LEUKOPLAST® STRONG

HEAVYWEIGHT FABRIC DRESSING



For flexible comfort that stretches with every movement, Leukoplast® strong is a strong and durable dressing that protects and cushions the wound, providing the practical answer to everyday minor injuries.

INDICATIONS

- Minor acute wounds like cuts, lacerations, abrasions
- After blood drawing or vaccination

FEATURES AND BENEFITS

- Lasting and durable adhesive
- Robust substrate made of 100% cotton
- Low adherent absorbent wound pad
- Breathable
- Sterile (except for roll format)

Dimensions	Code	Box Contents	Selling Unit
Roll Format			
4 cm x 5 m	7645900	1 roll	Box
6 cm x 5 m	7645901	1 roll	Box
8 cm x 5 m	7645902	1 roll	Box
Pre-Cut Format			
 2.2 cm x 7.2 cm	7646007	100 dressings	Box
 2.2 cm x 3.8 cm	7646005	100 dressings	Box
 3.8 cm x 3.8 cm	7646009	100 dressings	Box
 5 cm x 7.2 cm	7646013	100 dressings	Box
 3.8 cm x 7.5 cm Knuckle	7646010	100 dressings	Box
 4.5 cm x 5 cm Fingertip	7646012	50 dressings	Box
 4.5 cm x 7.5 cm Fingertip	7646011	100 dressings	Box
Doctor's Set	7645908	101 dressings	Box
Bulk			
 2.2 cm x 7.2 cm	7646000	7000 dressings	Case
 5 cm x 7.2 cm	7646004	4000 dressings	Case
 4.5 cm x 5 cm Fingertip	7646002	4850 dressings	Case
 4.5 cm x 7.5 cm Fingertip	7646003	4875 dressings	Case
 3.8 cm x 7.5 cm Knuckle	7646001	4875 dressings	Case

LEUKOPLAST® BARRIER

WATERPROOF & BARRIER TO BACTERIA PLASTIC DRESSING



Leukoplast® barrier is waterproof and provides a barrier to the risk of contamination and secondary bacterial infections. Patients can wash and bathe as normal without compromising the protection.

INDICATIONS

- Minor acute wounds like cuts, lacerations, abrasions
- After blood drawing or vaccination

FEATURES AND BENEFITS

- Waterproof and barrier to bacteria
- Breathable
- Skin friendly acrylic adhesive
- Sterile
- Not made with natural rubber latex

	Dimensions	Code	Box Contents	Selling Unit
	2.2 cm x 7.2 cm	7645208	100 dressings	Box
	2.2 cm x 3.8 cm	7645200	100 dressings	Box
	2.4 cm	7645203	100 dressings	Box
	3.8 cm x 3.8 cm	7645202	100 dressings	Box
	3.8 cm x 6.3 cm	7645204	100 dressings	Box
	5 cm x 7.2 cm	7645205	100 dressings	Box

LEUKOPLAST® UNIVERSAL

WATER-RESISTANT PLASTIC DRESSING



Leukoplast® universal water resistant adhesive dressing protects minor everyday injuries in wet or dirty environments. The flexible perforated film lets the skin breathe and ensures high wearing comfort.

INDICATIONS

- Minor acute wounds like cuts, lacerations, abrasions
- After blood drawing or vaccination

FEATURES AND BENEFITS

- Flexible and conformable
- Low adherent absorbent wound pad
- Water and dirt resistant
- Sterile
- Not made with natural rubber latex

	Dimensions	Code	Box Contents	Selling Unit
	1.9 cm x 7.2 cm	7646201	100 dressings	Box

CUTICELL® CLASSIC

NON-ADHERENT DRESSING IMPREGNATED WITH PARAFFIN



Cuticell® Classic is a low-adherent primary wound contact layer dressing made of open weave cotton gauze impregnated with soft paraffin to facilitate dressing change and to support a moist environment for the wound.

INDICATIONS

For superficial exuding wounds such as

- First and second-degree burns
- Lacerations and abrasions
- Donor and recipient skin graft sites

FEATURES AND BENEFITS


- Low-adherent and soothing to minimize pain at dressing change
- Supports a moist environment to help healing
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm x 5 cm	7253801	50 dressings	Box
5 cm x 5 cm	7253800	5 dressings	Box
10 cm x 10 cm	7253803	100 dressings	Box
10 cm x 10 cm	7253802	10 dressings	Box
10 cm x 40 cm	7253804	10 dressings	Box
10 cm x 7 m*	7253806	1 roll	Box
15 cm x 2 m*	7253807	12 rolls	Box

* Not individually wrapped

CUTICELL®

NON-ADHERENT DRESSING IMPREGNATED WITH NON- MEDICATED OINTMENT

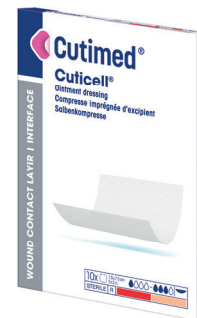

Cuticell® is a low-adherent primary wound contact layer made of smooth small meshed acetate fabric impregnated with non-medicated ointment basis. The result: a dressing that ensures the wound is not disturbed and helps maintain a moist environment that supports wound healing.

INDICATIONS

For superficial exuding wounds such as

- First and second-degree burns
- Lacerations and abrasions
- Radiation injuries
- Donor and recipient skin graft sites


FEATURES AND BENEFITS

- Low-adherent and soothing to minimize pain at dressing change
- Supports a moist environment to help healing
- Does not disturb wound healing
- Does not charge the wound with excess ointment
- Simple handling without release paper
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
7.5 cm x 7.5 cm	7253901	50 dressings	Box
7.5 cm x 7.5 cm	7253900	10 dressings	Box
7.5 cm x 20 cm	7253903	50 dressings	Box
7.5 cm x 20 cm	7253902	10 dressings	Box
20 cm x 40 cm	7253904	25 dressings	Box

CUTICELL® CONTACT

SILICONE WOUND CONTACT LAYER

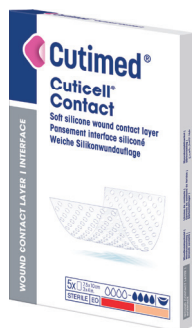

Cuticell® Contact is a silicone wound contact layer that has a perforated transparent structure and is highly conformable. Its optimally perforated structure ensures even highly viscous exudate can easily pass through.

INDICATIONS

For superficial exuding wounds such as

- First and second-degree burns
- Lacerations and abrasions
- Radiation injuries
- Donor and recipient skin graft sites

FEATURES AND BENEFITS

- Low-adherent and soothing to minimize pain at dressing change
- Supports a moist environment to help healing
- Allows exudate to drain freely into an absorbent secondary dressing
- High transparency for easy wound inspection
- Reduced risk of maceration
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
5 cm × 7.5 cm	7268000	5 dressings	Box
7.5 cm × 10 cm	7268001	5 dressings	Box
10 cm × 18 cm	7268002	5 dressings	Box
15 cm × 25 cm	7268003	5 dressings	Box

CUTIMED® DEBRICLEAN

DEBRIDEMENT PAD



Cutimed® Debriclean is a debridement pad. Due to its unique combination of gentle looped monofilament fibres and abrasive fibres, bacteria is absorbed and firm slough removed, making Cutimed® Debriclean a suitable debridement pad for wound bed preparation.

INDICATIONS

- Diabetic ulcers
- Venous and arterial ulcers
- Pressure sores
- Post-operative wounds and wounds healing by secondary intention
- Superficial minor burns
- The absorption of exudate, slough and dry skin flakes


FEATURES AND BENEFITS

- White, looped mono-filament fibres for gentle debridement
- Blue, more abrasive fibres for stronger debridement option (if required)
- Perforated adhesive layer supports absorption/rinsing
- Foam layer supports absorption / rinsing during use. Also aids with the cleaning efficiency of the pad
- Hand loop for easy, comfortable and accurate use
- Hard blister pack for ease of use when moistening and disposing

Dimensions	Code	Box Contents	Selling Unit
9 cm x 13 cm	7617000	5 dressings	Box

CUTIMED® SORBACT®

HYDROPHOBIC MICROBE BINDING ANTIMICROBIAL DRESSING

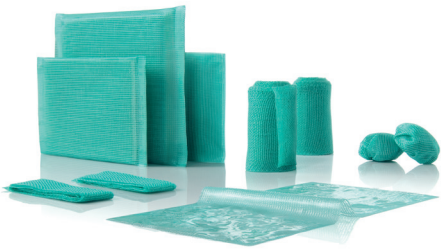

Cutimed® Sorbact® utilizes a physical mode of action versus a chemical mode of action. The hydrophobic interaction safely removes bacteria without cell death or negatively impacting the healing process. Other antimicrobial dressings on the market utilize alternative antimicrobials to kill bacteria or fungi, leaving behind bacterial cell debris and endotoxins.

INDICATIONS




For all unclean, colonized and infected exuding wounds

- Chronic wounds, such as venous, diabetic or pressure ulcers
- Postoperative and dehisced wounds
- Traumatic wounds
- Wounds after excision of fistulas and abscesses


FEATURES AND BENEFITS

- No healing delays due to bacterial death, cell debris or endotoxin release
- No known bacterial resistance has been described
- No cytotoxicity
- No known risk of allergies
- Icons on box to choose the right product
- Sterile
- Not made with natural rubber latex

The dressing surface of Cutimed® Sorbact® is coated with a highly hydrophobic fatty acid derivate which wound pathogens find very attractive. When the dressing is moist, wound bacteria and fungi are naturally drawn to the hydrophobic surface. They become irreversibly bound, unable to move back to the wound bed. The bound pathogens are removed with each dressing change, which helps clean contaminated, colonized and infected wounds by reducing the overall bacterial load.

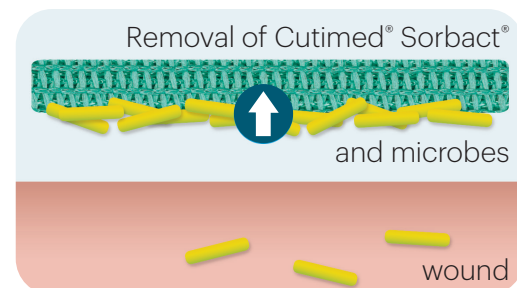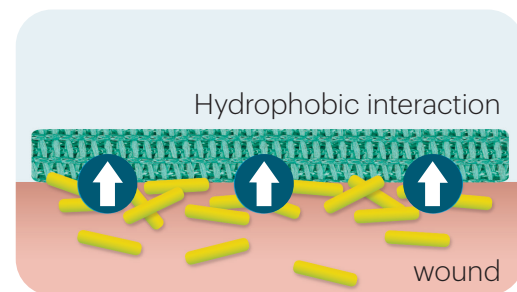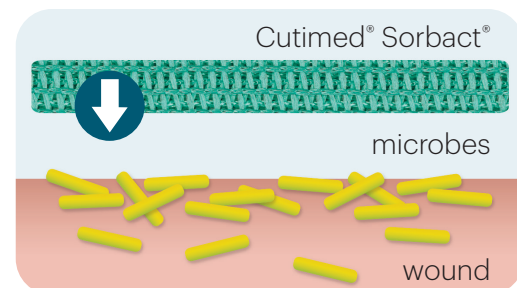


Wound Depth	Wound Phase	Exudate
☐ Superficial	☐ Necrotic	☐ Low
☐ Deep	☐ Infected	☐ Low to moderate
☐ Superficial + deep	☐ Sloughy	☐ Moderate to high
	☐ Granulation	☐ High to Excessive
	☐ Epithelialization	

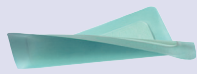Description	Dimensions	Code	Box Contents	Selling Unit
Cutimed® Sorbact® pad With absorbent core, ideal under TCC  	7 cm x 9 cm	7216101	5 dressings	Box
	7 cm x 9 cm	7216100	40 dressings	Box
	10 cm x 10 cm	7216201	5 dressings	Box
	10 cm x 10 cm	7216200	40 dressings	Box
	10 cm x 20 cm	7216300	20 dressings	Box
Cutimed® Sorbact® swab Most versatile; for any exudate level, deep or superficial wounds  	4 cm x 6 cm (11 x 16 unfolded)	7216401	5 dressings	Box
	4 cm x 6 cm (11 x 16 unfolded)	7216400	40 dressings	Box
	7 cm x 9 cm (17 x 28 unfolded)	7216501	5 dressings	Box
	7 cm x 9 cm (17 x 28 unfolded)	7216500	40 dressings	Box
Cutimed® Sorbact® ribbon For use in cavity wounds of all kinds  	2 cm x 50 cm	7216600	20 dressings	Box
	5 cm x 200 cm	7216700	10 dressings	Box
Cutimed® Sorbact® round swab For deep wound dressing or superficial swabbing  	3.5 cm (round)	7216800	70 dressings (14 x 5)	Box
Cutimed® Sorbact® gel Impregnated with hydrogel; for dry wounds  	7.5 cm x 7.5 cm	7261100	10 dressings	Box
	7.5 cm x 15 cm	7261101	10 dressings	Box
Cutimed® Sorbact® Hydroactive Absorbent hydrogel core  	7 cm x 8.5 cm	7264603	10 dressings	Box
	14 cm x 14 cm	7264604	10 dressings	Box
	19 cm x 19 cm	7264605	10 dressings	Box
	14 cm x 24 cm	7264607	10 dressings	Box
	24 cm x 24 cm	7264606	5 dressings	Box
Cutimed® Sorbact® Hydroactive B Absorbent hydrogel core and skin-friendly adhesive border  	7 cm x 8.5 cm	7993300	10 dressings	Box
	14 cm x 14 cm	7993302	10 dressings	Box
	19 cm x 19 cm	7993303	10 dressings	Box
	14 cm x 24 cm	7993304	10 dressings	Box
Cutimed® Siltec® Sorbact® Super-absorbent foam dressing with silicone adhesive border  	7.5 cm x 7.5 cm	7325100	10 dressings	Box
	10 cm x 10 cm	7325116	10 dressings	Box
	12.5 cm x 12.5 cm	7325101	10 dressings	Box
	15 cm x 15 cm	7325102	10 dressings	Box
	17.5 cm x 17.5 cm	7325103	5 dressings	Box
	22.5 cm x 22.5 cm	7325104	5 dressings	Box
	Sacrum 17.5 cm x 17.5 cm	7325105	5 dressings	Box
	Sacrum 23 cm x 23 cm	7325106	5 dressings	Box
	Oval 7 cm x 10 cm	7325118	10 dressings	Box
Cutimed® Sorbion® Sorbact® Super-absorbent core with gelling effect to retain very high levels of exudate  	10 cm x 10 cm	7269800	10 dressings	Box
	10 cm x 20 cm	7269801	10 dressings	Box
	20 cm x 20 cm	7269802	10 dressings	Box
	20 cm x 30 cm	7269803	10 dressings	Box

Hydrofera^{BLUE}

NON-CYTOTOXIC, ANTIBACTERIAL WOUND DRESSINGS

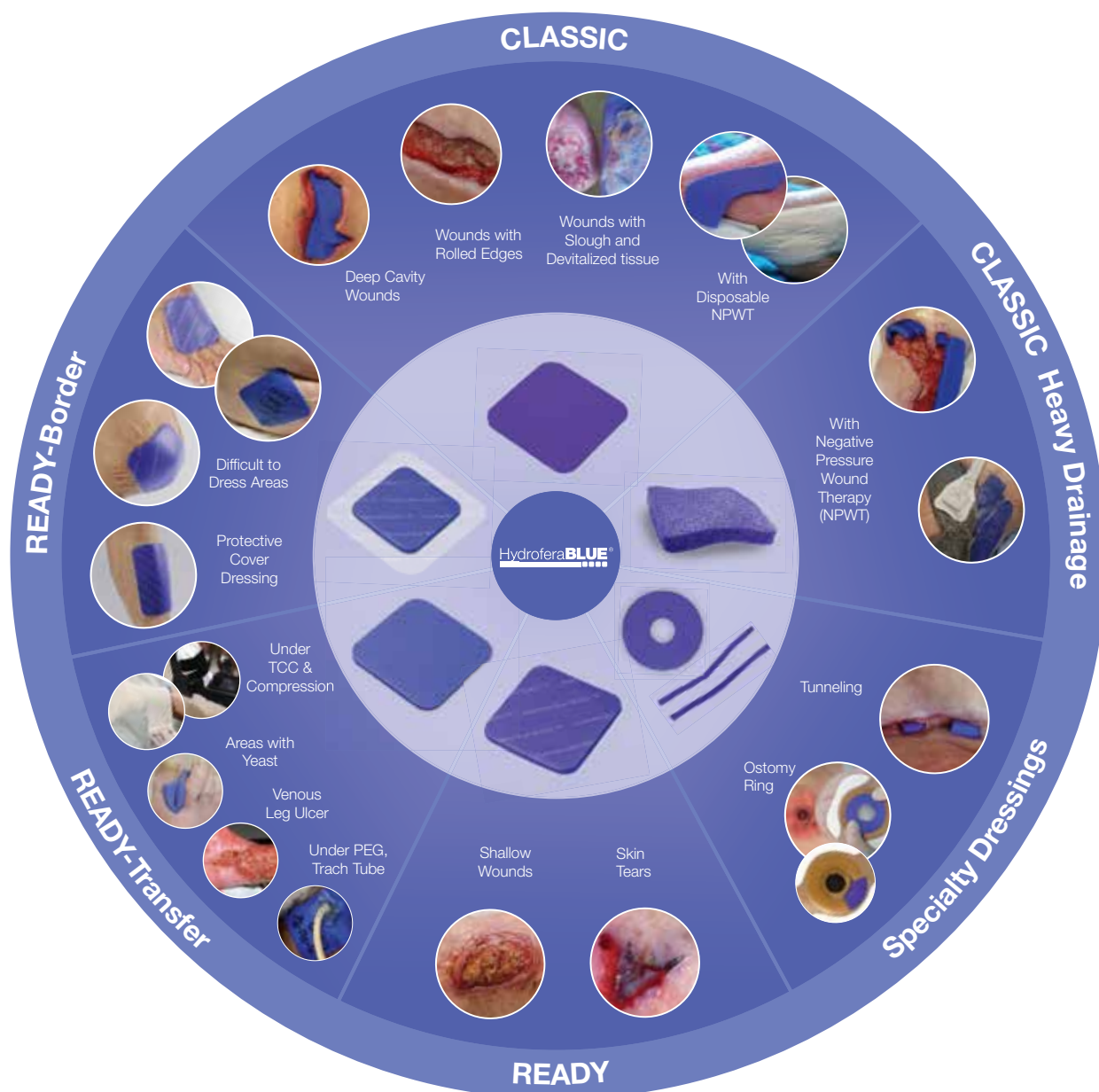

- Non-Cytotoxic
- Natural Negative Pressure*
- Natural Autolytic Debridement*

- Antibacterial
- Flattens Rolled Wound Edges*
- Disrupts Biofilm*

*CLASSIC Dressings

Ordering Information:

Hydrofera Blue Classic antibacterial dressings

**Product No**
Standard Dressings

HB2214

HB4414

HB6614

HBT0906

Product Size

2" x 2" (5 cm x 5 cm)

4" x 4" (10.2 cm x 10.2 cm)

6" x 6" (15.2 cm x 15.2 cm)

9-mm Tunneling Dressing (1.2 g)

Quantity/Box

10

10

10

10 pair

**Heavy Drainage Dressings**

HBHD4450

HBHD6675

4" x 4" x 0.5" thick (10.2 cm x 10.2 cm x 1.2 cm)

6" x 6" x 0.75" thick (15.2 cm x 15.2 cm x 1.9 cm)

5

5

**Ostomy Dressing**

HBRF2650

2.5" (6.4 cm) Diameter Ostomy Dressing

10

Hydrofera Blue READY antibacterial dressings



HBRS2520

HBRS4520

HBRS8820

2.5" x 2.5" (6.4 cm x 6.4 cm)

4" x 5" (10.2 cm x 12.7 cm)


8" x 8" (20.3 cm x 20.3 cm)

10

10

10

Hydrofera Blue READY-Transfer antibacterial dressings

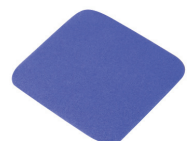

HBRT2525

HBRT4050

HBRT8080

2.5" x 2.5" (6.4 cm x 6.4 cm)

4" x 5" (10.2 cm x 12.7 cm)

8" x 8" (20.3 cm x 20.3 cm)

10

10

10

Hydrofera Blue READY-Border dressings



HBRB4040

HBRB5959

HBRB6080

4" x 4" (10.2 cm x 10.2 cm)

5.9" x 5.9" (15 cm x 15 cm)

6" x 8" (15 cm X 20.3 cm)

10

10

10

CUTIMED® SILTEC®

FOAM DRESSING WITH SILICONE WOUND CONTACT LAYER



Cutimed® Siltec® consistently provides reliable results, regardless of exudate level or viscosity. With its silicone wound contact layer and super-absorbent particles, it offers outstanding fluid management capabilities, but also offers patients key benefits for increased quality of life and compliance.

INDICATIONS

- For all exuding wounds such as venous, diabetic or pressure ulcers

FEATURES AND BENEFITS

- Reduced risk of maceration
- Atraumatic dressing changes
- Visible saturation
- Strikethrough and leakage prevention
- Comfortable wear
- Icons on box to choose the right product*
- Sterile
- Not made with natural rubber latex

Description	Dimensions	Code	Box Contents	Selling Unit
Cutimed® Siltec® With very gentle FeatherTack adherence  	5 cm x 6 cm	7328500	10 dressings	Box
	10 cm x 10 cm	7328501	10 dressings	Box
	10 cm x 20 cm	7328502	10 dressings	Box
	15 cm x 15 cm	7328503	10 dressings	Box
	20 cm x 20 cm	7328504	5 dressings	Box
	Heel 16 cm x 24 cm	7328601	5 dressings	Box
Cutimed® Siltec® PLUS With SoftTack adherence  	5 cm x 6 cm	7328800	10 dressings	Box
	10 cm x 10 cm	7328801	10 dressings	Box
	10 cm x 20 cm	7328802	10 dressings	Box
	15 cm x 15 cm	7328803	10 dressings	Box
	20 cm x 20 cm	7328804	5 dressings	Box
	Heel 16 cm x 24 cm	7328805	5 dressings	Box
Cutimed® Siltec® B (Border) With silicone adhesive border  	7.5 cm x 7.5 cm	7328400	10 dressings	Box
	10 cm x 10 cm	7328405	10 dressings	Box
	10 cm x 22.5 cm	7328406	10 dressings	Box
	12.5 cm x 12.5 cm	7328401	10 dressings	Box
	15 cm x 15 cm	7328402	10 dressings	Box
	17.5 cm x 17.5 cm	7328403	5 dressings	Box
	22.5 cm x 22.5 cm	7328404	5 dressings	Box
Cutimed® Siltec® Sacrum For sacral wounds  	17.5 cm x 17.5 cm	7328700	5 dressings	Box
	23 cm x 23 cm	7328701	5 dressings	Box
Cutimed® Siltec® L (Light) For low exuding wounds  	5 cm x 6 cm	7328300	10 dressings	Box
	10 cm x 10 cm	7328301	10 dressings	Box
	15 cm x 15 cm	7328302	10 dressings	Box
Cutimed® Siltec® Sorbact® ** Combined with Sorbact® antimicrobial technology; with skin-friendly silicone adhesive border  	** See page 20			

* See legend on page 19

CUTIMED® SORBION®

SUPER-ABSORBENT WOUND DRESSING

Cutimed® Sorbion® dressings have the excellent ability of quickly absorbing large amounts of exudate and locking it within their inner core, even under compression, therefore reducing the frequency of dressing changes. Cutimed® Sorbion® dressings also have the potential to remove chronicity-causing factors and thereby support wound healing; they provide gentle debridement, reduce the concentration and activity of MMPs and immobilize micro-organisms. The unique fiber matrix offers stability even when highly saturated, thus preventing the formation of lumps and pressure points and ensuring maximum wound contact.

INDICATIONS

- For all moderately to highly exuding wounds such as venous, diabetic or pressure ulcers

FEATURES AND BENEFITS

- High absorption capacity, long dressing wear time
- Fewer dressing changes (up to 4 days wear time)
- Securely locks in exudate, even under compression
- Removes wound debris and locks in wound germs
- Maintains a moist wound environment by gel formation
- No binders or adhesives; well tolerated
- No risk of leakage, high tear strength
- Able to apply on both sides
- Less odour formation
- Sterile
- Not made with natural rubber latex



Description	Dimensions	Code	Box Contents	Selling Unit
Cutimed® Sorbion® Sachet S The classic form suited for many wound types and areas of application  	7.5 cm x 7.5 cm	7323200	10 dressings	Box
	10 cm x 10 cm	7323206	10 dressings	Box
	10 cm x 20 cm	7323209	10 dressings	Box
	15 cm x 15 cm	7323212	10 dressings	Box
	20 cm x 20 cm	7323215	10 dressings	Box
	20 cm x 30 cm	7323218	10 dressings	Box
Cutimed® Sorbion® Sachet Multi Star Flexible form for applications on toes, heel, foot, sacrum, chest or wound cavities  	8 cm	7323701	10 dressings	Box
	14 cm	7323703	10 dressings	Box
Cutimed® Sorbion® Sachet XL For large difficult-to-dress wounds on lower leg, sacral region, thorax, thigh, lower part of abdomen  	25 cm x 45 cm	7324000	10 dressings	Box
Cutimed® Sorbion® Border Combined with adhesive border for easier application**  NEW 	10 cm x 10 cm	7101000	10 dressings	Box
	15 cm x 15 cm	7101002	10 dressings	Box
	20 cm x 20 cm	7101003	10 dressings	Box
	10 cm x 22.5 cm	7101004	10 dressings	Box
	Sacral	7101008	10 dressings	Box
Cutimed® Sorbion® Sana With particularly atraumatic surface for pain-sensitive patients  	12 cm x 12 cm	7323303	10 dressings	Box
	12 cm x 22 cm	7323306	10 dressings	Box
	22 cm x 22 cm	7323309	10 dressings	Box
Cutimed® Sorbion® Sana Multi Star With particularly atraumatic surface for pain-sensitive patients  	11 cm	7323801	10 dressings	Box
	17 cm	7323803	10 dressings	Box

* See legend on page 19

** This product not yet licensed for sale in Canada

CUTIMED® CAVITY

ABSORBENT FOAM DRESSING FOR DEEP WOUNDS



Cutimed® Cavity is highly absorbent and conformable, making it the ideal choice for deep exuding wounds.

INDICATIONS

- For all deep exuding wounds such as venous, diabetic or pressure ulcers
- Wounds after excision of fistulas and abscesses

FEATURES AND BENEFITS

- Soft and conformable
- Atraumatic
- Non-adhesive
- Easy to apply and remove
- Highly absorbent
- Icons on box to choose the right product*
- Sterile
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Selling Unit
5 cm x 6 cm	7262100	10 dressings	Box
10 cm x 10 cm	7262101	10 dressings	Box
15 cm x 15 cm	7262103	5 dressings	Box

* See legend on page 19

CUTIMED® TOTAL CONTACT CAST KITS

PRESSURE OFFLOADING CAST



Our Cutimed® Total Contact Cast Kits have been developed in partnership with healthcare professionals dealing with diabetic foot ulcers every day. The result is a recipe that offers the most effective TCC through an easy standardized technique. Our kits combine specifically chosen and proven materials to provide an intimate close fit and proper pressure redistribution for an optimized healing environment.

INDICATIONS

- Diabetic foot ulcers
- Charcot joint injuries
- Post-op immobilization following deformity reconstructions

FEATURES AND BENEFITS

- Supports increased patient compliance
- No secondary boot required
- Optimized healing environment
- Standardized technique
- Convenient all-in-one kit contains all material for one cast
- Cost-effective
- Not made with natural rubber latex

Dimensions	Code	Selling Unit
Cutimed TCC Kit (with plaster)	7349900	Box
Cutimed Off-Loader Select (no plaster)	7800901	Box

LEUKOPLAST® WOUND CLOSURE STRIP

ADHESIVE CLOSURE STRIPS



Leukoplast® Wound Closure Strip's prevalent benefits include atraumatic removal, low infection risk and good cosmetic results.

INDICATIONS

- Facial lacerations or incisions
- Thin, fragile or sensitive skin
- Wounds closed with intracutaneous sutures
- Replacement of staples and sutures

FEATURES AND BENEFITS

- Atraumatic removal
- Tension-free adaptation of wound edges
- Good cosmetic results
- Excellent tolerance
- Breathable
- Sterile
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
XS - 6 mm x 38 mm	7274000	300 strips	Box
S - 3 mm x 75 mm	7274001	250 strips	Box
M - 6 mm x 75 mm	7274002	150 strips	Box
L - 6 mm x 100 mm	7274003	500 strips	Box
XL - 12 mm x 100 mm	7274004	300 strips	Box
XXL - 25 mm x 100 mm	7274005	100 strips	Box

CUTIMED® GEL

TRANSPARENT AMORPHOUS HYDROGEL



Cutimed® Gel is a clear, amorphous hydrogel with pure formulation to reduce the risk of allergic reaction, and its superior moisture donation allows for less frequent dressing changes and effective debridement.

INDICATIONS

- To support autolytic debridement in necrotic wounds
- To provide moisture to dry wounds

FEATURES AND BENEFITS

- No preservatives
- Icons on box to choose the right product*
- Sterile
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Selling Unit
Tube 15 g	7261001	10 tubes	Box
Tube 25 g	7261002	10 tubes	Box

* See legend on page 19

Cover-Roll® Stretch	5
Cuticell® / Cuticell® Classic	17
Cuticell® Contact.....	17
Cutimed® Cavity	25
Cutimed® DebriClean	18
Cutimed® Gel	26
Cutimed® Siltec.....	23
Cutimed® Sorbact®	19
Cutimed® Sorbion®	24
Cutimed® Total Contact Cast Kit	25
Easifix® / Easifix® Cohesive.....	10
Elastomull®	9
Hydrofera Blue®	21
Hypafix®	5
Hypafix® skin sensitive	7
Hypafix® transparent	6
Leukomed®	11
Leukomed® Control	13
Leukomed® IV	8
Leukomed® skin sensitive	12
Leukomed® Sorbact®	14
Leukomed® T	7
Leukomed® T plus	13
Leukoplast® barrier	16
Leukoplast® elastic	14
Leukoplast® Sleek.....	9
Leukoplast® soft white	15
Leukoplast® strong	15
Leukoplast® universal	16
Leukoplast® Wound Closure Strip	26

