



Compression for Empowering Self-Care



JOBST® FarrowWrap®

Adjustable Wrap Compression Systems



Easy-to-use compression solutions help to promote independent self-care

Compression therapy is a vital component in the management of chronic swelling from lymphatic and venous conditions.

For certain patients, however, management can be difficult to maintain independently. It is therefore important to offer patients suitable compression solutions that are efficient and easy-to-use, to help improve therapy outcomes and empower patients to confidently care for their own needs.

Choosing the right compression¹:

- Easy to apply and remove
- Is comfortable for the patient and stays in place
- Offers variety in patient choice
- Delivers therapeutic compression and has high stiffness
- Is aesthetically acceptable

¹Harding, K., Dowsett, C., Flax, L et al. Simplifying venous leg ulcer management: consensus recommendations. Wounds International, 2015. <https://tinyurl.com/jrxfcj> (accessed 26 February 2017).
Properties of an ideal compression system.

Innovative solutions designed for improved self-care: **adjustable wrap compression systems**

Give your patients the choice and variety needed to take control of their own care and support them to achieve better therapy outcomes with adjustable wrap compression systems.

In oedema management, adjustable wrap compression systems (or compression wraps) can be used as a first-choice treatment option and offer an alternative to multi-layer bandaging or hosiery.

They have similar short-stretch properties to multi-layer bandaging in that they have low resting pressure and a high working pressure. As a result, they may create peak pressures that stimulate lymph flow. They are designed for easy use which enables patients to independently self-apply and adjust as their swelling reduces.


Which of your patients would benefit from a **wrap compression system**?



Chronic Oedema

Patients with swelling resulting from lymphoedema and venous oedema:

- With fluctuations in limb swelling
- In decongestive and / or maintenance phases

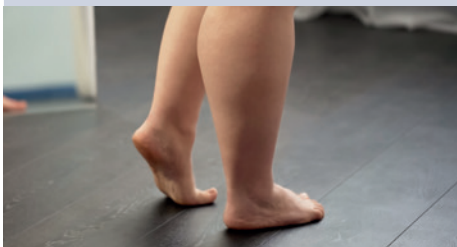

Larger body sizes

Patients unable to don or unsuitable for compression garments / bandages through:

- Obesity
- Overweight or larger body sizes
- Irregular shape of limbs
- Skin folds

Venous leg ulcer

Patients with open wounds between the knee and the ankle:

- Enables patient to change dressings as required
- Allows patient to remove to carry out personal hygiene routines

Physical limitations

Patients unable to apply or remove compression hosiery / bandaging due to:

- Arthritis or weak hand strength
- Back problems
- Sensitive or fragile skin at risk of breakdown
- Relying on carer-support

JOBST® FarrowWrap®

Easy-to-attach VELCRO® brand hook and loop support easy put on and take off.



Easy Donning

Short-stretch technology provides low resting and high working pressures to enhance lymphatic and venous return.



Short Stretch Technology

VELCRO® brand hook and loop system allows patients to respond better to fluctuating oedema by simply tightening or loosening the wrap.



Adjustable

Discreet design that's less bulky than multi-layer bandaging and can be worn under normal clothing.



Low Profile Design


* Overlapping bands not on JOBST FarrowWrap 4000

² Observational Study JOBST® FarrowWrap® 2019, data on file

The **easy choice** to support patient **self-care** and **empowerment**

Give your patients the tools they need to better manage their swelling and respond confidently to fluctuating oedema levels, with a solution that takes the guesswork out of applying compression safely.

JOBST FarrowWrap garments are a range of adjustable wrap compression systems that are available in a variety of styles, fabrics, sizes and compression classes. Made with durable materials, they are designed to help reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers. They are designed to provide the same strong support as multi-layer bandaging but in a simplified, intuitive design.

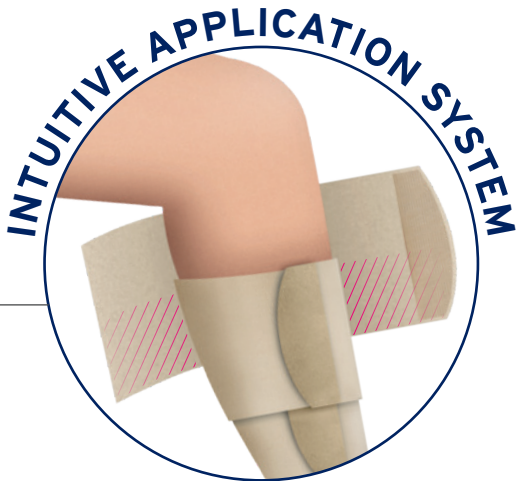

Overlapping bands replicate the principles of multi-layer bandaging.

Apply and adjust **confidently**

JOBST FarrowWrap was engineered for easy application and can offer all the benefits of multilayer bandaging without the complexity of application. The overlapping material* supports consistent self-application without the need for specialised intervention by health care professionals. The overlap provides the support needed to efficiently control oedema and help reduce the risk of swelling forming between the bands.

JOBST® FarrowWrap® Lite

Made with a double laminated fabric which is durable and soft. Contains a lower level of compression than other JOBST FarrowWrap products and is suitable for patients with **mild to moderate** oedema.

- Support patient when reduced compression is required
- Can support patients with sensitive skin
- Liner included



Colours



Tan



Black
(only available
for armpiece)

Compression level

20 - 30 mmHg

Styles available on Drug Tariff



Footpiece



Legpiece



Kneepiece



Thighpiece*



Armpiece



Ambidextrous
Hand Gauntlet

*Thighpiece comes with knee piece

Garment Care



Machine
wash 30°



Do not
bleach



Do not
tumble dry



Lay flat
to dry



Do not
iron



Do not
dry clean

JOBST® FarrowWrap® Strong

Made from durable fabric with a soft inner layer.
Recommended for patients with **moderate to severe oedema**.

- Suitable for skin folds and shape distortion
- Reliable hold for fluctuating oedema
- Liner included



Colours



Tan

Compression level

30 - 40 mmHg

Styles available on Drug Tariff



Footpiece



Legpiece



Kneepiece



Thighpiece*

*Thighpiece comes with knee piece

Garment Care



Machine
wash 30°



Do not
bleach



Do not
tumble dry



Lay flat
to dry



Do not
iron



Do not
dry clean

JOBST® FarrowWrap® Classic

Made from more rigid material than JOBST FarrowWrap Strong and recommended for patients with **moderate to severe** oedema.

- Suitable for irregular shaped limbs and deep skin folds
- Designed to sit flat against skin folds and not dig into the skin
- Liner included



Colours



Tan

Compression level

30 - 40 mmHg

Styles available on Drug Tariff



Footpiece



Legpiece



Thighpiece*

*Thighpiece comes with knee piece

Garment Care



Hand wash in
warm water



Do not
bleach



Do not
tumble dry



Lay flat
to dry



Do not
iron



Do not
dry clean




JOBST® FarrowWrap® 4000


Made with an inner sleeve to aid donning. Ideal for patients with **moderate to severe** venous and lymphatic conditions with or without a **venous leg ulcer** present.

- Designed for easy application with just four bands
- Helps promote healing of venous ulceration
- Comes with a JOBST FarrowHybrid compression sock (20-30mmHg wide)

Note: Open wounds must be covered with an appropriate dressing before the compression garment is applied.

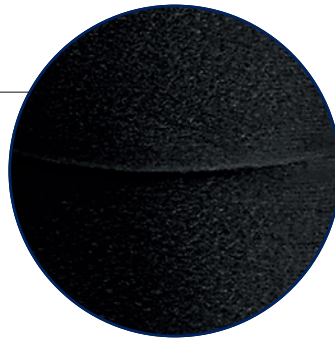

Colours

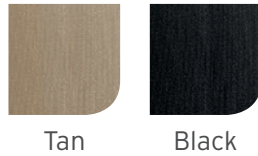

Tan

Black

Compression level

30 - 40 mmHg

Styles available on Drug Tariff



Legpiece

Garment Care



Machine wash 30°



Do not bleach



Do not tumble dry



Lay flat to dry



Do not iron



Do not dry clean

When you order a JOBST FarrowWrap 4000 you will receive 1 x JOBST FarrowHybrid (20-30mmHg)


JOBST FarrowHybrid	X-small	Small	Medium	Large
Size received	Medium-wide	Medium-wide	Large-wide	Large-wide*

*Do not apply JOBST FarrowHybrid if calf circumference exceeds 60 cm. When a large JOBST FarrowWrap 4000 is ordered, an additional non-compressive liner will be provided to accommodate calf circumference greater than 60 cm. Consider a JOBST FarrowWrap footpiece in this circumstance.

JOBST® FarrowHybrid®

Treatment of **mild to moderate** oedema in the foot.

- Provides even compression around the foot to ankle
- Non-compressive above the ankle
- Ability to wear normal shoes
- Available in a regular or wide foot
- Supplied in pairs

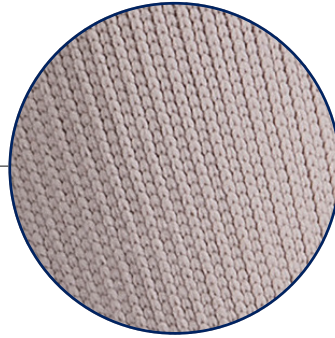

Colours



Tan



Black
(20-30mmHg,
wide only)

Compression level

20 - 30 mmHg

30 - 40 mmHg

Styles available on Drug Tariff



JOBST FarrowHybrid

Measurements Table

UK	Small	Medium	Large	X-Large
Shoe size	2-5	6-8	9-11	12-13

Regular - **A'** = 20-27 cm

Wide - **A'** = 28-35 cm

Garment Care



Machine
wash 30°



Do not
bleach



Do not
tumble dry



Lay flat
to dry



Do not
iron



Do not
dry clean

JOBST® FarrowWrap® Toe Cap and Glove

Suitable for patients with **mild to moderate** oedema in the hands and toes. Can be worn in conjunction with JOBST® FarrowWrap® footpiece or gauntlet.

- Can support patients with sensitive skin
- Fine soft seams for greater wearing comfort
- Durable and soft double laminated fabric
- Can be trimmed to patient requirements



Colours



Tan



Black

Compression level

15 - 20 mmHg

20 - 30 mmHg

Styles available on Drug Tariff



Toe Cap



Glove

Garment Care



Machine wash 30°



Do not bleach



Do not tumble dry



Lay flat to dry



Do not iron



Do not dry clean



Compression self-care tips for your patients



Washing the wrap

To prevent sticking, patients should be advised to fold the VELCRO® brand hook and loop fastener back on itself before washing. A special wash bag should further protect the wrap.



Pay attention to wounds

Before applying JOBST FarrowWrap, patients should ensure any wounds are thoroughly cleaned. They will also need to cover any open wounds that are present in the area being treated with a suitable wound dressing before applying compression.



Skin care

Application of professionally recommended moisturising creams will help your patients avoid infections and achieve the best possible treatment outcome.



The right shoes

Safe footwear is important to reduce the risk of patient falls and to help protect JOBST FarrowWrap from any unnecessary wear and tear.



Diet

Because obesity may worsen pre-existing lymphoedema, following a healthy and balanced diet is generally recommended.



Storage

To ensure a longer garment life-span, JOBST FarrowWrap should be properly stored. Instruct your patients to fold the VELCRO® brand hook and loop tabs back on themselves to avoid sticking.