

JOBST®

Compression for
those who love to
**Move
Freely**

Kimberly,
Secondary lymphoedema patient

JOBST® Confidence

The Next Generation of Made-to-Measure
Flat-Knit Compression Garments

**NEW
& UNIQUE**



Make a difference for your patients with mild to moderate lymphoedema

For patients with mild to moderate lymphoedema / chronic oedema and lipoedema, compression garments are often difficult to tolerate. Whilst these patients understand that they would benefit from compression therapy, most flat-knit garments are designed for use with more severe swelling. This can make patient compliance a challenge.

To successfully guide patients through their personal journey, it is important to help them find a garment that supports them in achieving their health and therapy goals, while also enabling them to live the way they want.

How to define mild to moderate lymphoedema?¹



STAGE

Early onset of the condition where there is accumulation of tissue fluid that subsides with limb elevation. The oedema may be pitting at this stage.



STAGE

Limb elevation alone rarely reduces swelling, and pitting is manifested. Later there may or may not be pitting as tissue fibrosis is more evident.



STAGE

The tissue is hard (fibrotic) and pitting is absent. Skin changes such as thickening, hyperpigmentation, increased skin folds, fat deposits, and warty over-growths develop.

MILD

MODERATE

SEVERE

¹Modified from: The Diagnosis and Treatment of Peripheral Lymphedema. 2016 Consensus Document of the International Society of Lymphology (ISL). Lymphology 2016; 49: 170-184

Understanding your patient's needs to support a **successful therapy**

Good compliance requires an optimal compression garment. Helping your patients find a garment that addresses their most important needs, such as comfort or the desire to live their lives as normally as possible, should be the highest priority to ensure happy patients and consistent treatment.

Working together with your patients and considering their individual needs can help them overcome some of the barriers to garment use and set them on the path for successful therapy.

Support patient's needs with **JOBST Confidence**



JOBST Confidence

Innovative knitting method allows for high ability to **conform to individual body shapes**



Made with a supple conforming fabric that also gives the reassurance of **firm support**



Advanced moisture management system with double-layer construction for **optimal wearing comfort**




Flexible, conforming fabric is designed to make putting on and taking off easier



Inspired by **patient needs** and developed with **next-generation technology**


Unique built-in moisture management



Going the extra step to achieve optimal comfort, JOBST Confidence garments also integrate a unique material construction and yarn composition to offer optimal moisture management. Only JOBST Confidence utilises a high-performance double-layer fabric to wick away moisture from the skin and towards the outer layer of the garment. This is designed to keep the skin dry and leave patients feeling comfortable all day long.

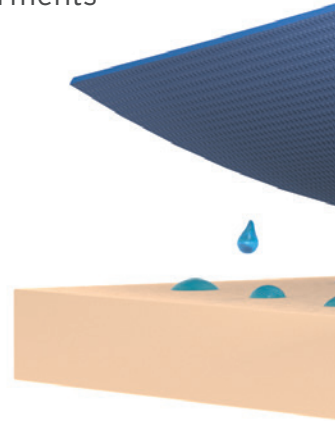

Softer yarns to support a pleasant feeling on the skin

JOBST Confidence is created with supple yarns that have been used with a double-layered knitted construction. This delivers a pleasant feeling on the skin and may help provide the flexibility and comfort that is desired by your patients when wearing compression.

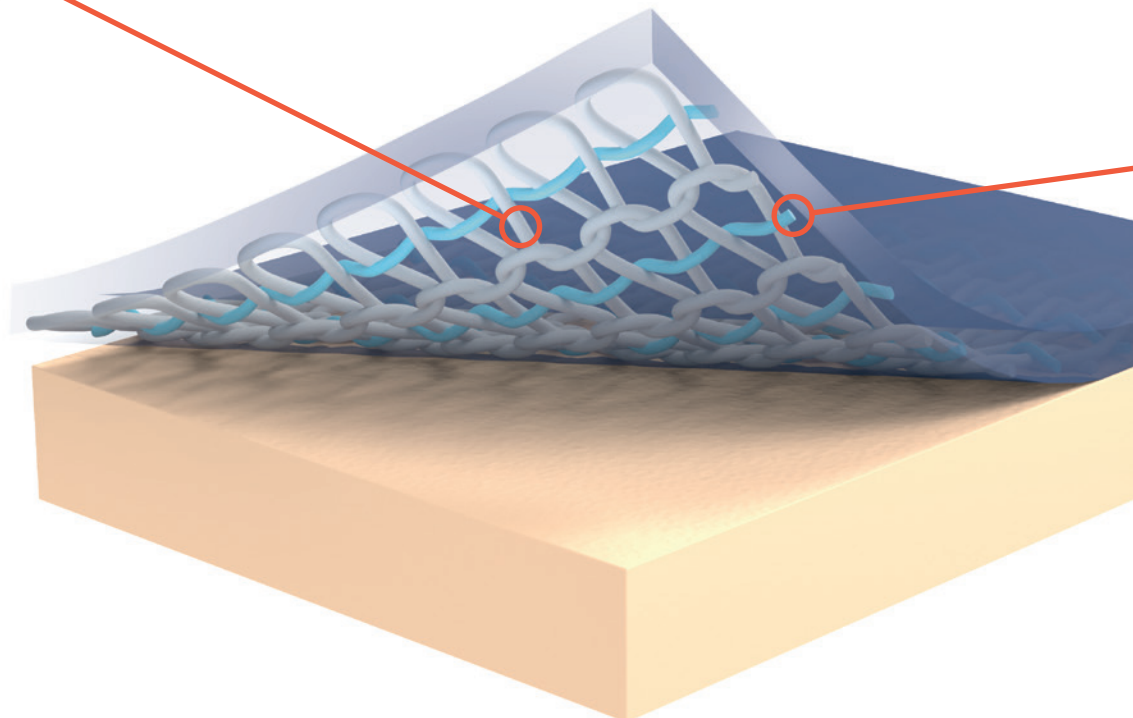

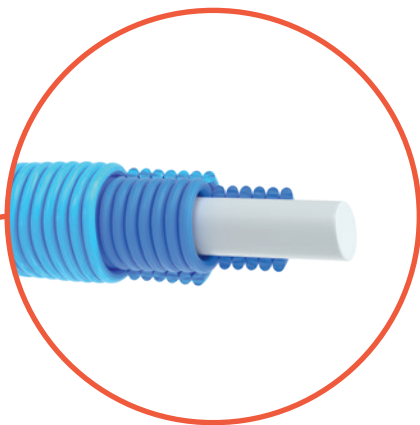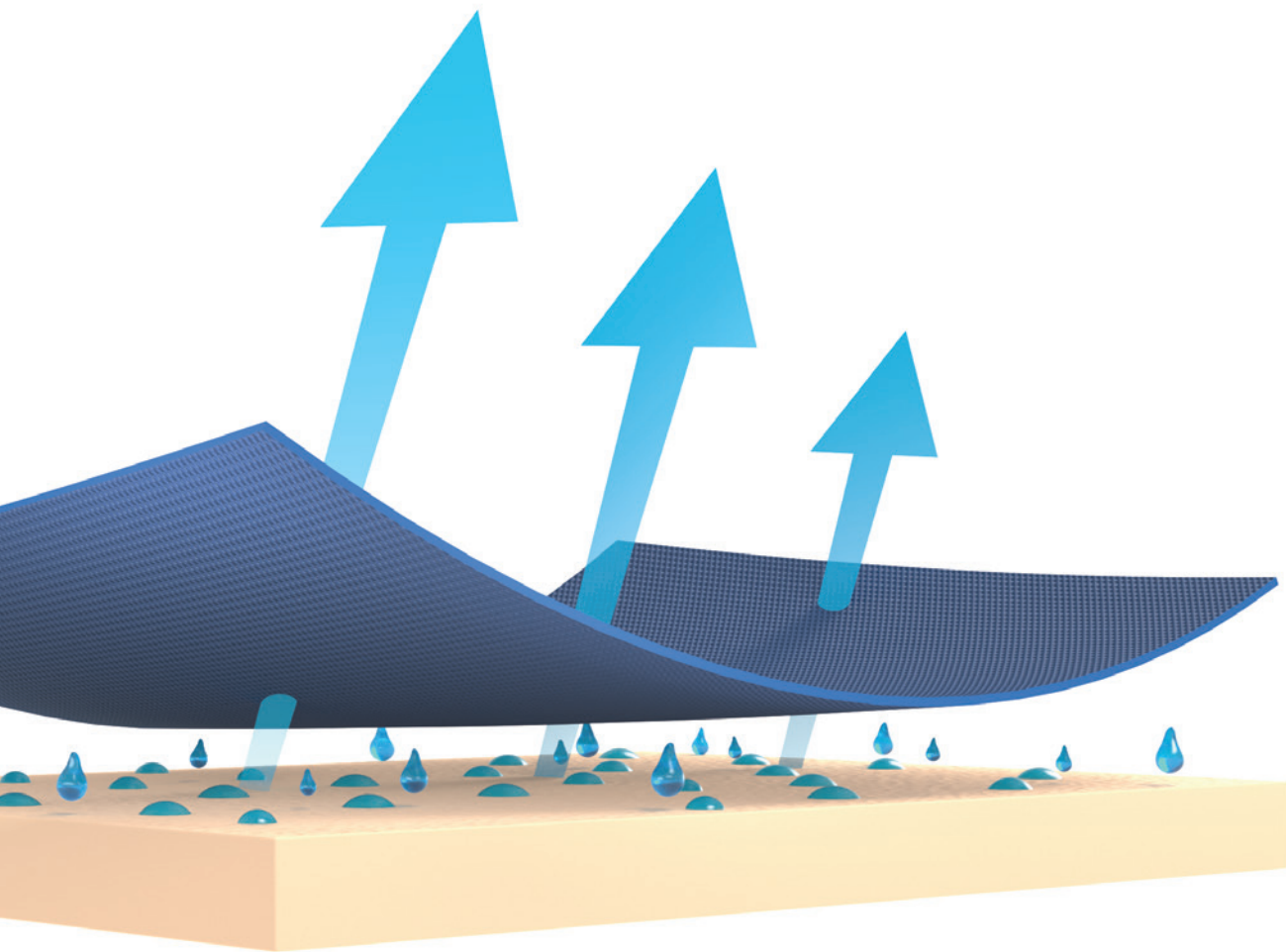


Firm support that adapts to individual needs

Your patients need the reassurance of firm support that they can rely on. The innovative method utilised in JOBST Confidence garments means that for the first time ever, the properties of the yarn adapt to the real body size of each patient. This always delivers the optimal compression level according to RAL standard.

Introducing a **new era of comfort** with the **next-generation** of flat-knit technology

JOBST Confidence helps your patients feel unrestricted in body and mind, with a one-of-a-kind fit garment that is as unique as they are. The new generation of flat-knit technology used in JOBST Confidence allows for a precise fit to the real anatomical body shapes of each of your individual patients. This personally refined fit not only provides comfort, but also supports your patients with a greater degree of movement.

**INNOVATIVE
TECHNOLOGY**

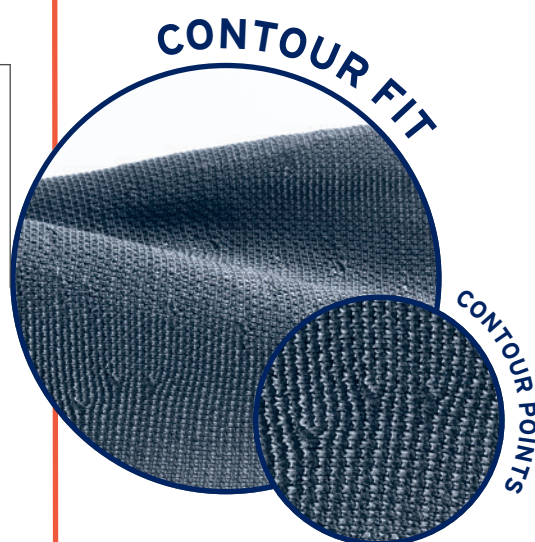

The key to the advanced comfort that JOBST Confidence provides your patients lies in the optimised knitting process. The innovative next-generation technology allows garments to anatomically fit the real body shapes of your patients more precisely than ever before.

This knitting process results in unique **Contour Points**, which show where the garment has been individually tailored to allow your patients to move more freely in their daily lives.

The new Contour Fit technology

JOBST Confidence has been expertly developed with a new knitting technology that, unlike traditional flat-knit, allows the fit of the garment to adapt in more positions than just the back. This supports a true personalisation of garment fit like never before. Overall, the garment is designed to sit comfortably on the skin whilst providing the right level of therapeutic compression required for each patient's needs.





“ I want to live
my best life,
with compression! ”

Kimberly, secondary lymphoedema patient

Break the barriers to patient compliance with **JOBST Confidence**

Positively changing your patients' lives and literally getting them back on their feet is a rewarding part of the job. To best help them along their journey, it is important for your patients to find a compression garment that best suits their needs and supports them in overcoming the challenges that are often associated with successful lymphedema therapy.

JOBST Confidence is designed to not only support your patient's journey towards improved health and therapeutic outcomes, but also to enable them to live their best lives with lymphoedema.


JOBST Confidence is designed to:

- provide a greater level of comfort
- be more skin-friendly
- provide enhanced movement
- support easy donning
- deliver improved climate control
- offer a range of colours and design options

Empowering your patients to self-care with **confidence**

Educate and understand lymphoedema

Your patients will have many questions at the start of their lymphoedema diagnosis and may struggle to accept compression therapy as part of their new daily lives. Providing content and resources, and maintaining regular communication with your patients can help them understand the disease better and empower them to proactively manage their own care.

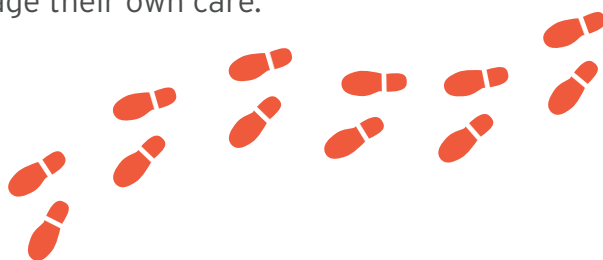


Set goals for success

Changing daily routines to suit the exceptional needs of a chronic disease like lymphoedema can be challenging at first. Helping your patients establish a sustainable management plan and to set short-term goals can help prevent them being overwhelmed with the new routines. Reviewing these goals together on a regular basis can support their journey and keep them motivated to wear compression.



Establish a self-care routine

Once your patient has received their new JOBST Confidence garment, spend some time to make sure they are comfortable applying it independently. Also discuss the washing and care instructions so that they feel confident to continue their care at home.

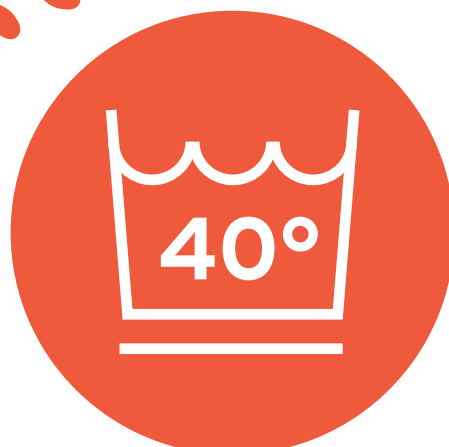



JOBST Confidence -

A complete range that's as individual as your patients


Colours




Beige



Jeans heather

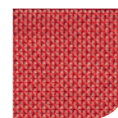

Red heather

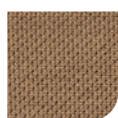

Caramel



Anthracite heather



Black

Styles



Knee high



Thigh high

Compression classes


Lower limb

CCL 1 ▶ 18 - 21 mmHg


CCL 2 ▶ 23 - 32 mmHg

CCL 3 ▶ 34 - 46 mmHg

Design options

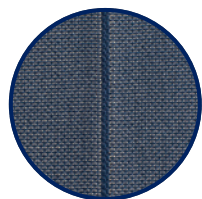


Deco line

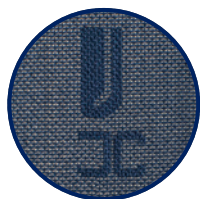

Initials

Comfort options

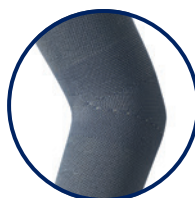

Knee functional zone



Slant toe



Closed toe



Ankle functional zone