



DEFY GRAVITY



JOBST® COMPRESSION

Upper Limb Compression Garments

CONTENTS

Upper Limb Compression Garments

JOBST® BELLA LITE 4

Circular knit Ready-To-Wear (RTW)

Armsleeve

Gauntlet

Combination Sleeve

Glove

JOBST® BELLA STRONG 6

Circular knit Ready-To-Wear (RTW)

Armsleeve Regular

Armsleeve Long

Gauntlet

Glove

JOBST® ELVAREX 8

Flat Knit Ready-To-Wear (RTW)

Ready-To-Wear Sleeves Bias finish

Ready-To-Wear Sleeves With Bra Loop

Ready-To-Wear Sleeves With Body Strap

Ready-To-Wear Gauntlet and Glove

MEASURING GUIDE 10

Armsleeve

Gloves

Gauntlet

Gloves and Gauntlets


THINKING ABOUT EARLY INTERVENTION?

JOBST® BELLA™ LITE Circular knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Made from soft yarns providing high wearing comfort
- Breathable ultra-thin fabric
- Fine seams
- Fingers on glove can be cut to length without fraying
- Glove is suitable for wearing on either hand
- Latex free
- Available in beige




Ordering Guide

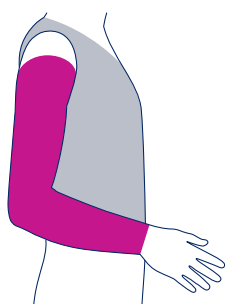


JOBST® Bella Lite Armsleeve

Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long
Product Code	75276-05	75276-00	75277-05	75277-00	75278-05	75278-00
cC	14 - 20 cm	14 - 20 cm	16 - 22 cm	16 - 22 cm	18 - 24 cm	18 - 24 cm
cD	18 - 23 cm	18 - 23 cm	23 - 29 cm	23 - 29 cm	29 - 34 cm	29 - 34 cm
cF	22 - 40 cm	22 - 40 cm	27 - 45 cm	27 - 45 cm	33 - 51 cm	33 - 51 cm

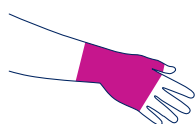


JOBST® Bella Lite Gauntlet

Size	Small	Medium	Large
Product Code	75282-00	75283-00	75284-00
cA	15 cm - 19 cm	19 cm - 22 cm	21 cm - 26 cm
cC	14 cm - 20 cm	16 cm - 22 cm	18 cm - 24 cm

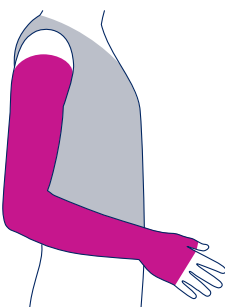

JOBST® Bella Lite Combination Sleeve

Size	Small Regular	Small Long	Medium Regular	Medium Long	Large Regular	Large Long
Product Code	77680-00	77680-03	77680-01	77680-04	77680-02	77680-05
cA	15 - 19 cm	15 - 19 cm	19 - 22 cm	19 - 22 cm	22 - 26 cm	22 - 26 cm
cD	18 - 23 cm	18 - 23 cm	23 - 29 cm	23 - 29 cm	29 - 34 cm	29 - 34 cm
cF	22 - 40 cm	22 - 40 cm	27 - 45 cm	27 - 45 cm	33 - 51 cm	33 - 51 cm

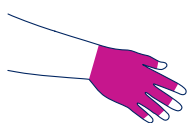

JOBST® Bella Lite Glove

Size	1	2	3	4	5	6	7	8
Product Code	71026-00	71026-01	71026-02	71026-03	71026-04	71026-05	71026-06	71026-07
cA	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm



MILD TO MODERATE LYMPHOEDEMA

JOBST® BELLA™ STRONG

Circular knit Ready-To-Wear (RTW)


- CCL2 20-30mmHg
- Indicated for mild to moderate oedema
- Skin friendly multi fibre nylon material
- Utilises advanced comfort technology to regulate humidity, quickly wicks moisture, keeping you cool and comfortable
- Innovative sizing system - armsleeves can fit approx 80% of patients and gauntlets approx 90% of patients
- Available in beige



Ordering Guide


Bella Strong armsleeves are available in two lengths- Regular and Long

JOBST® Bella Strong Armsleeve **Regular** Length

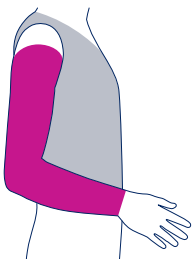


Size	1	2	3	4	5	6	7	8	9	10
Product Code	72736-01	72736-02	72736-03	72736-04	72736-05	72736-06	72736-07	72736-08	72736-09	72736-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

JOBST® Bella Strong Armsleeve **Long** Length

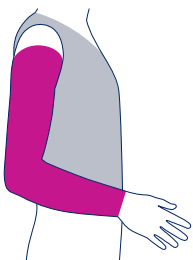


Size	1	2	3	4	5	6	7	8	9	10
Product Code	72737-01	72737-02	72737-03	72737-04	72737-05	72737-06	72737-07	72737-08	72737-09	72737-10
cC	14 - 18 cm	14 - 18 cm	14 - 18 cm	16 - 19 cm	17 - 20 cm	14 - 18 cm	17 - 20 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm
cD	17 - 21 cm	17 - 21 cm	21 - 25 cm	21 - 25 cm	21 - 25 cm	25 - 29 cm	25 - 29 cm	25 - 29 cm	29 - 33 cm	29 - 33 cm
cG	22 - 31 cm	31 - 41 cm	25 - 34 cm	28 - 38 cm	34 - 43 cm	29 - 38 cm	33 - 43 cm	38 - 48 cm	32 - 42 cm	36 - 46 cm

JOBST® Bella Strong **Gauntlet**

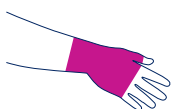

Size	1	2	3	4	5	6	7	8
Product Code	73694-40	73694-41	73694-42	73694-43	73694-44	73694-45	73694-46	73694-47
cA	15 - 17 cm	17 - 19 cm	17 - 19 cm	19 - 21 cm	19 - 21 cm	21 - 23 cm	21 - 23 cm	23 - 26 cm
cC	14 - 18 cm	14 - 18 cm	18 - 21 cm	14 - 18 cm	18 - 21 cm	16 - 19 cm	19 - 22 cm	19 - 22 cm

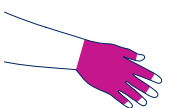

JOBST® Bella Strong **Glove**

Size	1	2	3	4	5	6	7	8
Product Code	75291-10	75291-11	75291-12	75291-13	75291-14	75291-15	75291-16	75291-17
cA	16.6 - 18.0 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
cC	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm



JOBST® ELVAREX - MODERATE LYMPHOEDEMA

JOBST® ELVAREX™ Flat Knit Ready-To-Wear (RTW)

- CCL2 20-30mmHg
- Indicated for moderate oedema

Two special features make JOBST® Elvarex™ the product of choice for lymphoedema treatment:

- “Double knit” exerts a light massaging effect on the initial lymphatics
- Low-stretch fabric


Other features include:

- Good skin tolerance due to air permeability (knitted construction), and cotton covering of all threads (cotton component against skin)
- Durable and comfortable to wear
- Graduated compression
- Flat knitting process with only one seam, provides good anatomical fit, especially for elbow region in sleeves
- Ease of donning due to short stretch thread and high cotton content




Ordering Guide

JOBST® Elvarex® Ready-To-Wear Armsleeves Bias finish (no strap)

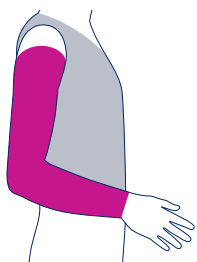


Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	76101-11	76101-10	76101-09	76101-08	76101-07	76101-06
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm

JOBST® Elvarex® Ready-To-Wear Sleeves with Bra Loop

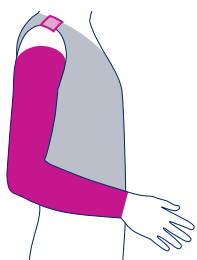


Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	79435-17	79435-16	79435-15	79435-14	79435-13	79435-12
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm

JOBST® Elvarex® Ready-To-Wear Sleeves With Body Strap

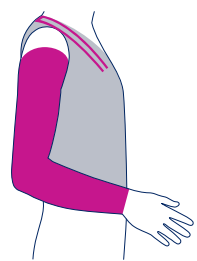

Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Product Code	79435-12	79435-22	79435-21	79435-20	79435-19	79435-18
cC	16 - 18 cm	18 - 20 cm	20 - 23 cm	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD	23 - 26 cm	26 - 28 cm	28 - 30 cm	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE	24 - 26 cm	26 - 29 cm	29 - 32 cm	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF	26 - 29 cm	29 - 33 cm	33 - 37 cm	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG	31 - 34 cm	34 - 38 cm	38 - 44 cm	31 - 34 cm	34 - 38 cm	38 - 44 cm


JOBST® Elvarex® Ready-To-Wear Gauntlet and Glove

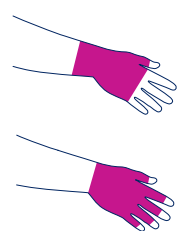

Size	Small Right	Medium Right	Large Right	Small Left	Medium Left	Large Left
Glove Product Code	76091-17	76091-16	76091-15	76091-14	76091-13	76091-12
Gauntlet Product Code	76091-23	76091-22	76091-21	76091-20	76091-19	76091-18
cA (Metacarpal heads)	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm
cB (Widest Palm)	17 - 19 cm	19 - 21 cm	21 - 23 cm	17 - 19 cm	19 - 21 cm	21 - 23 cm
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 22 cm	16 - 18 cm	18 - 20 cm	20 - 22 cm
Thumb base	5 - 6 cm	6 - 7 cm	7 - 8 cm	5 - 6 cm	6 - 7 cm	7 - 8 cm
Fingers 2 base	5 - 6 cm	6 - 7 cm	6 - 7 cm	5 - 6 cm	6 - 7 cm	6 - 7 cm
Fingers 3 base	5 - 6 cm	6 - 7 cm	6.5 - 7 cm	5 - 6 cm	6 - 7 cm	6.5 - 7 cm
Fingers 4 base	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm
Fingers 5 base	4 - 5 cm	5 - 6 cm	6 - 7 cm	4 - 5 cm	5 - 6 cm	6 - 7 cm

Measuring Guide

JOBST® Bella Lite Armsleeves

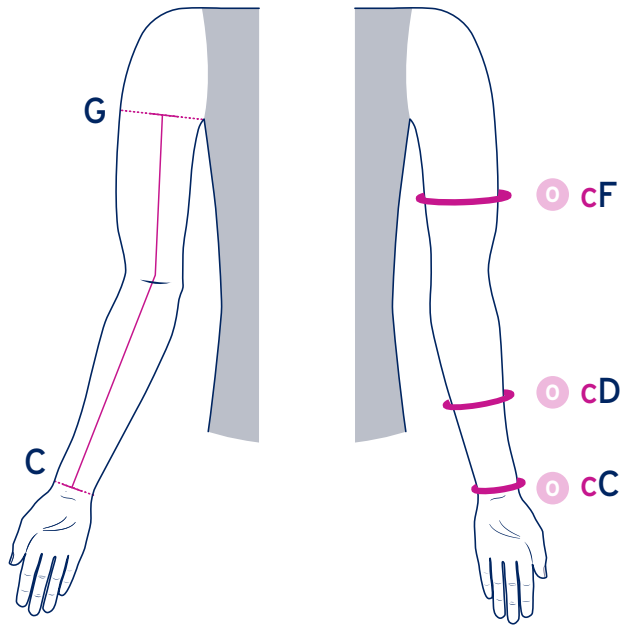


Length Measurements (C-G)

Regular up to 43 cm.

Long greater than 43 cm.

Circumference Measurements Bella Lite			
Size	Wrist cC	Mid Lower cD	Mid Upper cF
Small Reg	14 - 20 cm	18 - 23 cm	22 - 40 cm
Small Long	14 - 20 cm	18 - 23 cm	22 - 40 cm
Medium Reg	16 - 22 cm	23 - 29 cm	27 - 45 cm
Medium Long	16 - 22 cm	23 - 29 cm	27 - 45 cm
Large Reg	18 - 24 cm	29 - 34 cm	33 - 51 cm
Large Long	18 - 24 cm	29 - 34 cm	33 - 51 cm

JOBST® Bella Strong Armsleeves

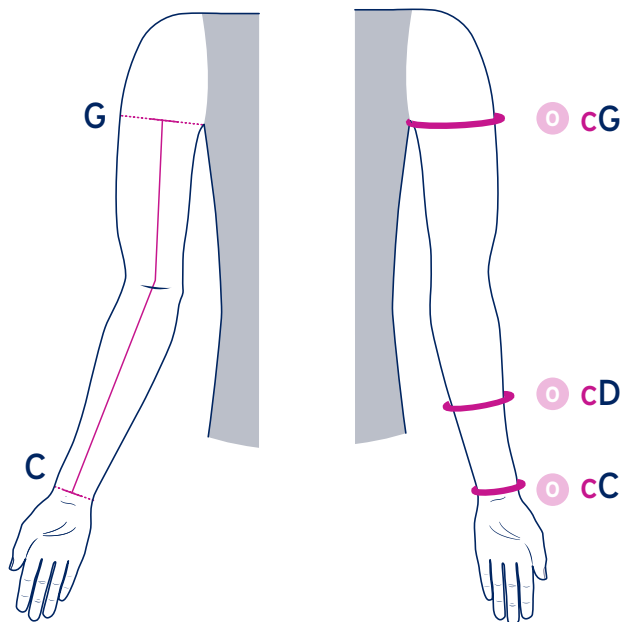

Measuring Circumference

STEP 1 Measure axilla circumference **cG**
This leaves 1 or 2 choices.

STEP 2 Measure mid lower circumference **cD**
This leaves 2 or 3 choices.


STEP 3 Measure wrist circumference **cC**
This leaves 1 or 2 choices.

If there are still 2 options left, we recommend choosing the larger size for ease of donning, overall comfort and ultimate compliance.

Either size selection will deliver the desired therapy.

Circumference Measurements Bella Strong			
Size	Wrist (cC)	Mid Forearm (cD)	Axilla (cG)
1	14 - 18 cm	17 - 21 cm	22 - 31 cm
2	14 - 18 cm	17 - 21 cm	31 - 41 cm
3	14 - 18 cm	21 - 25 cm	25 - 34 cm
4	16 - 19 cm	21 - 25 cm	28 - 38 cm
5	17 - 20 cm	21 - 25 cm	34 - 43 cm
6	14 - 18 cm	25 - 29 cm	29 - 38 cm
7	17 - 20 cm	25 - 29 cm	33 - 43 cm
8	18 - 21 cm	25 - 29 cm	38 - 48 cm
9	16 - 19 cm	29 - 33 cm	32 - 42 cm
10	19 - 22 cm	29 - 33 cm	36 - 46 cm

JOBST® Bella Lite and Strong Gloves

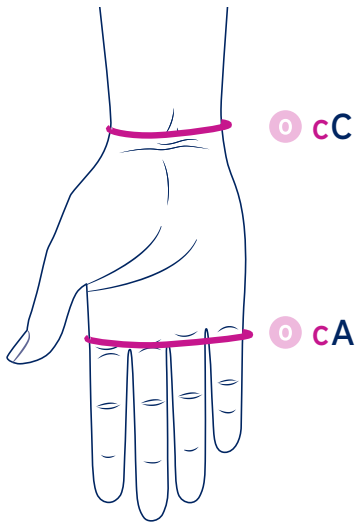


Measuring Circumference

- STEP 1** Measure palm circumference **cA**
This leaves 2 choices.
- STEP 2** Measure wrist circumference **cC**
Select Glove.

Circumference Measurements - Bella Lite and Bella Strong								
Size	1	2	3	4	5	6	7	8
A	16.6 - 18 cm	16.6 - 18 cm	18 - 19.5 cm	18 - 19.5 cm	18 - 19.5 cm	19.5 - 21.5 cm	19.5 - 21.5 cm	21.5 - 23.1 cm
C	15.6 - 16.7 cm	16.7 - 18.1 cm	15.9 - 17 cm	17 - 18.5 cm	18.5 - 20.4 cm	17.2 - 18.7 cm	18.7 - 20.6 cm	18.3 - 20.1 cm

JOBST® Bella Lite and Strong Gauntlet

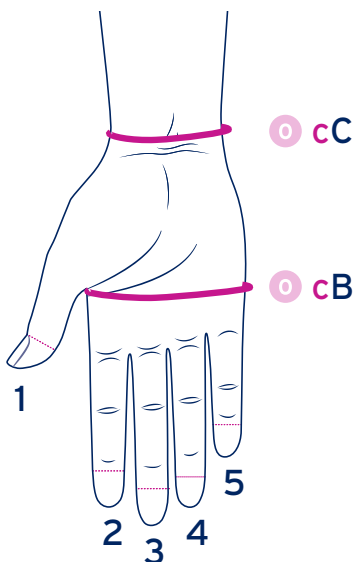

JOBST® Bella Strong Measuring Circumference

- STEP 1** Measure palm circumference
This leaves 2 choices.
- STEP 2** Measure wrist circumference
Select gauntlet

Circumference Measurements Bella Lite		
Size	Palm cB	Wrist cC
Small	15 cm - 19 cm	14 cm - 20 cm
Medium	19 cm - 22 cm	16 cm - 22 cm
Large	21 cm - 26 cm	18 cm - 24 cm

Circumference Measurements Bella Strong		
Size	Palm cB	Wrist cC
1	15 - 17 cm	14 - 18 cm
2	17 - 19 cm	14 - 18 cm
3	17 - 19 cm	18 - 21 cm
4	19 - 21 cm	14 - 18 cm
5	19 - 21 cm	18 - 21 cm
6	21 - 23 cm	16 - 19 cm
7	21 - 23 cm	19 - 22 cm
8	23 - 26 cm	19 - 22 cm

TIPS


Mark the measurement points on the limb with a JOBST® Skin Marking Pencil



Take the corresponding circumference and length measurements using the JOBST® tape measure

Measuring Guide

Elvarex® Ready-To-Wear Gloves and Gauntlets

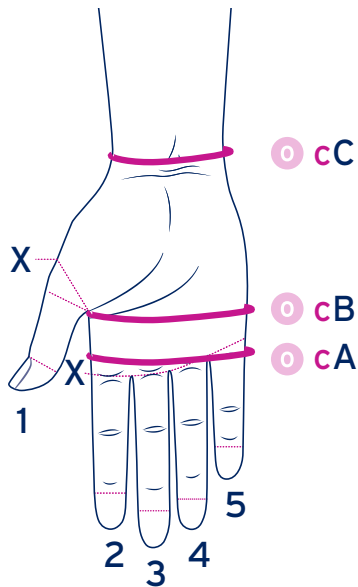


Measuring Circumferences

- STEP 1** The patient's hand should be placed palm down and the fingers should be slightly apart. As the hand is sensitive to pressure, the measuring tape should not be pulled tightly. Circumferential measures should follow the contour of the palm.
- STEP 2** Measure and record the circumference at A - over the MCP heads (as per diagram).
- STEP 3** Measure and record the circumference at B - distal to the thumb web (as per diagram).
- STEP 4** Measure and record the circumference at C, the wrist - distal to the styloid process (as per diagram).
- STEP 5** Measure and record the circumference at (X), the base of the thumb.
- STEP 6** If a glove with fingers is required, measure and record the circumference at (X) the base of each finger (as per diagram).

Compare the measurements you have obtained with the following table to determine the correct size. Where measurements vary greatly, a custom-made glove is necessary.

Lengths A-B 3 cm, A-C 9 cm, A-C1 12 cm

Circumference Measurements			
Measurement Point	Small	Medium	Large
cA (Metacarpal heads)	16 - 18 cm	17.5 - 19.5 cm	19.5 - 20.5 cm
cB (Widest Palm)	17 - 19 cm	19 - 21 cm	21 - 23 cm
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 22 cm
Thumb base	5 - 6 cm	6 - 7 cm	7 - 8 cm
Fingers 2 base	5 - 6 cm	6 - 7 cm	6 - 7 cm
Fingers 3 base	5 - 6 cm	6 - 7 cm	6.5 - 7 cm
Fingers 4 base	4.5 - 5.5 cm	5.5 - 6.5 cm	6.5 - 7.5 cm
Fingers 5 base	4 - 5 cm	5 - 6 cm	6 - 7 cm

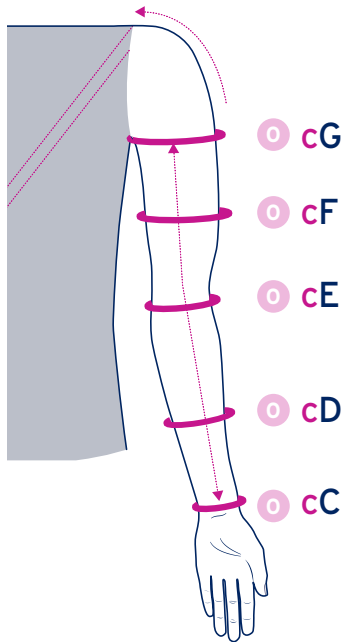

Length Measurements

Suitable for up to 42 cm for C-G, and 45 cm for C-G with attachment

Measuring Circumferences

- STEP 1** The patient should be seated, arm on table with palm facing upwards and elbow flexed to approximately 40°.
- STEP 2** Measure and record the circumference at C, the wrist. NOTE: At the wrist and elbow, the tape should never be pulled tightly, however, slightly firmer pressure may be used over fleshy areas, i.e. D, F and G points).
- STEP 3** Measure and record the circumference at D, the mid -point of the lower arm.
- STEP 4** Measure and record the circumference at E, the elbow. (Ensure the arm is slightly bent at [40°] before taking this measurement.)
- STEP 5** Measure and record the circumference F, the mid-point between the axilla and the elbow.
- STEP 6** Measure and record the circumference at G, (take the measurement approx 1 cm below axilla).

Compare the measurements you have obtained with the following table to determine the correct size.

Where measurements vary across all three sizes, or are smaller or larger than the tabled measurements, a custom-made sleeve will be required.

Circumference Measurements			
Measurement Point	Small	Medium	Large
cC (Wrist)	16 - 18 cm	18 - 20 cm	20 - 23 cm
cD (Mid Forearm)	23 - 26 cm	26 - 28 cm	28 - 30 cm
cE (Elbow)	24 - 26 cm	26 - 29 cm	29 - 32 cm
cF (Mid Upper arm)	26 - 29 cm	29 - 33 cm	33 - 37 cm
cG (Axilla)	31 - 34 cm	34 - 38 cm	38 - 44 cm

