



DEFY GRAVITY



JOBST® COMPRESSION

Compression Wrap Therapy

CONTENTS

Compression Wrap Therapy

EASY-TO-USE COMPRESSION SOLUTIONS

JOBST® FarrowWrap®	140
Product Range	
Foot Piece	
Toe Cap	
Leg Piece	
Knee Piece	
Thigh Piece	
Liners & Accessories	
Upper Limb	
JOBST® FarrowWrap®	
APPLICATION TECHNIQUES	162
JOBST® FarrowWrap®	
CARE GUIDELINES	164

JOBST® FarrowWrap®

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Easy Donning

Short-stretch technology provides low resting and high working pressures to enhance lymphatic and venous return.



Short Stretch Technology

VELCRO® brand hook & loop system allows patients to respond better to fluctuating oedema by simply tightening or loosening the wrap.



Adjustable

Discreet design is less bulky than multi-layer bandaging and can be worn under normal clothing.



Low Profile Design

THE EASY CHOICE TO SUPPORT PATIENT SELF-CARE AND EMPOWERMENT

Give your patients the tools they need to better manage their swelling and respond confidently to fluctuating oedema levels, with a solution that takes the guesswork out of applying compression safely.

JOBST® FarrowWrap® garments are a range of adjustable compression devices that are available in a variety of styles, fabrics, sizes and compression classes. Made with durable materials, they are designed to help reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers. They are designed to provide the same strong support as multi-layer bandaging but in a simplified, intuitive design.



Overlapping bands replicate the principles of multi-layer bandaging.

Apply and adjust confidently

JOBST® FarrowWrap® was engineered for easy application and can offer all the benefits of multi-layer bandaging without the complexity of application. The overlapping material supports consistent self-application without the need for specialised intervention by health care providers. The overlap provides the support needed to efficiently control oedema and help reduce the risk of swelling forming between the bands.



THE JOBST® FarrowWrap®

Product Range

LITE



Indicated for mild to moderate chronic oedema/lymphoedema

A double laminated fabric suitable for mild to moderate compression needs, making it especially suitable for palliative and paediatric patients. The fabric is both durable and breathable.

- Conforms to limb shape
- Good choice for patients who have neuropathy or peripheral vascular disease (PVD)
- Recommended for paediatric patients
- Suitable for sensitive skin and as such palliative care patients
- Machine washable and can be tumble dried on low heat

20-30 mmHg

STRONG



Indicated for moderate to severe chronic oedema/lymphoedema with a good limb shape

Premium fabric with strong compression and soft inner layer suitable for patients with a good limb shape and moderate to severe oedema.

- Less rigid than CLASSIC
- Contoured bands conform to the limb and resist sliding
- Highly breathable and durable
- Suitable for minor skin folds and distortion
- Material less likely to pill or fray
- Machine washable and can be tumble dried on low heat

30-40 mmHg

CLASSIC



Indicated for moderate to severe chronic oedema/lymphoedema

The original material that has been clinically proven over many years of use on thousands of patients worldwide. The rigid nature of the material offers performance beyond the compression range making it excellent for patients with moderate to severe compression levels.

- Suitable for oedema with skin folds and irregular limb shapes
- Durable, rigid fabric with a comfortable feel
- Reinforced spine for easier donning
- Less likely to roll down and will not work into skin folds
- Hand wash only and lay flat to dry

30-40 mmHg

Product Assortment

	Foot Piece	Leg Piece	Foot Compression Liner	Thigh Piece	Arm Piece	Gauntlet
Mild to Moderate 20-30mmHg						
JOBST® FarrowWrap® LITE	RTW	RTW		RTW	RTW	RTW
JOBST® FarrowHybrid® ADI			RTW			
Moderate to Strong 30-40mmHg						
JOBST® FarrowWrap® STRONG	RTW	RTW		RTW		
JOBST® FarrowWrap® CLASSIC	RTW	RTW		RTW		

RTW: Ready-To-Wear

	Leg Piece	Arm or Leg Piece	Toe Cap/Glove
Liners and accessories			
JOBST® FarrowWrap® LITE			RTW
JOBST® FarrowWrap® GarmentGrip™ Roll	RTW	RTW	



EASY-TO-USE COMPRESSION SOLUTIONS HELP TO PROMOTE INDEPENDENT SELF-CARE

Compression therapy is a vital component in the management of chronic swelling from venous and lymphatic conditions. Conventional treatment focuses on reducing symptoms with multi-layer bandaging.

For certain patients, this management system can be difficult to maintain independently. It is therefore important to offer these patients suitable compression solutions that are efficient and easy-to-use, to help improve therapy outcomes and empower patients to confidently care for their own needs.

Choosing the right compression¹:

- Easy to apply and remove**
- Is comfortable for the patient and stays in place**
- Offers variety in patient choice**
- Delivers therapeutic compression and has high stiffness**
- Is aesthetically acceptable**

¹Harding, K., Dowsett, C., Flax, L et al. Simplifying venous leg ulcer management: consensus recommendations. Wounds International, 2015. <https://tinyurl.com/jrxfcjrj> (accessed 26 February 2017).
Properties of an ideal compression system.

INNOVATIVE SOLUTIONS DESIGNED FOR IMPROVED SELF-CARE: **ADJUSTABLE COMPRESSION DEVICES**

Give your patients the choice and variety needed to take control of their own care and support them to achieve better therapy outcomes with adjustable compression devices.

In oedema management, adjustable compression devices (or compression wraps) can be used as a first-choice treatment option and offer an alternative to multi-layer bandaging or hosiery.

JOBST® FarrowWrap has short-stretch properties similar to multi layering bandaging, delivering low resting pressure and a high working pressure.

They have similar short-stretch properties to multi-layer bandaging in that they have low resting pressure and a high working pressure. As a result, they may create peak pressures that produce a massaging effect and stimulate lymph flow. JOBST® FarrowWrap are designed for easy use, enabling patient to independently apply and adjust the compression according to their needs and tolerance.

Which of your patients would benefit from **JOBST® FarrowWrap**?



OEDEMA

Patients with swelling resulting from lymphoedema and venous oedema:

- With fluctuations in limb swelling
- In decongestive and/or maintenance phases



LARGER BODY SIZES

Patients unable to don or unsuitable for compression garments/bandages through:

- Severe obesity
- Overweight or larger body sizes
- Irregular shape of limbs
- Skin folds and lobules



VENOUS LEG ULCER

Patients with open lesions between the knee and the ankle:

- Most common type of leg ulcer, accounting for more than 90% of all cases
- Can develop after a minor injury, where persistently high pressure in the veins of the legs has damaged the skin



PHYSICAL LIMITATIONS

Patients unable to apply or remove compression hosiery/bandaging due to:

- Arthritis or weak hand strength
- Back problems
- Sensitive or fragile skin at risk of breakdown

JOBST® FarrowWrap® FOOT PIECE



LITE Foot Piece

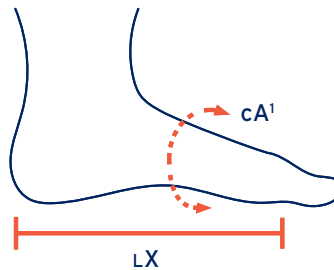


STRONG Foot Piece



CLASSIC Foot Piece

How to measure RTW Foot Piece size chart






		XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces						
A¹	Mid-foot	22 - 24 cm	25 - 27 cm	28 - 30 cm	31 - 34 cm	35 - 40 cm
X	Regular	15 - 17 cm	16 - 19 cm	18 - 20 cm	19 - 22 cm	21 - 24 cm
X	Long	18 - 20 cm	20 - 22 cm	21 - 23 cm	23 - 25 cm	25 - 27 cm

The length is measured from the most posterior aspect of the heel to the base of the 5th toe.

The midfoot measurement is a circumferential measurement taken at the midfoot point

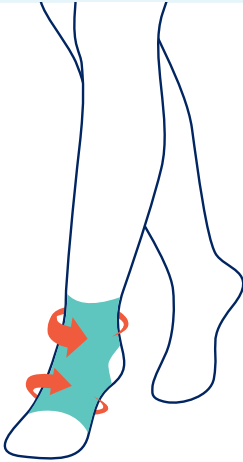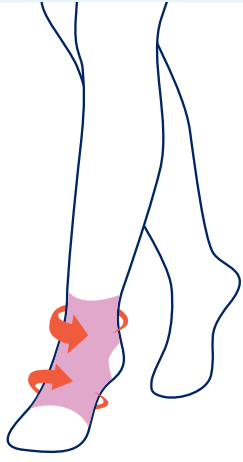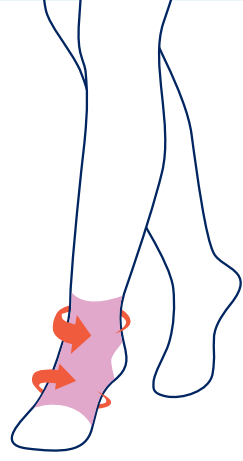Ordering Information

	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Foot Piece				
	76652-05	XSmall	Regular	Tan
	76652-06	Small	Regular	Tan
	76652-07	Medium	Regular	Tan
	76652-08	Large	Regular	Tan
	76652-09	XLarge	Regular	Tan
	76652-00	XSmall	Long	Tan
	76652-01	Small	Long	Tan
	76652-02	Medium	Long	Tan
	76652-03	Large	Long	Tan
	76652-04	XLarge	Long	Tan
JOBST® FarrowWrap® Strong (30-40mmHg) Foot Piece				
	76646-04	XSmall	Regular	Tan
	76646-05	Small	Regular	Tan
	76646-06	Medium	Regular	Tan
	76646-07	Large	Regular	Tan
	76646-08	XLarge	Regular	Tan
	76646-09	XSmall	Long	Tan
	76646-00	Small	Long	Tan
	76646-01	Medium	Long	Tan
	76646-02	Large	Long	Tan
	76646-03	XLarge	Long	Tan
JOBST® FarrowWrap® Classic (30-40mmHg) Foot Piece				
	76656-05	XSmall	Regular	Tan
	76656-06	Small	Regular	Tan
	76656-07	Medium	Regular	Tan
	76656-08	Large	Regular	Tan
	76656-09	XLarge	Regular	Tan
	76656-00	XSmall	Long	Tan
	76656-01	Small	Long	Tan
	76656-02	Medium	Long	Tan
	76656-03	Large	Long	Tan
	76656-04	XLarge	Long	Tan

JOBST® FarrowWrap® TOE CAP

Ready-to-wear Toe Cap which can be worn on either right or left foot.

Fine seams, with ultra thin fabric, allows air to pass to enhance patient comfort.


The toes can be easily trimmed with scissors to exactly the required length.

Measurement Table

Size	cA
XSmall	20 - 22 cm
Small	22 - 24 cm
Medium	24 - 26 cm
Large	26 - 28 cm

Ordering Information

Size	Product Code	Compression	Colour
XSmall	72947-00	20 - 30 mmHg	Beige
Small	72947-01	20 - 30 mmHg	Beige
Medium	72947-02	20 - 30 mmHg	Beige
Large	72947-03	20 - 30 mmHg	Beige

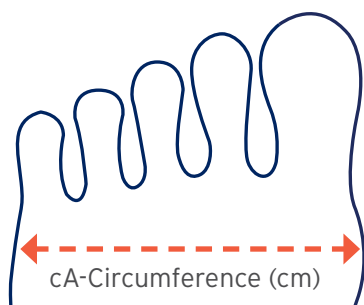



JOBST® FarrowWrap® LEG PIECE



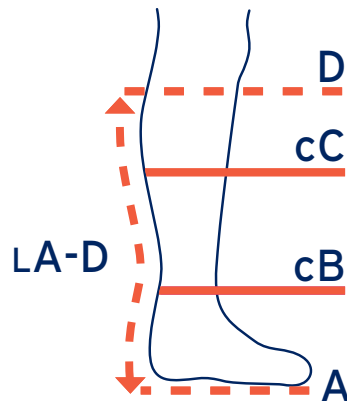
LITE Leg Piece



STRONG Leg Piece



CLASSIC Leg Piece






How to measure
RTW Leg Piece size chart

	XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces					
C Calf	36 - 43 cm	42 - 50 cm	48 - 58 cm	53 - 63 cm	58 - 68 cm
B Ankle	21 - 25 cm	25 - 30 cm	30 - 36 cm	36 - 42 cm	42 - 50 cm
A-D Regular	35 - 39 cm	37 - 41 cm	39 - 43 cm	41 - 45 cm	42 - 46 cm
A-D Tall	40 - 44 cm	42 - 46 cm	44 - 48 cm	46 - 50 cm	47 - 51 cm

Length A-D = Floor to 2 finger widths below crease of knee. Measure posteriorly following contour of leg.

Calf circumference taken at the widest part of the calf- this will be measure C. Ankle circumference taken at the narrowest part of the ankle- this will be measure B

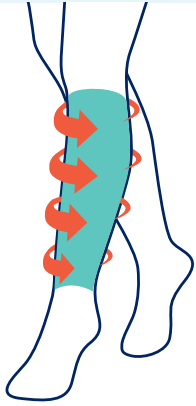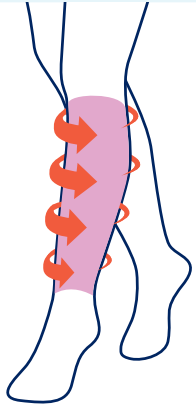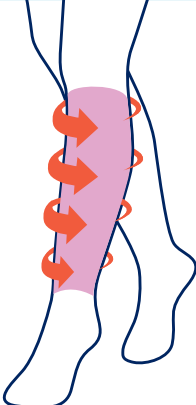Ordering Information

	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Leg Piece				
	76654-00	XSmall	Regular	Tan
	76654-01	Small	Regular	Tan
	76654-02	Medium	Regular	Tan
	76654-03	Large	Regular	Tan
	76654-04	XLarge	Regular	Tan
	76654-05	XSmall	Tall	Tan
	76654-06	Small	Tall	Tan
	76654-07	Medium	Tall	Tan
	76654-08	Large	Tall	Tan
	76654-09	XLarge	Tall	Tan
JOBST® FarrowWrap® Strong (30-40mmHg) Leg Piece				
	76648-00	XSmall	Regular	Tan
	76648-01	Small	Regular	Tan
	76648-02	Medium	Regular	Tan
	76648-03	Large	Regular	Tan
	76648-04	XLarge	Regular	Tan
	76648-05	XSmall	Tall	Tan
	76648-06	Small	Tall	Tan
	76648-07	Medium	Tall	Tan
	76648-08	Large	Tall	Tan
	76648-09	XLarge	Tall	Tan
JOBST® FarrowWrap® Classic (30-40mmHg) Leg Piece				
	76657-00	XSmall	Regular	Tan
	76657-01	Small	Regular	Tan
	76657-02	Medium	Regular	Tan
	76657-03	Large	Regular	Tan
	76657-04	XLarge	Regular	Tan
	76657-05	XSmall	Tall	Tan
	76657-06	Small	Tall	Tan
	76657-07	Medium	Tall	Tan
	76657-08	Large	Tall	Tan
	76657-09	XLarge	Tall	Tan

JOBST® FarrowWrap® KNEE PIECE

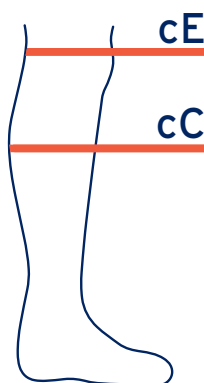
Lower Extremity



Adjustable Knee Piece, easy to apply & comfortable to wear.

Can be worn in combination with JOBST® FarrowWrap® Leg Pieces and Thigh Pieces, or as a single unit (e.g. in combination with an Elvarex knee high stocking)

Available in two fabrics: LITE and STRONG





How to measure

RTW Knee Piece size chart

	XSmall	Small	Medium	Large	XLarge
5 cm above patella	47 - 52 cm	53 - 59 cm	60 - 66 cm	67 - 73 cm	74 - 80 cm
Calf	36 - 43 cm	42 - 50 cm	48 - 58 cm	53 - 63 cm	58 - 68 cm

The patient needs to be measured at two points, C (widest calf) and E.

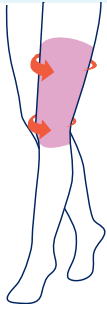Ordering Information

	Ref - No.	Size	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Knee Piece			
	73367-00	XSmall	Tan
	73367-01	Small	Tan
	73367-02	Medium	Tan
	73367-03	Large	Tan
	73367-04	XLarge	Tan
JOBST® FarrowWrap® Strong (30-40mmHg) Knee Piece			
	73368-00	XSmall	Tan
	73368-01	Small	Tan
	73368-02	Medium	Tan
	73368-03	Large	Tan
	73368-04	XLarge	Tan



JOBST® FarrowWrap® THIGH PIECE



LITE Thigh Piece



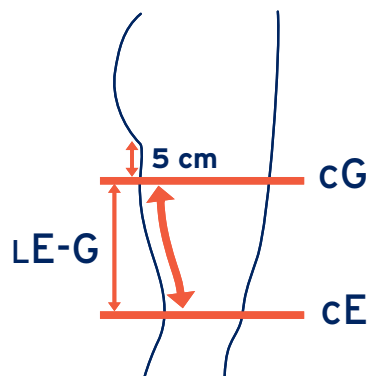
STRONG Thigh Piece



CLASSIC Thigh Piece

How to measure

RTW Thigh Piece size chart



	XSmall	Small	Medium	Large	XLarge
Ready-to-wear pieces					
G Groin	60 - 70 cm	65 - 75 cm	70 - 80 cm	75 - 90 cm	85 - 100 cm
E Above Knee	47 - 52 cm	53 - 59 cm	60 - 66 cm	67 - 73 cm	74 - 80 cm
E-G Short	16 - 20 cm	16 - 20 cm	16 - 20 cm	16 - 20 cm	16 - 20 cm
E-G Regular	21 - 26 cm	21 - 26 cm	21 - 26 cm	21 - 26 cm	21 - 26 cm
E-G Tall	27 - 31 cm	27 - 31 cm	27 - 31 cm	27 - 31 cm	27 - 31 cm

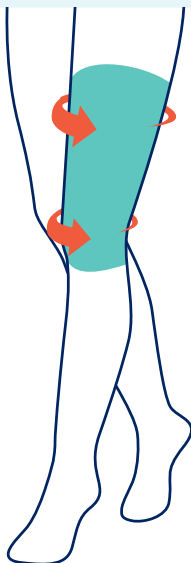
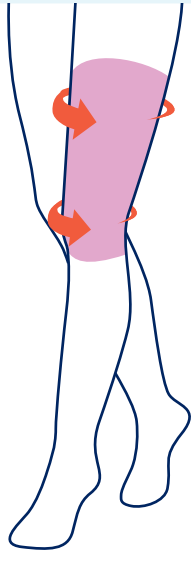
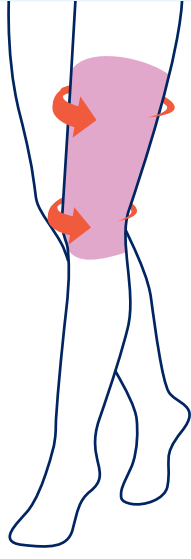
There are three measurements required for the Thigh Piece.

Above knee circumference - measure 5cm above the knee, this is your E measure

Groin circumference - measure at the gluteal fold (where the buttock meets the leg), this will be your G measure

Thigh length - measure the length of the thigh medially (inside leg) between E and G following the contour of the leg, this will be your EG measure

Ordering Information

	Ref - No.	Size	Length	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Thigh Piece				
	76655-05	XSmall	Short	Tan
	76655-06	Small	Short	Tan
	76655-07	Medium	Short	Tan
	76655-08	Large	Short	Tan
	76655-09	XLarge	Short	Tan
	76655-00	XSmall	Regular	Tan
	76655-01	Small	Regular	Tan
	76655-02	Medium	Regular	Tan
	76655-03	Large	Regular	Tan
	76655-04	XLarge	Regular	Tan
	76655-10	XSmall	Tall	Tan
	76655-11	Small	Tall	Tan
	76655-12	Medium	Tall	Tan
	76655-13	Large	Tall	Tan
76655-14	XLarge	Tall	Tan	
JOBST® FarrowWrap® Strong (30-40mmHg) Thigh Piece				
	76649-05	XSmall	Short	Tan
	76649-06	Small	Short	Tan
	76649-07	Medium	Short	Tan
	76649-08	Large	Short	Tan
	76649-09	XLarge	Short	Tan
	76649-00	XSmall	Regular	Tan
	76649-01	Small	Regular	Tan
	76649-02	Medium	Regular	Tan
	76649-03	Large	Regular	Tan
	76649-04	XLarge	Regular	Tan
	76649-10	XSmall	Tall	Tan
	76649-11	Small	Tall	Tan
	76649-12	Medium	Tall	Tan
	76649-13	Large	Tall	Tan
76649-14	XLarge	Tall	Tan	
JOBST® FarrowWrap® Classic (30-40mmHg) Thigh Piece				
	76658-06	XSmall	Short	Tan
	76658-07	Small	Short	Tan
	76658-08	Medium	Short	Tan
	76658-09	Large	Short	Tan
	76658-10	XLarge	Short	Tan
	76658-00	XSmall	Regular	Tan
	76658-01	Small	Regular	Tan
	76658-02	Medium	Regular	Tan
	76658-03	Large	Regular	Tan
	76658-04	XLarge	Regular	Tan
	76658-12	XSmall	Tall	Tan
	76658-13	Small	Tall	Tan
	76658-14	Medium	Tall	Tan
	76658-15	Large	Tall	Tan
	76658-16	XLarge	Tall	Tan

JOBST® FarrowWrap® LINERS & ACCESSORIES

Liners



FarrowHybrid® is a patented foot compression garment that provides compression in the foot up to the ankle.

- 20-30mmHg Compression
- From the ankle and above, it serves as a liner
- Available in standard and wide sizes
- Great alternative to a Foot Piece
- Low profile, easily fits into a shoe

	Small	Medium	Large	XLarge
Male Shoe Size	3 - 6	7 - 9	10 - 12	13 - 15
Female Shoe Size	4 - 7	8 - 10	11 - 13	14 - 16

Note the maximum calf circumference for Hybrid Liners is 60 cm

Hybrid Liners

Measure the mid-foot A circumference. Two mid-foot sizes available: 20-27 cm (regular foot) and 28-35 cm (wide foot). Request the patient's shoe size (US Sizing).

Accessories

Ref - No.	Size	Style	Limb	Colour
JOBST® Accessories (20-30mmHg)				
76672-00	14 cm X 80 cm	GarmentGrip	Thigh	Beige
76672-01	14 cm X 120 cm	GarmentGrip	Thigh Long	Beige
76672-03	20 cm X 120 cm	GarmentGrip	Wide	Beige
76672-04	30 cm X 150 cm	GarmentGrip	Extra Wide	Beige
76672-02	14 cm X 550 cm	GarmentGrip	Extra Long	Beige
76673-00	10 cm X 150 cm	Farrow Short Stretch		Beige
76673-01	8 cm X 150 cm	Farrow Short Stretch		Beige

JOBST® Farrow GarmentGrip can be cut to size to fit the shape and contour of the body and VELCRO® brand hook-and-loop fasteners can be added to secure in place. For triangular shaped legs, it is recommended to be worn over the leg Piece to secure the wrap system in place.

JOBST® Farrow GarmentGrip is washable and therefore reusable. It is available for the upper and lower limb in differing roll lengths and extra wide widths and can also be used to manage mid-line oedema.

TIP: On triangular shaped legs we recommend JOBST® Farrow Garment Grip be worn over our Leg Piece to help hold the garment in place

Hybrid Liners

Ref - No.	Fabric	Size	Female shoe size	Male shoe size	Colour	Mid-foot Circumference regular or wide
JOBST® Knee High Liner (20-30mmHg)						Regular
76664-00	Hybrid ADI	Small	US Female Shoe 4 - 7	US Male Shoe 3 - 6	Beige	Mid-foot 20 - 27 cm
76664-01	Hybrid ADI	Medium	US Female Shoe 8 - 10	US Male Shoe 7 - 9	Beige	Mid-foot 20 - 27 cm
76664-02	Hybrid ADI	Large	US Female Shoe 11 - 13	US Male Shoe 10 - 12	Beige	Mid-foot 20 - 27 cm
76664-03	Hybrid ADI	XLarge	US Female Shoe 14 - 16	US Male Shoe 13 - 15	Beige	Mid-foot 20 - 27 cm
JOBST® Knee High Liner (20-30mmHg)						Wide
76664-12	Hybrid ADI	Small	US Female Shoe 4 - 7	US Male Shoe 3 - 6	Beige	Mid-foot 28 - 35 cm
76664-13	Hybrid ADI	Medium	US Female Shoe 8 - 10	US Male Shoe 7 - 9	Beige	Mid-foot 28 - 35 cm
76664-14	Hybrid ADI	Large	US Female Shoe 11 - 13	US Male Shoe 10 - 12	Beige	Mid-foot 28 - 35 cm
76664-15	Hybrid ADI	XLarge	US Female Shoe 14 - 16	US Male Shoe 13 - 15	Beige	Mid-foot 28 - 35 cm
JOBST® Knee High Liner (20-30mmHg)						Wide
76664-16	Hybrid ADI	Small	US Female Shoe 4 - 7	US Male Shoe 3 - 6	Black	Mid-foot 28 - 35 cm
76664-17	Hybrid ADI	Medium	US Female Shoe 8 - 10	US Male Shoe 7 - 9	Black	Mid-foot 28 - 35 cm
76664-18	Hybrid ADI	Large	US Female Shoe 11 - 13	US Male Shoe 10 - 12	Black	Mid-foot 28 - 35 cm
76664-19	Hybrid ADI	XLarge	US Female Shoe 14 - 16	US Male Shoe 13 - 15	Black	Mid-foot 28 - 35 cm

•Calf circumference up to 60 cm

Lower Limb Liners

Ref - No.	Open/Closed Toe	Size	Calf Circumference	Colour
JOBST® Knee High Liners - No compression				Regular
76666-21	Closed Toe	Standard	Up to 58 cm calf	Beige
76666-22	Closed Toe	Wide	52 - 70 cm calf	Beige
76669-00	Open Toe	Small	< 40 cm calf	Beige
76669-02	Open Toe	Medium	40 - 70 cm calf	Beige
76669-04	Open Toe	Large	70 - 120 cm calf	Beige
JOBST® Thigh High Liners - No compression				Wide
76670-00	Open Toe	Small	< 40 cm calf	White
76670-02	Open Toe	Medium	40 - 70 cm calf	White
76670-04	Open Toe	Large	70 - 120 cm calf	White

JOBST® FarrowWrap® UPPER LIMB

Hand - Suitable For 24-Hour Wear

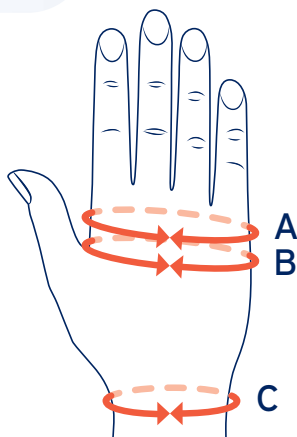


JOBST® FarrowWrap® LITE Hand Gauntlet 20-30 mmHg

JOBST® FarrowWrap LITE Hand Gauntlet is ambidextrous and comes with 4 mm and 8 mm foam squares for added compression to the dorsal hand. Suitable for mild to severe oedema.

Benefits:

1. Foam inserts help reduce oedema that can pool on the top of the hand
2. Ambidextrous, i.e. can be used on either hand
3. Durable and comfortable



How to measure

RTW Gauntlet size chart

	XSmall	Small	Medium	Large
A	16 - 18 cm	18 - 20 cm	20 - 22 cm	22 - 24 cm
B	17 - 19 cm	19 - 21 cm	21 - 23 cm	23 - 26 cm
C	15 - 18 cm	17 - 20 cm	19 - 22 cm	22 - 24 cm

Ordering Information

Ref - No.	Size
JOBST® FarrowWrap® Lite Hand Gauntlet: Ambidextrous, with foam	
76653-00	XSmall
76653-01	Small
76653-02	Medium
76653-03	Large



JOBST® FarrowWrap® UPPER LIMB

Arm Piece - Suitable For 24-Hour Wear



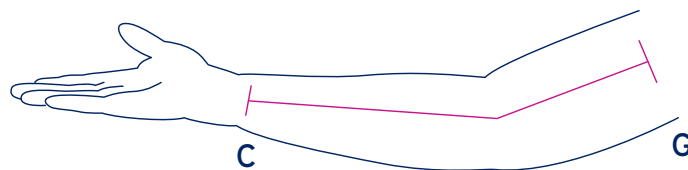
JOBST® FarrowWrap® LITE Arm Piece 20-30 mmHg compression

LITE RTW Arm Piece is an anatomically contoured effective solution to control upper extremity oedema. It is easy to apply and remove with one hand and designed for mild to moderate oedema control. Light and comfortable with just one layer but with a robust short-stretch performance.



How to measure

RTW Armsleeve size chart




	Length
Short	40 cm - 43 cm
Regular	44 cm - 47 cm
Long	48 cm - 51 cm

NOTE: Measure length along the lateral outer arm from the wrist to the axilla with a slight bend in the arm.

	Wrist	Elbow	Axilla
Small	14 - 18 cm	20 - 27 cm	22 - 31 cm
Medium	16 - 21 cm	25 - 34 cm	29 - 39 cm
Large	19 - 25 cm	30 - 40 cm	32 - 45 cm

Ordering Information

Ref - No.	Size	Option	Colour
JOBST® FarrowWrap® Lite (20-30mmHg) Arm Piece			Long
76650-05	Small	Left	Tan
76650-00	Small	Right	Tan
76650-01	Medium	Left	Tan
76650-02	Medium	Right	Tan
76650-03	Large	Left	Tan
76650-04	Large	Right	Tan
76650-17	Small	Left	Black
76650-18	Small	Right	Black
76650-19	Medium	Left	Black
76650-20	Medium	Right	Black
76650-21	Large	Left	Black
76650-22	Large	Right	Black
76655-12	Medium	Tall	Tan
76655-13	Large	Tall	Tan
76655-14	Xlarge	Tall	Tan
JOBST® FarrowWrap® Lite (20-30mmHg) Arm Piece			Regular
76650-05	Small	Left	Tan
76650-06	Small	Right	Tan
76650-07	Medium	Left	Tan
76650-08	Medium	Right	Tan
76650-09	Large	Left	Tan
76650-10	Large	Right	Tan
76650-23	Small	Left	Black
76650-24	Small	Right	Black
76650-25	Medium	Left	Black
76650-26	Medium	Right	Black
76650-27	Large	Left	Black
76650-28	Large	Right	Black
JOBST® FarrowWrap® Lite (20-30mmHg) Arm Piece			Short
76650-11	Small	Left	Tan
76650-12	Small	Right	Tan
76650-13	Medium	Left	Tan
76650-14	Medium	Right	Tan
76650-15	Large	Left	Tan
76650-16	Large	Right	Tan
76650-29	Small	Left	Black
76650-30	Small	Right	Black
76650-31	Medium	Left	Black
76650-32	Medium	Right	Black
76650-33	Large	Left	Black
76650-34	Large	Right	Black

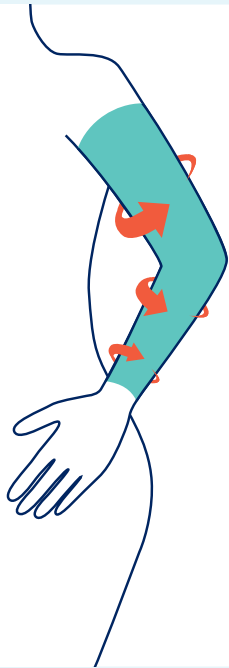

FarrowWrap® FOOT PIECE APPLICATION TECHNIQUES

Applicable for the following fabrics

- JOBST® FarrowWrap® Lite
- JOBST® FarrowWrap® Strong
- JOBST® FarrowWrap® Classic



1 Apply a liner. Either the open toe Soft Liner (white) or the closed toe Liner (beige).

With the Soft Liner, ensure you extend the fabric past the toes distally and past the knee proximally to allow sufficient fabric to fold back over the wraps once they have been applied. Unwrap the Foot Piece and lay it on the floor with the hole for the heel closest to you.



2 Place the foot on top of the Foot Piece, resting the heel over the hole in the Foot Piece.



3 To secure the foot, hold the Foot Piece flap closest to the toes on each side and pull both sides over the top of the foot to the opposite side. Apply at or near end stretch with the VELCRO®.

Now apply the remaining flap (the one closest to the ankle) in the same manner.



4 After securing the forefoot, hold one side of the ankle band and pull it over the top of the mid-foot to the opposite side and secure it in place at or near end stretch.



5 Repeat the same step with the other half of the ankle band on the opposite side.

When removing the Foot Piece, fold the VELCRO® over on itself and back on the same band.



6 **Care instructions (Strong and Lite fabric):** Machine wash in cold water with a mild detergent, tumble dry with no heat, do not iron, do not dry clean.

Care instructions (Classic fabric): Hand wash in cold water with mild detergent, lay flat to dry, do not iron, do not dry clean.

FarrowWrap® LEG PIECE APPLICATION TECHNIQUES



Applicable for the following fabrics

- JOBST® FarrowWrap® Lite
- JOBST® FarrowWrap® Strong
- JOBST® FarrowWrap® Classic



Apply a JOBST® Farrow liner and Foot Piece if applicable.

Unwrap the Leg Piece and lay the wrap on the floor behind your leg with the label side furthest back and facing up. The shortest bands will be closer to the foot.



Place the wrap with its spine positioned behind the leg starting from the ankle and finishing just below the knee.

Apply the top wrap as an anchor to hold the wrap in place. Then commence wrapping from the ankle up towards the knee.



Grab each side of the band closest to your ankle and pull forward wrapping the band around the lowest part of the leg at or near end stretch and affix with the VELCRO®. Undo the top band before applying the second last band to ensure the final band sits over the top. If wearing a Foot Piece, the bottom band should overlap the Foot Piece.



Repeat with each band moving up the leg, smoothing out any wrinkles as you go.

Try to ensure a 50% overlap between adjacent straps.

JOBST® FarrowWrap® CARE GUIDELINES



Wash your wrap/s weekly or more often if you experience wound leakage and wash your liner every day.

Prior to any washing fold the Velcro (Stick tabs) back on themselves. This prevents the build-up of fluff keeping the tabs in good condition and easy to apply.

Hand Washing

- After folding the tabs on themselves wash with warm water - 30 degrees and a mild soap or detergent.
- Roll firmly in a towel to remove excess water.

Washing Machine

- After folding the tabs on themselves place your wraps in a laundry bag
- Wash at 30 degrees using a mild soap or detergent.

Drying Your Wraps

- Dry flat and leave in a warm or shady room.



LITE Leg Piece



Do not tumble dry



Do not bleach



Do not iron



Do not use softeners



Do not dry clean