INSTRUCTIONS FOR USE

JOBST® Bella Lite & Strong

(EN) Instructions for Use

(DE) Anwendungshinweise

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(SV) Bruksanvisning

(NO) Bruksanvisning

(DA) Brugsanvisning

(NL) Handleiding

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(SL) Navodila za uporabo

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Brand/CCL	% Elastane (Spandex)	% Elastodiene (Latex)	% Polyamide (Nylon)	% Cotton	% Polyester	% Silicone
JOBST® Bella Strong Ready-to-Wear Armsleeve, 15-20 mmHg	16		84			
JOBST® Bella Strong Ready-to-Wear Armsleeve, 20-30 & 30-40 mmHg	19		58	2 (Wool)	15	6
JOBST® Bella Strong Ready-to-Wear Armsleeve with Silicone Band, 15-20 mmHg	18		74			8
JOBST® Bella Strong Ready-to-Wear Armsleeve with Silicone Band, 20-30 mmHg	20		72			8
JOBST® Bella Strong Ready-to-Wear Armsleeve with Silicone Band, 30-40 mmHg	23		69			8
JOBST® Bella Strong Ready-to-Wear Armsleeve Knit Welt, 15-20 mmHg	20		80			
JOBST® Bella Strong Ready-to-Wear Armsleeve Knit Welt, 20-30 mmHg	24		76			
JOBST® Bella Strong Ready-to-Wear Armsleeve Knit Welt, 30-40 mmHg	26		74			

FIBER CONTENT

Brand/CCL	% Elastane (Spandex)	% Elastodiene (Latex)	% Polyamide (Nylon)	% Cotton	% Polyester	% Silicone
JOBST® Bella Strong Ready-to-Wear Gauntlet, 15-20 mmHg	9		91			
JOBST® Bella Strong Ready-to-Wear Gauntlet, 20-30 mmHg	19		60	3 (Wool)	18	
JOBST® Bella Lite Ready-to-Wear Armsleeve, 15-20 mmHg	16		84			
JOBST® Bella Lite Ready-to-Wear Armsleeve with Silicone Band, 15-20 mmHg	17		80			3
JOBST® Bella Lite Ready-to-Wear Armsleeve, 20-30 mmHg	20		80			
JOBST® Bella Lite Ready-to-Wear Armsleeve with Silicone Band, 20-30 mmHg	22		75			3
JOBST® Bella Lite Ready-to-Wear Gauntlet, 15-20 mmHg	16		84			
JOBST® Bella Lite Glove, 15-20 mmHg	41		59			
JOBST® Bella Lite Ready-to-Wear Gauntlet, 20-30 mmHg	14		86			
JOBST® Bella Lite Ready-to-Wear Combined Armsleeve, 15-20 mmHg	19		81			
JOBST® Bella Lite Ready-to-Wear Combined Armsleeve with Silicone Band, 15-20 mmHg	15		77			8
JOBST® Bella Lite Ready-to-Wear Combined Armsleeve, 20-30 mmHg	21		79			
JOBST® Bella Lite Ready-to-Wear Combined Armsleeve with Silicone Band, 20-30 mmHg	17		75			8
JOBST® Bella Lite Glove, 20-30 mmHg	41		59			

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INSTRUCTIONS FOR USE

EN

JOBST® Bella Lite & Strong Ready-to-Wear Armsleeve

JOBST® Bella Lite: 15-20 mmHg and 20-30 mmHg

JOBST® Bella Strong: 15-20 mmHg, 20-30 mmHg and 30-40 mmHg

UK: Class 1 (CI 1) = 15-20 mmHg, Class 2 (CI 2) = 20-30 mmHg, Class 3 (CI 3) = 30-40 mmHg

Your healthcare professional has recommended this JOBST® Armsleeve for the management of your lymphedema or other edema.

Indications

For the management of lymphedema and other edema. Phlebitis, post thrombotic syndrome, vascular malformations, post liposuction (20-30 mmHg JOBST® Bella Lite only).

Contraindications

Ischemia (significantly reduced arterial flow), untreated septic phlebitis (infected veins), or conditions in which increased venous and lymphatic return is not desired.


Cautions

Consult your physician or healthcare professional if you experience skin infections, weeping dermatosis (skin weeps fluid), incompatibility to the fabric or impaired sensitivity of the limb. Should you encounter persistent pain, chronic discomfort, inflammation, swelling, or a loss of circulation in the limb, discontinue use and consult a physician.

Instructions for Donning

JOBST® Bella Lite Armsleeve


1. Fold Armsleeve on itself, from the top down. Note positioning of the wide band. Insert hand.
 2. Gently pull complete Armsleeve on to the elbow. The Armsleeve should now be positioned over the wrist and elbow portion of the arm.
 3. Gently pull the top of the Armsleeve up to its correct end location (top of the arm).
 4. Smooth out any wrinkles or folds that may appear.
- Optional: A donning aid may be used to help slide the sleeve over your hand.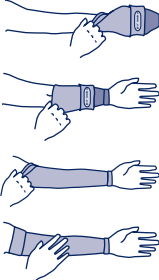

Instructions for Donning

JOBST® Bella Strong Armsleeve

1. Pull Donning Aid onto arm.
2. Fold back band on armsleeve so silicone is exposed.
3. Pull armsleeve onto arm until wrist cuff is on or above wrist bone.

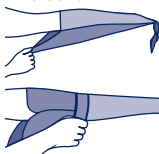4. Place loop over door knob or other secure, stationary object. Holding silicone band in place, slowly step away from door - allowing Donning Aid to be removed. Adjust armsleeve as needed.
5. Flip silicone band into place using a donning glove to adjust the armsleeve as needed.

JOBST® Bella Lite & Strong Ready-to-Wear Gauntlet

15-20 mmHg and 20-30 mmHg

UK: Class 1 (CI 1) = 15-20 mmHg, Class 2 (CI 2) = 20-30 mmHg

Your healthcare professional has recommended this JOBST® Gauntlet for the management of your lymphedema or other edema.

Indications

For the management of lymphedema and other edema.

Contraindications


Ischemia (significantly reduced arterial flow), untreated septic phlebitis (infected veins).

Cautions

Consult your physician or healthcare professional if you experience skin infections, weeping dermatosis (skin weeps fluid), incompatibility to the fabric, or impaired sensitivity of the limb. Should you encounter persistent pain, chronic discomfort, inflammation, swelling or a loss of circulation in the limb, discontinue use and consult a physician.

Instructions for Donning

1. Slip fingers into Gauntlet starting at the end that has the widest band.
2. Position Gauntlet so that it sits comfortably around the palm and thumb.
3. The Gauntlet should be positioned so that it ends around the knuckles.

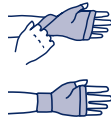JOBST® Bella Lite & Strong Ready-to-Wear Glove

15-20 mmHg and 20-30 mmHg

Your healthcare professional has recommended this JOBST® Glove for the management of your lymphedema or other edema.

Indications

For the control of mild to moderate hand swelling from acute and chronic edema, lymphatic and other swelling disorders.

Follow your healthcare provider's instructions.

INSTRUCTIONS FOR USE

EN

Contraindications

Severe arterial disease, moderate arterial disease in the presence of peripheral neuropathy, untreated vein infection (septic phlebitis), untreated and/or advancing skin infection (cellulitis), large blood clot in the vein (phlegmasia coerulea dolens).

Cautions

No potential risks or limitations identified. Consult your physician or healthcare professional if you experience skin infections, weeping dermatosis (skin weeps fluid), incompatibility to the fabric, or impaired sensitivity of the limb. Should you encounter persistent pain, chronic discomfort, inflammation, swelling or a loss of circulation in the limb, discontinue use and consult a physician.

Instructions for Donning

JOBST® Bella Lite Glove

- Open wounds should be covered before applying glove. Remove jewelry before applying. The Glove can be used on either hand:
- A. Right hand: label is visible on the wrist.
 - B. Left hand: turn the glove "inside out", inverting the fingers. The label is on the inside.
- Follow these steps when putting on your glove:
1. Slip fingers into glove.
 2. Position glove so that it sits comfortably around the palm and thumb.
 3. The glove should be positioned so that the fingers end near the beds of the fingernails.
 4. If the finger lengths are too long, mark the finger at the appropriate length, remove the glove, then trim the length of the glove finger to the marked area and reapply. Be careful not to remove to much of the finger length.

Instructions for Donning

JOBST® Bella Strong Glove

Follow these steps when putting on your glove:

1. Slip your hand through the wrist of the glove and into the fingers.
2. Adjust the fingers by carefully pulling on the fabric until the glove fits snugly into the web spaces between your fingers.
3. Be sure the fabric is smoothed out over the fingers and hand.

JOBST® Bella Lite Ready-to-Wear Combined Armsleeve

15-20 mmHg and 20-30 mmHg

Your healthcare professional has recommended this JOBST® compression garment for the management of your lymphedema or other edema.

Indications

For the management of lymphedema and other edema. Phlebitis, post thrombotic syndrome, vascular malformations, post liposuction (20-30 mmHg only).

Contraindications


Ischemia (significantly reduced arterial flow), untreated septic phlebitis (infected veins), or conditions in which increased venous and lymphatic return is not desired.

Cautions

Consult your physician or healthcare professional if you experience skin infections, weeping dermatosis (skin weeps fluid), incompatibility to the fabric or impaired sensitivity of the limb. Should you encounter persistent pain, chronic discomfort, inflammation, swelling or a loss of circulation in the limb, discontinue use and consult a physician.

Instructions for Donning

1. Fold Armsleeve part on itself, from the top down. Slip fingers into Gauntlet and position it so that it sits comfortably around the palm and thumb.
2. Gently pull complete Armsleeve part on to the elbow. The Armsleeve part should now be positioned over the wrist and elbow portion of the arm.
3. Gently pull the top of the Armsleeve part up to its correct end location (top of the arm).
4. Smooth out any wrinkles or folds that may appear.

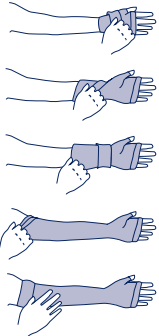Cautions JOBST® Bella

Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.

Patient measurements should be taken by a trained person to determine the correct product size needed and to decide whether a custom-made or ready-to-wear garment is required. When re-ordering a garment after 6 months we recommend remeasuring the patient as the limb circumference might have changed.

If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional. Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state.

INSTRUCTIONS FOR USE

EN

Care Instructions JOBST® Bella

Machine washable in warm water. Mild process. Use a delicate fabric detergent. Do not use fabric softener. Do not bleach. Tumble drying possible. Low temperature; exhaust temperature max. 50 °C. Do not iron. Do not dry clean. Flat drying. Drip line drying in the shade.

Special Care Instructions for JOBST® Bella Armsleeves with Silicone Band

1. Before washing the garment, turn the garment inside out.
2. Thoroughly hand wash the silicone band using a delicate fabric detergent. Concentrate on

removing any lotion, powder or hair from the silicone.
The use of certain skin care products, dry peeling skin or use in hairy areas, may prevent the silicone band from keeping the garment in place. JOBST® garments will retain their compression for 6 months under normal wear and care. Please protect your compression garments from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature.

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Customer Service U.S. 1-800-537-1063

Thank you for your trust.

Discover more about healthy legs and compression products on:

www.jobst.com

BSN medical Inc.
5825 Carnegie Blvd., Charlotte, NC 28209 USA

BSN medical GmbH
Quickbornstrasse 24, 20253 Hamburg, Germany

Australian Distributor: BSN medical (Aust.) Pty Ltd
L6, 211 Wellington Road, Mulgrave 3170 Victoria Australia

	Artwork ID JBR12076.02	Review Loop 1	Printing Colors  Pantone 281 C
Product Title JOBST REY JOBST® Bella Lite & Strong	Date of Review Loop 03.01.2019	Replaces Artwork ID JBR12076.01	
Contact artwork.coordination@bsnmedical.com	Technical Drawing for print ID -	Comment on Diecut -	
Contact Address BSN medical GmbH Quickbornstrasse 24 20253 Hamburg	Ref Number(s) 16013-90010-00		