INSTRUCTIONS FOR USE

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Dear Customer,

Thank you for choosing JOBST® FarrowWrap® / Farrow / FarrowHybrid®.

This assortment includes adjustable compression devices for upper and lower extremities, non-compressive liners, medical compression garments and related accessories.

1. JOBST FarrowWrap® / Farrow / FarrowHybrid® Features:

Left or Right

Adjustable

Short Stretch Technology

Trimable

Designed for easy donning

Durable

2. Indications

Please follow your healthcare provider's instructions.

JOBST® FarrowWrap® / Farrow / FarrowHybrid® products are designed to exert physical, external therapeutic compression to help specific venous and lymphatic conditions and swelling of other origins. They have a specific compression level which must be determined by your physician for the management of the following indications:

Lower limb

Compression: 20-30 mmHg and 15-20 mmHg

JOBST® Farrow Toe Cap

Mild to moderate swelling from venous, lymphatic and other swelling disorders; mild to moderate post thrombotic syndrome (PTS); chronic venous disease (CVD, C1-C5).

Compression: 30-40 mmHg

Moderate to severe swelling from venous, lymphatic and other swelling disorders; chronic venous disease (CVD, C3-C6); venous leg ulcer; mild to moderate post thrombotic syndrome (PTS).

Upper limb

Compression: 20-30 mmHg and 15-20 mmHg

JOBST® Farrow Glove

Mild to moderate swelling from lymphatic and other swelling disorders.

Compression: 30-40 mmHg

Moderate to severe swelling from lymphatic and other swelling disorders.

Non-compressive liners are stockings/sleeves for use under JOBST® FarrowWrap® adjustable compression devices to help protect the skin.

Accessories comprise Farrow GarmentGrip, used with JOBST® FarrowWrap® adjustable compression devices to help maintain hold and Farrow Foams, designed to evenly distribute the pressure and to add padding under FarrowWrap® adjustable compression devices.

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

3. Material composition

| Brand | % Polyamide (Nylon) | % Elastane (Spandex) | % Cotton | % Polyester | % Neoprene | % Polyurethane |
|---------------------------------------|---------------------|----------------------|----------|-------------|------------|----------------|
| Adjustable compression devices | | | | | | |
| JOBST® FarrowWrap® LITE | 82 | 18 | | | | |
| JOBST® FarrowWrap® STRONG | 82 | 18 | | | | |
| JOBST® FarrowWrap® CLASSIC | 32 | 5 | 47 | 16 | | |
| JOBST® FarrowWrap® BASIC | 89 | 11 | | | | |
| JOBST® FarrowWrap® 4000 | 89 | 11 | | | | |
| Non-compressive liners | | | | | | |
| JOBST® Farrow Liner | 94 | 6 | | | | |
| JOBST® Farrow Terry Cloth liner | | 15 | 85 | | | |
| Medical compression garments | | | | | | |
| JOBST® Farrow Glove | 59 | 41 | | | | |
| JOBST® Farrow Toe Cap | 59 | 41 | | | | |
| JOBST® FarrowHybrid® ADI | 83 | 17 | | | | |
| Accessories | | | | | | |
| JOBST® Farrow GarmentGrip Roll (Neo) | 48 | 5 | | | 47 | |
| JOBST® FarrowFoam Roll or Squares | | | | | | 100% |

4. Applications

Follow the product specific donning instructions provided in the back of the insert card. Please also follow your healthcare provider's instructions.

Application videos, are available at:

www.jobst.com

To avoid pressure marks, smooth out any wrinkles, while donning.

To achieve the indicated compression (mmHg), **JOBST FarrowWrap® garments should be stretched as follows:**

LITE & STRONG

Fix the VELCRO® brand pieces applying a moderate stretch which resembles an approximate stretch of the garment by 15%.

INSTRUCTIONS FOR USE

EN

CLASSIC & BASIC

Fix the VELCRO® brand pieces applying a gentle stretch which resembles an approximate stretch of the garment by 10%.

4000

Fix the VELCRO® brand pieces applying a stretch according to the values stated in the table below:

| XS | S | M | L |
|--------|---|--------|---|
| 15-20% | | 20-30% | |

PLEASE NOTE:

- Initial fitting should be done under the supervision of a health care professional, to determine comfortable compression level; based on patients need and tolerance.
- JOBST® FarrowWrap® TTF must be trimmed by a medical professional prior to any donning. If it has not been trimmed, please contact your provider for further information.
- Following application, if the patient exhibits signs of skin reddening, strictures, pressure marks, pain, reduced color or numbness in the extremities, the compression wrap system should be removed.
- It is recommended to wear a JOBST® FarrowWrap® brand liner under JOBST® FarrowWrap® adjustable compression devices to help protect the skin.
- Consult your health care professional with regards to combining different FarrowWrap® products. JOBST® FarrowWrap® kneepiece LITE should only be used in combination with JOBST® FarrowWrap® thighpiece LITE. JOBST® FarrowWrap® kneepiece STRONG should only be used in combination with JOBST® FarrowWrap® thighpiece STRONG and CLASSIC. It should be considered combining the JOBST® FarrowWrap® footpiece with a JOBST® FarrowWrap® legpiece for the management of legedema.
- It may be necessary to loosen the VELCRO® brand pieces of the garment during night-time wear. For JOBST® FarrowHybrid®, JOBST® Farrow glove and JOBST® Farrow Toe Cap it is recommended to consider removal when lying down. Please reach out to your health care professional for further advice.
- Jewelry in the affected area should be removed before donning.
- Excess liner can be folded down over the top of the compression garment.
- Garment may need to be tightened periodically to ensure correct fit.
- To prevent the VELCRO® brand pieces from sticking to other items, roll each band back onto itself and affix VELCRO® brand pieces.

5. Contraindications

JOBST® FarrowWrap®/Farrow/FarrowHybrid® products are contraindicated (should NOT be worn) if you have one or more of the following medical conditions:

- Untreated vein infection (septic phlebitis)
- Uncontrolled (decompensated) congestive heart failure
- Untreated and/or advancing skin infection (cellulitis) involving the area where the garment will be applied
- Phlegmasia coerulea dolens (large blood clot in the vein of the area treated)
- Patients unable to express pain or discomfort
- Circumference less than the indicated ranges (see sizing chart in the insert cards)
- Incompatibility to material

Compression 20-30 mmHg and 15-20mmHg for JOBST® Farrow Toe Cap/JOBST® Farrow Glove

- Severe arterial disease
- Moderate arterial disease in the presence of peripheral neuropathy

Compression 30-40 mmHg

- Moderate to severe arterial disease
- Mild arterial disease in the presence of peripheral neuropathy

Note: JOBST® Farrow products are designed to be used together and should not be used with non-JOBST® Farrow products.

6. Warnings and Precautions

Open wounds must be covered with an appropriate dressing before the liner and/or compression garment is put on. For lower limb garments, appropriate footwear must be worn when walking to help prevent falls.

Consult your physician if you have experienced or develop:

- Skin infection or weeping
- Mild to moderate arterial disease
- Loss of feeling in the area where the garment has been applied (peripheral neuropathy)
- History of blood clots
- Controlled CHF (congestive heart failure)
- Carpal Tunnel Syndrome (only for upper limb)
- Large bunions (only for JOBST® Farrow Toe Cap, JOBST® FarrowHybrid®)
- Immobility (only for JOBST® Farrow Toe Cap)

7. Size table

For standard sized garments (Ready-to-Wear & Trim-to-Fit): please consult the sizing chart in the insert card.

INSTRUCTIONS FOR USE

EN

8. Care instructions

| Brand | Washing | Bleaching | Drying | Ironing | Professional Cleaning |
|--------------------------------------|---------|-----------|--------|---------|-----------------------|
| JOBST® FarrowWrap® LITE | | | | | |
| JOBST® FarrowWrap® STRONG | | | | | |
| JOBST® FarrowWrap® CLASSIC | | | | | |
| JOBST® FarrowWrap® BASIC | | | | | |
| JOBST® FarrowWrap® 4000 | | | | | |
| JOBST® Farrow Glove | | | | | |
| JOBST® Farrow Toe Cap | | | | | |
| JOBST® Farrow Terry Cloth liner | | | | | |
| JOBST® Farrow Liner | | | | | |
| JOBST® FarrowHybrid® ADI | | | | | |
| JOBST® Farrow GarmentGrip Roll (Neo) | | | | | |
| JOBST® FarrowFoam Roll or Squares | | | | | |

Machine Wash at 30°C, mild process

Do not Bleach

Do not Dry Clean

Machine Wash at 40°C, mild process

Tumble Dry, low heat

Do not Iron

Hand Wash

Dry Flat

Do not Tumble dry

9. Storage and/or special handling conditions that apply

Please protect your compression garments from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature.

immediate use and no expiry date for storage is indicated on these.

10. Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical Inc. (BSN medical GmbH within the European Union) and the competent authority of your state.

The wearing period is not more than 6 months for JOBST® Farrow products. The manufacturing date/ expiry date for storage is printed on the package label next to the corresponding symbol. Custom Made products are products intended for

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| Brand | Washing | Bleaching | Drying | Ironing | Professional Cleaning |
|--------------------------------------|---------|-----------|--------|---------|-----------------------|
| JOBST® FarrowWrap® LITE | | | | | |
| JOBST® FarrowWrap® STRONG | | | | | |
| JOBST® FarrowWrap® CLASSIC | | | | | |
| JOBST® FarrowWrap® BASIC | | | | | |
| JOBST® FarrowWrap® TTF | | | | | |
| JOBST® FarrowWrap® 4000 | | | | | |
| JOBST® Farrow Glove | | | | | |
| JOBST® Farrow Toe Cap | | | | | |
| JOBST® Farrow Terry Cloth liner | | | | | |
| JOBST® Farrow Liner | | | | | |
| JOBST® FarrowHybrid® ADI | | | | | |
| JOBST® Farrow GarmentGrip Roll (Neo) | | | | | |
| JOBST® FarrowFoam Roll or Squares | | | | | |

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VELCRO® is registered trademark of Velcro BVBA.
Importer for the European Union: BSN-Jobst GmbH
Australian Distributor: BSN medical (Aust.) Pty Ltd.