

Fracture Management & Physiotherapy Catalogue



Essity – We Know Casting

When faced with orthopedic trauma, optimal care is essential for the best healing possible. When it comes to broken bones or injured ligaments, Delta-Cast®, AquaCast® and Actimove® are trusted partners. As the global market leader in Fracture Management, drawing on a strong brand heritage, we have a comprehensive and innovative portfolio to rely on.

Delta-Cast®, AquaCast® and Actimove® take pride in supporting customers through service and training, providing knowledge and tools to overcome any casting challenge that comes their way.

ESSITY'S WIDE RANGE OF THERAPY SOLUTIONS INCLUDE:

Splinting – Easy to handle and extremely versatile, this splinting system meets the demands of modern users.

Plaster of Paris – Well-fitting, strong and durable casts for a traditional application.

Cast Tape – Designed for ease of application and patient comfort in mind.

Padding & Stockinette – Natural and synthetic undercast materials that maximize patient comfort during the treatment with a cast.

Cast Removal – Durable saws that provide high cutting performance with low noise and vibration levels.

Casting Accessories – From scissors to spreaders, additional adhesive padding and cast shoes, there is a wide variety of accessories to aid technicians and patients.

Orthopedic Supports and Bracing – Supports that you need from sling, splints and walkers.



Fracture Management

SYNTHETIC CAST TAPES

Delta-Lite® Plus – Fiberglass Cast Tape	5
Delta-Cast® Conformable – Polyester Cast Tape	6
Delta-Cast® Elite – Polyester Cast Tape	7
Delta-Cast® Prints – Polyester Printed Cast Tape	8
Delta-Cast® Soft – Polyester Semi-Rigid Cast Tape	9

SYNTHETIC SPLINTS

Dynacast® Prelude – Fiberglass Splinting System	10
Delta-Cast® Prelude – Polyester Splinting System	11

PLASTER OF PARIS

Gypsona® – Plaster of Paris Bandages & Slabs	12
Gypsona® Roll – Plaster of Paris Splinting System	12

UNDERCAST MATERIAL

Delta Terry-Net™ S – Synthetic Terry Cloth Stockinette	13
Delta-Net® / Protouch® – Synthetic Stockinette	14
Protouch® – Cotton Stockinette	14
AquaCast® Liners	15
Saw Stop® DE-FLEX™ by Aquacast®	16
Hipster® Pantaloone Liner by Aquacast®	17
Delta-Dry® – Water Resistant Cast Padding and Stockinette	18
Delta-Dry® Pantaloone	19
Specialist® 100 / Specialist® 100 Sterile – Cotton Cast Padding	20
Sof-Rol® / Sof-Rol® Sterile – Natural Cast Padding	20
Soffban® Synthetic – Cast Padding	21
Delta-Rol® – Synthetic Cast Padding	21

ACCESSORIES

Delta Terry-Net™ – Adhesive Fleece Edger	22
Delta Terry-Net™ – Adhesive Felt Padding	22
Delta Terry-Net™ – Adhesive Terry Cloth/Foam Padding	23
Delta Terry-Net™ – Adhesive Hook & Stretch Loop Closures	23
Cast Cutters	24
Cast Dust Vacuums	24
Cast Saw Blades	25
Zip Stick	25
Delta-Cast® Serene™	26
Knee Rest	27
Adjustable Cast Stand	27
Casting Instruments	27
Tensoplast® STK – Skin Traction Kit	28
Actimove® Cast Shoe Rocker Sole	28

Fracture Management (continued)

ORTHOPEDIC SUPPORTS

Actimove® Standard Walker & Standard Air Walker	29
Actimove® Closed Shell Air Walker	30
Actimove® Sling – Versatile Upper Limb Support	31

Physiotherapy – Bandages and Tapes

STRAPPING TAPES

Strappal® – Superior Strapping Tape	33
Leukotape® P – High Adhesive Rigid Strapping Tape	33
Coach® by Actimove® Rigid Cloth Porous Adhesive Strapping Tape	34
Zonas® by Actimove® Rigid Cloth Porous Adhesive Strapping Tape	34

ELASTIC ADHESIVE BANDAGES AND TAPES

Elastikon® by Actimove® Elastic Adhesive Tape	34
Tensoplast® / Tensoplast® LF / Tensosport® – Robust Elastic Adhesive Bandage	35
Leukotape® K – Elastic Adhesive Tape for Pain Relief	35
Tensoplast® – Robust Elastic Adhesive Tape	36
Tensoplast® – Athletic Elastic Adhesive Tape	36
Lightplast® Pro – Light Elastic Adhesive Tape	37

COHESIVE BANDAGES

Co-Plus® LF – Elastic Cohesive Bandage	37
--	----

ELASTIC NON-ADHESIVE BANDAGES

Elastocrepe® – Non-Adhesive Elastic Bandage	38
---	----

TUBULAR BANDAGES

Tensogrip® – Tubular Elastic Support Bandage	38
--	----

Delta-Lite® Plus

Fiberglass Cast Tape

Delta-Lite® Plus provides extra rigid casts with smooth surfaces and edges. The advanced water activated resin system provides low tack properties allowing for ease of application.

FEATURES AND BENEFITS

- Smooth surface
- Rigid support
- Good conformability
- Excellent lamination
- Consistent and reliable working and set time
- Not made with natural rubber latex



Dimensions	Colour	Code	Box Contents	Selling Unit
2.5 cm x 1.8 m	White	7345800	10 rolls	Box
5 cm x 3.6 m	White	7345801	10 rolls	Box
7.5 cm x 3.6 m	White	7345802	10 rolls	Box
10 cm x 3.6 m	White	7345803	10 rolls	Box
12.5 cm x 3.6 m	White	7345804	10 rolls	Box
5 cm x 3.6 m	Dark Blue	7345820	10 rolls	Box
7.5 cm x 3.6 m	Dark Blue	7345821	10 rolls	Box
10 cm x 3.6 m	Dark Blue	7345822	10 rolls	Box
12.5 cm x 3.6 m	Dark Blue	7345823	10 rolls	Box
5 cm x 3.6 m	Green	7345825	10 rolls	Box
7.5 cm x 3.6 m	Green	7345826	10 rolls	Box
10 cm x 3.6 m	Green	7345827	10 rolls	Box
12.5 cm x 3.6 m	Green	7345828	10 rolls	Box
5 cm x 3.6 m	Red	7345830	10 rolls	Box
7.5 cm x 3.6 m	Red	7345831	10 rolls	Box
10 cm x 3.6 m	Red	7345832	10 rolls	Box
5 cm x 3.6 m	Light Blue	7345835	10 rolls	Box
7.5 cm x 3.6 m	Light Blue	7345836	10 rolls	Box
10 cm x 3.6 m	Light Blue	7345837	10 rolls	Box
12.5 cm x 3.6 m	Light Blue	7345838	10 rolls	Box
5 cm x 3.6 m	Black	7345845	10 rolls	Box
7.5 cm x 3.6 m	Black	7345846	10 rolls	Box
10 cm x 3.6 m	Black	7345847	10 rolls	Box
5 cm x 3.6 m	Mixed Colours*	7345850	10 rolls	Box
7.5 cm x 3.6 m	Mixed Colours*	7345851	10 rolls	Box
10 cm x 3.6 m	Mixed Colours*	7345852	10 rolls	Box
5 cm x 3.6 m	Pink	7345855	10 rolls	Box
7.5 cm x 3.6 m	Pink	7345856	10 rolls	Box
10 cm x 3.6 m	Pink	7345857	10 rolls	Box
5 cm x 3.6 m	Purple	7345860	10 rolls	Box
7.5 cm x 3.6 m	Purple	7345861	10 rolls	Box
10 cm x 3.6 m	Purple	7345862	10 rolls	Box
5 cm x 3.6 m	Yellow	7345865	10 rolls	Box
7.5 cm x 3.6 m	Yellow	7345866	10 rolls	Box
10 cm x 3.6 m	Yellow	7345867	10 rolls	Box

Reinforcing Slabs

5 cm x 20 cm	White	7348500	10 splints	Box
7.5 cm x 20 cm	White	7348501	10 splints	Box
7.5 cm x 38 cm	White	7227200	10 splints	Box
10 cm x 38 cm	White	7348503	10 splints	Box
7.5 cm x 76 cm	White	7348502	10 splints	Box
10 cm x 90 cm	White	7227201	10 splints	Box
12.5 cm x 90 cm	White	7348504	10 splints	Box

* Mixed Colours: Dark Blue, Red, Light Blue, Purple and Pink

Delta-Cast® Conformable

Polyester Cast Tape

Delta-Cast® Conformable is an easy-to-use polyester cast tape that produces versatile, lightweight and strong casts with high conformability. Ideal for applications where patient fit and comfort are critical.

FEATURES AND BENEFITS

- High conformability
- Smooth finish and soft edges
- Rigid immobilization
- High radiolucency
- Excellent lamination between layers
- Excellent for FCT casting or removable casts
- Not made with natural rubber latex



Dimensions	Colour	Code	Box Contents	Selling Unit
2.5 cm x 1.8 m	White	7228000	10 rolls	Box
5 cm x 3.6 m	White	7228001	10 rolls	Box
7.5 cm x 3.6 m	White	7228002	10 rolls	Box
10 cm x 3.6 m	White	7228003	10 rolls	Box
12.5 cm x 3.6 m	White	7228004	10 rolls	Box
5 cm x 3.6 m	Dark Blue	7228200	10 rolls	Box
7.5 cm x 3.6 m	Dark Blue	7228201	10 rolls	Box
10 cm x 3.6 m	Dark Blue	7228202	10 rolls	Box
5 cm x 3.6 m	Green	7228700	10 rolls	Box
7.5 cm x 3.6 m	Green	7228701	10 rolls	Box
10 cm x 3.6 m	Green	7228702	10 rolls	Box
5 cm x 3.6 m	Red	7228500	10 rolls	Box
7.5 cm x 3.6 m	Red	7228501	10 rolls	Box
10 cm x 3.6 m	Red	7228502	10 rolls	Box
5 cm x 3.6 m	Black	7228800	10 rolls	Box
7.5 cm x 3.6 m	Black	7228801	10 rolls	Box
10 cm x 3.6 m	Black	7228802	10 rolls	Box
5 cm x 3.6 m	Mixed Colours*	7228910	10 rolls	Box
7.5 cm x 3.6 m	Mixed Colours*	7228911	10 rolls	Box
10 cm x 3.6 m	Mixed Colours*	7228912	10 rolls	Box
5 cm x 3.6 m	Pink	7228100	10 rolls	Box
7.5 cm x 3.6 m	Pink	7228101	10 rolls	Box
10 cm x 3.6 m	Pink	7228102	10 rolls	Box
5 cm x 3.6 m	Purple	7228600	10 rolls	Box
7.5 cm x 3.6 m	Purple	7228601	10 rolls	Box
10 cm x 3.6 m	Purple	7228602	10 rolls	Box
5 cm x 3.6 m	Yellow	7228300	10 rolls	Box
7.5 cm x 3.6 m	Yellow	7228301	10 rolls	Box
10 cm x 3.6 m	Yellow	7228302	10 rolls	Box
5 cm x 3.6 m	Orange	7228400	10 rolls	Box
7.5 cm x 3.6 m	Orange	7228401	10 rolls	Box
10 cm x 3.6 m	Orange	7228402	10 rolls	Box

* Mixed Colours: Dark Blue, Red, Black, Purple and Pink

Delta-Cast® Elite

Polyester Cast Tape

Delta-Cast® Elite is a polyester cast tape that offers strength and durability. It features extensible yarns that provide excellent moulding capabilities and a smooth cast surface for patient comfort.

FEATURES AND BENEFITS

- Smooth finish and soft edges
- Rigid immobilization
- High radiolucency
- High conformability
- Excellent lamination between layers
- Excellent alternative to fiberglass cast tape
- Not made with natural rubber latex



Dimensions	Colour	Code	Box Contents	Selling Unit
2.5 cm x 1.8 m	White	7227904	10 rolls	Box
5 cm x 3.6 m	White	7227900	10 rolls	Box
7.5 cm x 3.6 m	White	7227901	10 rolls	Box
10 cm x 3.6 m	White	7227902	10 rolls	Box
12.5 cm x 3.6 m	White	7227903	10 rolls	Box
5 cm x 3.6 m	Dark Blue	7227600	10 rolls	Box
7.5 cm x 3.6 m	Dark Blue	7227601	10 rolls	Box
10 cm x 3.6 m	Dark Blue	7227602	10 rolls	Box
12.5 cm x 3.6 m	Dark Blue	7227603	10 rolls	Box
5 cm x 3.6 m	Green	7345400	10 rolls	Box
7.5 cm x 3.6 m	Green	7345401	10 rolls	Box
10 cm x 3.6 m	Green	7345402	10 rolls	Box
5 cm x 3.6 m	Red	7227500	10 rolls	Box
7.5 cm x 3.6 m	Red	7227501	10 rolls	Box
10 cm x 3.6 m	Red	7227502	10 rolls	Box
5 cm x 3.6 m	Light Blue	7345300	10 rolls	Box
7.5 cm x 3.6 m	Light Blue	7345301	10 rolls	Box
10 cm x 3.6 m	Light Blue	7345302	10 rolls	Box
5 cm x 3.6 m	Black	7227800	10 rolls	Box
7.5 cm x 3.6 m	Black	7227801	10 rolls	Box
10 cm x 3.6 m	Black	7227802	10 rolls	Box
5 cm x 3.6 m	Mixed Colours*	7345700	10 rolls	Box
7.5 cm x 3.6 m	Mixed Colours*	7345701	10 rolls	Box
10 cm x 3.6 m	Mixed Colours*	7345702	10 rolls	Box
5 cm x 3.6 m	Pink	7345500	10 rolls	Box
7.5 cm x 3.6 m	Pink	7345501	10 rolls	Box
10 cm x 3.6 m	Pink	7345502	10 rolls	Box
5 cm x 3.6 m	Purple	7227700	10 rolls	Box
7.5 cm x 3.6 m	Purple	7227701	10 rolls	Box
10 cm x 3.6 m	Purple	7227702	10 rolls	Box
5 cm x 3.6 m	Orange	7345600	10 rolls	Box
7.5 cm x 3.6 m	Orange	7345601	10 rolls	Box
10 cm x 3.6 m	Orange	7345602	10 rolls	Box

Reinforcing Slabs 3 Ply

5 cm x 20 cm	White	7454507	10 splints	Box
7.5 cm x 20 cm	White	7454508	10 splints	Box
7.5 cm x 38 cm	White	7454509	10 splints	Box

Reinforcing Slabs 5 Ply

10 cm x 38 cm	White	7454503	10 splints	Box
10 cm x 90 cm	White	7454504	10 splints	Box
12.5 cm x 90 cm	White	7454505	10 splints	Box

* Mixed Colours: Dark Blue, Red, Light Blue, Purple and Pink

Delta-Cast® Prints

Polyester Printed Cast Tape

Add personality to casts with colourful printed casting material. In addition to helping relieve patient anxiety and making casts more enjoyable to wear, the finished cast is strong and lightweight. Patterns are colourfast and will not interfere with radiographic viewing.

FEATURES AND BENEFITS

- Same advantages as Delta-Cast® Elite
- Unique style
- Long lasting, vibrant prints
- Not made with natural rubber latex

Dimensions	Colour	Code	Box Contents	Selling Unit
5 cm x 3.6 m	Pastel Prints	7227317	10 rolls	Box
7.5 cm x 3.6 m	Pastel Prints	7227307	10 rolls	Box
5 cm x 3.6 m	Teddy Bears	7227305	10 rolls	Box
5 cm x 3.6 m	Dinosaurs	7227302	10 rolls	Box
5 cm x 3.6 m	Soccer Balls	7227308	10 rolls	Box
7.5 cm x 3.6 m	Soccer Balls	7227309	10 rolls	Box
5 cm x 3.6 m	Camouflage	7227322	10 rolls	Box
7.5 cm x 3.6 m	Camouflage	7227306	10 rolls	Box
5 cm x 3.6 m	Paw Prints	7227320	10 rolls	Box
7.5 cm x 3.6 m	Paw Prints	7227321	10 rolls	Box
5 cm x 3.6 m	Pink Camo	7227340	10 rolls	Box
7.5 cm x 3.6 m	Pink Camo	7227341	10 rolls	Box



Delta-Cast® Soft

Polyester Semi-Rigid Cast Tape

Delta-Cast® Soft is a semi-rigid polyester cast tape, which allows for greater range of movement and earlier mobility to reduce some of the problems of rigid immobilization, such as muscle atrophy.

FEATURES AND BENEFITS

- Smooth finish and soft edges
- High radiolucency
- Provides support while allowing a limited range of motion
- Can be removed with scissors or by unwrapping
- Not made with natural rubber latex

Dimensions	Colour	Code	Box Contents	Selling Unit
2.5 cm x 1.8 m	White	7270802	10 rolls	Box
5 cm x 3.6 m	White	7270812	10 rolls	Box
7.5 cm x 3.6 m	White	7270822	10 rolls	Box
10 cm x 3.6 m	White	7270832	10 rolls	Box
12.5 cm x 3.6 m	White	7270842	10 rolls	Box
5 cm x 3.6 m	Blue	7270805	10 rolls	Box
7.5 cm x 3.6 m	Blue	7270815	10 rolls	Box
10 cm x 3.6 m	Blue	7270825	10 rolls	Box
5 cm x 3.6 m	Red	7270808	10 rolls	Box
7.5 cm x 3.6 m	Red	7270818	10 rolls	Box
10 cm x 3.6 m	Red	7270828	10 rolls	Box
5 cm x 3.6 m	Black	7270801	10 rolls	Box
7.5 cm x 3.6 m	Black	7270811	10 rolls	Box
10 cm x 3.6 m	Black	7270821	10 rolls	Box
5 cm x 3.6 m	Paw Prints	7270809	10 rolls	Box
7.5 cm x 3.6 m	Paw Prints	7270819	10 rolls	Box



Dynacast® Prelude

Fiberglass Splinting System

Dynacast® Prelude is a cost-effective splinting system of choice for every phase of immobilization. Its easy-stretch padding covers all edges, while its roll form eliminates any wasted product.

FEATURES AND BENEFITS

- No wrong side application
- Minimum water needed to activate
- Easy stretch padding covers all edges
- Can be opened for customizing, trimming or reinforcing
- Interlocking Single Layer Technology
- Additional sizes 2.5 cm and 15 cm
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Case Contents	Selling Unit
Roll Format				
2.5 cm x 4.6 m	7141800	1 roll	2 boxes	Case
5 cm x 4.6 m	7142000	1 roll	2 boxes	Case
7.5 cm x 4.6 m	7142100	1 roll	2 boxes	Case
10 cm x 4.6 m	7141500	1 roll	2 boxes	Case
12.5 cm x 4.6 m	7141600	1 roll	2 boxes	Case
15 cm x 4.6 m	7141700	1 roll	2 boxes	Case
Pre-Cut Splints				
7.5 cm x 30 cm	4722200	5 splints	-	Box
7.5 cm x 40 cm	4722300	5 splints	-	Box
10 cm x 30 cm	4722400	5 splints	-	Box
10 cm x 40 cm	4722500	5 splints	-	Box
10 cm x 76 cm	4722600	4 splints	-	Box
12.5 cm x 65 cm	4722700	4 splints	-	Box
12.5 cm x 76 cm	4723500	4 splints	-	Box
Dynacast® Prelude Solo - Unpadded Pre-Cut Splints				
5 cm x 25 cm	7296300	10 splints	-	Box
7.5 cm x 30 cm	7296301	10 splints	-	Box
7.5 cm x 87 cm	7296302	10 splints	-	Box
10 cm x 38 cm	7296303	10 splints	-	Box
10 cm x 75 cm	7296304	10 splints	-	Box
12.5 cm x 75 cm	7296305	10 splints	-	Box
15 cm x 30 cm	7296306	10 splints	-	Box



SUGGESTION

- Co-Plus® LF or Easifix® Cohesive bandages are ideal to hold Dynacast® Prelude splints in place and reduce the risk of slippage.





Delta-Cast® Prelude

Polyester Splinting System

Delta-Cast® Prelude is a splinting system with polyester substrate. In addition to all the great advantages as Dynacast® Prelude, Delta-Cast® Prelude offers a highly comfortable padding that allows skin to breathe, as well as smooth edges.

FEATURES AND BENEFITS

- Highly comfortable water resistant padding with 3D knit construction to allow skin to breath
- Quick drying padding reduces risk of skin maceration
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
2.5 cm x 4.6 m	7301801	1 roll	Box
5 cm x 4.6 m	7301802	1 roll	Box
7.5 cm x 4.6 m	7301803	1 roll	Box
10 cm x 4.6 m	7301804	1 roll	Box
12.5 cm x 4.6 m	7301805	1 roll	Box



Gypsona®

Plaster of Paris Bandages & Slabs

The traditional rich and creamy texture of Gypsona® allows intimate moulding and creates a smooth, appealing finished cast, making Gypsona® the world's leading brand.

FEATURES AND BENEFITS

- Leno gauze for extra stability
- Creamy texture
- Low plaster loss
- Central plastic core
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Case Contents	Selling Unit
Bandages - Extra Fast Setting / 2 minutes				
5 cm x 2.7 m	LPL23	24 rolls	3 boxes	Case
7.5 cm x 2.7 m	LPL33	12 rolls	6 boxes	Case
10 cm x 2.7 m	LPL43	12 rolls	6 boxes	Case
15 cm x 2.7 m	LPL63	12 rolls	6 boxes	Case
20 cm x 2.7 m	LPL83	12 rolls	4 boxes	Case
10 cm x 4.5 m	LPL45	12 rolls	6 boxes	Case
15 cm x 4.5 m	LPL65	12 rolls	4 boxes	Case
20 cm x 4.5 m	LPL85	12 rolls	4 boxes	Case
90 cm x 5.4 m	7198737	3 rolls	-	Case
Slabs - Extra Fast Setting / 2 minutes				
7.5 cm x 75 cm	L5330	50 splints	6 boxes	Case
10 cm x 75 cm	L5430	50 splints	6 boxes	Case
12.5 cm x 75 cm	L5530	50 splints	6 boxes	Case
5 ply - 10 cm x 18 m	LP542	1 folded slab	-	Box
5 ply - 15 cm x 18 m	LP562	1 folded slab	-	Box



Gypsona® Roll

Plaster of Paris Splinting System

The foam and cotton flannel design, combined with extra-fast setting plaster makes Gypsona® Roll a clinician's product of choice for low-cost splint applications.

FEATURES AND BENEFITS

- All-in-one splinting system
- Extra-fast setting
- Easy to cut for custom fitting
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Selling Unit
10 Layers			
7.5 cm x 7.3 m	5702	1 roll	Box
10 cm x 7.3 m	5703	1 roll	Box
12.5 cm x 7.3 m	5704	1 roll	Box
15 Layers			
10 cm x 7.3 m	5751	1 roll	Box
12.5 cm x 7.3 m	5752	1 roll	Box
15 cm x 7.3 m	5753	1 roll	Box

Delta Terry-Net™ S

Synthetic Terry Cloth Stockinette

Delta Terry-Net™ S synthetic terry cloth stockinette is constructed of breathable synthetic fibers to help avoid skin maceration. The terry-net design provides additional padding and stretches to three times its normal width to conform to complex anatomies.

FEATURES AND BENEFITS

- Soft synthetic fibers
- Stretches to 3 times normal width
- Available in pre-cut version with built-in thumb liner
- Dispensing box
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Case Contents	Selling Unit
Roll Format				
5 cm x 13.5 m	7230600	1 roll	2 boxes	Case
7.5 cm x 13.5 m	7230601	1 roll	2 boxes	Case
10 cm x 13.5 m	7230602	1 roll	2 boxes	Case
12.5 cm x 13.5 m	7230603	1 roll	-	Box
15 cm x 13.5 m	7230604	1 roll	-	Box
Pre-Cut Format				
5 cm x 41 cm - Short	7230802	10 stockinettes	-	Box
7.5 cm x 41 cm - Short	7230800	10 stockinettes	-	Box
5 cm x 48 cm - Long	7230803	10 stockinettes	-	Box
7.5 cm x 48 cm - Long	7230801	10 stockinettes	-	Box



Delta-Net® / Protouch®

Synthetic Stockinette

Delta-Net® / Protouch® synthetic stockinettes stretch to more than twice their widths to conform comfortably to complex anatomies.

FEATURES AND BENEFITS

- Synthetic fibers
- Good conformability
- Not made with natural rubber latex



Dimensions	Colour	Code	Box Contents	Case Contents	Selling Unit
Delta-Net®					
5 cm x 22.75 m	White	7230700	1 roll	2 boxes	Case
7.5 cm x 22.75 m	White	7230701	1 roll	2 boxes	Case
10 cm x 22.75 m	White	7230702	1 roll	2 boxes	Case
15 cm x 22.75 m	White	7230703	1 roll	-	Box
20 cm x 22.75 m	White	7230704	1 roll	-	Box
2.5 cm x 22.75 m	Black	7272300	1 roll	2 boxes	Case
5 cm x 22.75 m	Black	7272301	1 roll	2 boxes	Case
7.5 cm x 22.75 m	Black	7272302	1 roll	2 boxes	Case
10 cm x 22.75 m	Black	7272303	1 roll	2 boxes	Case
15 cm x 22.75 m	Black	7272304	1 roll	-	Box

Protouch®					
2.5 cm x 22.75 m	White	301001	1 roll	-	Box
5 cm x 22.75 m	White	301002	1 roll	-	Box
7.5 cm x 22.75 m	White	301003	1 roll	-	Box
10 cm x 22.75 m	White	301004	1 roll	-	Box
15 cm x 22.75 m	White	301006	1 roll	-	Box
20 cm x 22.75 m	White	301008	1 roll	-	Box



Protouch®


Cotton Stockinette

Protouch® cotton stockinette is designed for patient protection under plaster of Paris and synthetic casting materials.

FEATURES AND BENEFITS

- High-grade cotton
- Good conformability
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
2.5 cm x 22.75 m	307001	1 roll	Box
5 cm x 22.75 m	307002	1 roll	Box
7.5 cm x 22.75 m	307003	1 roll	Box
10 cm x 22.75 m	307004	1 roll	Box
15 cm x 22.75 m	307006	1 roll	Box
20 cm x 22.75 m	307008	1 roll	Box

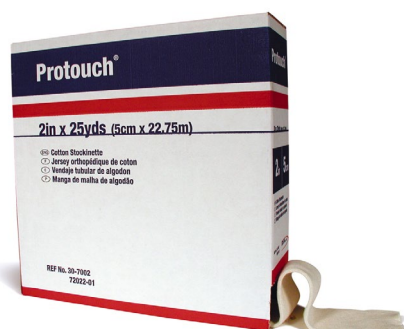

AquaCast® Liners

Waterproof Cast Padding

AquaCast® Liner is a comfortable waterproof, breathable cast padding for use under waterproof casting materials.

AquaCast® Form Fit Liner offers a unique, one-piece waterproof cast liner for short arm casts. Both products are moisture vapour permeable and made with pillows and channels to enhance comfort and facilitate drainage.

FEATURES AND BENEFITS

- Quick drying of cast allows patients to bathe, swim and exercise
- Decrease odour and enhance hygiene with the ability to thoroughly wash and rinse skin
- Supports favourable skin condition following cast removal
- Less itching due to smooth material surfaces
- Less recasting due to wet or damaged casts
- Not made with natural rubber latex

Description / Size	Code	Unit of Measure
AquaCast® Liner Rolls		
5 cm x 1.6 m	7336510	12 rolls / box
7.5 cm x 1.6 m	7336511	12 rolls / box
10 cm x 1.6 m	7336512	12 rolls / box
15 cm x 1.6 m	7336513	6 rolls / box
AquaCast® Form Fit Liners		
Form Fit Liner - Small	7336520	4 units / bag
Form Fit Liner - Medium	7336521	4 units / bag
Form Fit Liner - Large	7336522	4 units / bag



SAW STOP® DE-FLEX™

Protective Strip by AquaCast®

Saw Stop® DE-FLEX™ protective strips are tough, waterproof, flexible ribbons that provide an added layer of protection from a cast saw. Two styles are available, both of which are 1¼ inches wide and 10 feet long and feature adhesive backing for ease of use.

BLUE Saw Stop DE-FLEX Strips: this proprietary blue material is used in circumstances where the cast technician feels a layer of protection is needed. The material is designed to work with any padding material.

WHITE Saw Stop DE-FLEX Strips: this material provides all of the protection of our trademark Blue Saw Stop DE-FLEX Protective Strip, but is thinner, more conformable, more durable, and dries more rapidly. It should be used in the most demanding applications.

FEATURES AND BENEFITS

- Adhesive Backing for easy application
- Not made with natural rubber latex



Description	Colour	Code	Selling Unit
AquaCast® DE-FLEX Protective Strip			
3.175 cm x 3.05 m	Blue	7336570	1 / box
3.175 cm x 3.05 m	Blue	7336571	3 / box
3.175 cm x 3.05 m	Blue	7336572	10 / box
3.175 cm x 3.05 m	White	7336575	1 / box
3.175 cm x 3.05 m	White	7336576	3 / box
3.175 cm x 3.05 m	White	7336577	10 / box
Aquacast® Saw Stop DE-FLEX Zip Stick			
Regular	-	7336371	1 / box
Mini	-	7336372	1 / box



Hipster® Pantaloons® Liner

Waterproof Pantaloons


Hipster® Pantaloons® Liner by Aquacast® is a waterproof, breathable hip spica liner. It's innovative product design includes extra material in the perineal area to allow for easier cuffing. Slanted legs allow for better fit and less bunching of material.

FEATURES AND BENEFITS

- Single piece design creates a barrier that protects the padding from soiling
- Waterproof surface facilitates cleaning of excrement and urine from the skin
- Breathability keeps skin drier by allowing water, moisture and perspiration to evaporate
- With the addition of AquaCast® Liner Padding, the entire cast can be waterproof, further enhancing the user's experience

Size	Code	Age (months)	Unit of Measure
Pediatric Size 2 (waist ≤ 45.7 cm)	7336550	0-9	Single 1 / box
Pediatric Size 4 (waist ≤ 55.8 cm)	7336551	6-24	Single 1 / box
Pediatric Size 6 (waist ≤ 76.2 cm)	7336552	18-36	Single 1 / box
Pediatric Size 8 (waist ≤ 86.3 cm)	7336553	> 36	Single 1 / box
Adult Size 20	7336555	-	Single 1 / box
Adult Size 24	7336556	-	Single 1 / box
Adult Size 30	7336557	-	Single 1 / box
Pediatric Size 2 (waist ≤ 45.7 cm)	7336560	0-9	Double 2 / box
Pediatric Size 4 (waist ≤ 55.8 cm)	7336561	6-24	Double 2 / box
Pediatric Size 6 (waist ≤ 76.2 cm)	7336562	18-36	Double 2 / box
Pediatric Size 8 (waist ≤ 86.3 cm)	7336563	> 36	Double 2 / box
Adult Size 20	7336565	-	Double 2 / box
Adult Size 24	7336566	-	Double 2 / box
Adult Size 30	7336567	-	Double 2 / box

HIPSTER®
Pantaloons™
Liner by **AquaCast®**

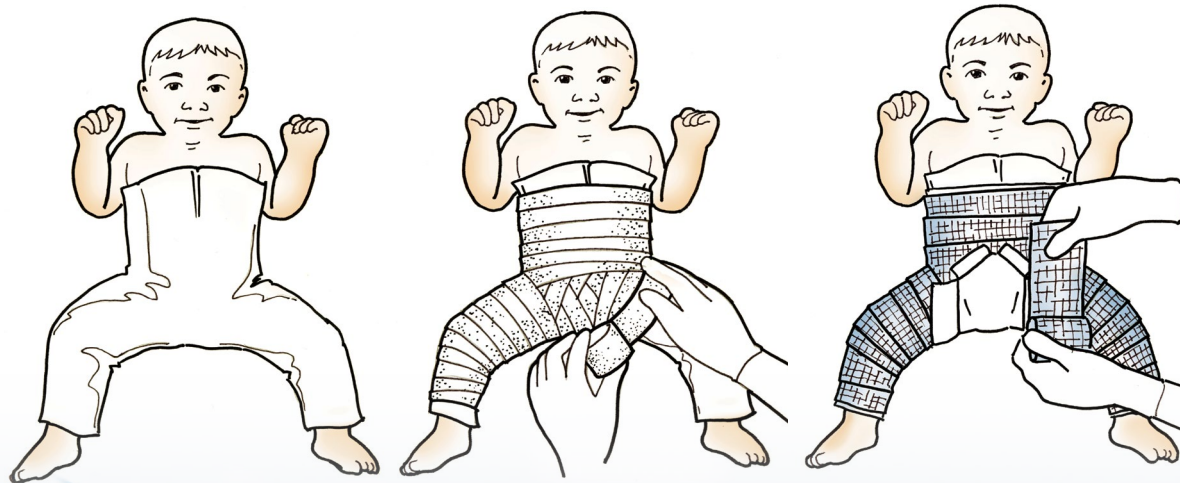

Delta-Dry®

Water Resistant Cast Padding and Stockinette

The patented Delta-Dry® material offers water resistance. The open knit structure of the fibers accelerate drainage after submerging in water. The patient's body temperature will then evaporate residual moisture quickly through the airy mesh (high MVTR). In this way, cast and skin dry quickly and skin feels dry in about 90 minutes after daily hygiene.

FEATURES AND BENEFITS

- Allows patients to shower, swim, bathe and wash hands
- Helps reduce odour under the cast
- Excellent conformability to body contours
- Provides comfortable, fast-drying casts
- Simple and quick application
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Selling Unit
Padding			
5 cm x 2.4 m	7344300	12 rolls	Box
7.5 cm x 2.4 m	7344301	12 rolls	Box
10 cm x 2.4 m	7344302	12 rolls	Box
Delta-Dry® Stockinette			
2.5 cm x 10 m	7456400	1 roll	Box
5 cm x 10 m	7456401	1 roll	Box
7.5 cm x 10 m	7456402	1 roll	Box
10 cm x 10 m	7456403	1 roll	Box
12.5 cm x 10 m	7456404	1 roll	Box
Delta-Dry® Softliner - Padded Stockinette			
2.5 cm x 10 m	7277901	1 roll	Box
5 cm x 10 m	7277902	1 roll	Box
7.5 cm x 10 m	7277903	1 roll	Box
10 cm x 10 m	7277904	1 roll	Box
12.5 cm x 10 m	7277905	1 roll	Box


[Table of Contents](#)

Delta-Dry® Pantaloons

Water Resistant Pantaloons

This pre-shaped, soft and water-resistant liner was developed on the basis of the proven, patented Delta-Dry® technology to simplify the treatment of Developmental Dysplasia of the Hip or Femur Fractures. Its stretch properties allow for an easy application and minimize creasing for a reduced risk of pressure marks.

FEATURES AND BENEFITS


- Comes in 3 sizes in a pre-cut shape
- Water resistance helps maintain patient's hygiene¹
- Enables bathing and flushing
- High water vapour permeability - dries in 90 minutes
- Permits an easy application, a suitable fit with little creasing
- Soft, flexible and cushioned for infant skin
- Not made with natural rubber latex

Description / Size	Code	Unit of Measure
Size 1 (thigh ≤ 25 cm)	7336500	1 / each
Size 2 (thigh 25-29.5 cm)	7336501	1 / each
Size 3 (thigh ≥ 29.5 cm)	7336502	1 / each



UNDERCAST MATERIAL

FRACTURE MANAGEMENT

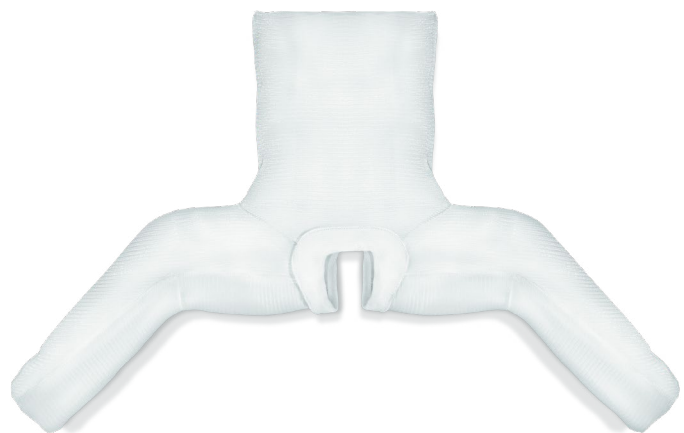

1. Wolff et al., J Pediatr Orthop 1995; 15(3): 386-388.

Specialist® 100 / Specialist® 100 Sterile

Cotton Cast Padding

Specialist® 100 has the product features cotton users desire; soft material that feathers and self-bonds to create a smooth undercast surface. Specialist® 100 is hand-tearable and applies easily.

FEATURES AND BENEFITS

- Breathable material helps to avoid skin maceration
- 100% cotton
- Low linting
- Hand-tearable
- Available in sterile version
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Case Contents	Selling Unit
Specialist® 100				
5 cm x 3.7 m	7230200	24 rolls	6 bags	Case
7.5 cm x 3.7 m	7230201	12 rolls	6 bags	Case
10 cm x 3.7 m	7230202	12 rolls	6 bags	Case
15 cm x 3.7 m	7230203	6 rolls	6 bags	Case
Specialist® 100 Sterile				
5 cm x 3.7 m	7234600	50 rolls	-	Box
7.5 cm x 3.7 m	7234601	50 rolls	-	Box
10 cm x 3.7 m	7234602	25 rolls	-	Box
15 cm x 3.7 m	7234603	25 rolls	-	Box



Sof-Rol® / Sof-Rol® Sterile

Rayon Absorbant Cast Padding

Sof-Rol® is a highly absorbent rayon material. This soft natural material creates a comfortable undercast surface for either synthetic or natural casting materials.

FEATURES AND BENEFITS

- Highly absorbent
- Easy application
- 100% viscose rayon
- Available in sterile version
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Case Contents	Selling Unit
Sof-Rol®				
5 cm x 3.6 m	9052	24 rolls	6 bags	Case
7.5 cm x 3.6 m	9033	12 rolls	6 bags	Case
10 cm x 3.6 m	9034	12 rolls	6 bags	Case
15 cm x 3.6 m	9036	6 rolls	6 bags	Case
Sof-Rol® Sterile				
5 cm x 3.6 m	7234510	20 rolls	-	Box
7.5 cm x 3.6 m	7234511	20 rolls	-	Box
10 cm x 3.6 m	7234512	20 rolls	-	Box
15 cm x 3.6 m	7234513	20 rolls	-	Box



Soffban® Synthetic

Cast Padding

Soffban® Synthetic is made of breathable synthetic material that does not hold moisture against the skin. It offers excellent conformability and provides cushioning to reduce the risk of rubbing and cast sores.

FEATURES AND BENEFITS

- Easy to tear and apply
- Excellent conformability
- Provides good cushioning reducing the risk of rubbing and cast sores
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Case Contents	Selling Unit
5 cm x 2.7 m	7146401	12 rolls	24 bags	Case
15 cm x 2.7 m	7146501	12 rolls	8 bags	Case
7.5 cm x 2.7 m	7146701	12 rolls	16 bags	Case
10 cm x 2.7 m	7148605	12 rolls	12 bags	Case

Delta-Rol®

Synthetic Cast Padding

Delta-Rol® is made of breathable synthetic material that wicks moisture away from the skin. Its conformable stretch allows narrow widths fitting around small anatomies without cutting and tearing.

FEATURES AND BENEFITS

- Conformable stretch
- Low linting
- Non-absorbent synthetic fibers
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Case Contents	Selling Unit
5 cm x 3.6 m	6882	24 rolls	3 bags	Case
7.5 cm x 3.6 m	6883	12 rolls	3 bags	Case
10 cm x 3.6 m	6884	12 rolls	3 bags	Case
15 cm x 3.6 m	8666	6 rolls	3 bags	Case

Delta Terry-Net™

Adhesive Fleece Edger

Delta Terry-Net™ adhesive fleece edger's unique stretch allows the user to pad easily around corners and contours, ensuring smooth cast edges. It also features high quality adhesive for secure placement.

FEATURES AND BENEFITS

- Highly conformable
- Unique stretch
- Secure adhesive
- Not made with natural rubber latex



Dimensions	Colour	Code	Box Contents	Selling Unit
3 cm x 13.5 m	Black	7229513	1 roll	Case

Delta Terry-Net™

Adhesive Felt Padding

Delta Terry-Net™ Adhesive Cloth/Foam Padding provides added protection and cushioning by combining a layer of soft terry cloth bonded to 0.32 cm/1/8" foam. The adhesive backing makes attaching the padding simple and easy.

FEATURES AND BENEFITS

- Soft orthopedic felt
- Adhesive back
- Not made with natural rubber latex



Dimensions	Code	Box Contents	Selling Unit
58 cm x 100 cm	54012	1 sheet	Box

Delta Terry-Net™

Adhesive Terry Cloth/Foam Padding

Delta Terry-Net™ Adhesive Felt Padding is an adhesive orthopedic felt, used for added patient comfort. Its adhesive backing makes attaching the padding simple and easy. It may also be used as a cast edge padding.

FEATURES AND BENEFITS

- Open cell foam
- Adhesive back
- Comfortable terry cloth cushion
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
58 cm x 100 cm	55012	1 sheet	Box



Delta Terry-Net™

Adhesive Hook & Stretch Loop Closures

This easy-to-use hook and loop system features high quality adhesive to ensure secure placement.

FEATURES AND BENEFITS

- Soft stretch loop
- High quality adhesive
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
Hook			
2.5 cm x 9 m	7229500	1 roll	Box
5 cm x 9 m	7229502	1 roll	Box
Loop			
2.5 cm x 13.7 m	7229501	1 roll	Box
5 cm x 13.7 m	7229503	1 roll	Box



American Orthopaedic™

Cast Cutter

FEATURES AND BENEFITS

- Specially designed for rugged use: the perfect tool with power and endurance
- Internally lubricated for longer service
- Switch is designed and placed so that it will not break if dropped
- Hex drive provides 4 blade rotations extending blade life by 50% compared to similar competitive models
- One year limited warranty



Description	Code	Selling Unit
Cast Cutter 115V HEX D	0295200	1 / each

American Orthopaedic™

Cast Dust Vacuum

FEATURES AND BENEFITS - Portable Cast Dust Vacuum

- Lightweight and mobile; easy movement from room to room or patient to patient
- Unique lift-out disposable filter cartridge makes disposal quicker, easier and cleaner than messy bags
- Quiet operation for patient comfort
- Function that allows the saw and vacuum to be turned on simultaneously with contact of the cast

Description	Code	Selling Unit
Vacuum Portable 115V Black	0295300	1 / each
Vacuum 115V Cast Dust AM	0295400	1 / each
American Ortho HEPA Disposable Cartridge Filter for 0295-300	0295175	1 / each
HEPA Filter Cartridge for 0295-400	0295410	1 / each



FEATURES AND BENEFITS - Cast Dust Vacuum

- Unique disposable filter cartridge
- Self-contained unit conveniently stores cast spreader, scissors, replacement blades and the filter storage caps
- Lightweight construction and convenient handle provide easy mobility
- Versatile; attaches easily to most cast cutters



Cast Saw Blades

FEATURES AND BENEFITS

- “Universal Hub” fits most popular brands of cast cutters, both pin and hex drive
- Hexagon drive provides 6 blade rotations for hexagon drive cast cutters



1. Electropolish blades.



2. PTFE coated blades.



3. Titanium Nitride
“Gold” coated blades.



4. Dicronite® coated
blades.


All blades are manufactured with stainless steel, hardened and supplied with different finishes to increase blade life.

Number	Code	Finish	Hub Design	Size	Selling Unit	Recomended Use
1	4183110	Electropolish	Universal	5 cm	6 / pkg.	Plaster, Wood, Bone
1	4183112	Electropolish	Universal	6.35 cm	6 / pkg.	Plaster, Wood, Bone
2	4183114	PTFE Coated	Universal	5 cm	1 / each	Plaster, Synthetic Cast Material
2	310265	PTFE Coated	Universal	6.35 cm	1 / each	Plaster, Synthetic Cast Material
3	310168	Titanium Nitride “Gold”	Universal	6.35 cm	1 / each	Plaster, Synthetic, Composites, Plastics
4	0295225	Dicronite	Universal	5 cm	1 / each	Plaster, Synthetic, Composites, Plastics
4	0295226	Dicronite	Universal	6.35 cm	1 / each	Plaster, Synthetic, Composites, Plastics

Zip Stick

Cast Removal Aid

This convenient cast aid provides additional patient protection during cast removal. The flexible stick slides under the cast and provides a smooth path for scissors or cast saw. Standard and metric rule adds functionality.

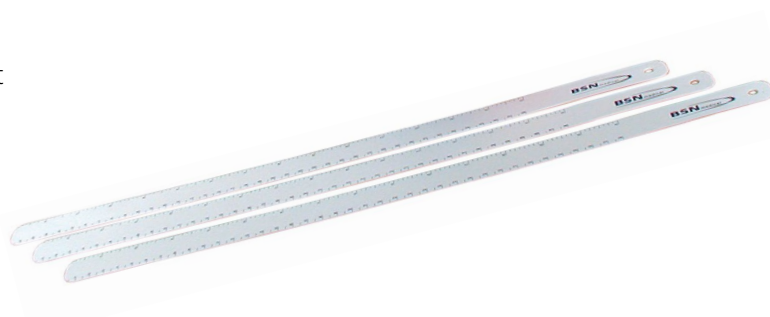

Description	Code	Selling Unit
Zip Stick White	7204629	1 / each

Delta-Cast® Serene™

Cast Remover

FEATURES AND BENEFITS

For you:

- **Cleaner:** Static blade does not generate airborne particles, keeping the cast room air clean without the need for a vacuum system
- **Easy to use:** Weight of the balanced device rests on the cast – enabling almost effortless one-handed operation
- **Quieter:** Minimizes the risk of long-term exposure to loud noises for a less stressful work environment
- **Effective:** Effortlessly and fully cuts through up to 10 layers of synthetic cast material
- **Simple maintenance:** Blade lasts for up to 200 cuts, is quick to change and easy to clean
- **Portable:** Easy to move from room-to-room with no cumbersome vacuum system

For your patients:

- **Quieter:** Less noisy than traditional cast saws, allowing for easier communication and reduced anxiety for patients
- **Pediatric-friendly:** A concealed blade that does not produce vibrations or heat – making it less intimidating for patients, especially children
- **Safer:** Operates without an oscillating blade and includes a protective foot and hidden blade, helping to mitigate the risk of cuts and burns
- **Cleaner:** Cuts casts without producing airborne particles providing a cleaner environment for patients

Description	Code	Selling Unit
1 Delta-Cast® Serene™ Kit	7628900	1 device 1 extra blade 1 power supply
Delta-Cast® Serene™ PIN	7628905	1 PIN
Delta-Cast® Serene™ Protective Foot	7628906	1 protective foot
Delta-Cast® Serene™ Replacement Blades	7628907	10 blades
Delta-Cast® Serene™ Power Supply	7628910	1 power supply



Casting Accessories



This sturdy foam knee rest is covered in washable vinyl for easy clean-up. The knee rest is used to aid in positioning a patient's limb for casting and to provide additional patient comfort during the casting process.

Description	Code	Selling Unit
Knee Rest	7309300	Unit
Adjustable Cast Stand	58050000	Unit

This adjustable cast stand makes applying casts easier and more comfortable for the patient. The upright can be raised or lowered from 31.8 to 53.3 cm to accommodate clinician and patient. Includes reusable plastic drip pan.



Casting Instruments

Casting instruments from Essity are effective for cutting casting material. They are handcrafted with high quality stainless steel to provide precision performance.

Description	Code	Selling Unit
(A) Medical Cast Spreader	7204611	1 / each
(B) Fiskars Powerarc Scissors	7204658	1 / each
(C) Clean Cut Scissors	7229402	1 / each
(D) Bohler Serrated Edge scissors 235 mm E	7293500	1 / each
(E) Lister Scissors 180 mm	7293200	1 / each
(F) Lister Scissors 200 mm	7293300	1 / each
(G) Small Precision Scissors	7293400	1 / each



Casting Accessories

Tensoplast® Skin Traction Kit

Adhesive skin traction system applied directly on the skin for non-invasive fracture reduction. Simple and effective, Tensoplast® Skin Traction Kit is used with weights (not included) and offers a reliable traction. (Not made with natural rubber latex)

Description	Code	Box Contents	Case Contents	Selling Unit
Skin traction kit (Adult)	7154400	1 kit	12 boxes	Case
Skin traction kit (Child)	7155400	1 kit	12 boxes	Case



Actimove® Cast Shoe Rocker Sole

The rocker sole helps ease pressure when ambulating and allows an easy roll through the gait pattern.

FEATURES AND BENEFITS

- Patients can feel safe while walking due to the non-slip sole
- Easy to apply and remove
- Protects the cast from water as the shoe is water resistant
- Lightweight
- Suitable for those sensitive to latex, as this product is not made with natural rubber latex
- For stock keeping, wearable or usable on either left or right foot

Description	Dimensions	Colour	Code	Selling Unit
Cast shoe rocker sole	Pediatric Infant	Black	6500480	Unit
Cast shoe rocker sole	Extra Small	Black	6500481	Unit
Cast shoe rocker sole	Small	Black	6500482	Unit
Cast shoe rocker sole	Medium	Black	6500483	Unit
Cast shoe rocker sole	Large	Black	6500484	Unit
Cast shoe rocker sole	Extra Large	Black	6500485	Unit



Actimove® Standard Walker and Standard Air walker

Orthopedic Walkers

Standard Walker

FEATURES AND BENEFITS

- Designed to stabilize and protect the foot, ankle and lower leg
- New design for better wearing comfort – at least 20% lighter than previous version
- Circumferential strapping for increased stability, a shock absorbing sole to minimize impact and improve comfort
- Rocker bottom to simulate natural gait
- Five sizes available in two heights

Standard Air Walker

FEATURES AND BENEFITS

- Integrated pump and valve system designed to easily adjust desired compression level
- New design for better wearing comfort – at least 20% lighter than previous version
- Circumferential strapping for increased stability, a shock absorbing sole to minimize impact and improve comfort
- Rocker bottom to simulate natural gait
- Five sizes available in two heights

Size	Shoe Size Men	Shoe Size Women	Code	Selling Unit
Standard Walker Low				
Extra Small	< 4.5	< 6	7627050	Unit
Small	4.5 - 7	6 - 8	7627051	Unit
Medium	7.5 - 10	8.5 - 11	7627052	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627053	Unit
Extra Large	> 12.5	> 13.5	7627054	Unit
Standard Walker High				
Extra Small	< 4.5	< 6	7627055	Unit
Small	4.5 - 7	6 - 8	7627056	Unit
Medium	7.5 - 10	8.5 - 11	7627057	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627058	Unit
Extra Large	> 12.5	> 13.5	7627059	Unit
Standard Air Walker Low				
Extra Small	< 4.5	< 6	7627250	Unit
Small	4.5 - 7	6 - 8	7627251	Unit
Medium	7.5 - 10	8.5 - 11	7627252	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627253	Unit
Extra Large	> 12.5	> 13.5	7627254	Unit
Standard Air Walker High				
Extra Small	< 4.5	< 6	7627255	Unit
Small	4.5 - 7	6 - 8	7627256	Unit
Medium	7.5 - 10	8.5 - 11	7627257	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627258	Unit
Extra Large	> 12.5	> 13.5	7627259	Unit



20%
lighter




[Table of Contents](#)

Actimove® Closed Shell Air Walker

Closed Shell Orthopedic Air Walker

Lightweight closed shell walker with integrated pump and valve system. Designed to offer the highest protection for the lower leg, ankle and foot.

The high-quality rigid outer skeleton for protection of the foot, ankle and lower leg. Circumferential strapping, a shock absorbing sole and breathable materials provide great comfort for the patient. Walking with Actimove® Walker products feels natural through the rocker bottom which simulates a physiological gait.

FEATURES AND BENEFITS

- Integrated pump and valve system designed to easily adjust desired compression level
- Air bladder embedded within the lining, less likely to be punctured
- Adjustable toe guard provides protection in the toe area
- Not made with natural rubber latex

Size	Shoe Size Men	Shoe Size Women	Code	Selling Unit
Closed Shell Air Walker Low				
Extra Small	< 4.5	< 6	7627630	Unit
Small	4.5 - 7	6 - 8	7627631	Unit
Medium	7.5 - 10	8.5 - 11	7627632	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627633	Unit
Extra Large	> 12.5	> 13.5	7627634	Unit

Closed Shell Air Walker High				
Extra Small	< 4.5	< 6	7627635	Unit
Small	4.5 - 7	6 - 8	7627636	Unit
Medium	7.5 - 10	8.5 - 11	7627637	Unit
Large	10.5 - 12.5	11.5 - 13.5	7627638	Unit
Extra Large	> 12.5	> 13.5	7627639	Unit

Description	Size	Code	Colour	Selling Unit
Actimove Walker Accessory				
Heel Wedge	Universal	7627796	Black	Unit



Actimove® Sling

Versatile Upper Limb Support

The Actimove® Sling on a roll is a convenient support system for the upper extremity. The combination of a strap with the strong Y-tab hook closures offers users and patients an easy-to-adjust custom fit for every patient with one single product.

FEATURES AND BENEFITS

- Highly versatile; one product, many applications
- Economical; replaces several products and sizes
- Stretch resistant foam keeps arm in desired position
- Strong Y-tab hook fasteners allow easy positioning adjustments of the arm
- Strap width remains constant during use which reduces tension and increases patient comfort
- Not made with natural rubber latex

Description	Dimensions	Code	Box Content	Selling Unit
Dispenser roll*	5.5 cm x 12 m	7285918	2 rolls	Box
Y-Tabs	-	7346600	14 hooks	Bag

* Includes 28 Y-Tab hook fasteners



Choosing the Right Bandage or Tape



ELASTIC BANDAGES

- Lightplast® Pro
- Co-Plus® LF
- Elastocrepe®
- Tensogrip®

ELASTIC ADHESIVE TAPES AND BANDAGES

- Tensoplast® / Tensosport®
- Leukotape® K
- Elastikon® by Actiomove®

RIGID STRAPPING TAPES

- Strappal®
- Leukotape® P
- Coach® by Actimove®
- Zonas® by Actimove®



Strappal®

Superior Strapping Tape

Constructed from viscose fabric combined with a zinc oxide adhesive, Strappal® is a high quality strapping tape offering support and efficient stabilization of joint structures.

INDICATIONS

- Rigid support for joints or strapping
- Limitation of movement

FEATURES AND BENEFITS

- Hand-tearable
- Rigid backing material
- Serrated edges
- Not made with natural rubber latex

Dimensions	Code	Box Contents	Selling Unit
1.25 cm x 10 m	7149615	36 rolls	Box
2.5 cm x 10 m	7148700	36 rolls	Box
4 cm x 10 m	7149000	24 rolls	Box
5 cm x 10 m	7149300	18 rolls	Box
7.5 cm x 10 m	7149401	6 rolls	Box



Leukotape® P

High Adhesive Rigid Strapping Tape

Leukotape® P is a high strength rigid strapping tape, with a strong zinc oxide adhesive. It is porous and hand-tearable, yet offers support for sprains and strains.

INDICATIONS

- Sprains & strains
- Ideal for patellofemoral taping
- Posture correction

FEATURES AND BENEFITS

- Porous
- Hand-tearable
- High strength adhesive

Dimensions	Code	Box Contents	Selling Unit
3.8 cm x 13.7 m	7616800	1 roll	Box





Rigid Cloth Porous Adhesive Strapping Tape

Support and conformity helps protect against sprains and injury. Adheres securely yet tears easily by hand.

FEATURES AND BENEFITS

- High tensile strength
- 100% cotton back cloth allows perspiration to escape

***Caution:** This product contains dry natural rubber.



Rigid Cloth Porous Adhesive Strapping Tape

For prophylactic use in active sport. Ideal for conditions requiring immobilization and support. Adheres securely yet tears by hand.

FEATURES AND BENEFITS

- Highest tensile strength
- 100% cotton back cloth allows perspiration to escape

***Caution:** This product contains dry natural rubber.



Elastic Adhesive Tape

Soft cotton elastic tape that conforms to joints such as elbows, knees and shoulders. Reliable compression for support of sprains, strains and muscle injuries.

FEATURES AND BENEFITS

- Allows skin to breathe and moisture to pass through
- Treatment of conditions which require limitation of movement, e.g. sprains, sports injuries
- Prophylactic use in active sport
- Retention of dressings and devices, e.g. post-orthopedic and general surgery or over a muscular wound

***Caution:** This product contains dry natural rubber.

Dimensions	Code	Colour	Selling Unit
3.8 cm x 13.7 m / 1.5" x 15 yds	6500001	White	32 rolls/case (Speed Pack)



Dimensions	Code	Colour	Selling Unit
2.5 cm x 9.1 m / 1" x 10 yds	6500104	White	12 rolls/box, 12 boxes/case 144 rolls/case



Dimensions	Code	Colour	Selling Unit
2.5 cm x 2.2 m / 1" x 2.5 yds	7308016	Cream	12 rolls/box, 12 boxes/case 144 rolls/case
7.6 cm x 2.2 m / 3" x 2.5 yds	7308017	Cream	4 rolls/box, 12 boxes/case 48 rolls/case
10.2 cm x 2.2 m / 4" x 2.5 yds	7308018	Cream	6 rolls/box, 6 boxes/case 36 rolls/case



Tensoplast®

Robust Elastic Adhesive Bandage

This porous elastic adhesive bandage is made from a high-quality extensible cloth, providing compression and support. The reliable adhesive keeps the bandage firmly adhered even throughout lengthy treatment.

INDICATIONS

- Firm support for injured joints
- To control effusion and swelling
- Retention of wound dressings over irregular areas and requiring compression

FEATURES AND BENEFITS

- Porous adhesive that allows skin to breathe
- Firm reliable adhesive
- Fluffy edges for comfort
- Central yellow line for accurate overlap

Dimensions*	Colour	Code	Box Contents	Case Contents	Selling Unit
Tensoplast®					
5 cm x 4.5 m	Cream	7205039	12 rolls	-	Box
7.5 cm x 4.5 m	Cream	7205040	12 rolls	-	Box
10 cm x 4.5 m	Cream	7205038	12 rolls	-	Box
5 cm x 4.5 m	Cream	7205037	1 roll	12 boxes	Case
7.5 cm x 4.5 m	Cream	7205043	1 roll	12 boxes	Case
10 cm x 4.5 m	Cream	7205042	1 roll	12 boxes	Case
Tensoplast® LF					
5 cm x 4.5 m	Cream	7623200	12 rolls	-	Box
7.5 cm x 4.5 m	Cream	7623201	12 rolls	-	Box
10 cm x 4.5 m	Cream	7623202	12 rolls	-	Box
Tenosport®					
5 cm x 4.5 m	Bleached	7205015	36 rolls	Box	Case
7.5 cm x 4.5 m	Bleached	7205016	24 rolls	Box	Case

*Stretched



Leukotape® K

Elastic Adhesive Tape

Leukotape® K is an elastic adhesive tape belonging to the group of kinesiology tapes. Kinesiology tapes may enhance proprioception, support lymphatic drainage, and help to reduce pain. Also available new larger size 25m co-branded as Leukotape K by Actimove.

INDICATIONS

Usage in kinesiology-type taping techniques

- Application fields include:
 - Lymphedema
 - Muscular pain
 - Disordered tonicity

FEATURES AND BENEFITS

- Full-day training available
- High wearing comfort and skin-friendly
- Durable; stays in place for up to 9 days
- Water-repellent: can be worn in shower and during sports
- Easy to apply
- Not made with natural rubber latex

Dimensions	Colour	Code	Box Contents	Pack Contents	Selling Unit
5 cm x 25 m	Beige	6507807	1 roll	1 box	Pack
2.5 cm x 5 m	Beige	7297810	1 roll	5 boxes	Pack
5 cm x 5 m	Beige	7297811	1 roll	5 boxes	Pack
7.5 cm x 5 m	Beige	7297812	1 roll	5 boxes	Pack
2.5 cm x 5 m	Red	7297815	1 roll	5 boxes	Pack
5 cm x 5 m	Red	7297816	1 roll	5 boxes	Pack
7.5 cm x 5 m	Red	7297817	1 roll	5 boxes	Pack
2.5 cm x 5 m	Blue	7297820	1 roll	5 boxes	Pack
5 cm x 5 m	Blue	7297821	1 roll	5 boxes	Pack
7.5 cm x 5 m	Blue	7297822	1 roll	5 boxes	Pack
5 cm x 5 m	Black	7297823	1 roll	5 boxes	Pack
5 cm x 5 m	Pink	7297825	1 roll	5 boxes	Pack



Tensoplast®

Robust Elastic Adhesive Tape

This porous elastic adhesive tape is made from a high-quality extensible cloth, providing firm controlled compression and strong support. The reliable adhesive keeps the bandage firmly adhered even throughout lengthy treatment.

INDICATIONS

- For secure fixation or taping with maximum conformability and comfort
- Sprains & strains requiring supportive treatment
- Retention of wound dressings over irregular areas and requiring compression
- Support bandaging of thumb and finger injuries

FEATURES AND BENEFITS

- Porous adhesive that allows skin to breathe
- Fully-spread adhesive assures firm adhesion

Dimensions*	Code	Box Contents	Case Contents	Selling Unit
2.5 cm x 4.5 m	7206902	1 roll	6 boxes	Case
5 cm x 4.5 m	7205218	6 rolls	-	Box
7.5 cm x 4.5 m	7205219	4 rolls	-	Box

*Stretched



Tensoplast®

Athletic Elastic Adhesive Tape

Tensoplast® is a light elastic adhesive tape which is suitable for medium support or compression. Its adhesive properties are long lasting and provide controlled support.

INDICATIONS

- Sprains & strains
- Correction of false position & posture

FEATURES AND BENEFITS

- High adhesive properties
- Provides medium support and compression
- Water repellent
- Air permeable
- Thinner substrate

Dimensions*	Code	Box Contents	Selling Unit
5 cm x 4.5 m	04412001	24 rolls	Box
7.5 cm x 4.5 m	04413001	16 rolls	Box
10 cm x 4.5 m	04414001	16 rolls	Box

*Stretched



Lightplast® Pro

Light Elastic Adhesive Tape

Lightplast® Pro is a hand-tearable light elastic adhesive tape, suitable for light support or compression. The unique fabric backing helps to conform to body movement, and make it ideal for all-purpose taping.

INDICATIONS

- Sprains & strains
- Prophylactic use against sport injuries

FEATURES AND BENEFITS

- Hand-tearable
- Good adhesive strength
- Lightweight
- Conforms well
- Lengthwise elastic

Dimensions*	Code	Box Contents	Selling Unit
2.5 cm x 4.5 m	7695200	48 rolls	Box
3.75 cm x 4.5 m	7695300	32 rolls	Box
5 cm x 4.5 m	7695400	24 rolls	Box
7.5 cm x 4.5 m	7695500	16 rolls	Box
5 cm x 6.8 m	7695600	24 rolls	Box
7.5 cm x 6.8 m	7695700	16 rolls	Box

*Stretched



Co-Plus® LF

Elastic Cohesive Bandage

Co-Plus® LF flexible elastic cohesive bandage provides non-slip light support and compression. Because of its unique construction, Co-Plus® LF adheres to itself and not to skin, aiding in patient comfort. No clips or fasteners required.

INDICATIONS

- Light support and compression
- Retention of splinting system

FEATURES AND BENEFITS

- Secure dressing fixation
- Non-slip support
- Air-permeable
- Hand-tearable
- Not made with natural rubber latex



Dimensions*	Colour	Code	Box Contents	Selling Unit
Co-Plus® LF - Latex Free				
2.5 cm x 4.5 m	Beige	7210060	72 rolls	Box
5 cm x 4.5 m	Beige	7210016	36 rolls	Box
7.5 cm x 4.5 m	Beige	7210017	24 rolls	Box
10 cm x 4.5 m	Beige	7210018	18 rolls	Box
15 cm x 4.5 m	Beige	7210019	12 rolls	Box
5 cm x 4.5 m	Mixed Colours**	7210033	36 rolls	Box
7.5 cm x 4.5 m	Mixed Colours**	7210020	24 rolls	Box
10 cm x 4.5 m	Mixed Colours**	7210021	18 rolls	Box

*Stretched

**Mixed Colours: Neon Pink, Dark Blue, Light Blue, Purple, Neon Green and Red

Elastocrepe®

Non-Adhesive Elastic Bandage

Elastocrepe® is made of high quality stretch cotton/ rayon fabric with non-fraying edges. The special weave helps avoid excessive stretching and provides targeted compression.

INDICATIONS

- Sprains & strains
- Edema control (post-cast removal)

FEATURES AND BENEFITS

- Lightweight and conformable
- Non-fray edges
- Air-permeable
- Provides support due to longitudinal elasticity
- Central yellow line for accurate overlap
- Not made with natural rubber latex (except clips)

Dimensions*	Code	Box Contents	Selling Unit
5 cm x 4.5 m	7208247	12 rolls	Bag
7.5 cm x 4.5 m	7208249	12 rolls	Bag
10 cm x 4.5 m	7208243	12 rolls	Bag
15 cm x 4.5 m	7208267	6 rolls	Bag

*Stretched



Tensogrip®

Tubular Elastic Support Bandage

Tensogrip® is an elasticized, tubular support bandage, ideal for the effective management of strains, sprains and soft tissue injuries. It provides pressure and support over varying body contours.

INDICATIONS

- Light support and compression
- Retention of wound dressings

FEATURES AND BENEFITS

- Provides light support, light compression and dressing retention
- Horizontal stretch
- Easily cut to size
- Air-permeable

Dimensions	Code	Box Contents	Selling Unit
4.5 cm x 10 m, Size A	7151300	1 roll	Box
6.25 cm x 10 m, Size B	7151400	1 roll	Box
6.75 cm x 10 m, Size C	7151900	1 roll	Box
7.5 cm x 10 m, Size D	7151500	1 roll	Box
8.75 cm x 10 m, Size E	7152000	1 roll	Box
10 cm x 10 m, Size F	7151600	1 roll	Box
12 cm x 10 m, Size G	7152100	1 roll	Box
17.5 cm x 10 m, Size J	7151700	1 roll	Box
21.5 cm x 10 m, Size K	7152300	1 roll	Box
32.5 cm x 10 m, Size L	7152200	1 roll	Box
37.5 cm x 10 m, Size M	7152400	1 roll	Box





 ca.customerservice@essity.com

 essity.com

Essity
1275 North Service Road West, Suite 800
Oakville, ON Canada L6M 3G4
Tel. 1-877-978-5526 Fax 1-877-978-9703
136589E ©Essity Hygiene and Health AB, 2026 D26