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Cutimed® PROTECT

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Medical skin protection – Cream

Description of the product:

Cutimed® PROTECT is a barrier cream that protects the skin from bodily fluids. It protects dry, chafed, red, or irritated skin by moisturizing the skin and providing a long lasting barrier. The cream will not impair adhesion of adhesive dressings or medical tapes.

Indications:

Cutimed® PROTECT Cream protects skin from damage and irritation, particularly at-risk areas associated with incontinence. Cutimed® PROTECT Cream safeguards skin from damage associated with chronic incontinence. It moisturises and protects dry skin.

Contraindications:

Cutimed® PROTECT Cream should not be used on skin where an infection is present or used if there is a known allergic sensitivity to the ingredients.

Warning:

If skin condition does not improve within 7 days (or worsens), consult your doctor. Keep out of reach of children. For external use only. In case of ingestion, seek medical help. Avoid contact with eyes. Cutimed® PROTECT Cream may increase the adherence of some adhesive products.

Directions for use:

- (1) Gently cleanse skin.
- (2 – 3) Apply Cutimed® PROTECT Cream thinly to cover entire affected area.
- (4) Repeat as necessary.
- Please note that if the area feels greasy, too much product has been applied.
- Cutimed® PROTECT Cream shall be applied by healthcare professionals only.

Ingredients:

Purified Water, Ethylhexyl Isnonanoate, Disiloxane, Polyisobutylene, VP Eicosene Copolymer, Hydroxyethyl Acrylate / Sodium Acryloyl Dimethyl Taurate Copolymer, Butylene Glycol, Oleth-3 Phosphate, Phenoxyethanol, Hydrogenated Styrene/Isoprene Copolymer, Alantoin, Carbomer, Triethanolamine, Methylparaben, Propylparaben, Disodium EDTA.

Storage:

Cutimed® PROTECT Cream should be stored in dry conditions. Do not expose to direct sunlight. Store between 5°C and 30°C.

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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